FARGO

01-3520

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FI

EN**SYMBOLS**

In this Instruction for Use warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger or basic information.

Gives basic information, recommendations and information for efficient, trouble-free use.

WARNING / CAUTION - to indicate that caution is necessary when operating the device. To indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

Attention! This sign is used for very important parts concerning your safety. Please look at the operating instructions.

SAFETY INFORMATION**WARNING**

The rollator is only suitable for single occupancy.

**CAUTION**

Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's toys, etc. Do not reach for items further than your arm will extend.

**WARNING**

It is forbidden to smoke or to be close to the flames, using a rollator.

WARNING

The rollator is not intended for use by the blind.

CAUTION

Use slow speeds on gradients. Do not exceed the maximum gradient stated. Do not use escalators. Do not reverse down a gradient.

CAUTION

Risk of getting fingers caught. There is always a risk of getting, e.g. fingers or arms, caught in the moving parts of the rollator.

WARNING

It is forbidden to exceed the maximum safe load of product.

CAUTION

While unpacking, be aware of the risk of injury from catching your fingers between the spokes and the fork.

WARNING

When using a rollator, use both brakes simultaneously to prevent the unbraked wheel from twisting and causing the rollator user to fall.

GENERAL

INTRODUCTION

This Instruction for use contains important information about the handling of the product. To ensure safety when using the product, read the Instruction for use carefully and follow the safety instructions. FARGO 4-wheel rollator is a walking device for indoor and outdoor use perfect for compensation and improvement respectively of restricted ability of walking.

PREFACE

Thank you for the confidence shown in Kid-Man®. Every product is factory checked and left us in faultless condition. We wish you much success and we hope UAB Kasko Group | Kid-Man® rollator will make a positive impact to your daily living.

WARRANTY

We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB Kasko Group | Kid-Man® (hereinafter called Manufacturer) after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, Manufacturer, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

LIMITATION OF LIABILITY

Manufacturer accepts no liability for damage arising from:

- Non-compliance with the Instruction for use;
- Incorrect use;
- Natural wear and tear;
- Incorrect assembly or set-up by the purchaser or a third-party technical modifications;
- The usage of unapproved 3rd party accessories instead of approved Manufacturer's accessories;
- Unauthorized modifications and/or use of unsuitable spare part.

STANDARDS AND REGULATIONS

The 4-wheel rollator complies the requirements of:

- a) REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009 and repealing Council Directives 90/385/EEC and 93/42/EEC.
- b) (ISO 11199-2:2021) Assistive products for walking manipulated by both arms - Requirements and test methods - Part 2: Rollators (ISO 11199-2:2021) EN ISO 11199-2:2021.

SERVICE LIFE

The expected service life is 5 (five) years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and in tended use, stated in this Instruction for Use.

After then, 4-wheel rollator must be recycled. Do not throw your rollator away with common waste (see chapter “DISPOSAL”).

INFORMATION ABOUT MANUFACTURER

Manufacturer: UAB “Kasko Group” (registered trade mark Kid-Man®)

Address A.Vivulskio 12D, Vilnius, Lithuania,

Phone +370 5 265 0000

E-mail info@kid-man.com

Website www.kid-man.com

INFORMATION ABOUT PRODUCT

Product: **4-WHEEL ROLLATOR**

Model: **FARGO**

REF: **01-3520**

INTENDED PURPOSE AND INDICATIONS

4-WHEEL ROLLATOR FARGO is a walking device for indoor and outdoor use, perfect for compensation and improvement respectively of restricted ability of walking.

i This 4-wheel rollator needs to fit to your specific health condition.

CONTRA-INDICATIONS

There are no contraindications known, when using the 4-wheel rollator as intended.

INTENDED USERS

FARGO 4-WHEEL ROLLATOR is intended to compensate and improve mobility of a person with restricted ability of walking with a maximum weight capacity of 135 kg.

WARNING

Risk of accidents. It is forbidden to exceed the maximum safe load of 4-wheel Rollator.


CAUTION

The 4-wheel rollator is appropriate for use by others. In order to do this, the 4-wheel rollator must first be thoroughly cleaned and disinfected. Then, the all components of the 4-wheel rollator must be get checked by an authorised service dealer for general condition, wear and shall get repaired if necessary.

PRODUCT LABEL

The identification label is attached to the frame of your 4-wheel rollator (see picture).

1. Product
2. Model
3. Maximum load
4. Manufacturer / contact details
5. Manufacturer date
6. UDI / SN code
7. CE mark
8. Consult instructions for use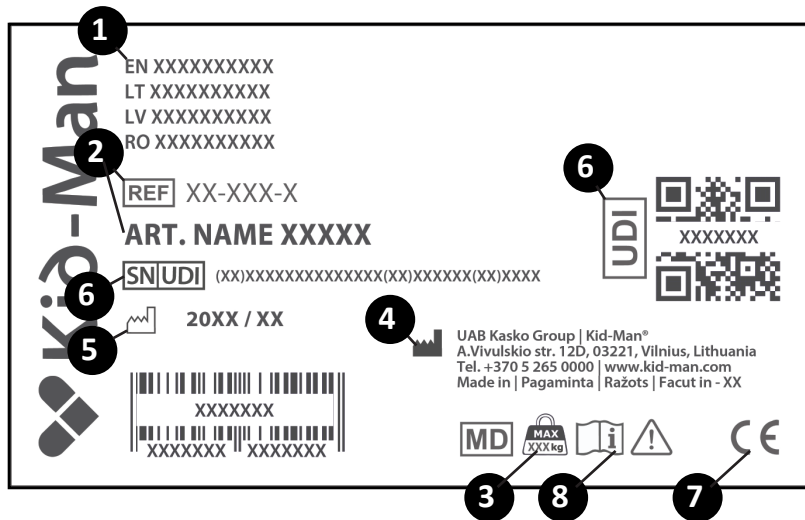

CONTENT OF PACKAGE

The 4-wheel rollator FARGO is delivered in a cardboard packaging and is ready for use. After unpacking, please keep the cardboard packaging if possible. You may need it later for storage or for the return of the device.


Once parcel is received, please check whether it is not damaged during transportation.

Content of package:

1. 1 x folded rollator
2. 1 x backrest
3. 4 x knob for rollator assembly
4. 1 x bag
5. 1 x walking stick holder
6. Instruction for use

i If you see signs of damage, please make note to transport company on the same time.

i Please check content of package and if any of below parts are missing, get in touch with your local dealer.

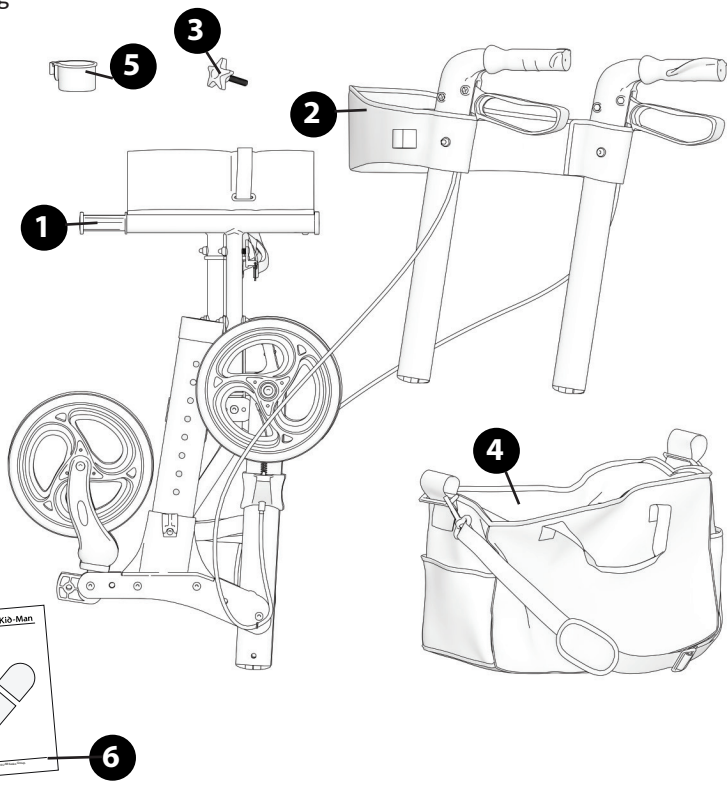COMPONENTS AND FUNCTIONS

Overview

1. Handle
2. Brake
3. Knob for rollator assembly
4. Frame
5. Rear wheel
6. Caster
7. Bag
8. Brake wire
9. Seat
10. Backrest
11. Height adjustment button for handle



UNFOLDING AND FOLDING THE ROLLATOR

- When unpacking 4-wheel rollator need to unscrew knobs near to the folded wheels. When wheel are unbend - insert knob again back to screw hole.
- To unfold the 4-wheel rollator:
 - 1) Position the 4-wheel rollator in front of you;
 - 2) With your hands flat, press the seat frame down, so that the seatsurface flattens out and the seat is completely set between the frame (Picture1).
- To fold the 4-wheel rollator:
 - 1) Hold the 4-wheel rollator handle;
 - 2) Raise the seat by using the lifting strap to lift the 4-wheel rollator. The Rollator will fold as you lift it off the ground. (Picture 2).


**WARNING**

Danger of trapping your finger while folding or unfolding 4-wheel rollator!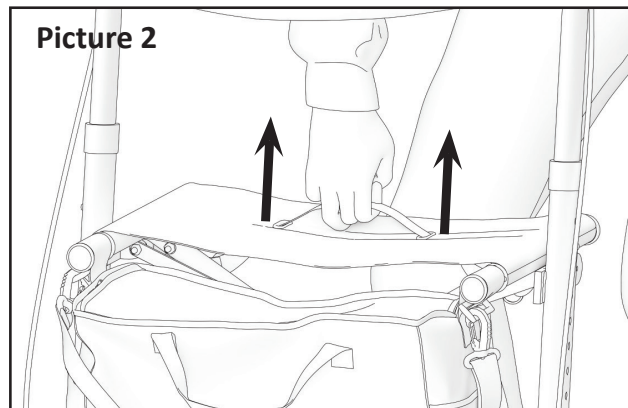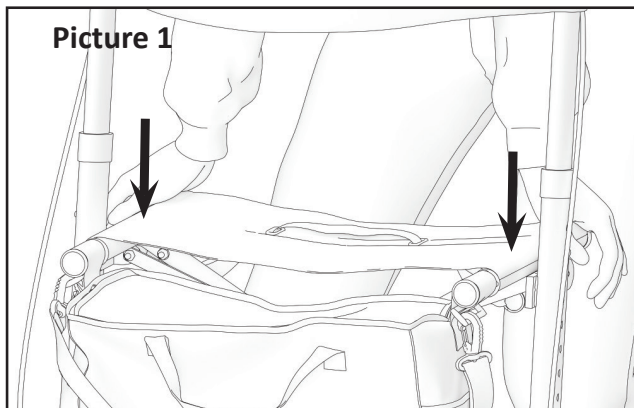


SITTING ON A ROLLATOR

Make sure the rollator is on the parking brake set before you sit on the seat of your rollator. When you sit down, please make sure that your feet are between the rear wheels.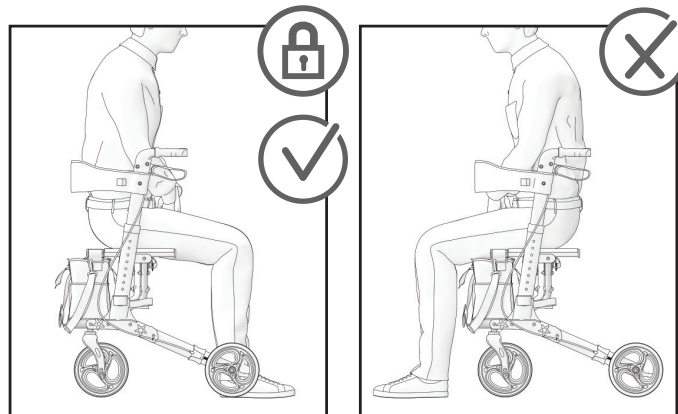

HANGING BAG

To put textile bag on the rollator, need to hang it on two bag holders that are in the front of the rollator.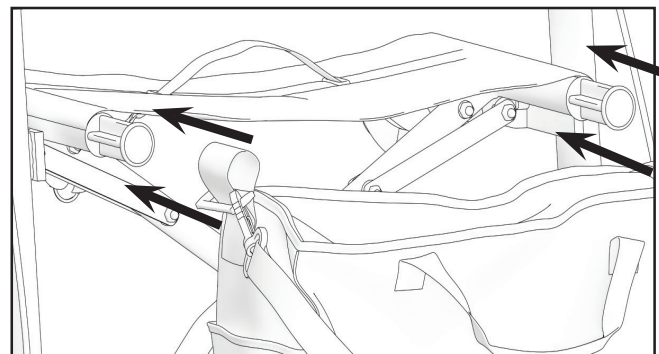


OPERATION**WARNING**

There are no positions of points intended to carry additional loads.


BRAKING

The efficiency of the brakes can be affected by abrasion of the wheels. Therefore, test the brakes regularly. For desired braking strength adjust the brake either manually at the screw on the housing of the brake or with the suitable tool (not included) on the brake wire (See Picture 3).
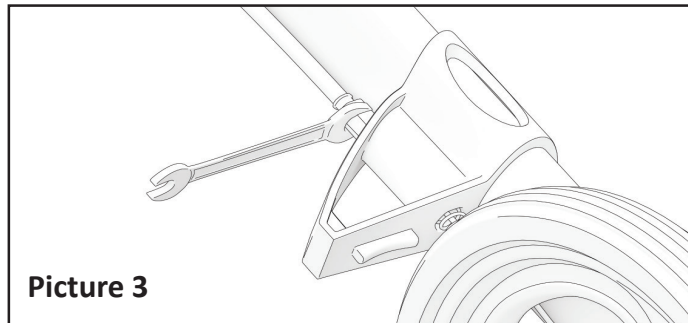
**TO OPERATE THE BRAKES**


To activate the brakes, lift the brake levers up and hold them there (as indicated by the arrow).

To release the brakes, release the levers (Picture 4).

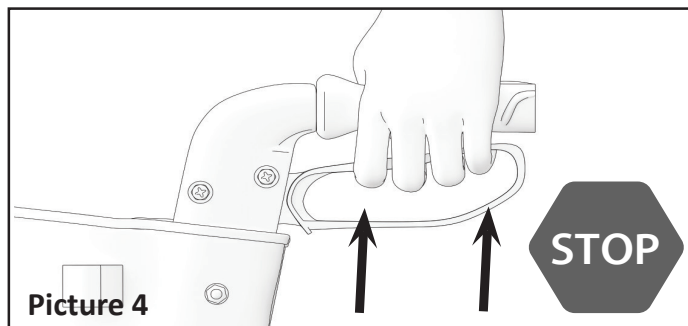To engage parking brakes of the 4-wheel rollator, depress the brake levers until it locks and remains in the down position, as shown by the arrow (Picture 5).

 **WARNING**
Before settling on your seat, make sure the brakes are engaged.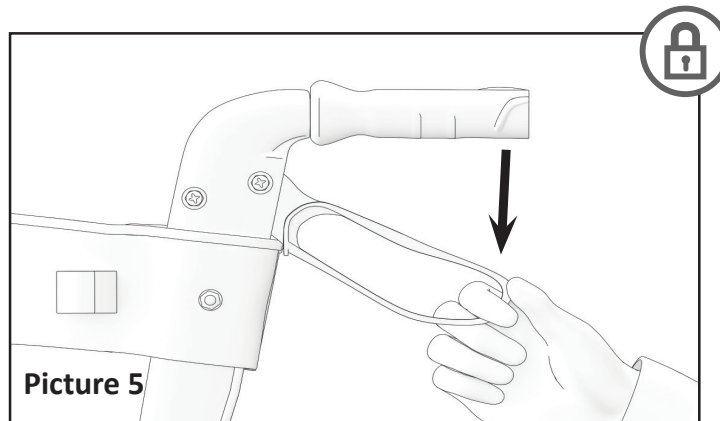


ADJUSTING THE HEIGHT OF THE HANDLES

Using the metal button on the left and right sides of the 4-wheel rollator. (Picture 6).

 **CAUTION**
Make sure both handles are set in the same position.

Picture 6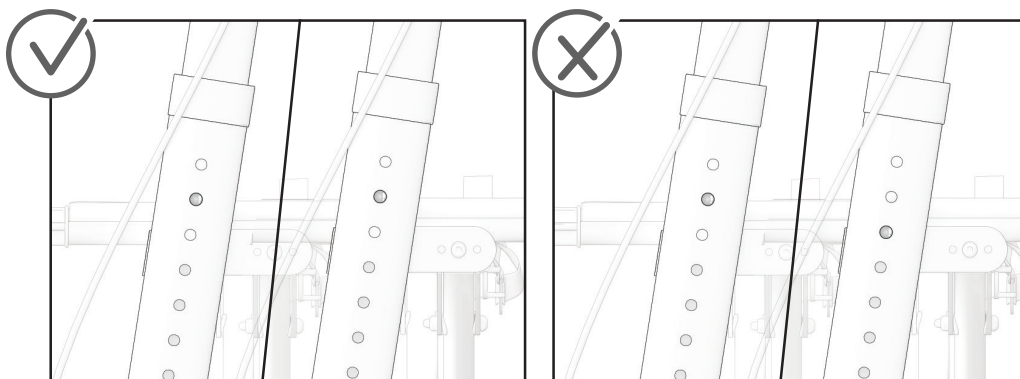


WALKING

When you walk with your rollator make sure you walk upright, with the rollator near your body. It will give you the correct body posture (Picture 7).

**WARNING**


The rollator is only suitable for single occupancy.

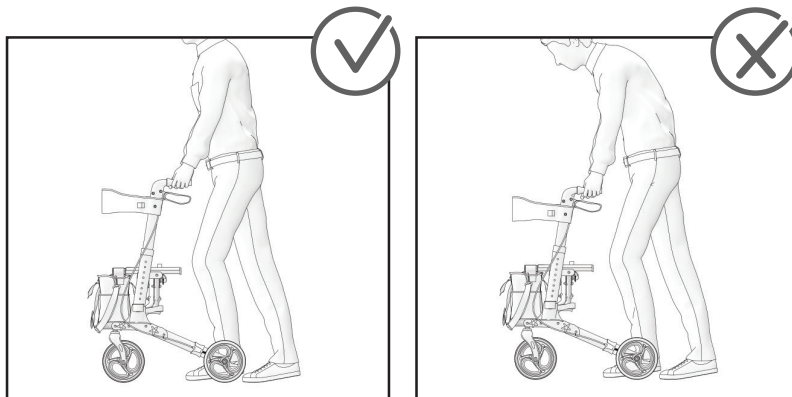Picture 7**TIPPING AID**

When you want to lift the rollator, use the tipping aid mounted next to rear wheels. Firstly, place your foot on the tipping aid. By pushing the tipping aid down, the casters are lifted thus allowing using to cross vertical obstacle. Then using the rollator normally, please keep your feet far from tipping aid.

**WARNING**

Do not attempt to ride over curbs or obstacles. Doing so may cause your rollator to tip over and cause bodily harm to you or damage to the rollator.

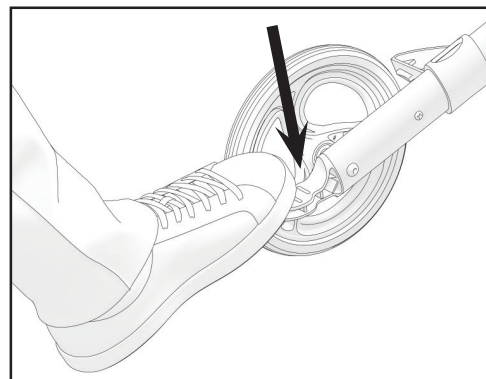Picture 8

MAINTENANCE

To ensure safe and reliable operation, carry out the following checks and maintenance regularly or have it carried out by another person. Below You will find recommended maintenance's schedule.

	Before use	Once a week	Once a month	Once a year
Check that wheels are seated correctly		X		
Visual check		X		
Check the folding mechanism			X	
Check the brakes	X		X	
Clean the wheels			X	
Have 4-wheel rollator checked by an authorized dealer				X

WARNING

Repair, cleaning, or adjustment of 4-wheel rollator should be performed by authorized service person while person is not sitting or using the 4-wheel rollator.

CLEANING AND DISINFECTION

Should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent.

- i** Sand and seawater can damage the bearings and steel parts of rollator can rust if the surface is damaged.
- i** If the 4-wheel rollator is exposed by sand and seawater for short periods, clean it after every trip to the beach.

STORAGE

The product should be stored in a dry environment, away from direct sunlight.

ENVIRONMENTAL CONDITIONS

Keep 4-wheel rollator under these conditions:

- Temperature limitation - 20° C to + 40° C;
- Relative humidity 20% to 75%;

RE-USE

The 4-wheel rollator is suitable for re-use. The following actions should be taken:

Cleaning and disinfection (see chapter „CLEANING AND DISINFECTION “).

Inspection (see chapter „MAINTENANCE “).

The 4-wheel rollator should be adapted for the new user in accordance with the service documentation.

DISPOSAL

Be environmentally aware and dispose of your 4-wheel rollator properly. Disposal is subject to national and local regulations. To have 4-wheel rollator properly recycled, please contact your local community, disposal facility or our service department.

INCIDENTS

Any serious incident that has occurred in relation to the device should be reported to UAB Kasko Group | Kid-Man® and the competent authority of the Member State in which the user and/or patient is established.

Contacts

UAB Kasko Group | Kid-Man® (registered trade mark Kid-Man®)

Address A.Vivulskio 12D, Vilnius, Lithuania,

Phone +370 5 265 0000

E-mail info@kid-man.com

Website www.kid-man.com

SPECIFICATION

Maximum safe load	135 kg
Overall weight	8.8 kg
Height of handgrips (7 positions)	810 - 930 mm
Overall width	670 mm
Overall length	810 mm
Seat height	545 mm
Width between handrips	430 mm
Size of caster	200 x 35 mm

i All materials used are protected against corrosion. We use only REACH compliant materials and components.

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