## TULIS

---

ENG USER MANUAL

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EE KASUTUSJUHEND

RO MANUAL DE UTILIZARE

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<b>EN</b>	USER MANUAL .....	8-9
<b>LT</b>	VARTOTOJO VADOVAS .....	10-11
<b>LV</b>	LIETOTĀJA ROKASGRĀMATA .....	12-13
<b>EE</b>	KASUTUSJUHEND .....	14-15
<b>RO</b>	MANUAL DE UTILIZARE .....	16-17
<b>SL</b>	UPORABNIŠKI PRIROČNIK .....	18-19
<b>CZ</b>	UŽIVATELSKÁ PŘÍRUČKA .....	20-21
<b>FI</b>	KÄYTTÖOPAS .....	22-23

**1.****2.****3.\*****4.****\***

Accessories (3, 5, 6, 7) not included. To purchase - contact Your local dealer.  
Priedai (3, 5, 6, 7) nepridedami. Norėdami įsigyti, kreipkitės į vietinį platintoją.  
Piederumi (3, 5, 6, 7) nav iekļauti. Lai iegādātos, sazinieties ar vietējo izplatītāju.  
Tarvikud (3, 5, 6, 7) ei kuulu komplekti. Ostmiseks võtke ühendust kohaliku edasimüüjaga.

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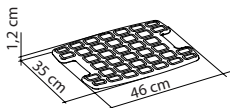
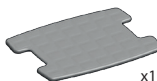
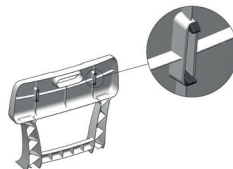
 150 kg

### INTENDED USAGE:

The products of the Commode Chairs line allow the user to achieve greater autonomy and independence while using the toilet and the shower, reducing muscle fatigue due to standing position as well as slip and injury risks. The commode chair can be set up in three different configurations:

- Placed directly on the WC as an alternative to the direct use of it, allowing the end user to keep a wider and more comfortable seat than those typical of the toilet bowl.
- The use in combination with the bucket, available separately as an accessory, makes the seat suitable for the proper use as a commode even outside the bathroom.
- By equipping the rigid cushion available separately as an accessory, the commode can be used as a shower chair.

The legs are height adjustable, so the user can choose the specific length according to their needs. The legs can also be configured so that the front ones are shorter than the rear ones. This promotes maintaining a healthier posture and reduces the bending of knees and hips. The seat is provided with anti-slip caps, and given its flexibility, it also maintains stability on bumpy floors, like those typical of many shower trays. The versions provided with backrest and/or armrests give additional safety and comfort. The commode can be easily moved thanks to its reduced weight. The seat can be completed with a soft cushion and a soft cushion for the backrest to make the use more comfortable. The soft cushions, armrests, backrest, and buckets are available separately as accessories.

### INDICATIONS AND CONTRAINDICATIONS:

The products of the Commode Chairs KING line are aimed to elderly people, wheelchair users, people with reduced mobility of the lumbar, knee, or hip, and all those who have difficulty maintaining the standing position during the shower stages of daily personal hygiene or that are in need of more stability while using the WC. The user is required to have autonomy of balance during sitting and standing procedures as well as to have adequate ability to keep sitting. The commode chair must not be used if the user has problems related to balance or perception disorders.

### SAFETY WARNING AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceeded.
- Before every use, be sure the commode chair is correctly assembled in a stable position.
- The commode chair can be used only on a surface that is free from obstacles.
- During assembly, strictly adhere to what is shown in the User Manual.
- Before positioning the commode on the WC, check that the bucket supports are in the closed position.
- The risk of finger injuries may increase during the mounting process.
- Do not swing on the shower seat; this may lead to slipping, overturning, or subsidence of the legs. Always keep the shower seat on its 4 legs.
- Do not modify the shower seat and its components in any way.
- Do not use the commode chair if damaged or altered in any way.
- After use, the commode chair should be removed from the shower tray to avoid any obstruction.
- Do not use the commode chair for different purposes from the intended one. The commode chair must not be stored outdoors.

### MAINTENANCE INFO:

Product maintenance consists of cleaning and sanitation procedures. In order to guarantee safe use and an adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components, and the tightness of the screws must be checked. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.

2. Remove the evident dirt before proceeding with the most accurate disinfection.
  3. Use water or non-aggressive detergent for the cleaning procedure.
  4. Wipe with a dry and clean cloth.
- Alternatively, the product can be cleaned in domestic dishwashers at the maximum temperature. The soft cushion must be cleaned strictly by hand.

#### REUSE REQUIREMENTS:

The product is suitable for reuse by other users. Before reusing by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

#### CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the regular household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose of it.

#### INCIDENT REPORTING:

If there is a serious incident involving this device, contact the manufacturer of this product at [info@kid-man.com](mailto:info@kid-man.com) and the responsible authority in your country.

ITEM NUMBER	DESCRIPTION
1.	Seat
2.	Backrest
3.	Bucket with lid
4.	Armrest
5.	Sof cushion holed
6.	Soft cushion for seat
7.	Soft cushion for backrest

ICON	DESCRIPTION
	Made in Italy
	Tools Free Assembly
	Dishwasher Friendly
	Max User Weight
	2 Years Warranty

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