TUALETO KĖDĖ
TUALETES KRESLS
TUALETT TOOL
COMMODE CHAIR
ТУАЛЕТНЫЙ СТУЛ
SCAUN TOALETA CU SEZUT MOALE
TOALETNÍ ŽIDLE
TOALETNI STOL

Naudojimo ir priežiūros instrukcija
Lietošanas un kopšanas instrukcija
Kasutusjuhend
User manual
Руководство по эксплуатации
Manual de utilizare
Uživatelský manuál
Uporabniški priročnik

04-7410

04-7420

- Prieš naudodami atidžiai perskaitykite naudojimo ir priežiūros instrukciją!
- Pirms lietojiet, lūdzu izlasiet uzmanīgi lietošanas un kopšanas instrukciju!
- Enne kasutamist lugege hoolikalt kasutusjuhendit!
- Observe instructions for use! Please read these instructions carefully before use and keep it for later reference!
- Прежде чем использовать внимательно прочтите инструкцию!
- Respectați instrucțiunile de utilizare! Citiți aceste instrucțiuni cu atenție înainte utilizării și păstrați-le pentru referințe ulterioare!
- Dodržujte návod k použití! Před prvním použitím si pečlivě přečtěte tento návod a uchovejte ho pro pozdější použití!
- Upoštečajte navodila za uporabo! Pred uporabo natančno preberite ta navodila in jih shranite za poznejšo uporabo!

PRATARMĒ / IEVADS / EESSŌNA / PREFACE / ПРЕДИСЛОВИЕ / PREFAŢĂ / PŘEDMLUVA / PREDGOVOR

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EN Model 04-7420 comes already assembled. Model 04-7410 needs to be folded out as it is showed in Pic. No. 1

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EN The bucket and lid are accommodated on the seat. To add the bucket, just hang it on special metal bracket, however on model 04-7410 bucket to be installed from the back of seat on special metal bracket (Pic. No 2).

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EN Commode is perfectly suited for side transfer (for instance, from bed to commode). Before side transfer, lift the armrest as showed in picture (Pic. No. 3). Once seated, both armrest should be in down position.

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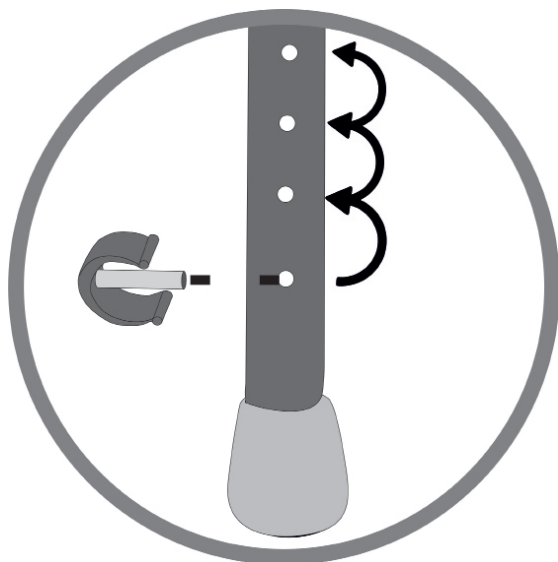
EN Commode chair C-clips should be checked periodically if they fix legs properly (model 04-7420), in order to avoid injuries and ensure safety.

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EN The commode chair should be stored in a dry environment, away from direct sunlight. Should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent

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EN We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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