**01-3900**

**01-3203**

**01-3016W5**

**VAIKŠČIOJIMO RĖMAS | STAIGĀŠANAS RĀMIS | KÄIMISRAAM | WALKING FRAME | РАМА ДЛЯ ХОДЬБЫ |  
HOJICA**

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**EN****PREFACE**

- Thank you for the confidence shown in Kid-Man. Every product is factory checked and left us in faultless condition, We wish you much success and we hope Kid-Man product will make a positive impact to your daily living.

**INDICATIONS**


- Kid-Man walking frame are suitable for everyday indoor and outdoor use. All Kid-Man handcarts are manufactured to the highest quality standards and are CE marked as medical devices. Walking frame made of anodized aluminium tubes.

**SET UP**


- Walking frame come fully assembled and ready to be used.

**USAGE**


- Hold padded armrest and pull both sides outwards until push button gets in place. The click sound will appear. Ready to be used walking frame gets „U“ shaped. (Pic. No. 2) Once frame in opened, please set the height of frame according user's needs. (Pic. No. 3)
- Once unused, it is recommended to close the frame. With one hand pushing plastic holder in the middle of frame, fold one side in and repeat the same with other side. (Pic. No. 1) Closed frame takes less space and is more convenient for storage.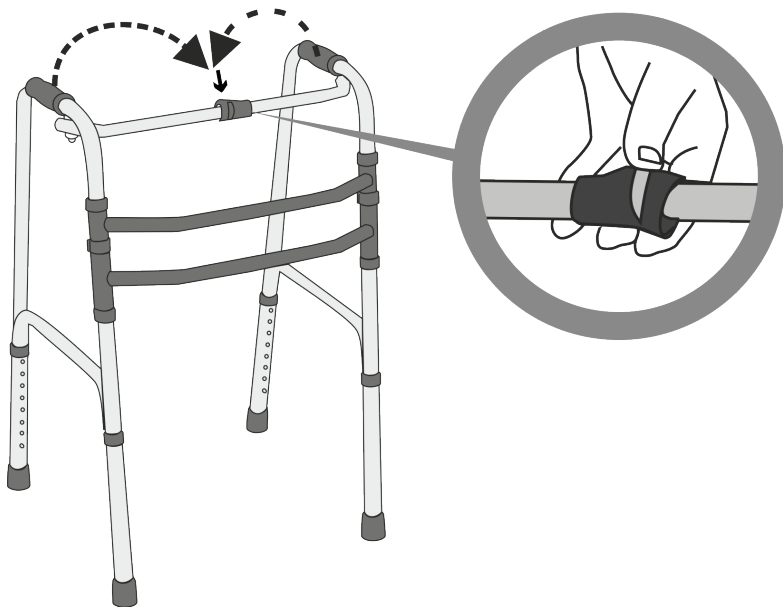

Pic. No. 1

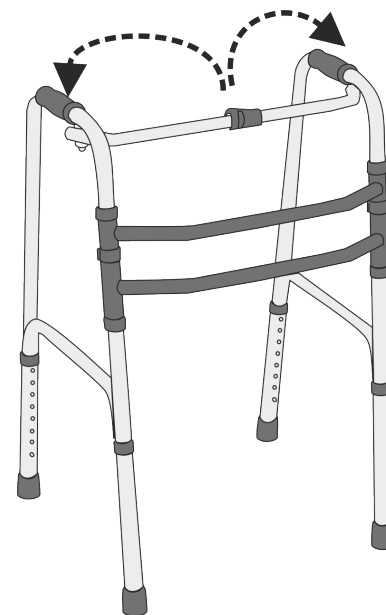
Pic. No. 2

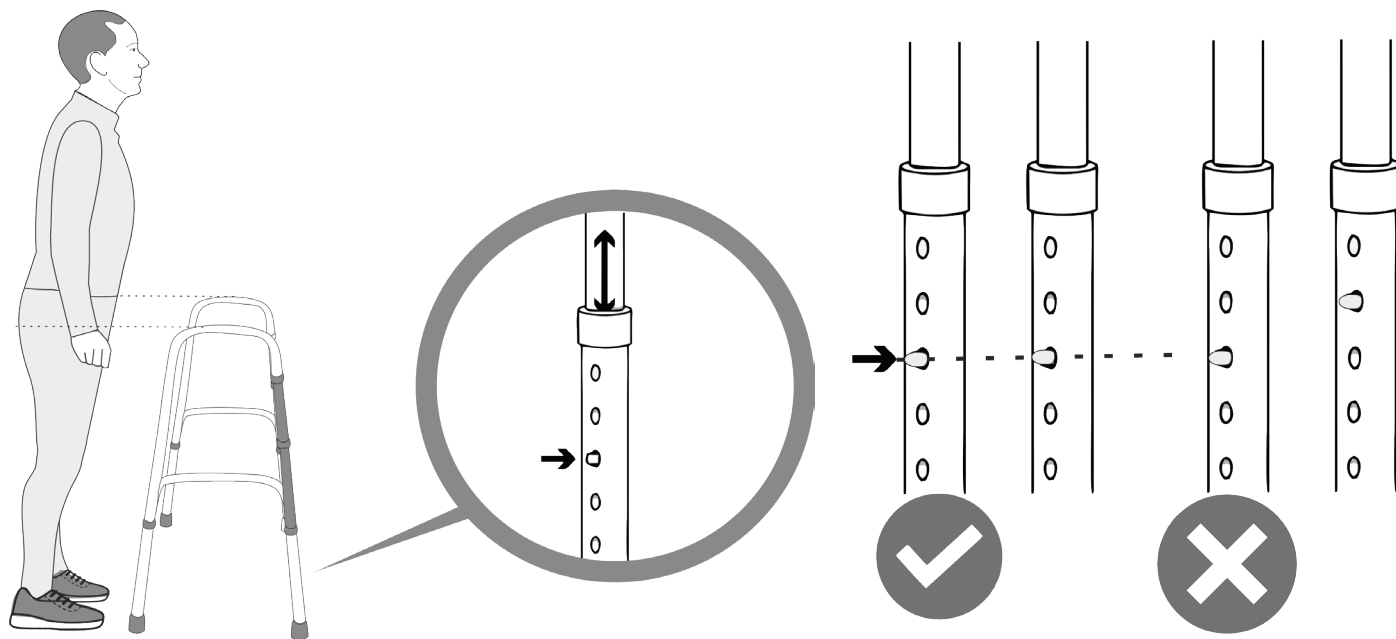Pic. No. 3





• **Warning!** Before using device, make sure all feet are set in the same height.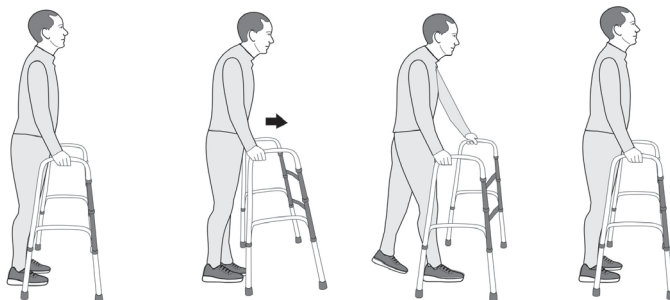

01-3900

01-3203

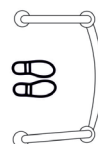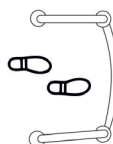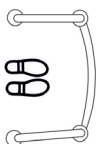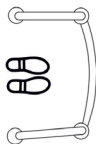01-3016W5

### CARE AND MAINTENANCE

- Only original spare parts can be used for replacement. Please provide model and serial number to dealer or contact manufacturer directly. Manufacturer and dealer is not responsible for any damage occurred to product or user if non original spare parts are used and (or) repair was done in unauthorized facility.

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**EN** We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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