01-3900

01-3203

01-3016W5

VAIKŠČIOJIMO RĖMAS | STAIGĀŠANAS RĀMIS | KÄIMISRAAM | WALKING FRAME | РАМА ДЛЯ ХОДЬБЫ | HODULJA | CADRU DE MERS

Naudojimo ir priežiūros instrukcija | Lietošanas un kopšanas instrukcija | Kasutusjuhend | User manual
Руководство по эксплуатации | Priročnik za uporabnike | Manual de utilizare

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EN**PREFACE**

- Thank you for the confidence shown in Kid-Man®. Every product is factory checked and left us in faultless condition, We wish you much success and we hope Kid-Man® product will make a positive impact to your daily living.

INDICATIONS


- Kid-Man® walking frame are suitable for everyday indoor and outdoor use. All Kid-Man® handcarts are manufactured to the highest quality standards and are CE marked as medical devices. Walking frame made of anodized aluminium tubes.

SET UP


- Walking frame come fully assembled and ready to be used.

USAGE


- Hold padded armrest and pull both sides outwards until push button gets in place. The click sound will appear. Ready to be used walking frame gets „U“ shaped. (Pic. No. 2) Once frame is opened, please set the height of frame according to user's needs. (Pic. No. 3)
- Once unused, it is recommended to close the frame. With one hand pushing plastic holder in the middle of frame, fold one side in and repeat the same with other side. (Pic. No. 1) Closed frame takes less space and is more convenient for storage.

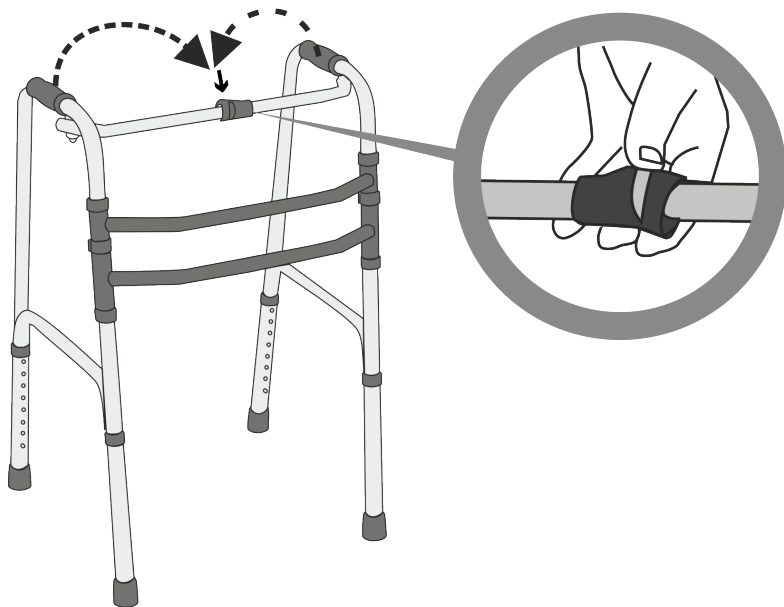

Pic. No. 1





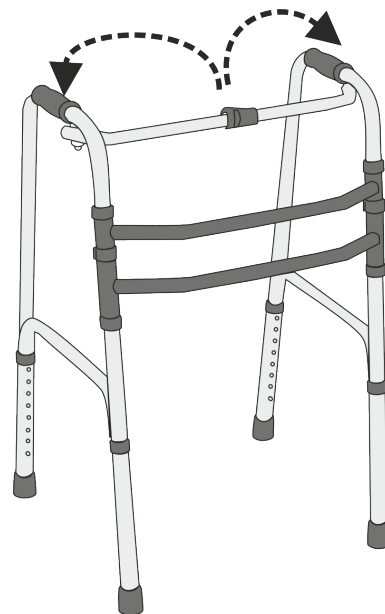
Pic. No. 2

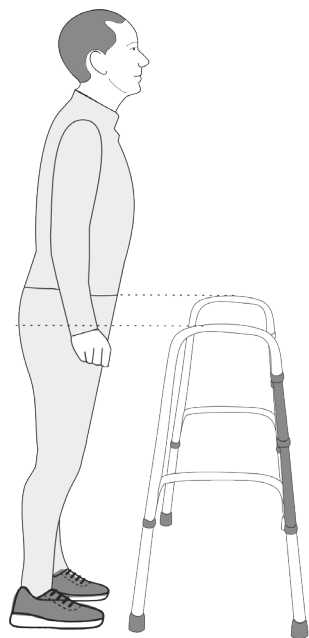Pic. No. 3

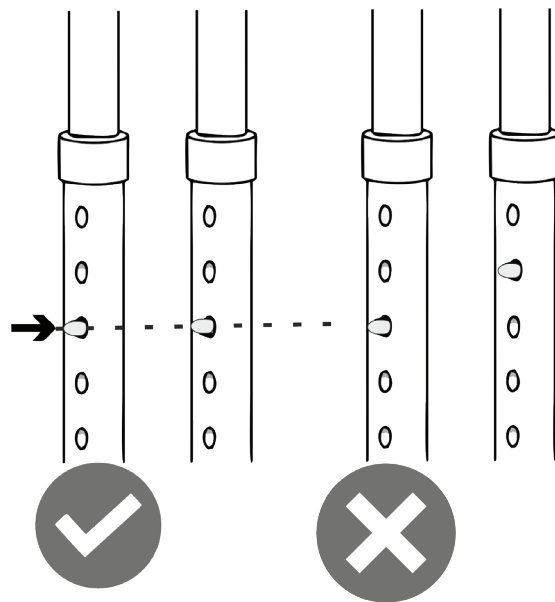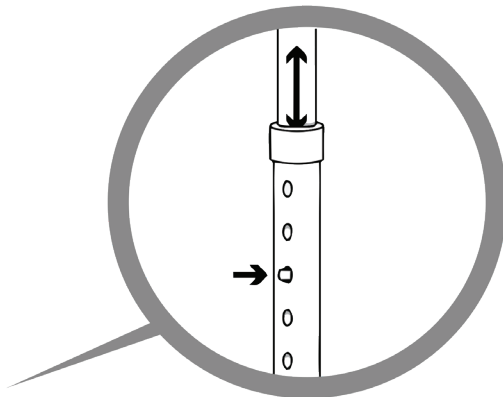
• **Warning!** Before using device, make sure all feet are set in the same height.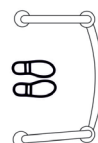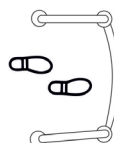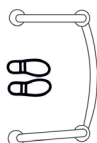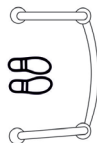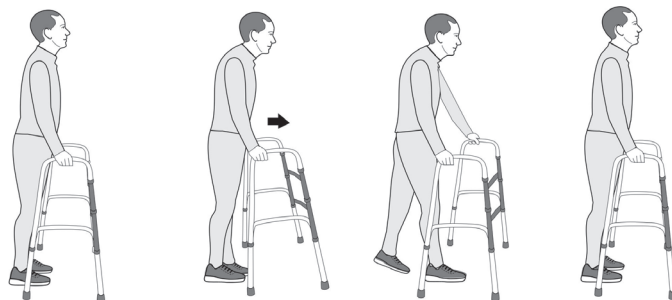

01-3900

01-3203

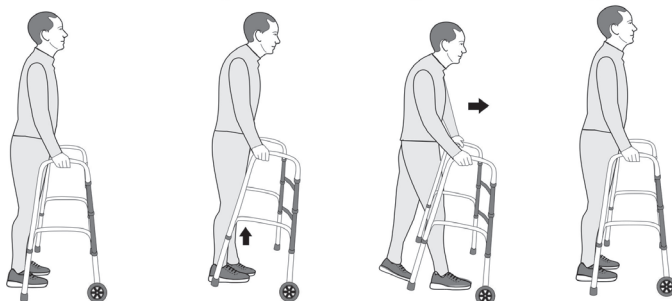01-3016W5

CARE AND MAINTENANCE

- It is recommended to clean with a damp cloth soaked in warm water. Do not use aggressive cleaning agents or solvents.

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EN We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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