**DUŠO SUOLIUKAI**  
**DUŠAS SOLIŅI**  
**DUŠI ISTE**  
**SHOWER BENCHES**  
**СКАМЕЙКИ ДЛЯ ДУША**  
**SCAUN DE DUS CU FIXARE PE**  
**SPRCHOVÉ LAVIČKY**  
**STOLI ZA PRHANJE**

Naudojimo ir priežiūros instrukcija  
Lietošanas un kopšanas instrukcija  
Kasutusjuhend  
User manual  
Руководство по эксплуатации  
Manual de utilizare  
Uživatelská příručka  
Navodila za uporabo in vzdrževanje

**01-5106**

**01-5105**

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Observe instructions for use! Please read these instructions carefully before use and keep it for later reference!

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**EN** Thank you for the confidence shown in Kid-Man. Every product is factory checked and left us in faultless condition, We wish you much success and we hope Kid-Man product will make a positive impact to your daily living.

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**EN** Wall mountable shower benches to be used both in bath or small shower. Frame is made of aluminum, seat is plastic.

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**EE** Bath benches are intended to be wall mounted in shower room. Seat is foldable, requires little space on a wall. 01-5105 bench has height adjustable legs, however model 01-5106 comes w/o legs.

**EN** Shower benches are intended for use in the bathroom, mounted on the wall. The seat is foldable and requires little space on the wall. Model 01-5105 has height-adjustable legs, while model 01-5106 comes without legs.

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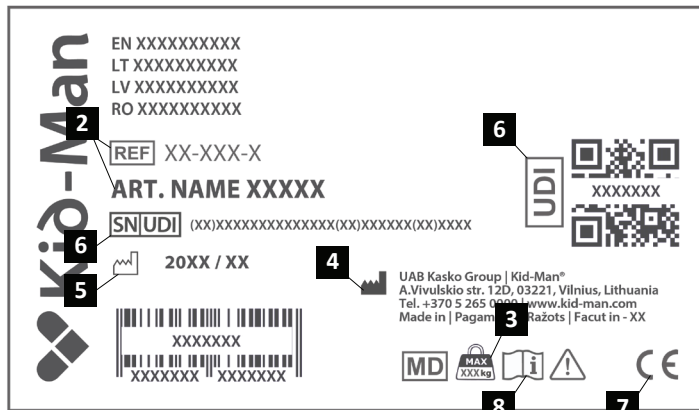
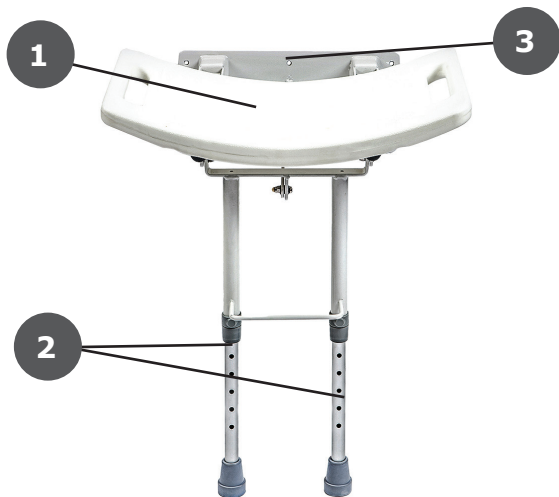
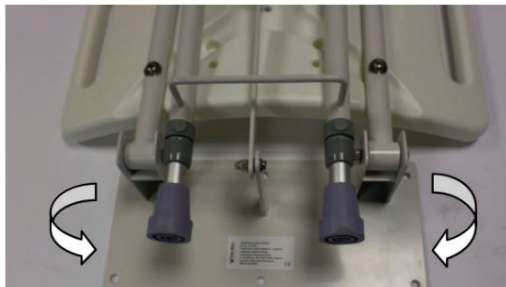
**EN** Benches comes fully assembled, however before use need to bend out (in model 01-5106 need pull the fixing handle out and in the same time bend out seat and release handle after). It is recommended to mount bench on hard wall, see Pic No 1 and No 2. Please be cautious, improper mounting may cause serious injuries.

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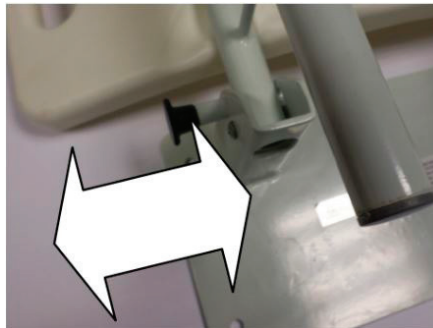
**EN** Bench should be mounted on the concrete wall with 6x8 mm screws clockwise, before mounting drill the holes in diameter suitable for selected mounting hardware. Mounting type and hardware should be selected depending on individual surface. We strongly recommend not to mount bench on non solid surface such as wood or gypsum wall to avoid injuries!

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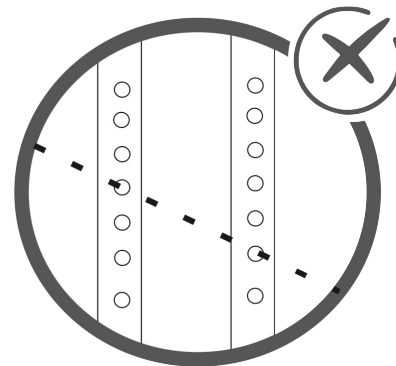
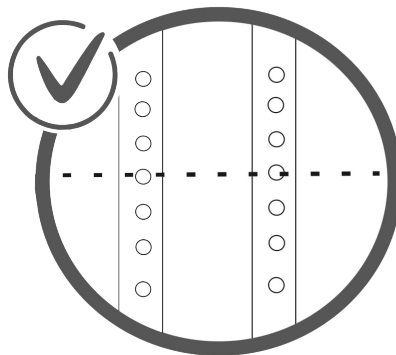
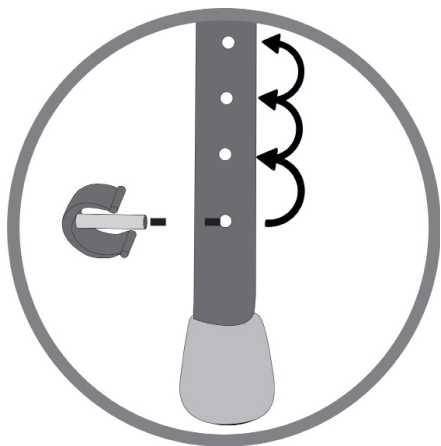
**EN** Only original spare parts can be used for replacement. Please provide model and serial number to dealer or contact manufacturer directly. Manufacturer and dealer is not responsible for any damage occurred to product or user if non original spare parts are used and (or) repair was done in unauthorized facility.

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**EN** We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group” after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group”, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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The quality management system of Kasko Group,  
UAB meets the standards ISO 9001 and ISO 14001

Data / Datum / Kuupäev / Date /  
Дата / Datum / Data / Datum  
2014-12-15

Paskutinio atnaujinimo data / Izdevuma datums / Viimase  
uuenduse kuupäev / Date of last update / Дата последнего  
обновления / Data ultimei actualizării / Datum poslední aktualizace  
2025-07-31