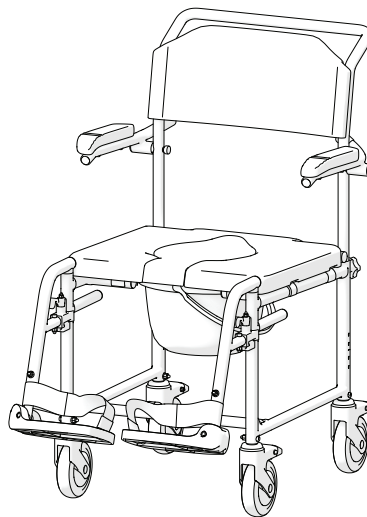
AKVA

01-5209A, 01-5209B

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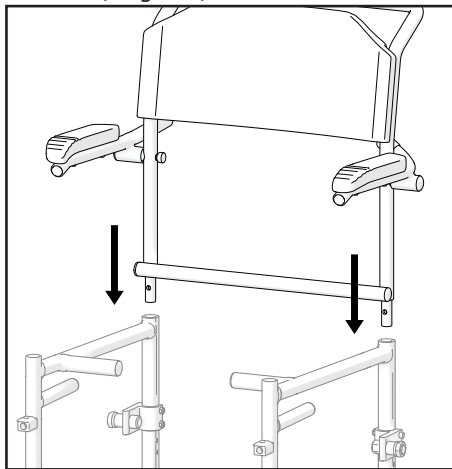
EN This Instruction for use MUST be given to the user of the product. BEFORE using this product, this Instruction for use MUST be read and saved for future reference.

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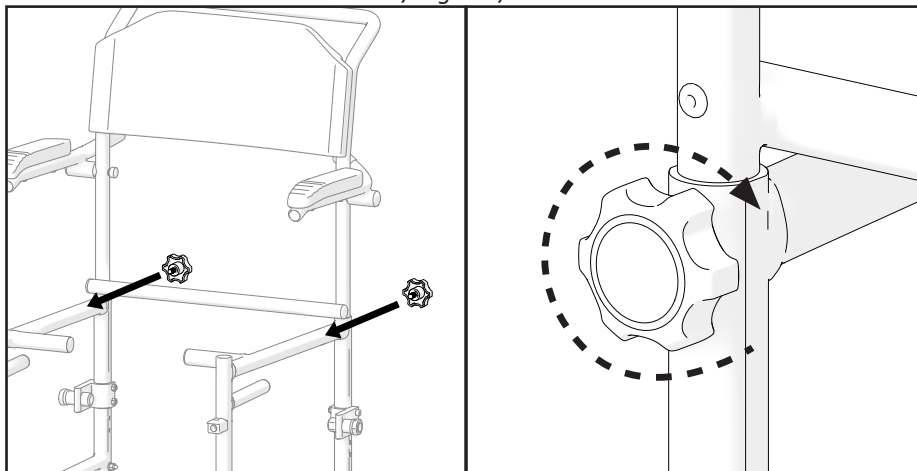
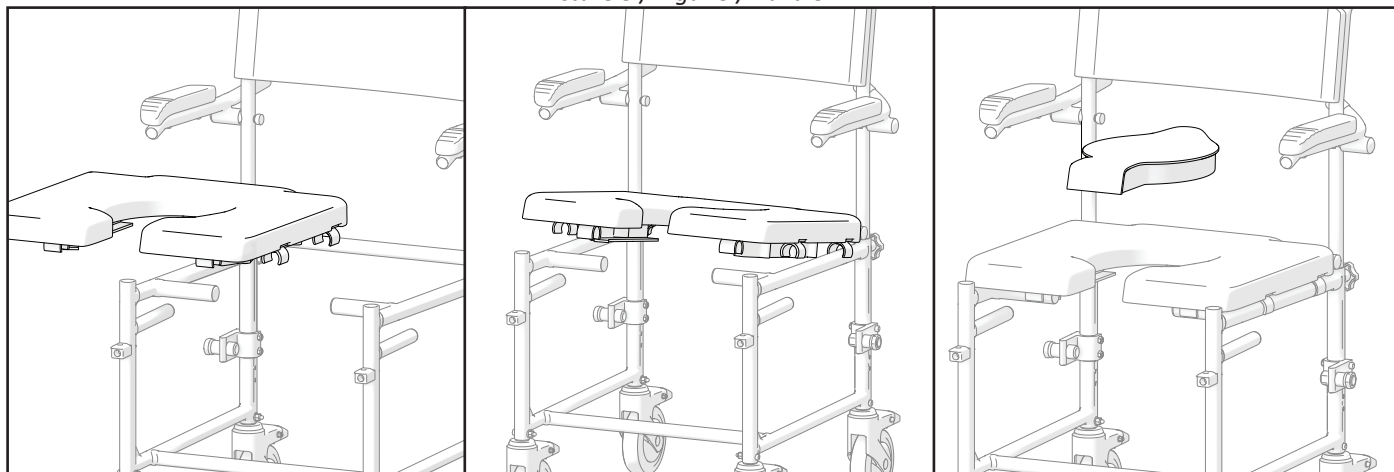
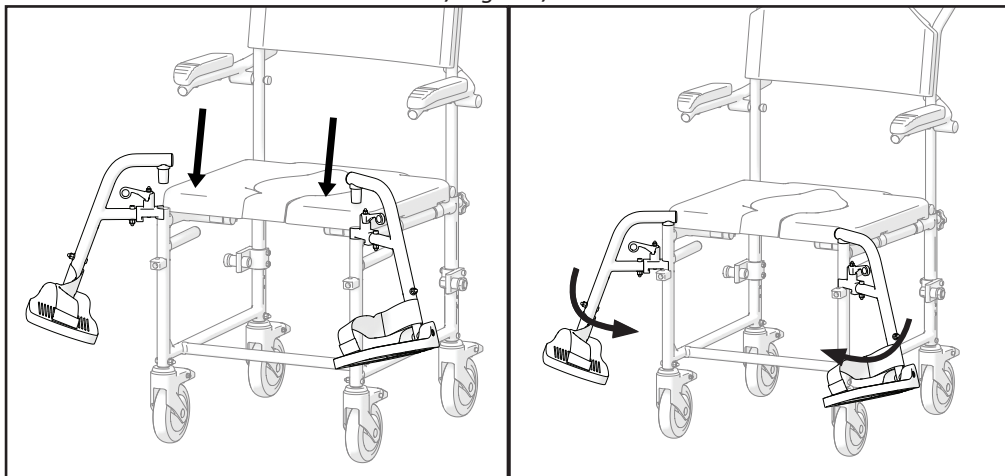
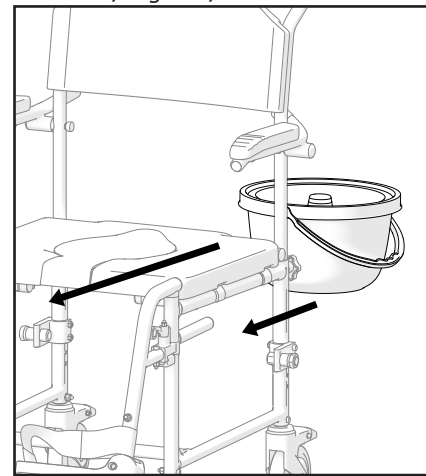
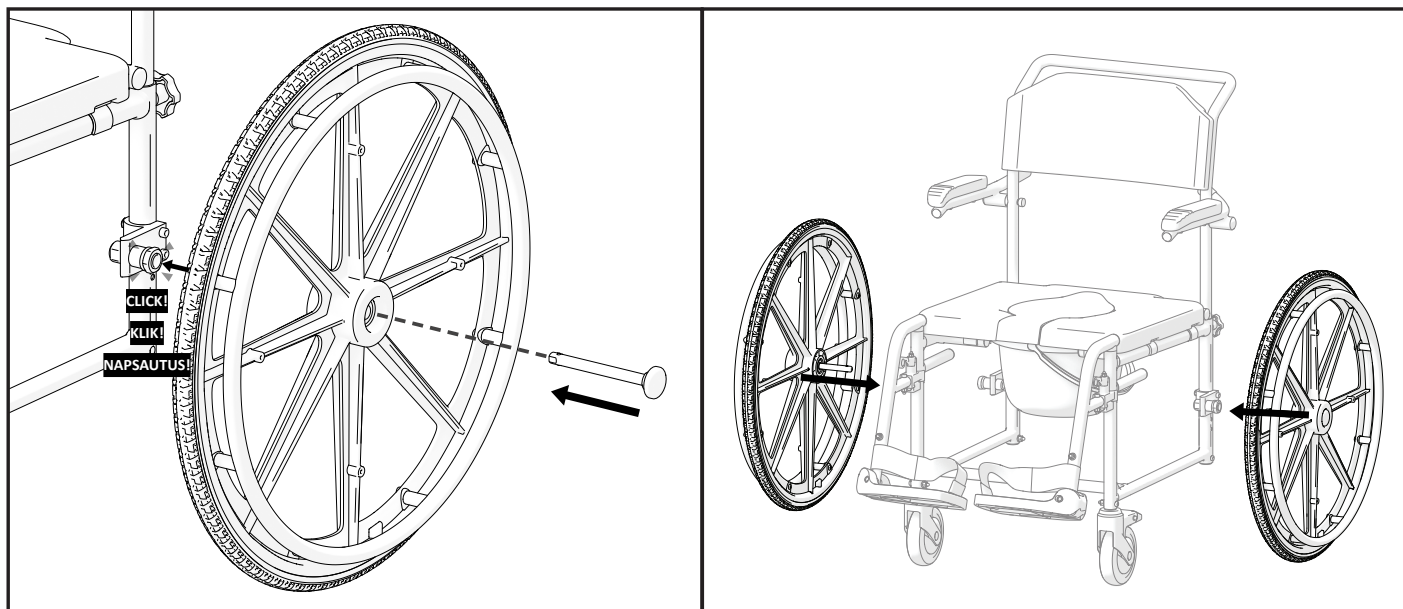
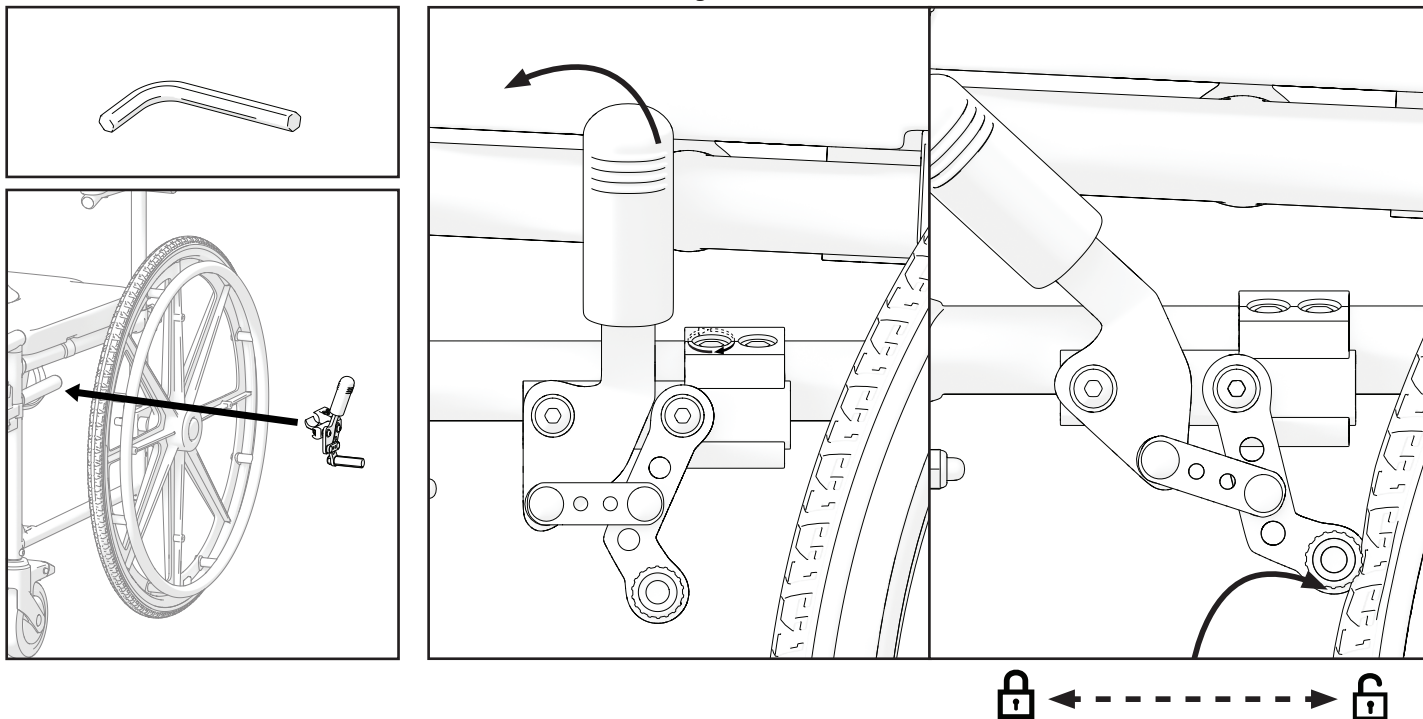
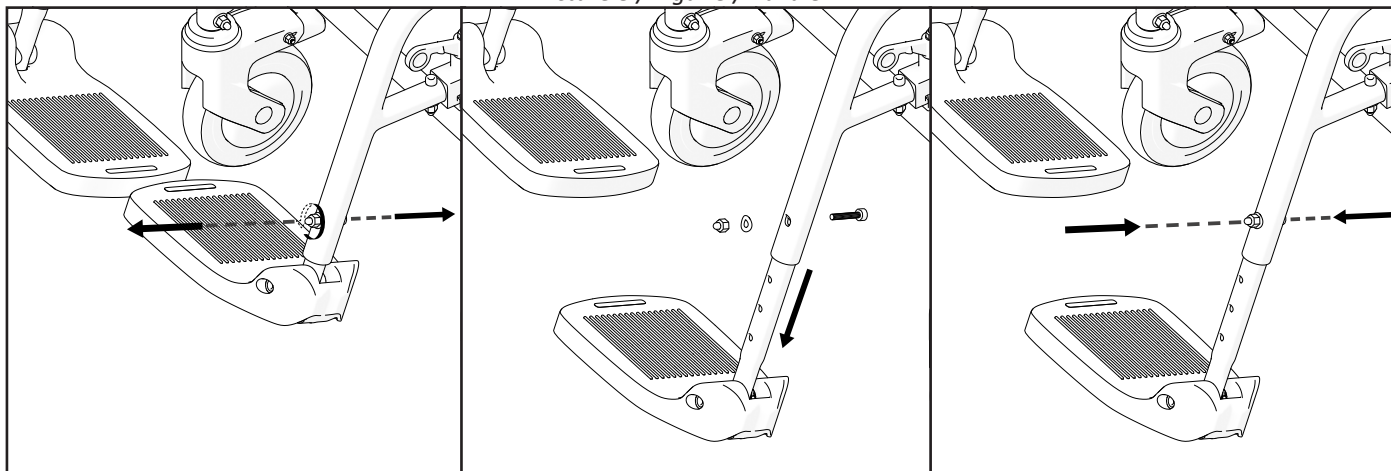
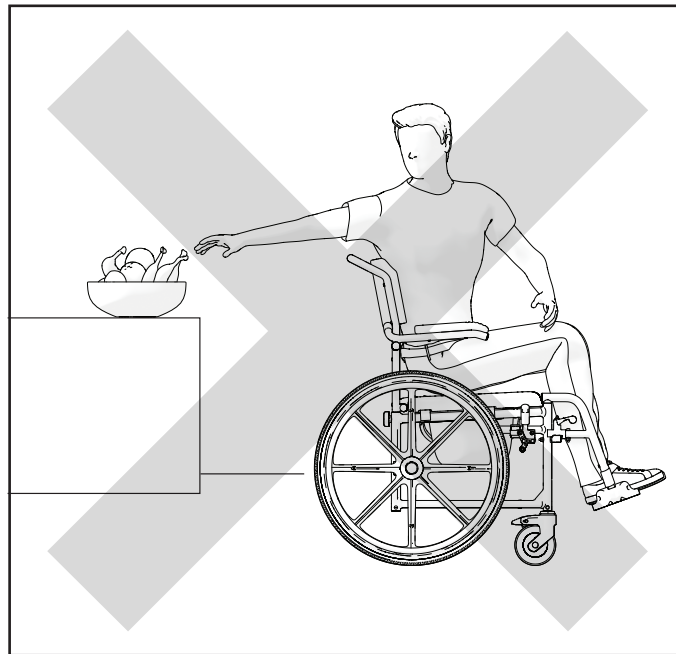
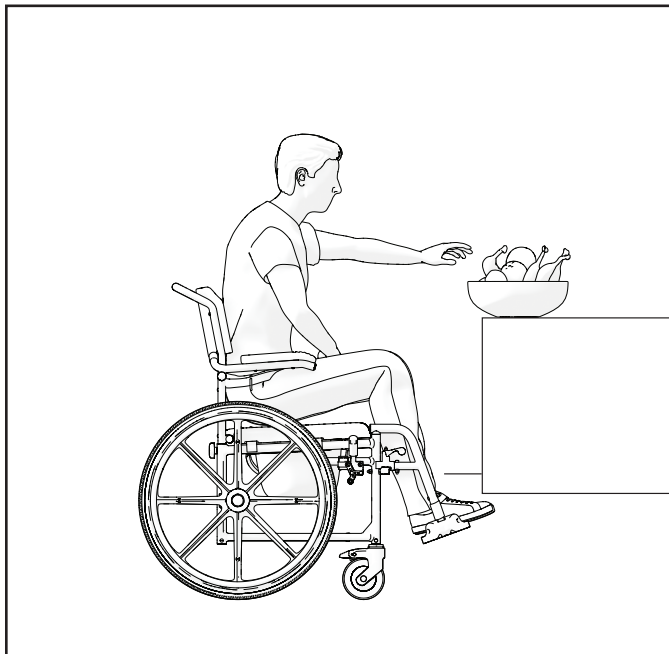
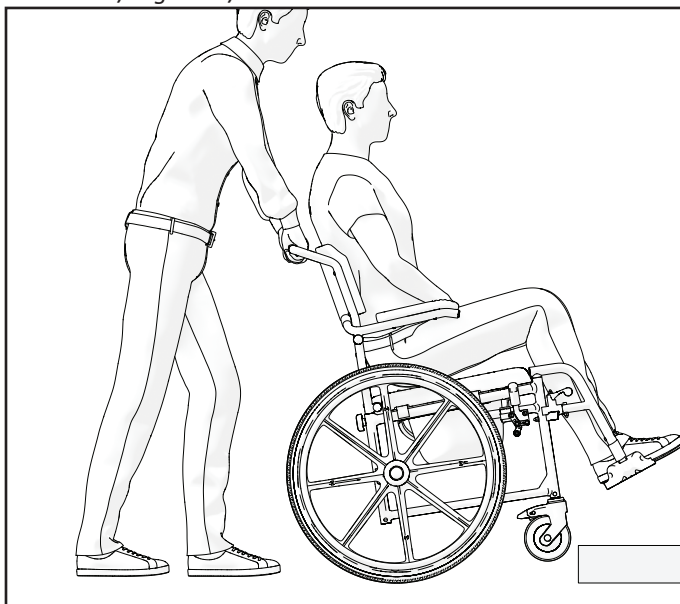
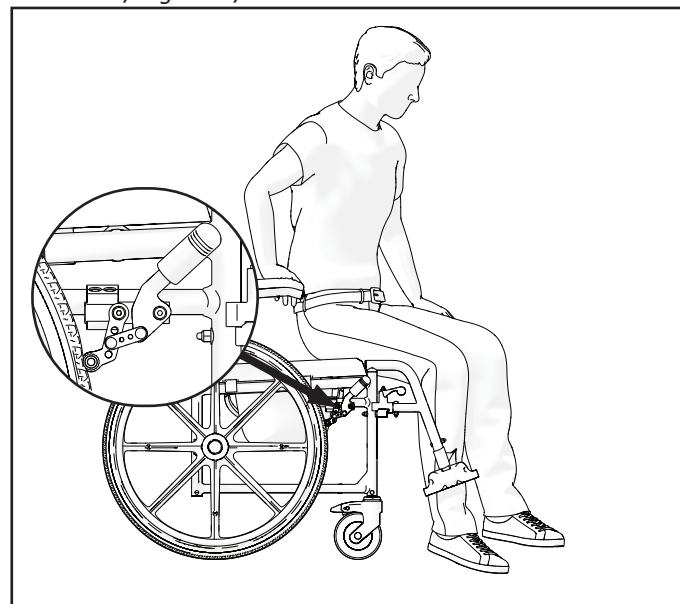
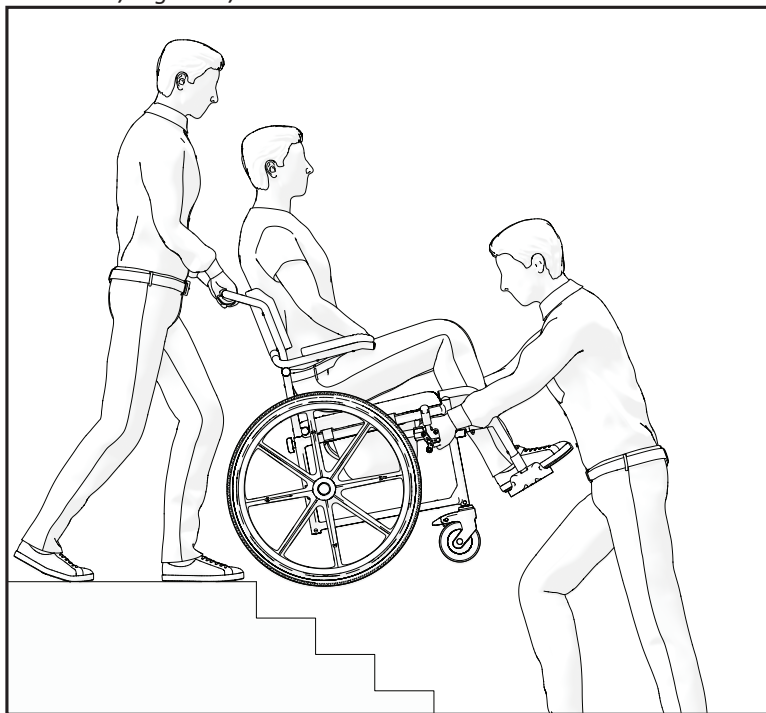
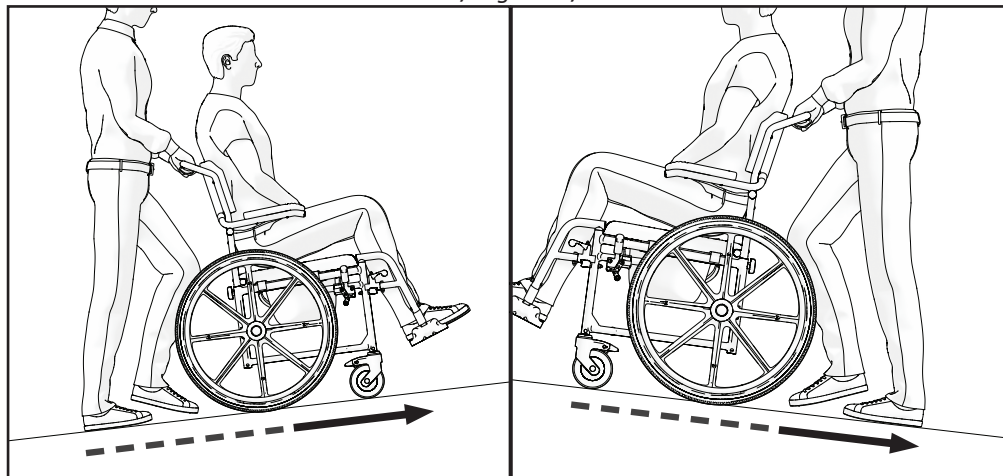
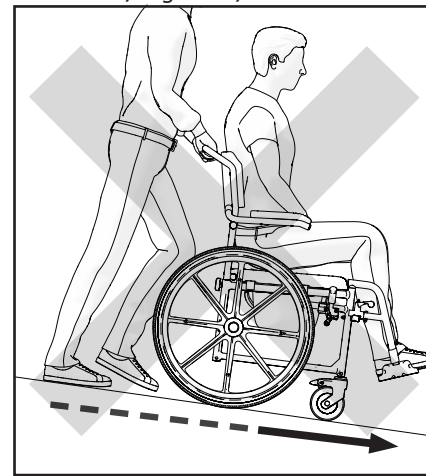
EN**SYMBOLS**

In this Instruction for Use, warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger or basic information.

Gives basic information, recommendations, and guidance for efficient, trouble-free use.

WARNING / CAUTION - to indicate that caution is necessary when operating the device. To indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

ATTENTION! This sign is used for very important parts concerning your safety. Please look at the operating instructions.

SAFETY INFORMATION**WARNING**

Any other or incorrect use could lead to a hazardous situation.

**WARNING**

Risk due to a driving style that is unsuitable for the conditions. There is a risk of skidding on wet ground, gravel, or uneven terrain. Always adjust your speed and driving style to the conditions (weather, surface, individual ability, etc.).

**WARNING**

Risk of injury. In a collision, you could sustain injuries to body parts that extend beyond the shower-commode chair (e.g., feet or hands). Avoid unbraked collisions; never drive into an object head-on; drive carefully through narrow passages.

-  **WARNING**
Risk of the shower-commode chair getting out of control. At high speed, you could lose control of your chair and overturn. Avoid collisions in general. Do not reverse down a gradient.
-  **WARNING**
Risk of burning. The shower chair must not be permanently exposed to temperatures greater than +40 °C. Otherwise, the frame as well as the seat, backrest, and armrests can heat up, which might cause burns.
-  **CAUTION**
Risk of getting fingers caught. There is always a risk of getting, e.g., fingers or arms, caught in the moving parts of the shower chair.
-  **WARNING**
Risk of accidents. Safety devices that are incorrectly set or no longer working (e.g., brakes) can cause accidents. Always check that the safety devices are working before you use the shower chair, and have them checked regularly by the authorized dealer.
-  **WARNING**
Risk of accidents. For proper and safe use, we recommend you follow the safety instructions while using your shower chair. This shower chair, AKVA, is designed to ensure stability while seated. Every movement on the shower chair affects the center of gravity, which may cause the chair to tip over.
-  **WARNING**
Risk of choking. Please protect children under the age of five years from easy access to the Akva shower chair. Due to small elements used in its construction, children could swallow them and choke.
-  **CAUTION**
The shower chair is only suitable for single occupancy.
-  **CAUTION**
Keep your feet on the footplates when moving. Do not stand on the footplates. Always maintain proper balance. Users should not shift their center of gravity outside the seating area.

**CAUTION**

We recommend that you consult your healthcare professional for advice about transferring to and from the shower-commode chair. The parking brakes should always be applied when transferring. The brakes are not suitable for slowing the chair down.

**CAUTION**

Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's toys, etc. Do not reach for items beyond your arm's length.

**WARNING**

Risk of accidents. Never leave an unoccupied shower-commode chair AKVA on an incline or in a place where it may cause injury to nearby people or damage to surrounding objects.

**WARNING**

The design of your shower-commode chair AKVA may differ slightly from the model described in this manual due to ongoing product development.

**WARNING**

Do not attempt to lift the shower-commode chair AKVA by any removable parts. Lifting by removable parts may result in injury to the user or damage to the shower-commode chair AKVA.

**WARNING**

Do not attempt to ride over curbs or obstacles. Doing so may cause your shower-commode chair AKVA to tip over, resulting in bodily harm or damage to the chair.

**WARNING**

All servicing, cleaning, and adjustments should be performed only when the shower-commode chair AKVA is unoccupied.

**WARNING**

Do not exceed the maximum safe load of the shower-commode chair AKVA.

GENERAL

INTRODUCTION

This Instruction for Use contains important information about handling the product. To ensure safety when using the product, read this Instruction for Use carefully and follow the safety instructions.

PREFACE

Thank you for the confidence shown in Kid-Man®. Every product is factory checked and left us in faultless condition. We wish you much success and hope that the UAB Kasko Group | Kid-Man® product will make a positive impact on your daily life.

WARRANTY

We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB Kasko Group | Kid-Man® (hereinafter called Manufacturer) after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If the issue is not covered under warranty and the customer agrees, the Manufacturer will, if possible, repair the issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

LIMITATION OF LIABILITY

Manufacturer accepts no liability for damage arising from:

- Non-compliance with the Instruction for use;
- Incorrect use;
- Natural wear and tear;
- Incorrect assembly or setup by the purchaser or a third-party technical modifications;
- The usage of unapproved 3rd party accessories instead of approved Manufacturer's accessories;
- Unauthorized modifications and/or use of unsuitable spare part.

STANDARDS AND REGULATIONS

The shower-commode chairs AKVA comply with the requirements of:

a) REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009, and repealing Council Directives 90/385/EEC and 93/42/EEC.

SERVICE LIFE

The expected service life is 5 (five) years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this Instruction for Use.

After then, the device must be recycled. Do not throw your product away with common waste (see chapter “DISPOSAL”).

INFORMATION ABOUT MANUFACTURER

Manufacturer: UAB “Kasko Group” (registered trade mark Kid-Man®)

Address: A. Vivulskio 12D, Vilnius, LT-03221, Lithuania

Phone: +370 5 265 0000

E-mail: info@kid-man.com

Website: www.kid-man.com

INFORMATION ABOUT PRODUCT

Product: Shower - commode chair

Model: AKVA

REF: 01-5209B ; 01-5209A

WARNING

Risk of accidents and serious injury. The shower-commode chair is not intended for use in motor vehicles.

INTENDED PURPOSE AND INDICATIONS

Shower-commode chair is intended for the transportation of persons with permanent or temporary mobility disorders. Shower-commode chairs can be used independently or with the help of an attendant.

The 01-5209B version is equipped with push rims. There are three different intended uses: as a transport chair, a commode chair, or a shower chair. Thanks to the corrosion-resistant materials used, the chair is suitable for use in shower rooms. The chair is designed for indoor use.

CONTRA-INDICATIONS

There are no known contraindications when using the shower-commode chair AKVA as intended.

INTENDED USERS

The shower-commode chair is intended to provide mobility for persons aged 15 and up (adolescents and adults), with a maximum weight capacity of 120 kg.

WARNING

Risk of accidents. It is forbidden to exceed the maximum safe load of the shower-commode chair.

The device is intended to provide mobility for persons limited to a sitting position and only when assisted by another person.

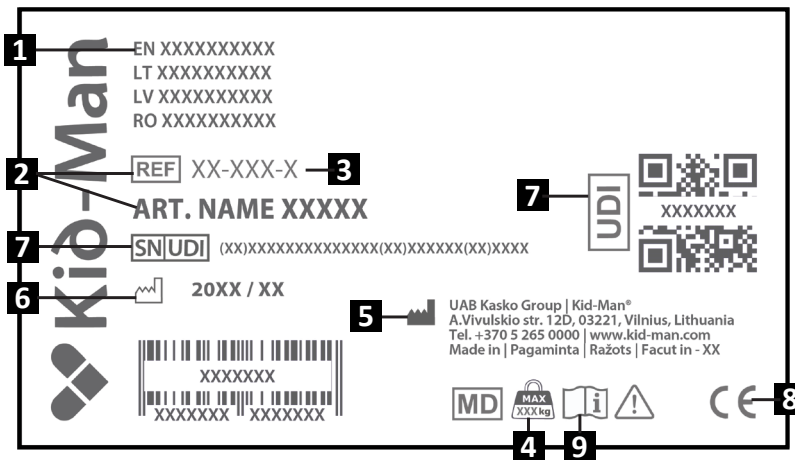
CAUTION

The shower-commode chair is suitable for use by multiple users. In order to do this, the chair must first be thoroughly cleaned and disinfected. Then, all components of the shower-commode chair must be checked by an authorised service dealer for general condition and wear, and repaired if necessary.

PRODUCT LABEL

The identification label is attached to the frame of your product (see picture).

1. Product
2. Model
3. Size (seat width)
4. Maximum load
5. Manufacturer / contact details
6. Date of manufacture
7. UDI / SN code
8. CE mark
9. Consult instructions for use

CONTENTS OF THE PACKAGE


The AKVA shower-commode chairs are delivered in cardboard packaging and are ready for assembly. After unpacking, please keep the packaging if possible, as you may need it later for storage or for returning the product. Once the parcel is received, please check whether it was damaged during transportation.

- i** If you notice any signs of damage, please report it to the transport company immediately.
- i** Please check the contents of the package. If any of the parts listed below are missing, contact your local dealer.

AKVA | 01-5209A

Contents of the package:

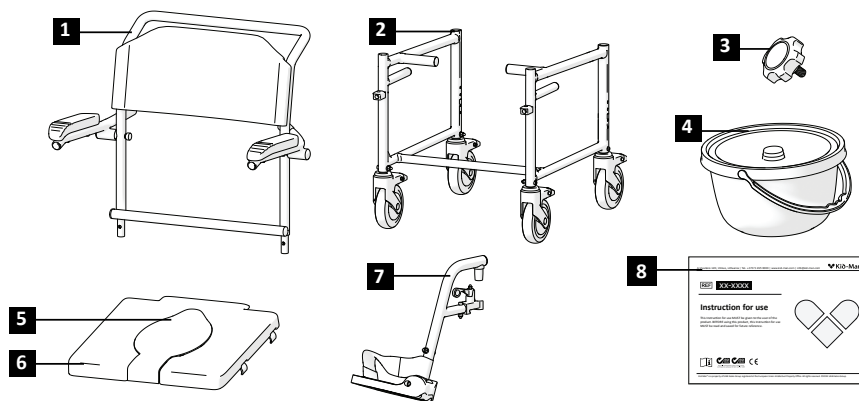
- 1 x armrest, backrest and push handle
- 1 x frame with front and rear castors
- 1 x tightening knob
- 1 x bucket with cover
- 1 x seat cover
- 1 x seat
- 2 x legrest
- 1 x instruction for use

AKVA | 01-5209B

Contents of the package:

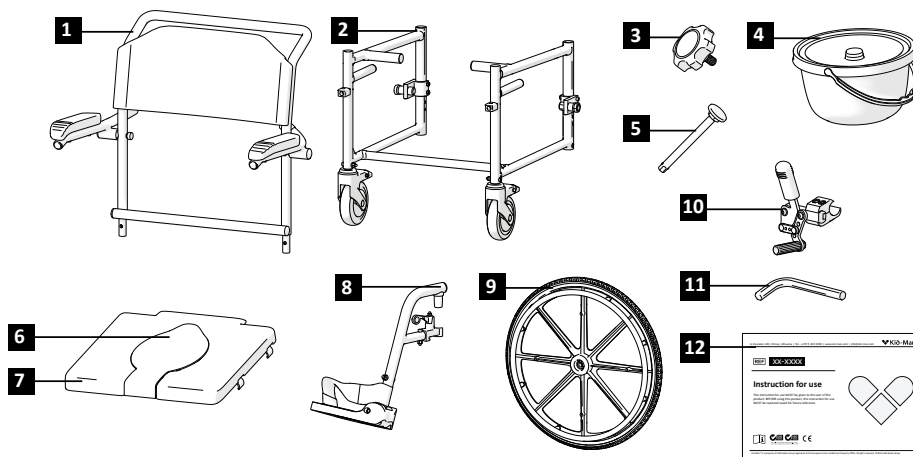
- 1 x armrest, backrest and push handle
- 1 x frame with front castors
- 1 x tightening knob
- 1 x bucket with cover
- 2 x quick release axle
- 1 x seat cover
- 1 x seat
- 2 x legrest
- 2 x rear wheel
- 2 x braking system
- 1 x key
- 1 x instruction for use

COMPONENTS AND FUNCTIONS


AKVA | 01-5209A Overview

- | | |
|-----------------|--------------------------|
| 1. Backrest | 6. Castor brake |
| 2. Front castor | 7. Attendant push handle |
| 3. Armrest | 8. Seat |
| 4. Legrest | |
| 5. Rear castor | |

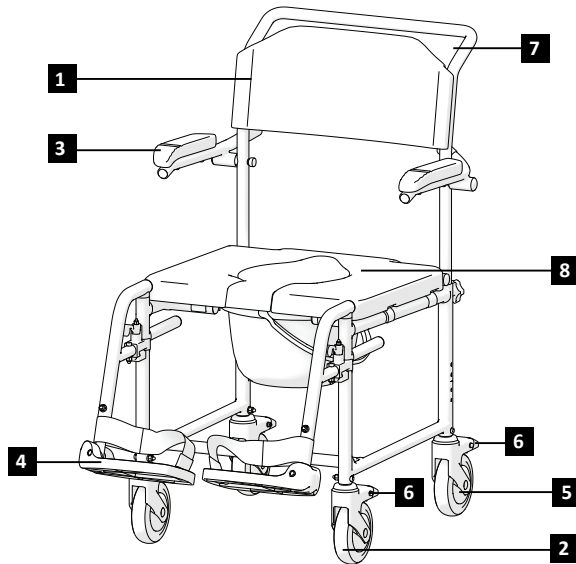AKVA | 01-5209B Overview

- | | |
|-----------------|--------------------------|
| 1. Backrest | 6. Brake lever |
| 2. Front castor | 7. Castor brake |
| 3. Armrest | 8. Attendant push handle |
| 4. Legrest | 9. Seat |
| 5. Rear wheel | |

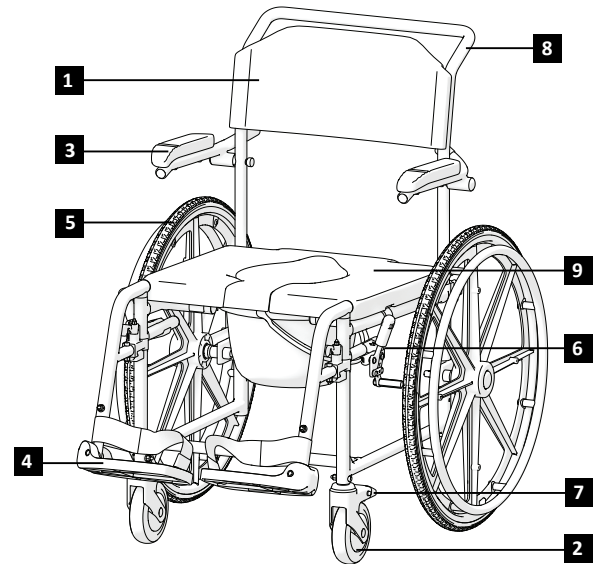Design of your shower-commode chair can slightly differ from a chair described in this manual due to continuous development of the product.

SETTING UP THE SHOWER-COMMODE CHAIR

To set up the shower-commode chair:

- | | |
|---|---|
| 1) Assemble the backrest and armrest part to the bottom frame of the chair (Picture 1). | 5) Insert the bucket (Picture 5). |
| 2) Tighten the knobs (Picture 2). | 6) Insert the quick release axle into the rear wheel (Picture 6). |
| 3) Place the seat and then place the seat cover (Picture 3). | 7) Attach the rear wheels by pressing the quick-release axle (Picture 6). |
| 4) Put legrest in their place (Picture 4). | 8) Using the key attach the brake levers (Picture 7). |

i **You do not need to use excessive force.** If you push too hard, you may damage the legrest (e.g., break it).

 **WARNING**
Danger of trapping your finger! Be careful not to grasp the seat pipe.

 **WARNING**
Never lift the shower-commode chair by its removable parts (seat, legrests). Make sure that all removable parts are detached. The removable parts of the shower-commode chair are: legrests, backrest, and cushion. For instructions on how to remove them, please refer to this Instruction for Use. Once the parts are removed, you can easily lift the chair by holding the frame with both hands.

OPERATION

i This shower-commode chair is designed to ensure stability while seated. Every movement on the shower-commode chair affects the center of gravity, which may cause the chair to tip over. To increase your safety, you may use a seatbelt (not included).

i There are no designated points on the chair intended for carrying additional loads.

LEANING FORWARDS

WARNING

Risk of falling out. If you lean forward out of the shower-commode chair you could fall out of it. Never bend too far forward and do not shift forward in your seat to reach an object. Do not bend forward between your knees to pick up something off the floor.

REACHING AN OBJECT FROM THE SHOWER-COMMODE CHAIR

Only the arms may be extended over the shower-commode chair seat. The body and head should stay inside the limits of the seat (Picture 9).

MOVING ONTO THE SIDEWALK

Although the device is intended for indoor use, here are some tips on **how to operate it outdoors**:

We recommend having an assistant help move onto the sidewalk.

Firstly, the accompanying person should position the shower-commode chair straight in front of the curb side. Then, he should tilt the shower-commode chair backwards until the steering wheels reach the curb.

Lastly, he should push the shower-commode chair straight forward until the back wheels hit the curb and then push it again until the back wheels go over the curb (Picture 10).

TRANSFER

Transfer can be performed alone, assuming you are strong enough.

Firstly, set the shower-commode chair as close to the place, which you want to sit on, as possible, whereby the steering wheels point forward. Put on the brakes. Move your body towards the desired place (Picture 11).

If there is no support available during transfer, use a transfer board if possible. Fold the legrests up and move them aside to make room. Bend your body forward, while leaning on the two armrests, and stand up. Then move your body toward the place, where you want to sit, while distributing your weight to your arms and hands.

If you are capable of getting up and standing on your own, you may change seats by leaning forward. If you have trouble with this movement, we advise you to seek assistance from an accompanying person to ensure an easy and safe transfer. Firstly, engage both brakes and make sure both steering wheels point towards the front. Please bear in mind that you must not support yourself on the brakes. Grab the handle of the armrest firmly and support yourself on your arms, to stand up easily (before, make sure armrest are securely locked in place). The assistant may help you complete the transfer. Position yourself as close to the desired place as possible. When changing your position, please sit as far back as possible, to prevent screws from breaking, seat cushions from getting damaged, or the shower-commode chair from tipping over. Activate the two brakes; these brakes should in no way be used as support for the transfer. You must never support yourself on the footrests, while sitting or getting up.

STAIRS

We recommend seeking assistance from two people when moving the shower-commode chair over stairs - one positioned in front of the chair and one behind it - due to the complexity and risk involved.

The person behind the shower-commode chair should tip it backward until it is balanced. Then, they push the chair gently against the first step and firmly grasp the push handles to prepare for lifting. Meanwhile, the person in front should firmly hold both sides of the front frame and lift the chair upward, over the step. As the front person moves to the next step, the person behind follows, repeating the process step by step. Important: Do not lower the shower-commode chair until the last step has been cleared by the person in front. Carrying the chair down the stairs follows the same process, but in reverse order (Picture 12).

WARNING

Please do not grab shower-commode chair over the legrests, due to disassembly possibilities.

ASCENDING / DESCENDING A RAMP WITH ASSISTANCE

See (Picture 13 - 14).

ARMREST OPERATION

The shower-commode chair is equipped with flip-up armrests. We recommend flipping up the armrests when getting in or out of the chair from the side. While operating the shower-commode chair, the armrests should remain in the down position.

WARNING

Do not lift the shower-commode chair by holding the armrests, as this may cause serious damage to the product or injury.

BRAKING

Your shower-commode wheelchair (01-5209B) has brakes mounted on the frame on both sides. Once delivered, brakes are adjusted properly to ensure sufficient braking. In order to apply brakes, please hold braking lever and push it forward (Picture 7).

We recommend always use both brakes at the same time to ensure maximum safety.

Your shower-commode chair (01-5209A) has brakes mounted on all four castors.

WARNING

Risk of accidents. Safety devices that are incorrectly set or no longer working (e.g., brakes) can cause accidents.

Always check that the safety devices are working before you use the shower-commode chair and have them checked regularly by the authorized dealer.

LEGREST MOUNTING

Once delivered, legrests are packed in a separate small cardboard. To install them, remove them from the package. The legrests must be installed at the front of the chair. To add the legrest, place the legrest on the two posts located on the frame. Then swing the legrest inwards until it locks in position (Picture 4).

Do not mix up the left and right legrests!

After installation, the plastic footplates must face inward. When leaving the shower-commode chair, the legrests must be flipped away or completely removed. Pull the plastic lever upward and move the legrest outward.

The length of legrest can be adjusted to meet user's needs. To adjust the length of the legrest, loosen the screw (tool not included) and pull the inner tube inwards or outwards, depending on whether you want to increase or decrease the length. After selecting the required legrest length, the screw must be screwed back in (Picture 8).

MOVING

Either the chair user or an attendant can manually propel the chair. If the user is propelling the chair, they should use the hand rims attached to the rear wheels (just in version 01-5209B). The user can slow the chair by applying pressure to the hand rims using their hands. The chair should not be propelled by the user in environments where the user cannot stop the chair promptly and comfortably. If an attendant is propelling the chair, they should push using the push handles. We do not recommend using chair in escalators. Use slow speeds on gradients. Do not exceed the maximum gradient stated. Do not use escalators. Do not reverse down a gradient. Keep your feet on the footplates when moving. Do not stand on the footplates. Always maintain proper balance. Users should not move their centre of gravity out of the seating area.

 Self-propelling is only available for the 01-5209B version.

 **WARNING**
Risk due to driving style being unsuitable for the conditions. There is a risk of skidding on wet ground, gravel or uneven terrain. Always adjust your speed and driving style to the conditions (weather, surface, individual ability, etc.).

 **WARNING**
Risk of injury. In a collision you could sustain injury to parts of your body that extend beyond the shower-commode chair (e.g. feet or hands). Avoid an unbraked collision; Never drive into an object head-on; Drive carefully through narrow passages.

 **WARNING**
Risk due to shower-commode chair being out of control. At high speed you could lose control of your chair and overturn. Avoid collisions in general. Do not reverse down a gradient.

 **WARNING**
Risk of accidents. For proper and safe use, we recommend you follow safety instructions while using your chair. This chair was designed to ensure stability while seated. Every movement on the chair affects the point of gravity, which may cause the chair to tip over. To increase your safety, you may use seatbelt.

TRANSFER

We recommend you consult your healthcare professional for advice about transferring to and from the chair. The parking brakes should always be applied when transferring. The chair brakes and parking brakes are not suitable for slowing the chair down.

MAINTENANCE

To ensure safe and reliable operation, carry out the following checks and maintenance regularly or have them carried out by another person. The user or reliable person should routinely check the following items. It is recommended that an authorized dealer services the chair annually. During servicing, these items should be repaired, replaced, adjusted, and/or lubricated if required. Below you will find the recommended maintenance schedule.

	Before use	Once a week	Once a month	Once a year
Check the wheels		X		
Check that rear wheels are seated correctly		X		
Visual check		X		
Check the castor brakes and main brakes on frame (01-5209B)	X		X	
Have shower-commode chairs checked by an authorized dealer				X

WARNING

Repair, cleaning, or adjustment of the product must be performed by an authorized service person when the product is not in use.

CLEANING AND DISINFECTION

Use a moist cloth to clean the frame and plastic parts. The use of a mild detergent for more stubborn dirt is possible as well. The wheels can be cleaned with a wet synthetic fiber brush (the use of wire brush is not allowed). Seat and back pads can be washed off. Use mild detergent. Use a common non aggressive disinfectant. If you are unsure whether the product you are using is suitable for disinfection, please contact your local dealer.

i Sand and seawater can damage the bearings, and metal parts may rust if the surface is scratched or compromised. Exposure to sand and seawater should be limited to short periods only, and the shower-commode chair must be cleaned thoroughly after each use.

i The shower-commode chair may be disinfected by spraying or wiping with tested, approved disinfectants. Spray a soft cleaning and disinfecting product and follow the instructions given by the manufacturer.

STORAGE

The product should be stored in a dry environment, away from direct sunlight. It should always be kept clean and free of dust.

 **WARNING**
The product must not be permanently exposed to temperatures greater than +40°C. Otherwise, the frame and materials may overheat, which could cause burns or material damage.

 **WARNING**
Never place or store other items on top of the product.

After long-term storage (more than 3 months), the product must be inspected before use (see chapter “MAINTENANCE”).

ENVIRONMENTAL CONDITIONS

Keep the product under these conditions:
Temperature limitation –20° C to +40°C;
Relative humidity 20% to 75%;
Atmospheric pressure 800 hPa to 1060 hPa.

RE-USE

The product is suitable for re-use. The following actions must be carried out:
Cleaning and disinfection (see chapter CLEANING AND DISINFECTION); Inspection (see chapter MAINTENANCE);

The product should be adapted to the new user in accordance with the service documentation.

DISPOSAL

Be environmentally aware and dispose of the product properly. Disposal is subject to national and local regulations. To have the product recycled, please contact your local community, waste disposal facility, or our service department.

SPECIFICATION

Model	AKVA	
REF	01-5209A	01-5209B
Overall length (cm)	89.5	100
Overall width (cm)	60	60
Overall height (cm)	95	95
Seat width (cm)	46	46
Seat depth (cm)	42	42
Seat height (cm)	53	53
Footplate length (cm)	17	17
Footplate width (cm)	15	15
Legrest length (cm)	30 - 42	30 - 42
Legrest spacing (cm)	40	40
Armrest length (cm)	22.5	22.5
Armrest width (cm)	5	5
Size of front caster (cm)	12.5	12.5
Size of rear wheel (cm)	12.5	61
Weight (kg)	13	17.5
Maximum safe load (kg)	120	120

INCIDENTS

Any serious incident that has occurred in relation to the device should be reported to UAB Kasko Group | Kid-Man® and the competent authority of the Member State in which the user and/or patient are established.

CONTACTS:

Manufacturer: UAB “Kasko Group” (registered trade mark Kid-Man®)

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