# BEGO

01-3560GR; 01-3560TR; 01-3560RD; 01-3560BK

**4 WHEEL ROLLATOR**  
**ŠTIRIKOLESNI ROLATOR**

EN

SL

**EN** Instruction for use  
**SL** Navodila za uporabo


**EN** This Instruction for use **MUST** be given to the user of the product. **BEFORE** using this product, this Instruction for use **MUST** be read and saved for future reference  
**SL** Ta navodila za uporabo je **TREBA** izročiti uporabniku izdelka. **PRED** uporabo tega izdelka je **TREBA** skrbno prebrati ta navodila za uporabo in jih shraniti za morebitno prihodnjo uporabo.

Picture 1 / Slika 1

Picture 2 / Slika 2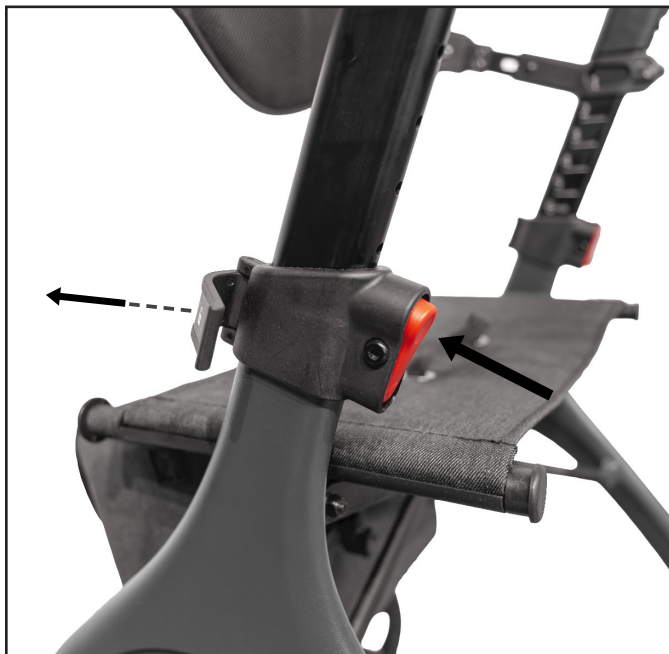


Picture 3 / Slika 3

Picture 4 / Slika 4


Picture 5 / Slika 5

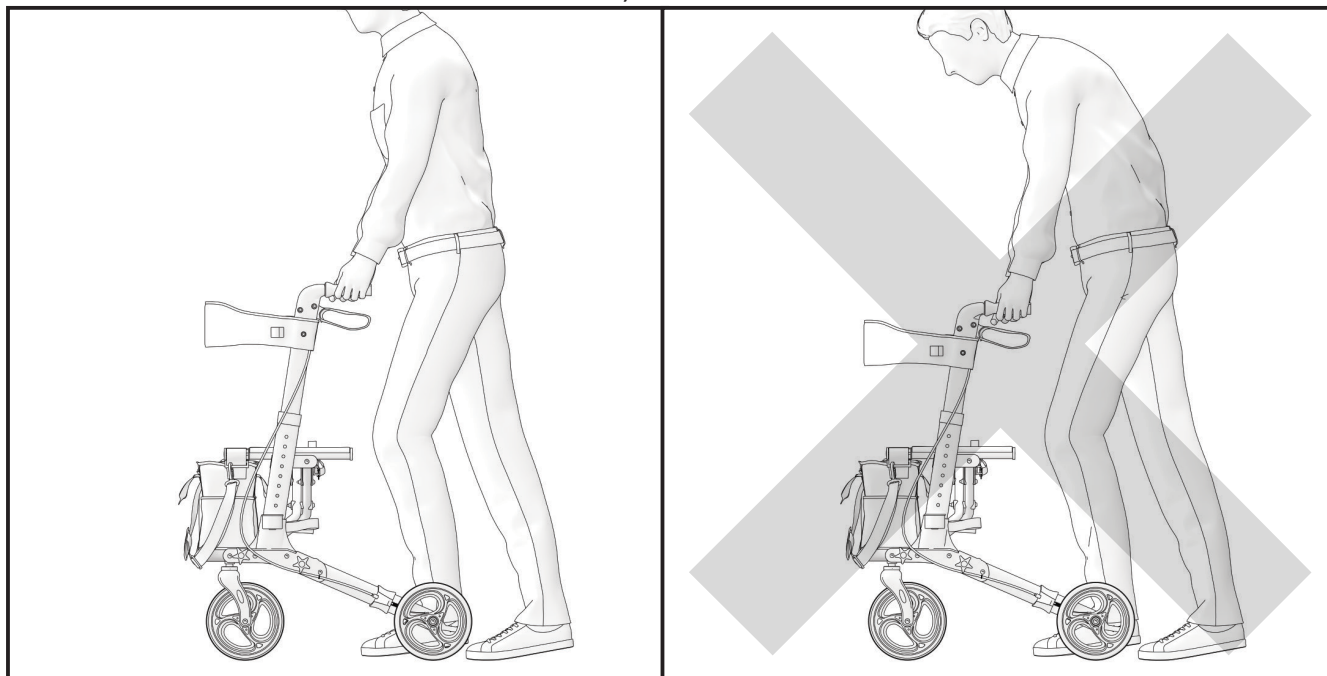
Picture 6 / Slika 6

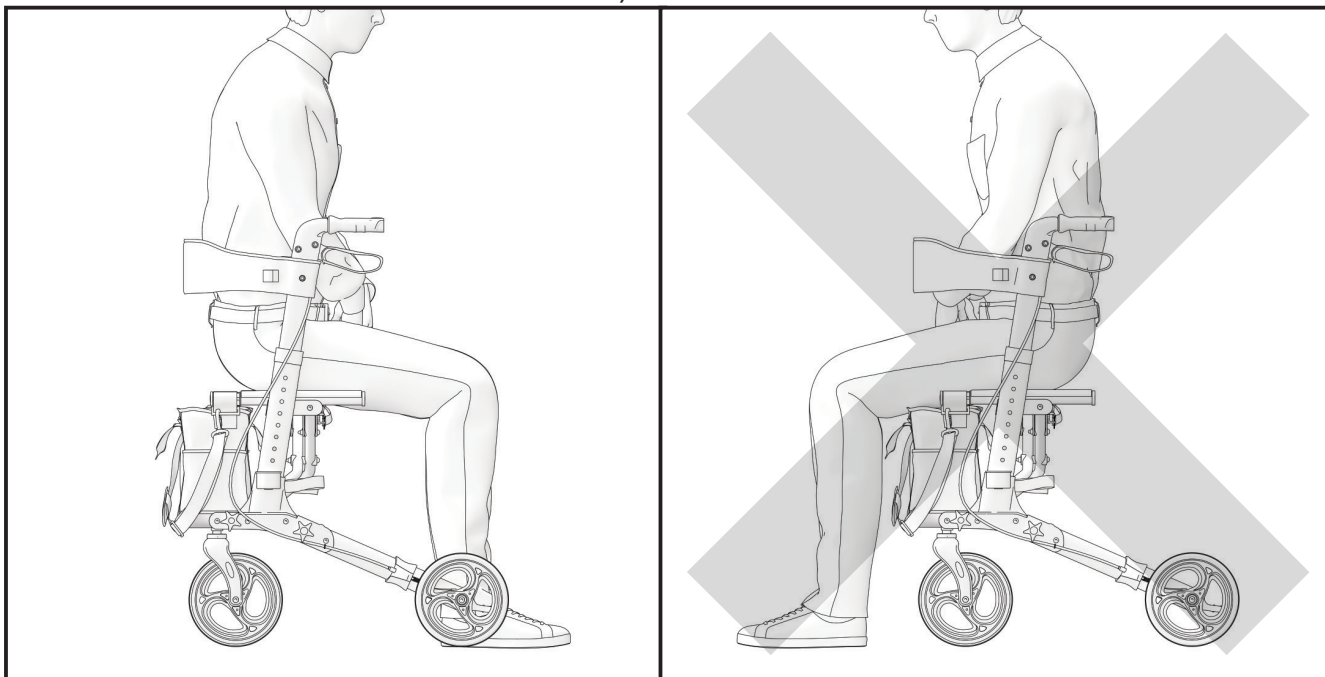
Picture 7 / Slika 7

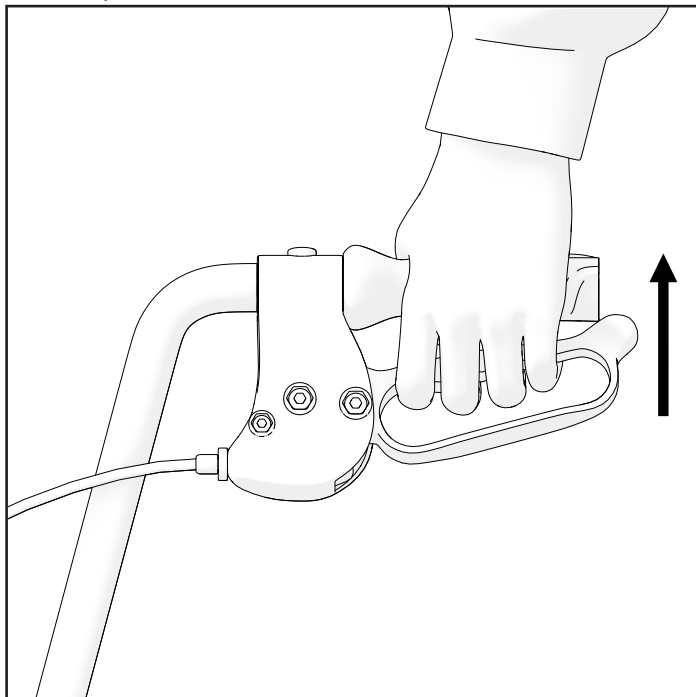Picture 8 / Slika 8

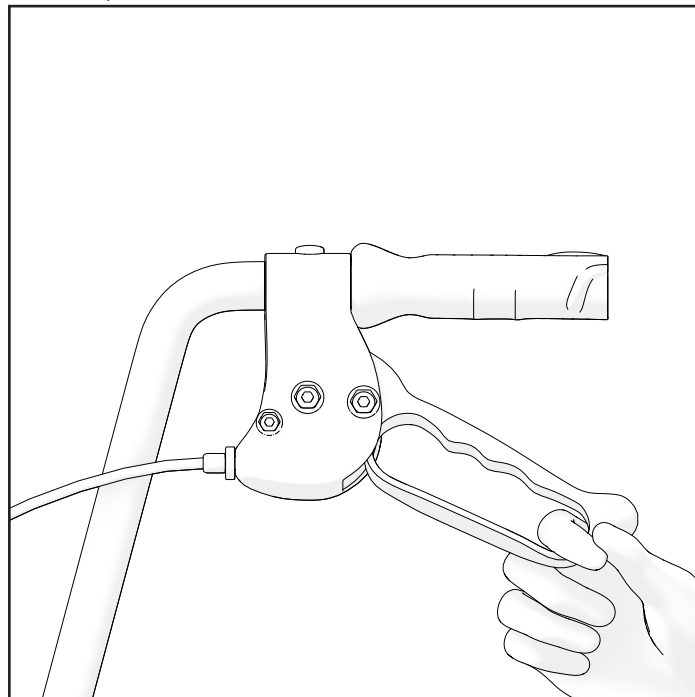**EN****SYMBOLS**

In this Instruction for Use, warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger or basic information.

Gives basic information, recommendations, and guidance for efficient, trouble-free use.

**WARNING / CAUTION** - to indicate that caution is necessary when operating the device. To indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

**ATTENTION!** This sign is used for very important parts concerning your safety. Please look at the operating instructions.

**SAFETY INFORMATION****WARNING**

The rollator is only suitable for single occupancy.

**WARNING**

Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's toys, etc. Do not reach for items further than your arm will extend.

**WARNING**

Do not attempt to ride over curbs or obstacles. Doing so may cause your rollator to tip over and cause bodily harm to you or damage to the rollator.

**WARNING**

It is forbidden to smoke or to be close to the flames, using a rollator.

**WARNING**

The rollator is not intended for use by the blind.

**WARNING**

**Risk of burning.** The rollator must not be permanently exposed to temperatures greater than +40 °C. Otherwise, the frame as well as the armrests can heat up, which might cause burns.

**CAUTION**

**Risk of getting fingers caught.** There is always a risk of getting, e.g., fingers or arms, caught in the moving parts of the rollator.

**WARNING**

**Risk of accidents.** Safety devices that are incorrectly set or no longer working (e.g., brakes) can cause accidents. Always check that the safety devices are working before you use the 4 wheel rollator, and have them checked regularly by the authorized dealer.

**WARNING**

**Risk of accidents.** For proper and safe use, we recommend you follow the safety instructions while using your rollator. BEGO, was designed to ensure stability while walking. Every movement on the rollator affects the center of gravity, which may cause the device to tip over.

**WARNING**

Use slow speeds on gradients. Do not exceed the maximum gradient stated. Do not use escalators. Do not reverse down a gradient.

**WARNING**

It is forbidden to exceed the maximum safe load of product.

**WARNING**

While unpacking, be cautious to avoid catching your fingers between the wheel and the fork.

**WARNING**

When using the 4 wheel rollator, use both brakes simultaneously to prevent the unbraked wheels from twisting and causing the user to fall.

## GENERAL

### INTRODUCTION

This Instruction for Use contains important information about handling the product. To ensure safety when using the product, read this Instruction for Use carefully and follow the safety instructions.

### PREFACE

Thank you for the confidence shown in Kid-Man®. Every product is factory checked and left us in faultless condition. We wish you much success and we hope UAB Kasko Group | Kid-Man® product will make a positive impact to your daily living.

### WARRANTY

We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB Kasko Group | Kid-Man® (hereinafter called Manufacturer) after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, Manufacturer, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

### LIMITATION OF LIABILITY

Manufacturer accepts no liability for damage arising from:

- Non-compliance with the Instruction for use;
- Incorrect use;
- Natural wear and tear;
- Incorrect assembly or setup by the purchaser or a third-party technical modifications;
- The usage of unapproved 3rd party accessories instead of approved Manufacturer's accessories;
- Unauthorized modifications and/or use of unsuitable spare part.

## STANDARDS AND REGULATIONS

The 4 wheel rollator complies the requirements of:

- a) REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009 and repealing Council Directives 90/385/EEC and 93/42/EEC.
- b) LST EN ISO 11199-2:2021 Abiem rankomis valdomos pagalbinės vaikščiojimo priemonės. Reikalavimai ir bandymo metodai. 2 dalis. Vaikštynės su ratukais (ISO 11199-2:2021) Assistive products for walking manipulated by both arms - Requirements and test methods - Part 2: Rollators (ISO 11199-2:2021) EN ISO 11199-2:2021.

## SERVICE LIFE

The expected service life is 5 (five) years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this Instruction for Use.

After then, the device must be recycled. Do not throw your product away with common waste (see chapter “DISPOSAL”).

## INFORMATION ABOUT MANUFACTURER

**Manufacturer:** UAB “Kasko Group” (registered trade mark Kid-Man®)

**Address:** A. Vivulskio 12D, Vilnius, LT-03221, Lithuania

**Phone:** +370 5 265 0000

**E-mail:** info@kid-man.com

**Website:** www.kid-man.com

## INFORMATION ABOUT PRODUCT

**Product:** 4 WHEEL ROLLATOR

**Model:** BEGO

**REF:** 01-3560GR; 01-3560TR; 01-3560RD; 01-3560BK

### WARNING

Risk of accidents and serious injury. The rollator is not intended for use in motor vehicles.

### INTENDED PURPOSE AND INDICATIONS

4 wheel rollator BEGO is a walking device for indoor and outdoor use, perfect for compensation and improvement respectively of restricted ability of walking.

**i** This 4 wheel rollator needs to fit to your specific health condition.

### CONTRA-INDICATIONS

There are no known contraindications when using the 4 wheel rollator BEGO as intended.

### INTENDED USERS

4 wheel rollator BEGO is intended to compensate and improve mobility of a person with restricted ability of walking with a maximum weight capacity of 135 kg.

**WARNING**

Risk of accidents. It is forbidden to exceed the maximum safe load of the 4 wheel rollator.


**CAUTION**

The 4 wheel rollator is appropriate for use by others. In order to do this, the rollator must first be thoroughly cleaned and disinfected. Then, the all components of the rollator must be get checked by an authorised service dealer for general condition, wear and shall get repaired if necessary.

PRODUCT LABEL

The identification label is attached to the frame of your product (see picture).

1. Product
2. Model
3. Size (seat width)
4. Maximum load
5. Manufacturer / contact details
6. Date of manufacture
7. UDI / SN code
8. CE mark
9. Consult instructions for use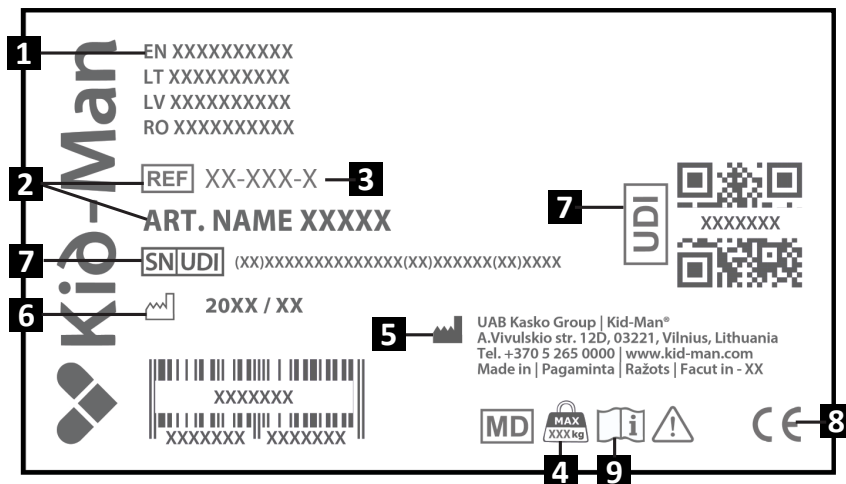


## CONTENT OF PACKAGE

The 4 wheel rollator BEGO is delivered in a cardboard packaging and are ready for use. After unpacking, please keep the cardboard packaging if possible. You may need it later for storage or for the return of the device. Once parcel is received, please check whether it is not damaged during transportation.

- i** If you see signs of damage, please make a note of it to the transport company at the same time.
- i** Please check the contents of the package, and if any of the parts listed below are missing, contact your local dealer.

### Content of package:

1. 1 x Folded rollator with backrest and brakes
2. 1 x Bag
3. Instruction for use

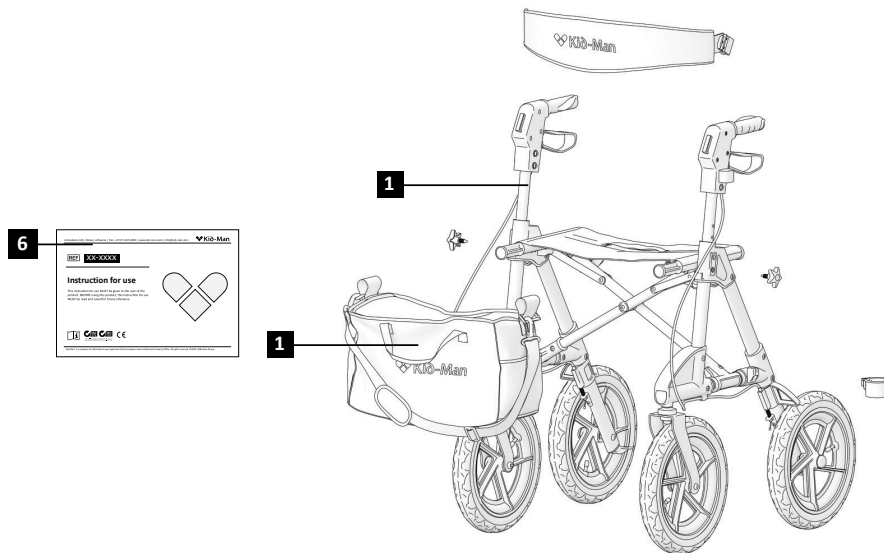## COMPONENTS AND FUNCTIONS

### Overview

1. Backrest
2. Seat
3. Bag
4. Front wheel
5. Handle
6. Brake
7. Length adjustable button for backrest
8. Height adjustable button for handles
9. Frame
10. Rear wheel

## SETTING UP THE ROLLATOR

### To unfold the rollator:

1. Ensure the rollator is on solid, even ground;
2. With your hands flat, press the seat frame down until the seat surface flattens out and is fully set between the frame (Picture 1);
3. Hang the textile bag onto the two bag holders at the front of the rollator.

### To fold the rollator:

1. Locate the folding strap on the seat;
2. Hold the rollator by the handles;
3. Raise the seat using the lifting strap to bring the rollator upward. As you lift, the rollator will automatically fold together (Picture 1).
4. Secure the folded position if necessary and store the rollator.

#### **WARNING**

Danger of trapping your finger!

## OPERATION

### **Braking**

The efficiency of the brakes can be affected by abrasion of the wheels. Therefore, test the brakes regularly. For desired braking strength adjust the brake either manually at the screw on the housing of the brake or with the suitable tool (not included) on the brake wire.

### **To operate the brakes**

To activate the brakes, squeeze the brake levers upward and hold them (Picture 6). To release the brakes, release the levers. To engage parking brakes of the 4 wheel rollator, depress the brake levers until they lock and remain in the place (Picture 7).

#### **WARNING**

Before settling on your seat, make sure the brakes are engaged!

## OPERATION

### **Adjusting the height and length of the backrest**

To adjust the backrest to an extended position, press and hold the adjustment button with one hand. Simultaneously, pull the backrest forward using the other hand (Picture 4 (a.)). Repeat the same procedure on both sides - left and right, to ensure symmetrical adjustment.

The backrest height can also be adjusted. To do so, loosen the locking knob and raise or lower the backrest to the desired position (Picture 4 (b.)). Perform the adjustment equally on both sides to ensure proper alignment.

### **Adjusting the height of the handles**

The height is adjusted in two steps. First, release the handle from the frame. Second, while pressing the red button with one hand, use the other hand to raise or lower the handle to the desired height (Picture 2). Repeat the same for both handles on the left and right sides. The height must be the same on both sides.

The height can be adjusted to 6 different positions. The positions are marked on the inner side of the handles (Picture 3).

### **Walking**

When you walk with your rollator make sure you walk upright, with the rollator near your body. It will give you the correct body posture (Picture 5).

### **Sitting on the rollator**

Make sure the rollator is secured with the parking brakes before sitting on the seat.

When seated, always place your feet between the rear wheels (Picture 6).

### MAINTENANCE

To ensure safe and reliable operation, carry out the following checks and maintenance regularly or have it carried out by another person. Below You will find recommended maintenance's schedule.

	Before use	Once a week	Once a month	Once a year
Check that the wheels are seated correctly		X		
Visual check		X		
Check the folding mechanism			X	
Check the brakes	X			
Clean the wheels			X	
Have 4 wheel rollator checked by an authorized dealer				X

#### WARNING

Repair, cleaning, or adjustment of the product must be performed by an authorized service person while the product is not in use.

#### CAUTION

Only original spare parts can be used for replacement. Please provide model and serial number to dealer or contact manufacturer directly. Manufacturer and dealer are not responsible for any damage occurred to product or user if non original spare parts are used an (or) repair was done in unauthorized facility.

### CLEANING AND DISINFECTION

Should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent.

- i** Sand and seawater can damage the bearings and steel parts of rollator can rust if the surface is damaged. If the 4 wheel rollator is exposed by sand and seawater for short periods, clean it after every trip to the beach.
- i** The rollator may be disinfected by spraying or wiping with tested, approved disinfectants. Spray a soft cleaning and disinfecting product and follow the instructions given by the manufacturer.

## STORAGE

The product should be stored in a dry environment, away from direct sunlight. It should always be kept clean and free of dust.

 **WARNING**  
The product must not be permanently exposed to temperatures greater than +40 °C. Otherwise, the frame and materials may overheat, which could cause burns or material damage.

 **WARNING**  
Never place or store other items on top of the product.

After long-term storage (more than 3 months), the product must be inspected before use (see chapter “MAINTENANCE”).

## ENVIRONMENTAL CONDITIONS

Keep the product under these conditions:  
Temperature limitation –20°C to +40°C;  
Relative humidity 20% to 75%;  
Atmospheric pressure 800 hPa to 1060 hPa.

## RE-USE

The product is suitable for re-use. The following actions must be carried out:  
Cleaning and disinfection (see chapter CLEANING AND DISINFECTION); Inspection (see chapter MAINTENANCE);

The product should be adapted for the new user in accordance with the service documentation.

## DISPOSAL

Be environmentally aware and dispose of the product properly. Disposal is subject to national and local regulations.  
To have the product recycled, please contact your local community, waste disposal facility, or our service department.

**SPECIFICATION**

Model	BEGO			
REF	01-3560GR	01-3560TR	01-3560RD	01-3560BK
Color	Lime green	Turkish blue	Berry red	Graphite black
Maximum safe load (kg)	135			
Weight (kg)	6.9			
Overall length (cm)	65			
Overall width (cm)	53			
Overall height (cm)	81.5 - 94			
Basket length (cm)	15.5			
Basket width (cm)	31.5			
Basket depth (cm)	26			
Width between armrest (cm)	44			
Wheel size (cm)	20 x 3			

## INCIDENTS

**Any serious incident that has occurred in relation to the device should be reported to UAB Kasko Group | Kid-Man® and the competent authority of the Member State in which the user and/or patient is established.**

### CONTACTS:

**Manufacturer:** UAB “Kasko Group” (registered trade mark Kid-Man®)

**Address:** A.Vivulskio 12D, Vilnius, LT-03221, Lithuania

**Phone:** +370 5 265 0000

**E-mail:** [info@kid-man.com](mailto:info@kid-man.com)

**Website:** [www.kid-man.com](http://www.kid-man.com)

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