BENCIS

02-6055

SHOWER CHAIR WITH REMOVABLE ARMRESTS
BRUSEBADESTOL MED AFTAGELIGE ARMLÆN
SUIHKUTUOLI IRROTETTAVILLA KÄSINOJILLA

EN

DK

FI

EN Instruction for use

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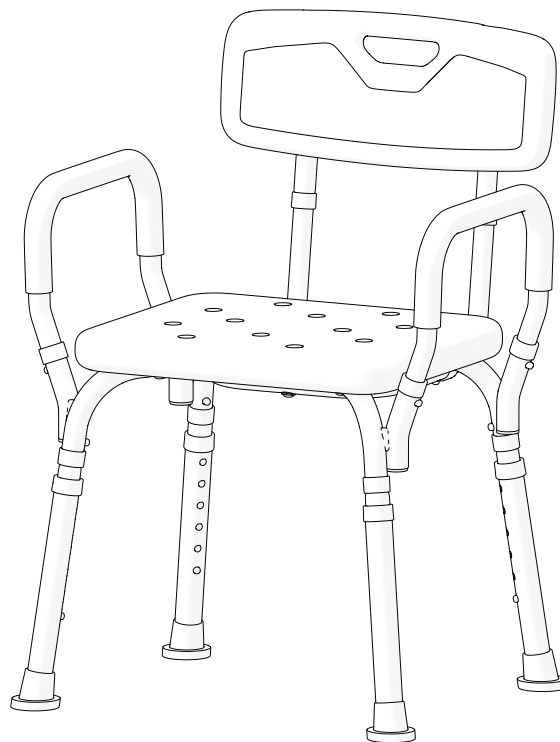
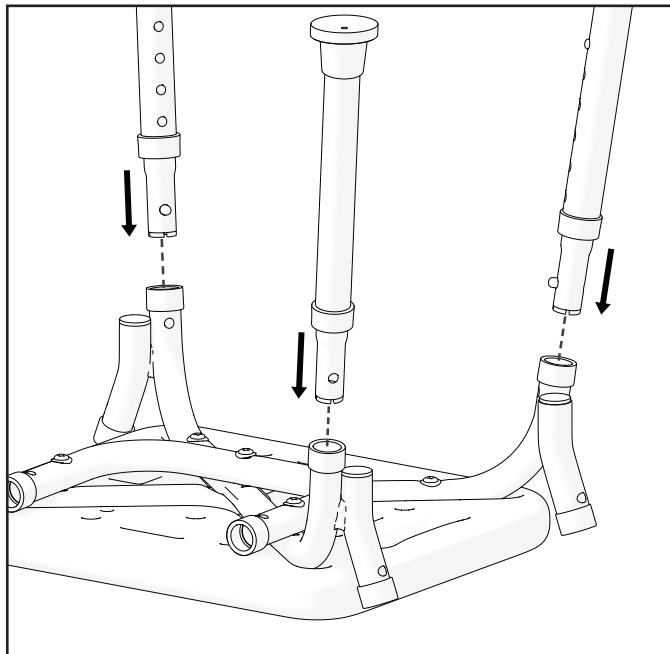
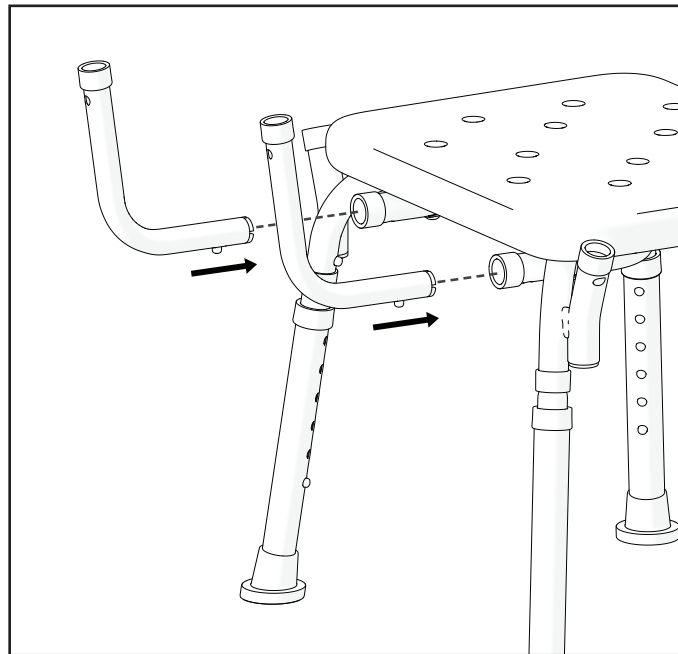
EN This Instruction for use **MUST** be given to the user of the product. **BEFORE** using this product, this Instruction for use **MUST** be read and saved for future reference.

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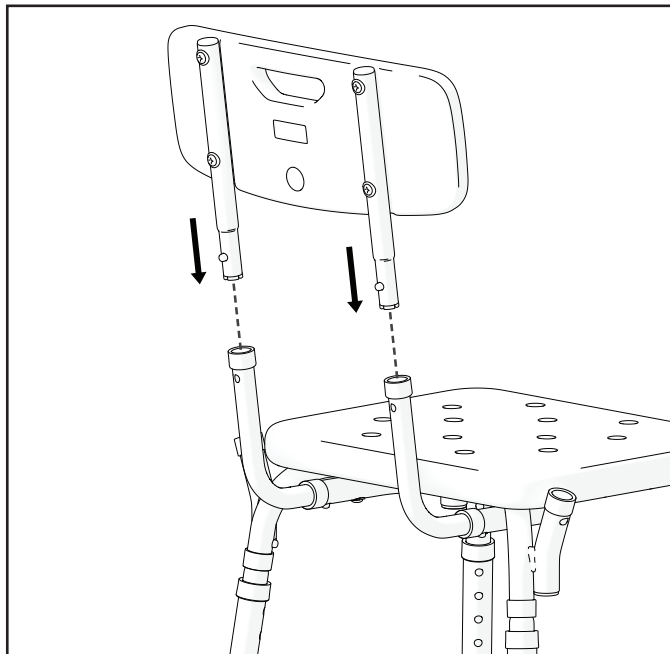
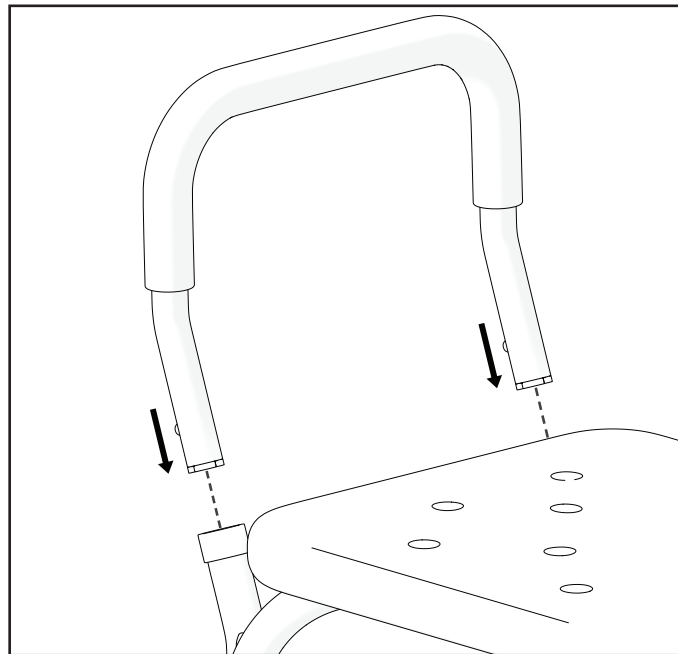
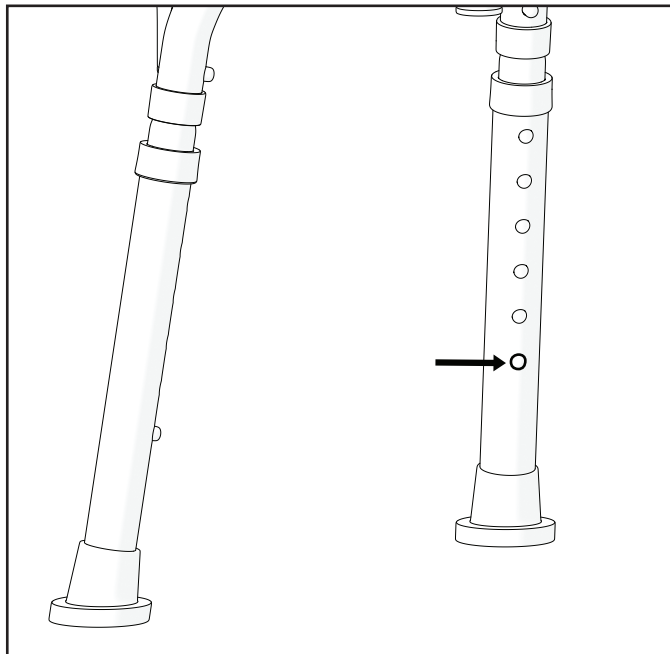
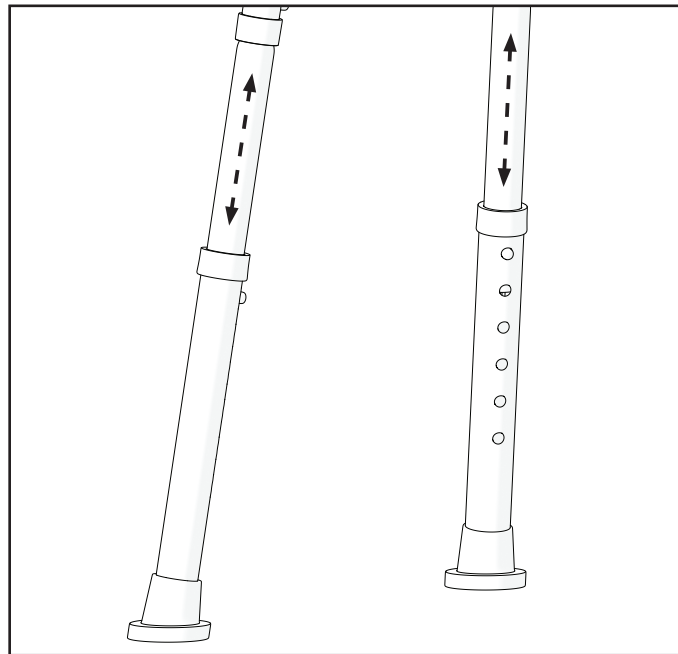
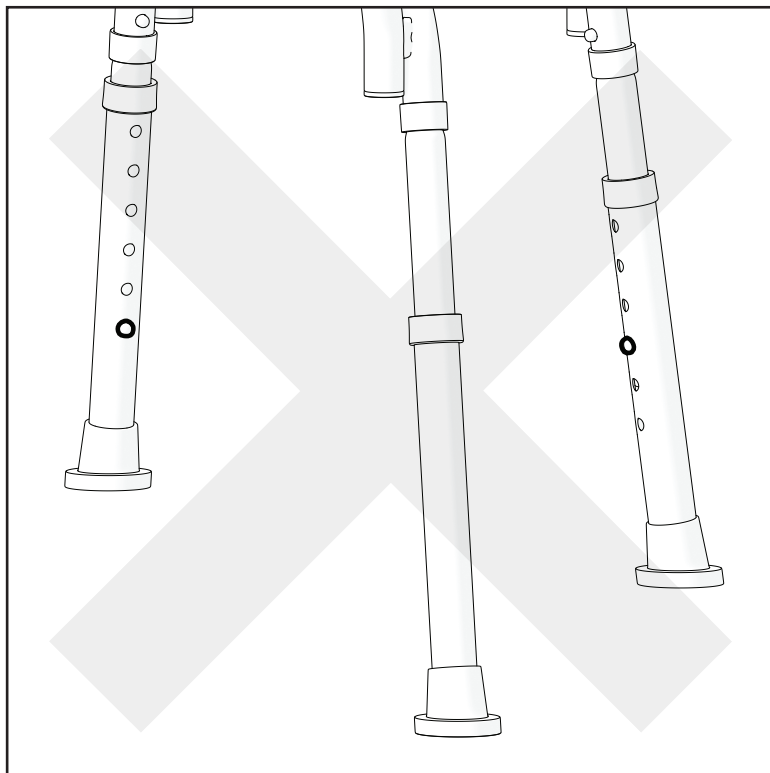
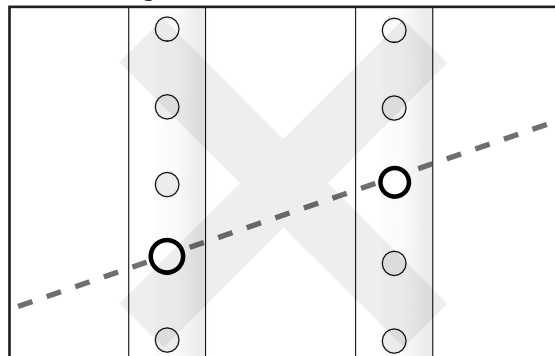
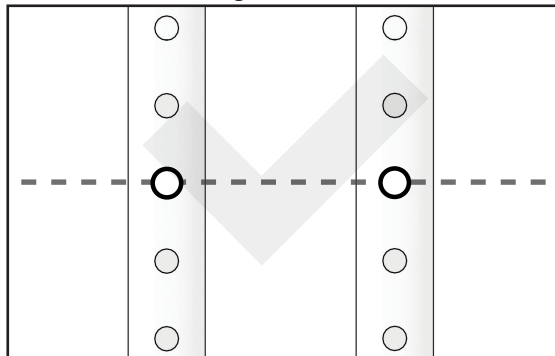
EN

SYMBOLS

In this Instruction for Use, warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger or basic information.

Gives basic information, recommendations, and guidance for efficient, trouble-free use.

WARNING / CAUTION - to indicate that caution is necessary when operating the device. To indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

ATTENTION! This sign is used for very important parts concerning your safety. Please look at the operating instructions.

SAFETY INFORMATION

**WARNING**

Risk of accidents. For proper and safe use, we recommend you follow safety instructions while using your shower chair. This chair was designed to be stable installed and ensure stability while seated. Every movement on the shower chair affects the point of gravity, which may cause the chair to tip over.

**CAUTION**

The shower chair is only suitable for single occupancy.

**WARNING**

All service, cleaning and adjustment should be performed while the shower chair is unoccupied.

WARNING

It is forbidden to exceed the maximum safe load of shower chair.

WARNING

Risk of burning. The shower chair must not be permanently exposed to temperatures greater than +40 °C. Otherwise, the frame as well as the seat, backrest, and armrests can heat up, which might cause burns.

CAUTION

Risk of getting fingers caught. There is always a risk of getting, e.g., fingers or arms, caught in the moving parts of the shower chair.

WARNING

Risk of choking. Please protect children under the age of five years from easy access to the BENCIS shower chair. Due to small elements used in its construction, children could swallow them or choke.

WARNING

The design of your shower chair BENCIS may differ slightly from the model described in this manual due to ongoing product development.

GENERAL

INTRODUCTION

This Instruction for Use contains important information about handling the product. To ensure safety when using the product, read this Instruction for Use carefully and follow the safety instructions.

PREFACE

Thank you for the confidence shown in Kid-Man®. Every product is factory checked and left us in faultless condition. We wish you much success and we hope UAB Kasko Group | Kid-Man® product will make a positive impact to your daily living.

WARRANTY

We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB Kasko Group | Kid-Man® (hereinafter called Manufacturer) after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, Manufacturer, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

LIMITATION OF LIABILITY

Manufacturer accepts no liability for damage arising from:

- Non-compliance with the Instruction for use;
- Incorrect use;
- Natural wear and tear;
- Incorrect assembly or setup by the purchaser or a third-party technical modifications;
- The usage of unapproved 3rd party accessories instead of approved Manufacturer's accessories;
- Unauthorized modifications and/or use of unsuitable spare part.

STANDARDS AND REGULATIONS

The shower chair BENCIS comply with the requirements of:

a) REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009, and repealing Council Directives 90/385/EEC and 93/42/EEC.

SERVICE LIFE

The expected service life is 5 (five) years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this Instruction for Use.

After then, the device must be recycled. Do not throw your product away with common waste (see chapter "DISPOSAL").

INFORMATION ABOUT MANUFACTURER

Manufacturer: UAB “Kasko Group” (registered trade mark Kid-Man®)

Address: A. Vivulskio 12D, Vilnius, LT-03221, Lithuania

Phone: +370 5 265 0000

E-mail: info@kid-man.com

Website: www.kid-man.com

INFORMATION ABOUT PRODUCT

Product: Shower chair with removable armrests

Model: BENCIS

REF: 02-6055

INTENDED PURPOSE AND INDICATIONS

The shower chair BENCIS is intended for use by persons with permanent or temporary mobility disorders who require support when bathing. The chair is designed to be used in a shower area or placed inside a bathtub. BENCIS may be used independently or with the help of an attendant. Thanks to its stable frame and non-slip feet, the chair provides a safe and reliable seating solution during bathing. The backrest and armrests are removable, allowing easier side transfer and individual adjustment for the user. The shower chair is designed for indoor use.

CONTRA-INDICATIONS

There are no known contraindications when using the shower chair BENCIS as intended.

INTENDED USERS

The shower chair BENCIS is intended to be used by persons with disabilities or limited mobility during bathing in a shower or bathtub. The chair has a maximum weight capacity of 100 kg.

WARNING

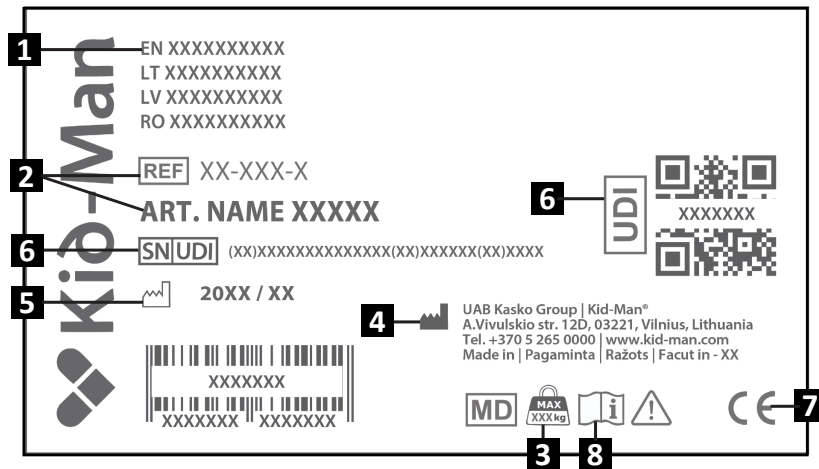
Risk of accidents.

It is forbidden to exceed the maximum safe load of the shower chair.

PRODUCT LABEL

The identification label is attached to the frame of your product (see picture).

1. Product
2. Model
3. Maximum load
4. Manufacturer / contact details
5. Date of manufacture
6. UDI / SN code
7. CE mark
8. Consult instructions for use

CONTENT OF PACKAGE

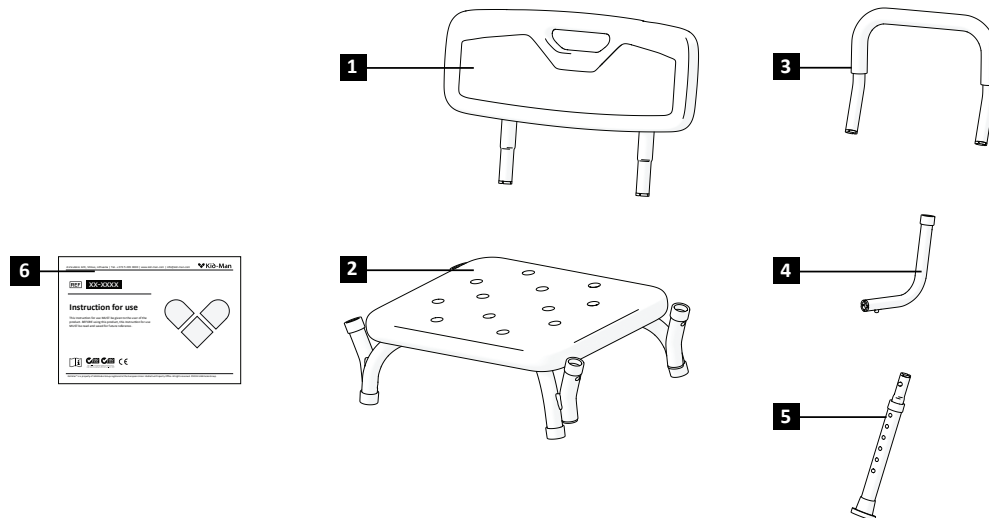
The BENCIS shower chair is delivered in cardboard packaging and is ready for use. After unpacking, please keep the packaging if possible, as you may need it later for storage or for returning the product. Once the parcel is received, please check whether it was damaged during transportation.

i If you see signs of damage, please make a note of it to the transport company at the same time.

i Please check the contents of the package, and if any of the parts listed below are missing, contact your local dealer.


Contents of the package:

1. 1 x Backrest
2. 1 x Seat
3. 2 x Armrest
4. 2 x L-shaped back tube
5. 4 x Leg
6. Instruction for use

COMPONENTS AND FUNCTIONS**Overview:**

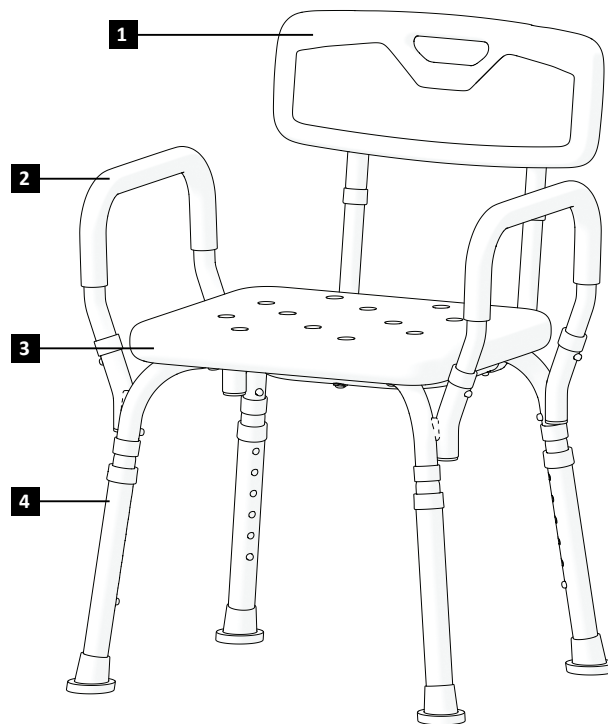1. Backrest
2. Armrest
3. Seat
4. Leg

The design of the device may differ slightly from the one described in this manual due to continuous product development.

SETTING UP THE SHOWER CHAIR

To set up the shower chair:

1. Place the seat face down on a level surface. Insert the four legs into the frame. Depress the spring-loaded button and push the tube in until the button “clicks” and locks into position (Picture 1).
2. Place the chair upright so that it is resting evenly on all four legs. Insert the L-shaped back tubes into the seat back holes until the spring-loaded button “clicks” into locked position (Picture 2).
3. Insert the backrest onto the L-shaped back tubes until the spring-loaded buttons “click” and lock into position (Picture 3).
4. Insert the armrest into the frame. Depress both spring-loaded buttons and push down until the buttons “click” and lock into position. Repeat this step for the opposite armrest (Picture 4).

Armrests should curve inwards towards each other.

OPERATION

Adjusting the height

Each leg has 6 height levels. Use the push buttons and numbered holes on the legs to set the stool to the desired seat height (Pictures 5–6). Ensure that the bath seat is level and stable, and that all push buttons are fully engaged before use (Pictures 7–9).

WARNING

Danger of trapping your finger!

MAINTENANCE

To ensure safe and reliable operation, carry out the following checks and maintenance regularly or have it carried out by another person. Below You will find recommended maintenance's schedule.

	Before use	Once a week	Once a month	Once a year
Check that the shower chair is seated correctly and is stable	X			
Visual check		X		
Check rubber tips		X		
Check if spring-loaded buttons are fixated	X			
Have shower chairs checked by an authorized dealer				X

WARNING

Repair, cleaning, or adjustment of the product must be performed by an authorized service person while the product is not in use.

CAUTION

Only original spare parts can be used for replacement. Please provide model and serial number to dealer or contact manufacturer directly. Manufacturer and dealer are not responsible for any damage occurred to product or user if non original spare parts are used an (or) repair was done in unauthorized facility.

STORAGE

The product should be stored in a dry environment, away from direct sunlight.

DISPOSAL

Be environmentally aware and dispose of the product properly. Disposal is subject to national and local regulations. To have the product recycled, please contact your local community, waste disposal facility, or our service department.

CLEANING AND DISINFECTION

CLEANING

- It is recommended to use disposable gloves.
- It is recommended to use a pH neutral detergent and warm water for cleaning before disinfecting.
- Use pH neutral detergent and warm water for cleaning.
- Starting from the top, clean the backrest and arms (remember to clean under the arms).
- Should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent.

DISINFECTION

- It is recommended to use disposable gloves.
- It is recommended to use chlorine-based disinfectant solution for disinfection.

ENVIRONMENTAL CONDITIONS

Keep the product under these conditions:
Temperature limitation -20° C to +40° C;
Relative humidity 20% to 75%;
Atmospheric pressure 800 hPa to 1060 hPa.

RE-USE

The shower chair is suitable for re-use. The following actions should be taken:
Cleaning and disinfection (see chapter “CLEANING AND DISINFECTION”);
Inspection (see chapter “MAINTENANCE”).

Before handing the product over to a new user, it should be adjusted and verified according to the service documentation, ensuring it is suitable and safe for the new user's needs.

SPECIFICATION

Model	BENCIS
REF	02-6055
Maximum safe load (kg)	100
Weight (kg)	3.2
Overall length (cm)	51.2
Overall width (cm)	53
Overall height (cm)	73.5 - 86
Seat width (cm)	40.4
Seat depth (cm)	33
Seat height (cm)	42 - 55
Height positions	6

INCIDENTS

Any serious incident that has occurred in relation to the device should be reported to UAB Kasko Group | Kid-Man® and the competent authority of the Member State in which the user and/or patient is established.

CONTACTS:

Manufacturer: UAB “Kasko Group” (registered trade mark Kid-Man®)

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