drive

DeVilbiss®
HEALTHCARE

Hermès

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User Manual
Benutzerhandbuch

LÈVE-PATIENT
PATIENT LIFT
PATIENTENLIFTER

CE

Lève-patient *Hermès*

SOMMAIRE

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Hermès patient lift

TABLE OF CONTENT

I - WARNINGS	14
II - USED SYMBOLS	14
III - SPECIFICATIONS	15
3.1 <i>Hermès</i>	15
3.2 Lifting assembly	16
IV - INTENDED USE	16
V - METHOD OF ASSEMBLY	16
5.1 General terminology	16
5.2 Unpacking	16
5.3 Assembly	17
VI - USING HERMÈS PATIENT LIFT	19
6.1 Commissioning	19
6.2 Charging	19
6.3 Possible problems	19
6.4 Battery pack	20
6.5 Optional class 3 medical weighing system	20
VII - TRANSFERRING A PATIENT	20
7.1 Base spacing	20
7.2 Transferring the patient from a bed to an armchair	20
7.3 Transferring the patient from an armchair to a bed	20
7.4 Transferring a patient lying on the floor	20
7.5 Moving	21
VIII - DRIVE DEVILBISS HEALTHCARE SLING RANGE, AND MAINTENANCE	21
8.1 Sling Checks	21
8.2 Cleaning	21
IX - HERMÈS PATIENT LIFT MAINTENANCE	21
9.1 Cleaning	21
9.2 Disinfecting	21
9.3 Daily maintenance	21
9.4 Routine maintenance	21
9.5 Preventive maintenance	21
9.6 Shipping, storage, and disposal	22
X - WEIGHT OF THE MAIN COMPONENTS (IN KGS)	22
XI - WARRANTY CONDITIONS AND LIABILITY	22
XII - BEFORE CONTACTING DRIVE DEVILBISS HEALTHCARE CUSTOMER SERVICE	23
XIII - MAINTENANCE BOOK	24

PLEASE CAREFULLY READ THIS USER MANUAL BEFORE USING THE DEVICE.

- **MAXIMUM PATIENT WEIGHT 250 KG.**
- **ONLY USE COMPATIBLE SLINGS**
- **THIS DEVICE SHOULD BE USED BY COMPETENT PERSONAL.**
- **NEVER USE A DEFECTIVE DEVICE.**
- **SERVICE AND MAINTENANCE REQUIREMENTS ARE MET.**
- **DO NOT ATTEMPT TO LIFT THE HOIST USING THE BOOM AS IT WILL DAMAGE THE ACTUATOR.**

I - WARNINGS

The owner must inform and show the user how to correctly use the patient lift detailing the precautions for use and the risks related to inappropriate use. Pinching areas should be explained in detail. The owner should ensure the user perfectly understands the safety instructions. When assembling and using the device, it is recommended to keep hands away from the pinching areas.

When the patient lift is exposed to heat source (sunlight through a window or radiator) for a long time, ensure there is no burn risk before using the patient lift.

The device should be used in an unobstructed area so that manoeuvres are not disturbed.

Although slings provided by Drive Devilbiss Healthcare are flame retardant, do not use the device in the vicinity of flames and do not smoke near the device.

Do not use or store the patient lift in a place that is too damp or close to splashing water (rain, shower, etc.).

Under certain circumstances the operation of the patient lift operation can be disturbed by radio and electromagnetic waves (proximity of a radiotelephone antenna, etc.). In case of dysfunction, do not use the patient lift.


The patient lift operation can interfere with sensitive electronic devices. In this case, do not use the patient lift.

Because of the component weight and dimensions, there are risks when dismantling and reassembling. Thus, it is highly recommended to be assisted and to beware of the pinching areas.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

II - USED SYMBOLS

	Maximum safety operating load
	Warning
	Read the use instructions
	Device compliant to european medical device regulation
	Device manufacturer
	Serial number
	The device is subject to Directive 2002/96/EC on waste electrical and electronic equipment (WEEE) and marketed after 13 August 2005.
	Medical device

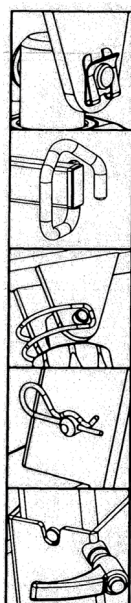- Check the safety clip is in place.
- Check the 4 clips of the hanger bar are present and functional.
- Check the safety ring is present and properly folded down.
- Check the axle locking pin is present
- Check the positioning stud is in the slot provided at the back of the mast support.

Mechanical disengagement of the actuator rod (red ring)	see photo 6 p.7
Battery power supply	24 VDC 2.9 Ah (included into the battery pack)
Built-in battery charger	
Battery charge indicator	with a LED
Buttons	Emergency stop and emergency lowering
Autonomy of the patient lift in number of cycles (lifting/lowering) with an average weight of 100 Kg, with new and charged batteries	40 cycles
CE marking year	2010
Complies with NF EN ISO 10535 standard	

IDENTIFICATION LABEL

The identification label is affixed on the cross bar.

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
REF SR2310000

Device name → **LEVE-PATIENT HERMES**

Manufacturing date (year/month) → **2023-01** **SN** 9XXXXXXX ← Serial number


= 250 kg **MD**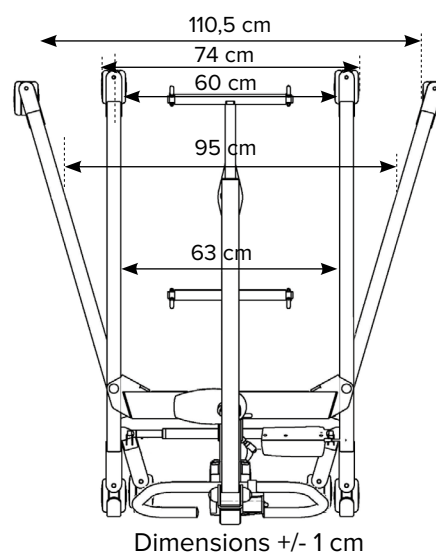

III - SPECIFICATIONS

3.1) *Hermès*

Dimensions (unfolded) L x w x h	1,356 x 741 x 1,320 mm
Lifting capacity	250 kg
Inner distance between the legs	600 mm min. to 1,100 mm max.
Turning diameter	3000 mm
Base height	114 mm
Lifting time	45 s (empty)
Patient lift weight	57 kg
Service life	10,000 cycles at 250 kg
Front castors	dia. 100 mm without brakes
Rear castors	dia. 100 mm equipped with a brake
Structure: steel - Coating:	
Electric actuator (lifting)	24 VDC 12,000 N
Electric actuator (spacing)	24 VDC 3,500 N

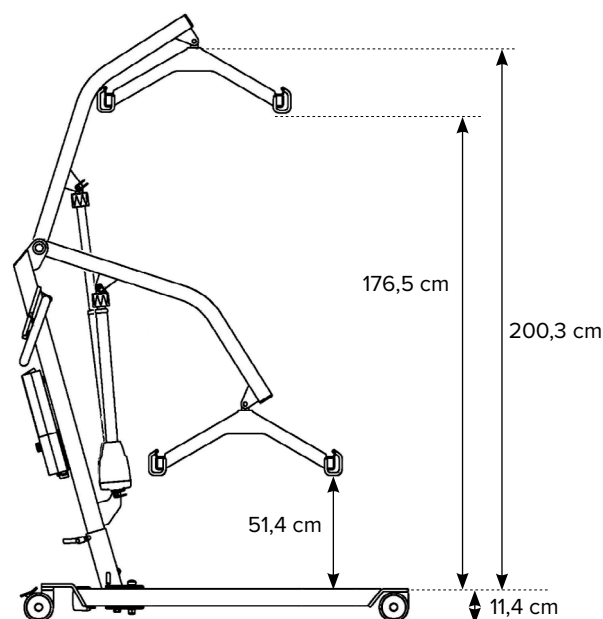Important: The patient lift must only be used with compatible slings. Please contact your local distributor for full information.

Operating force: - Finger-operated: < 5 N
- Hand-operated: < 105 N
- Foot-operated: < 300 N

3.2) Lifting assembly

Autonomous box: (control box + battery pack)

Dimensions	158 x 117 x 95 mm
Weight	3.16 kg
Primary protection	-
Output voltage	Direct 24 V
Output current	400 mA max
Number of outputs	2
Battery capacity	2,9 Ah
Battery voltage	24 VDC
Autonomy	40 cycles
Charging time	4 - 5 h
Emergency lowering button(1) (2) (photo #2, page 6). Charged/discharged battery indicator	On the control box
Protection	IPX4

Actuator :

Power supply	24 VDC
Max. center to center distance	1,070 mm
Max. stroke	350 mm
Max. thrust force	1,000 N
Lifting time	39 s
No-load speed	15 mm/s (no load) 10.4 mm/s (loaded)
Speed loaded	7 mm/s
Weight	5.3 kg
Protection	IPX4
Mechanical disengagement/ Safety nut	yes
Sound level	50 dB

External battery charger:

Power supply	100-240 VAC 50-60 Hz
Output supply	24 VDC
Protection	IPX5

(We reserve the right to change these specifications without notice).

IV - INTENDED USE

- The patient lift is intended for the transfer, in a sitting position, of patients with reduced mobility; it can be used to transfer a patient from a bed to an armchair or a shower or toilet seat and vice versa. It should not be used as mobility aid.


Not suitable for children under 12, for patients with mental disorders, and for patients whose morphology is incompatible with this kind of transfer.

- **Hermès** patient lift can be used to pick up a patient on the floor, but should not be used with a stretcher or any other hanger bar than the one provided with the patient lift.

- Do not use the patient lift to move heavy objects or to perform any activities other than patient transfer.

V - ASSEMBLY PRINCIPLE

5.1) General terminology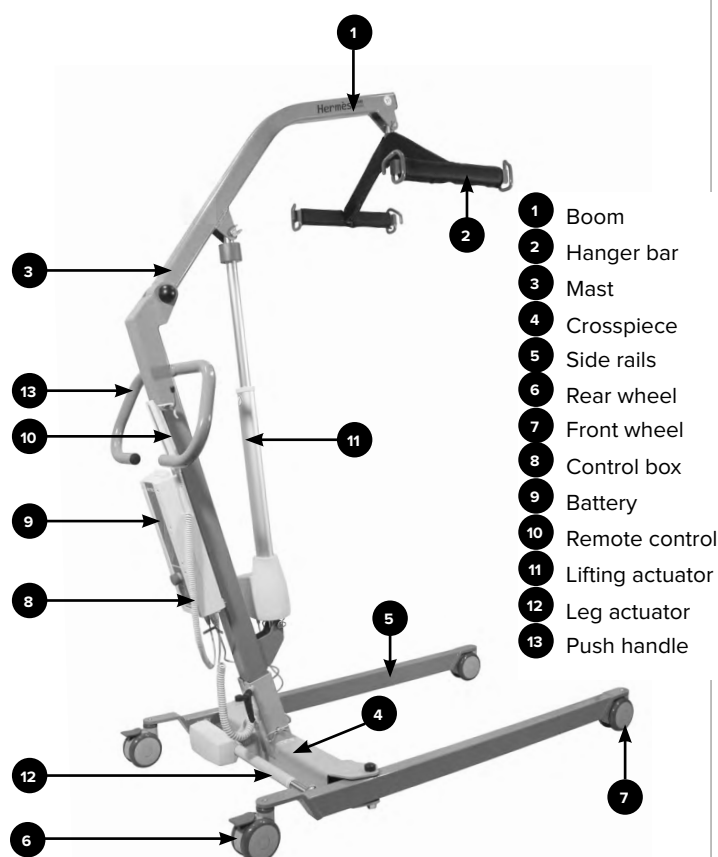

5.2) Unpacking

Make sure all the device elements have not been damaged during transport. Make sure the accessories are present: the lifting assembly (box with battery pack, battery charger, actuator, and remote control), a user manual, and possibly a sling if it was ordered. Pass the power cord through the anti-pull-out system located under the control box so that it cannot be pulled out (photo #4, page 19).

Note: Our device is assembled and tested in our manufacturing site, but its packaging requires disassembly.


5.3) Assembly

A - Provided parts:

- 1- *Hermès* structure with lifting kit: box (including battery pack) + battery charger, actuator, and remote control (photo #1),
- 2- *Hermès* user manual,
- 3- Possibly one sling, if it was ordered.

B - Structure installation :


- 1 - Check the components are not damaged.
Place the base (1) on the floor.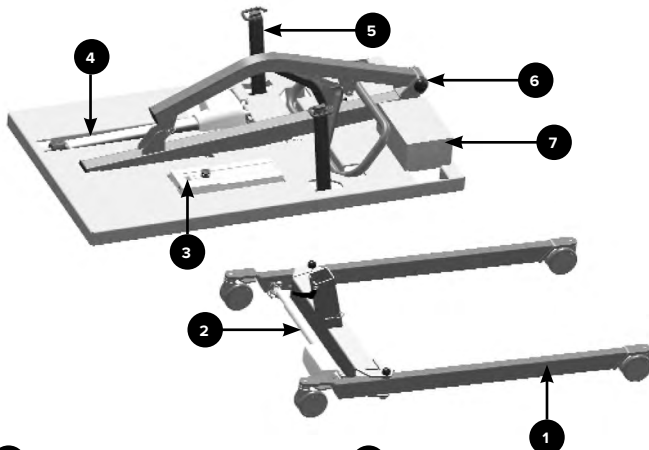

- | | |
|---------------------------|------------------------------|
| 1 Base | 5 Hanger bar |
| 2 Leg actuator | 6 Mast/boom assembly |
| 3 Battery and control box | 7 Weighing system (optional) |
| 4 Lifting actuator | |


- 2 - Take out the mast/boom/hanger bar assembly (**M+F+P**).
Remove the safety ring (**AS**) and the axle (**AX**)/safety clip (**C**) to disengage the hanger bar (**P**).

Put the parts aside.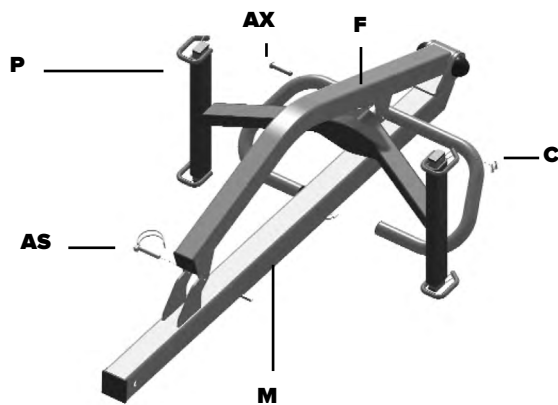

- 3 - Insert the mast (**M**) on the base (**1**) until the positioning stud is in its slot.
Install the axle (**A1**) and the axle locking pin.


Tighten the handle until the play between the mast (**M**) and the base (**1**) is totally reduced.

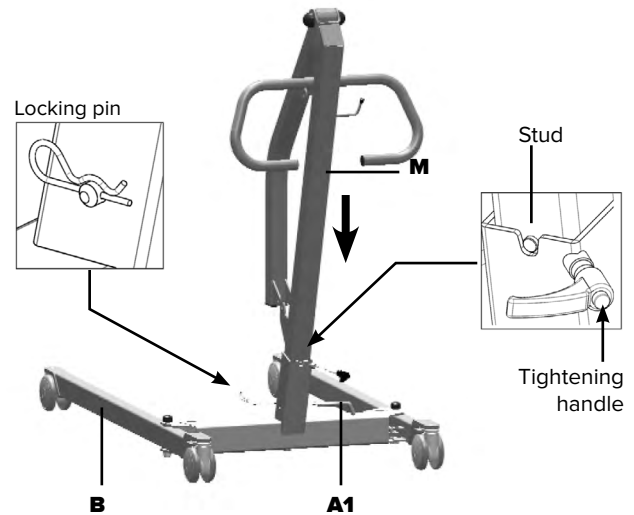- 4 - Assemble the lifting actuator: fix the lower part of the body on the mast (**M**) with the safety ring (**AS**), then fix the actuator head onto the boom (**F**) with the second safety ring (**AS**).


Caution: fold back the clips of the 2 safety rings (**AS**).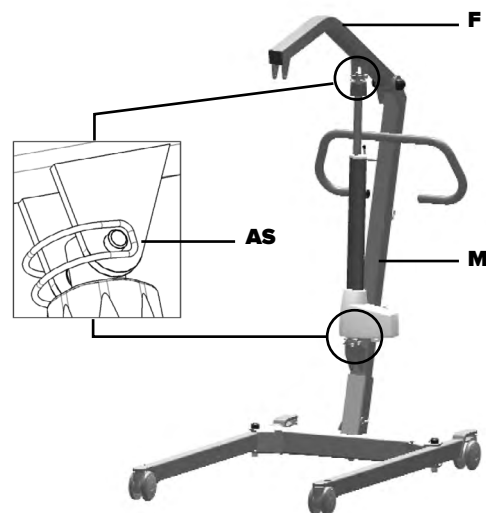

- 5 - In case of optional weighing system:


Place the weighing system at the end of the boom (**F**) using the axle (**AX**) and the safety clip (**C**).
Then assemble the hanger bar (**P**) at the end of the weighing system using the second axle (**AX**) and the safety clip (**C**).
Be careful when installing the safety clips (**C**), as they must be fully inserted in the groove of their axle (**AX**).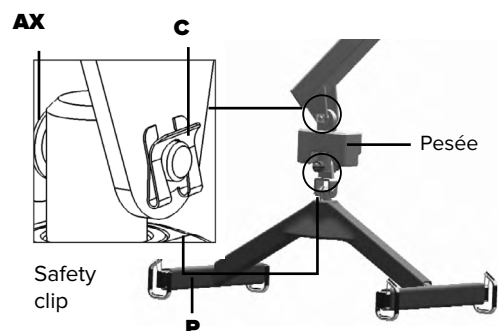


6 -Without weighing system:

Assemble the hanger bar (**P**) at the end of the boom (**F**) using the axle (**AX**) and the safety clip (**C**).


Be careful when placing the safety clip (**C**) as it must be fully inserted in the groove of the axle (**AX**).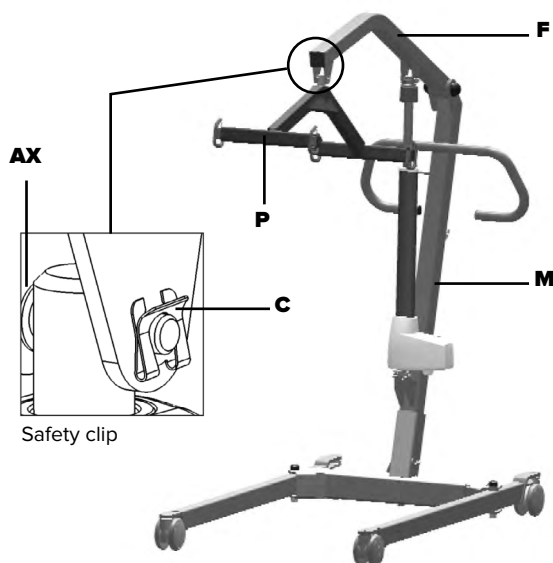

7 -Assemble the control box on the mast using a 3 hexagonal key by unscrewing then screwing the screw (**V**).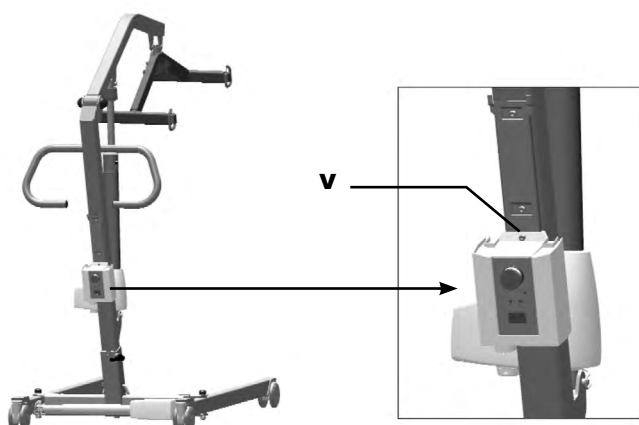

8 -Plug the actuator power cords and the remote control under the control box.

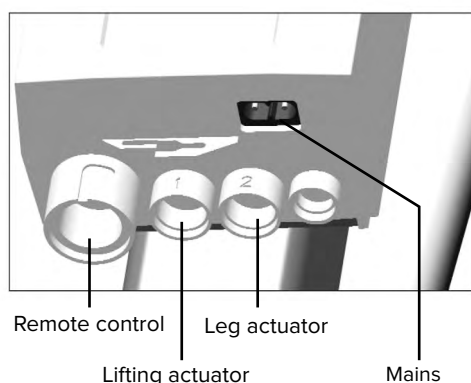9 -Clip the battery onto the control box.

Check all the fastenings: safety rings (AS), safety clip (C), axle (AX), and locking pin.

Perform dry runs before commissioning the patient lift.

Assembly must be performed by an experienced and qualified technician. Beware of the weight and bulk of the assembly and the possible risk of pinching.

The assembly of the patient lift is the sole responsibility of the customer. Drive DeVilbiss Healthcare cannot be held responsible for any malfunction caused by the assembly.

10 - Check the patient lift is correctly charged:

- "CHARGE" light off = battery charged
- "CHARGE" light on orange = charging in progress
- "ON" light on = box connected to the mains

CAUTION: before using the device, make sure all the shafts, screws, and pins handled during the installation are perfectly engaged in their respective supports, and they are correctly locked.

Hermès patient lift is operational only if you use a compatible sling and do not exceed the maximum patient weight of 250 Kg

Photo n°1 - Remote control

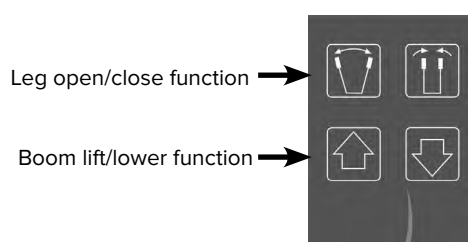
Photo n°2


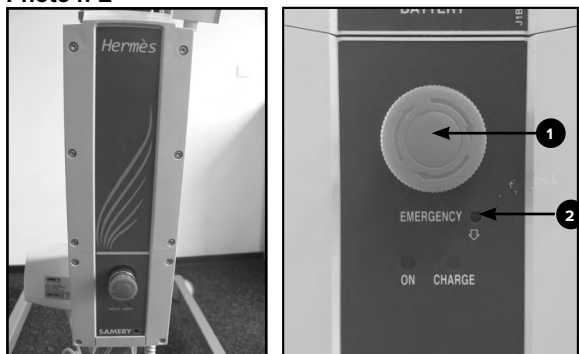
Photo n°3 - Supply box (including battery pack)

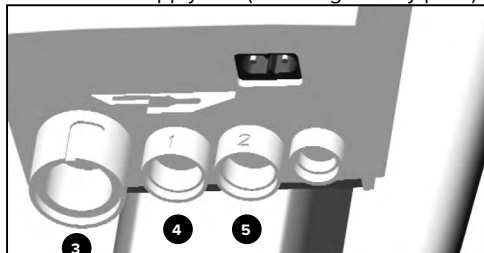- 1 -Emergency stop push button
- 2 -Electric emergency lowering button
- 3 -Remote control connection plug
- 4 -Lifting actuator connection plug
- 5 -Leg actuator connection plug
- 6 -Anti-pull-out system.
- 7 -Battery pack unlocking handle


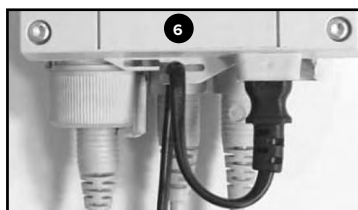
Photo n°4

Photo n°5 - Battery pack unlocking handle

Photo n°6 - Manual lowering ring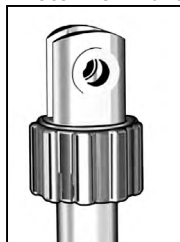

VI - USING HERMÈS PATIENT LIFT

WARNING

Hermès patient lift must be used exclusively indoors on a level, stable, and adherent floor; it must be handled by competent personnel trained in the use of this type of equipment.

6.1) Commissioning

When all the cables are connected:

- Totally charge the battery.
- Operate the patient lift with the remote control.
- Check the charge of the batteries.
- The patient lift is ready to be used.

Note: if an audible signal sounds, put the patient lift on charge.

6.2) Charging

CHARGE THE PATIENT LIFT EVERY EVENING.

Carry out this operation in a well-ventilated room, check the condition of the power cord before plugging it in.

- Connect the cord to a 230-volt mains socket.
- The ON light, green LED (mains presence) and the CHARGE light, orange LED, will light up.
- When charging is complete, the CHARGE light goes out.
- Unplug the cord from the mains socket.
- The patient lift is ready to be used.

YOU CANNOT USE THE PATIENT LIFT WHEN CHARGING.

6.3) Possible problems

If you cannot lower the patient with the remote control, use the emergency lowering:

- Electric: with the tip of a pen, press "EMERGENCY" on the control box or manual: turn the red lifting actuator ring.
- If an audible signal sounds when lifting or lowering, charge the patient lift (for 4 hours max.).
- When charging, if the ON light (mains presence) does not come on, check the connection of the power cord, and check if the mains socket properly works.
- If the remote control or the actuator does not work, check the connections.

IN CASE OF PROBLEM WHEN LIFTING OR LOWERING A PATIENT, PRESS THE EMERGENCY STOP BUTTON (photo #3). RESET IT BY PULLING IT.

IN CASE OF POWER SUPPLY PROBLEM, IT IS POSSIBLE TO MANUALLY LOWER OR LIFT THE BOOM BY PRESSING AND TURNING THE RED RING LOCATED AT THE END OF THE Actuator ROD (photo #5). DO NOT FORGET TO REPOSITION IT.

CAUTION: The actuator must only be used discontinuously, that is to say: 2 minutes operating, and then 18 minutes resting.

Do not exceed 2 minutes of continuous operation.

Any duty cycle exceeding the above cycle can damage the lifting assembly (actuator/control box). This damage is excluded from the warranty.

6.4) Battery pack

The battery included in the box of the battery pack does not require any special maintenance. Though, it should be regularly charged to maintain its service life (at least every 3 months).

With a second battery pack (available on request), the patient lift can work 24/24 hours.

6.5) Optional class 3 medical weighing system

A user manual for the weighing system is provided with these instructions.

(non contractual photo)

CAUTION: the maximum permissible load of 300 kg, indicated on the weighing system, applies to the weighing system only, and not to the patient lift to which it is attached (maximum lifting capacity of the patient lift: 250 kg).

The fastening system of the weighing system was adapted to *Hermès* patient lift. Drive DeVilbiss Healthcare does not accept any liability if the weighing system is used on a device other than the patient lifts manufactured by Drive DeVilbiss Healthcare.

When lifting, lowering or transferring a patient, never exert lateral forces on the hanger bar or on the patient.

The weighing system fasteners should never clash with on the patient lift boom or the hanger bar. THERE IS A RISK OF BREAKAGE.

The condition of the fasteners should be checked annually.

VII- TRANSFERRING A PATIENT

Before positioning and transferring the patient, make sure:

- The patient's weight does not exceed the maximum user weight for the patient lift (250 kg max.),
- The battery is charged and the patient lift is fully operational,
- The floor is flat and stable,
- The patient lift brakes are functional.
- Check the sling is compatible with *Hermès* patient lift and its size is appropriate for the patient.
- Check the sling fasteners and patient support components are in perfect condition.
- In the maintenance book included in this manual, check regular maintenance is performed once a year at least.
- If the patient is agitated, an attendant should secure the patient's movement.

CAUTION: Open the patient lift base as much as possible before lifting or moving the patient. Do not engage the brakes when transferring the patient. Do not continue lifting or lowering the patient if the 4 castors are not on the floor. Lifting

should be done very gradually, checking the patient is well positioned in the sling and there is no risk of tipping due to excessive overhang.

7.1) Leg spacing

- Before opening or closing the legs, which consists of the two side rails, make sure the brakes are not engaged.
- Press the button on the remote control to open or close the legs of the patient lift.

7.2) Transferring the patient from a bed to an armchair

- Position the patient on his/her side.
- Place the sling under the patient.
- Check the back part of the sling is correctly positioned (from the shoulders to the tail bone).
- Lay the patient on his/her back.
- Pass each leg support under each thigh.
- Cross the fastening buckles of each leg support.
- Position the patient lift at the patient's level.
- Hang each loop of the sling to the hanger bar hooks.
- The patient can now be transferred from the bed to an armchair.

7.3) Transferring the patient from a bed to an armchair

- Lean the patient forward while holding him/her.
- Slide the sling behind the patient's back up to the tail bone.
- Pass each leg support under each thigh crossing them.
- Cross the fastening buckles of each leg support.
- Position the patient lift at the patient's level.
- Hang each loop of the sling to the hanger bar hooks.
- The patient can now be transferred from an armchair to a bed.

Specific points:

- When positioning the sling, make sure it does not wrinkle at the leg level to prevent injury to the patient.

CAUTION! WHEN LIFTING/LOWERING, NEVER ENGAGE THE BRAKES SO THAT THE PATIENT LIFT CAN BE CORRECTLY POSITIONED IN THE AXIS OF THE PATIENT. IN CASE OF POWER SUPPLY PROBLEM, IT IS POSSIBLE TO MANUALLY LOWER OR LIFT THE BOOM BY PRESSING AND TURNING THE RED RING LOCATED AT THE END OF THE ACTUATOR ROD.

7.4) Transferring a patient lying on the floor

CAUTION: The patient's weight should not exceed 250 Kg and the patient lift should be positioned so that the patient's head and torso are inside the legs. Lift the patient very gradually, ensuring he/she remains well positioned within the sling.

7.5) Moving

Although it is not recommended, it is possible to use the patient lift to make a very short trip with the patient suspended (e.g. from bed to shower).

For this operation, use the handles of the patient lift. One person should handle and push the patient lift and two other persons should ensure the stability and safety of the patient. Care must be taken to avoid swinging the patient lift, which can cause the patient to fall. The front castors should be kept as far apart as possible and the patient should be moved very slowly as close to the ground as possible. The nature of the floor must also be taken into consideration and great care must be taken when crossing a doorstep.

Do not push or pull the slings to move the patient lift. Do not move a patient if agitated.

VIII- DRIVE DEVILBISS HEALTHCARE SLING RANGE, AND MAINTENANCE

Transfer slings for 4-point handle bar. Maximum load 250 Kg.

Do not exceed the maximum load of the patient lift.

A full range of slings is available for use with the *Hermès* patient lift. Please contact your local distributor for full details.

8.1) Sling Checks

(Before any use; to be performed by competent personnel)

Before each use, check the condition of the slings: no loose seams, the fabric and slings must not be frayed, no tears or traces of wear (transparent fabric weave).

8.2) Cleaning

(Whenever necessary; to be performed by competent personnel)

- Please refer to the cleaning instructions included with the sling.

IX - HERMÈS PATIENT LIFT MAINTENANCE

9.1) Cleaning

(Monthly or as required; to be carried out by competent personnel).

Unplug the charger and remove the battery pack before servicing and cleaning.

- Wipe down with a clean cloth moistened with a mild detergent and dilute with warm water.
- Wipe surfaces with clean water and a clean cloth, and allow to fully dry before use.
- Do not use too aggressive detergents, do not clean under running water.

9.2) Disinfecting

(Whenever necessary; to be performed by competent personnel).

- Mop up any fluid with paper towels.
- Wipe surfaces using clean water.
- Wipe down with a 0.1% Chlorine solution (1,000ppm). Wipe sur-

faces with clean water and a clean cloth and allow to fully dry before use.

In cases of blood spills or other bodily fluids it is recommended that a 1% Chlorine solution (10,000ppm) is used instead.

Suitable protective clothing must be worn when handling chlorine solutions.

9.3) Daily maintenance

- Charge the patient lift every night:
- CHARGE light, orange LED off = fully charged.
- CHARGE light, orange LED on = insufficiently charged or charging did not work.

9.4) Routine maintenance

(Monthly or as required; to be carried out by competent personnel). Routine maintenance consists in checking the patient lift functionality:

- With the patient lift on level ground, check the 4 castors are touching the floor.
- When the brakes are engaged, it must not be possible to move the patient lift (level and adherent floor).
- Operate the patient lift unloaded, and check the boom and side legs move smoothly and without noise.
- Handle the handles on the mast and check there is no play between the boom and the mast and between the mast and the side rails.
- Check the plastic covers and protective housings are not broken and there are no cracks or missing parts.
- Check the condition of the connecting cables: power cord, remote control, and actuator. If a cable shows the slightest trace of burning, wear, cut or if the wires are visible, quarantine the patient lift immediately and have the cables replaced a competent technician.

9.5) Preventive maintenance

Annually or as required; to be carried out by an authorised person or Drive Devilbiss Sidhil Service Technicians according with paragraphs 9.5.1, 9.5.2, 9.5.3 and 9.5.4 below.

- It is a requirement to carry out preventative maintenance annually.
- Battery pack to be changed approximately every 2-3 years, depending on use.
- Hanger bar suspension pin must be checked every year during maintenance.

Only authorised service personnel or Drive Devilbiss Sidhil Service Technicians should carry out annual preventative maintenance. No special tools are required to carry out annual maintenance, but tools used must be adequate and fit for purpose. All maintenance activities must be carried out according to this manual and LOLER requirements.

9.5.1) Mechanical control:

- Check the welds. Visual inspection.
- Check the alignments. Visual inspection.

9.5.2 Electrical control:

- Check the remote control.

- Check the charger.
- Check the remote control box and the actuator.
- Check the lifting/lowering speed.
- Check the emergency stop and lowering.
- Condition of the electric cables (change the cables if in doubt).

Replace the electric cables, and/or the components to which they are connected, if they show signs of wear, sheathing cuts, plug degradation or blackening of the plastics and sheathing around the connection areas. Replace defective components with original ones.

Note: Never disassemble the protective covers of the actuator, battery pack, remote control, battery charger, and control box.

9.5.3) Charging test:

- Fully charge the battery, and then try to lift a load. If the lifting time is longer than 1.30 minute, replace the battery. To extend its service life, the battery should be regularly charged for daily use, and at least every six months for sporadic use.

The battery should be changed every 2 or 3 years depending on use. Worn or malfunctioning parts must be replaced with original parts provided by Drive DeVilbiss Healthcare.

9.5.4) The slings:

The slings must be checked at least once a month, or more regularly if used intensively, to ensure the seams are not undone, the weft of the upholstery and slings is not visible and there are no torn parts. Never use a defective sling.

Worn or malfunctioning parts must be replaced with original parts manufactured by Drive DeVilbiss Healthcare.

Perform a load test before putting the patient lift back into service.

9.5.5) Maintenance book:

Every maintenance operation or intervention must be recorded in the maintenance book included in these instructions.

9.6) Shipping, storage, and disposal

- The patient lift must be returned or transported in its original packaging.
- Before storing the patient lift, thoroughly clean it and check its operation; store the patient lift in a dry room away from dust and moisture.
- During the storage period, the battery should be recharged every three months.
- When commissioning after storage, recharge the batteries and check their operation before any further use.
- Storage temperature: -10 to +50°C; Humidity: 10 to 80%.
- Service temperature: +10 to +40°C; Humidity: 30 to 80%.
- Defective parts, packaging, and batteries must be disposed of at authorised waste treatment centres.
- This symbol means the patient lift on which it is affixed is subject to Directive 2002/96/EC on waste electrical and electronic equipment (WEEE) and marketed after 13 August 2005.

The device in question must not be disposed of with household waste, and must be handed in to an appropriate waste collection point or a distributor.

- By following these guidelines, you are doing your bit for the environment by helping to preserve natural resources and protect human health.

X - WEIGHT OF THE MAIN COMPONENTS (IN KGS)

- Fully equipped patient lift..... 56.9
- Base equipped with electrical equipment..... 26.4
- Mast and boom equipped with electrical equipment 27
- Equipped hanger bar 3.5
- Equipped base without electrical equipment..... 24.1
- Equipped mast and boom without removable electrical equipment (only the control box is present)..... 18.5

XI - WARRANTY CONDITIONS AND LIABILITY

The Hermès patient lift is guaranteed for 5 years on structure, actuator, and control box, in normal use according to the instructions given in this manual. Negligence, handling errors or accidents are excluded from the warranty. The handset and battery are guaranteed for 24 months.

Any modification or addition of accessories without written consent from Drive DeVilbiss Healthcare will de facto exclude the patient lift from the warranty and from Drive DeVilbiss Healthcare's liability. In particular, we cannot be held liable for the use of incompatible slings.

The periodic checks and inspections mentioned in the previous paragraph must be carried out by authorised personnel or Drive DeVilbiss Sidhil Technicians and recorded in the maintenance book. , we cannot be held responsible if these checks and inspections are not carried out.

Only spare parts and components supplied by Drive DeVilbiss Healthcare can be used for repairs.

A list of spare parts is available on request by contacting Drive DeVilbiss Healthcare Customer Service.

XII - BEFORE CONTACTING DRIVE DEVILBISS HEALTHCARE CUSTOMER SERVICE

Calling the Customer Service can often be avoided!

If the patient lift is not working properly, please perform the following checks first:

PROBLEM	POSSIBLE CAUSE	SOLUTION
Nothing happens when pressing one of the buttons on the remote control.	▶ The remote control cord is not properly plugged in.	▶ Insert the plug correctly.
	▶ The remote control cord is cut.	▶ Replace the remote control.
	▶ The battery pack is not powered.	▶ Deactivate the red push-button.
	▶ The battery pack is discharged.	▶ Charge the battery pack.
	▶ The actuator connection plug is not correctly inserted.	▶ Insert the plug correctly.
	▶ The actuator power cord is cut.	▶ Replace the actuator.
The actuator makes an abnormal noise	▶ The duty cycle 2 min./18 min. was not respected.	▶ Contact Drive Devilbiss Healthcare for verification.
	▶ The pinion or nut supporting the lifting tube is damaged.	
The actuator works but very slowly	▶ The battery pack is discharged.	▶ Charge the battery pack
The batteries do not charge.	▶ The battery pack no longer holds a charge.	▶ Replace the battery pack.
	▶ The power cord is not correctly connected.	▶ Check the mains socket connection.
	▶ There is no voltage at the mains socket.	▶ Check the presence of mains voltage at the wall socket.
The CHARGE light (orange LED) does not switch off after charging	▶ The CHARGE light (orange LED) does not switch off after charging.	▶ Replace the battery.
The ON light (green LED) does not come on when the power cord is connected.	▶ The power cord is not connected correctly.	▶ Check the connection of the cord to the mains socket.
	▶ The power cord is cut.	▶ Replace the power cord.
	▶ There is no voltage at the mains socket.	▶ Check the presence of mains voltage at the wall socket.
The red emergency lowering ring turns on the actuator rod.	▶ The actuator rod is disengaged.	▶ Push the ring upwards.

XIII- MAINTENANCE BOOK

Patient lift serial number:.....

COMPANY :

This document should be photocopied and given to the technician every time the patient lift is serviced.
In case of incident, the liability can only be determined if all the sheets of this book are completed.

DATE	Type of intervention: check, inspection, quotation, repair, etc.	Intervention description	Technician's name and position

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