KABIS

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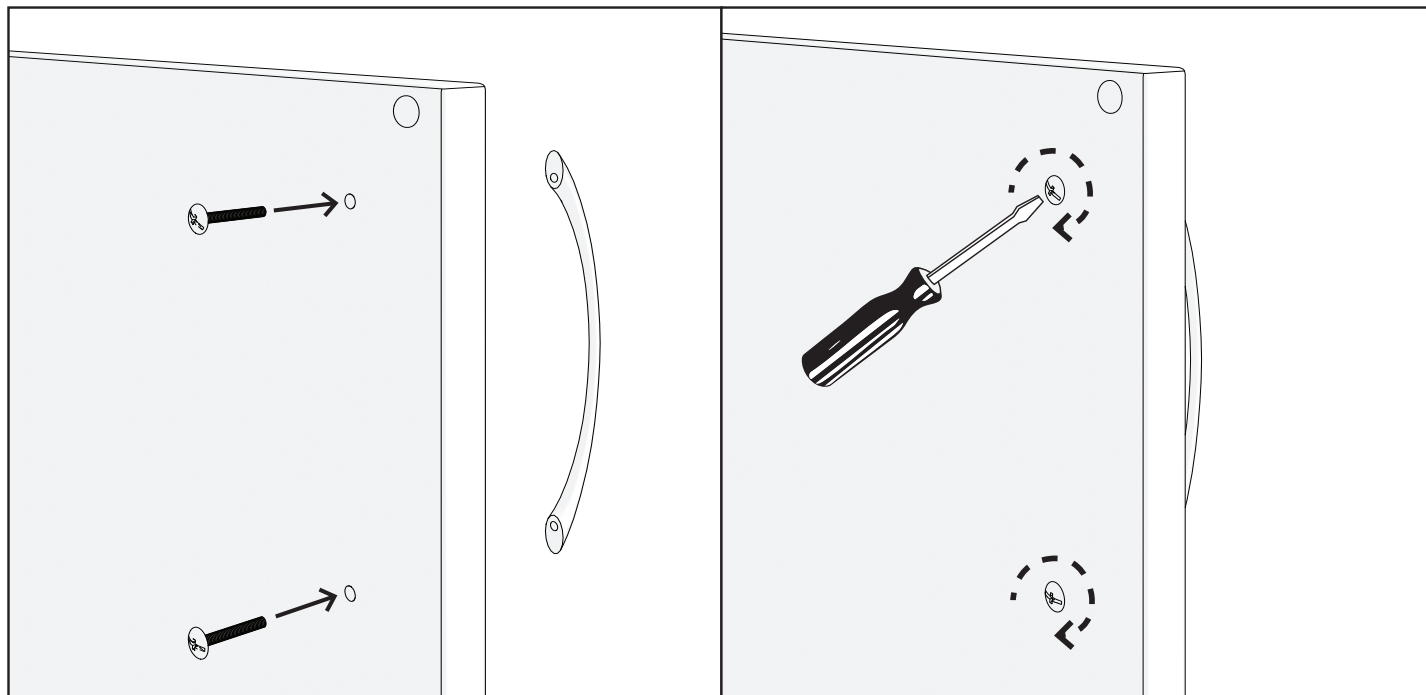
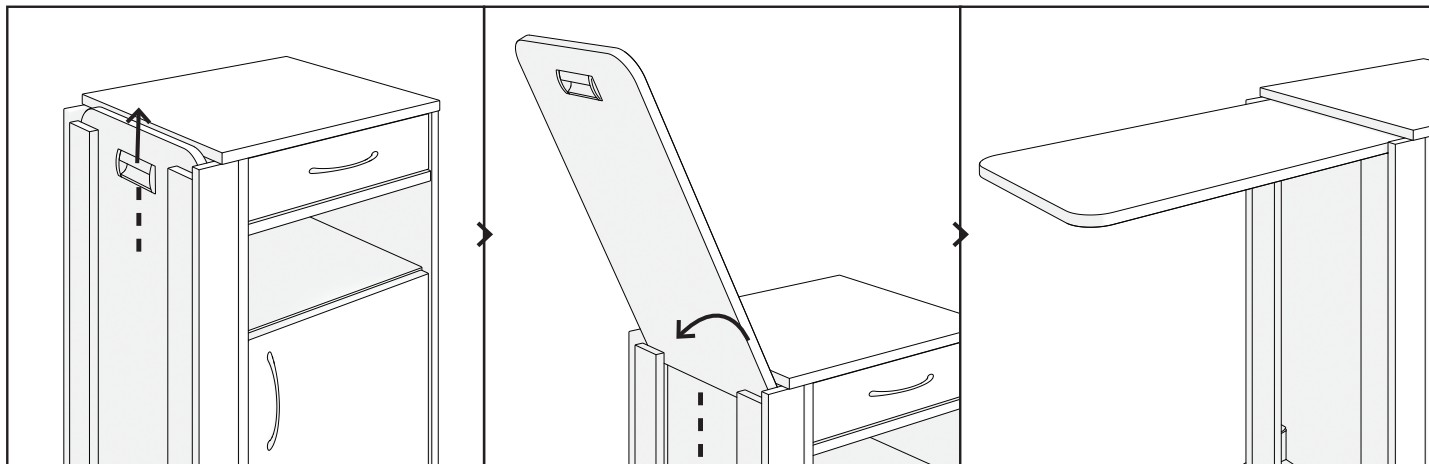
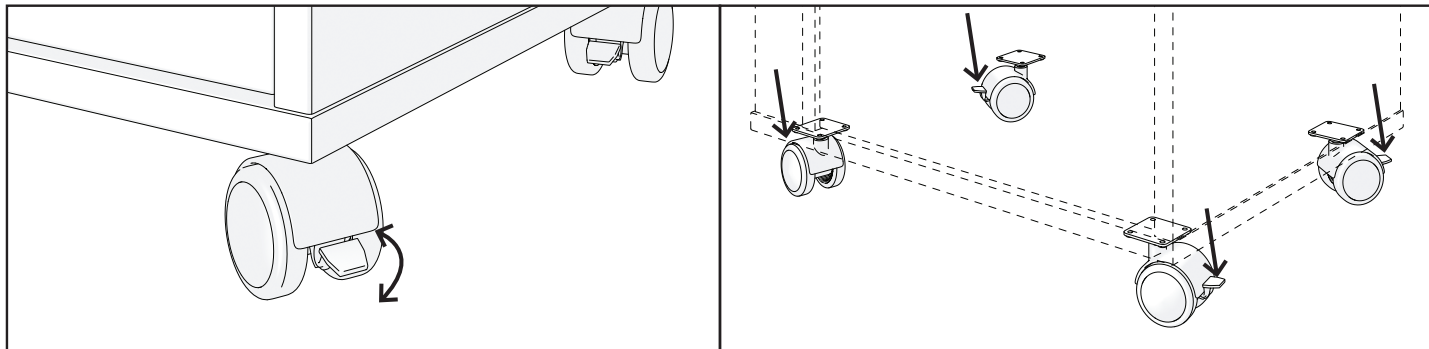
SYMBOLS

In this Instruction for Use warnings are indicated by symbols.

The warning symbols are accompanied by a heading that indicates the severity of the danger or basic information.

 Gives basic information, recommendations and information for efficient, trouble-free use.

 **WARNING / CAUTION** - to indicate that caution is necessary when operating the device. To indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

 **ATTENTION!** This sign is used for very important parts concerning your safety. Please look at the operating instructions.

SAFETY INFORMATION

 **WARNING**
For indoor use only!

 **WARNING**
Keep away from fire!

 **CAUTION**
It's recommended to place the bedside cabinet on the solid base, which would be air permeability.

GENERAL

INTRODUCTION

This Instruction for use contains important information about the handling of the product. To ensure safety when using the product, read the Instruction for use carefully and follow the safety instructions.

Bedside cabinet KABIS is a mobile, user-friendly, and hygienic storage unit designed for use in healthcare environments. It features a durable structure, easy-to-clean surfaces, and a multifunctional tabletop that can be lifted and swiveled for patient care support. The cabinet is equipped with double castors, two of which have locking mechanisms, ensuring both mobility and stability during use.

This instruction for use is available in multiple languages to comply with MDR 2017/745.

Please ensure that the correct language version is provided to the user in their respective Member State.

- i** Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models. If not otherwise stated, each section in this document refers to all models of the product.

PREFACE

Thank you for the confidence shown in Kid-Man. Every product is factory checked and left us in faultless condition.

We wish you much success and we hope UAB Kasko Group | Kid-Man® product will make a positive impact to your daily living.

WARRANTY

We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB “Kasko Group” after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB “Kasko Group”, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

LIMITATION OF LIABILITY

Manufacturer accepts no liability for damage arising from:

- Non-compliance with the Instruction for use;
- Incorrect use;
- Natural wear and tear;
- Incorrect assembly or set-up by the purchaser or a third-party technical modifications;
- The usage of unapproved 3rd party accessories instead of approved Manufacturer's accessories;
- Unauthorized modifications and/or use of unsuitable spare part.

STANDARDS AND REGULATIONS

The bedside cabinet KABIS is not classified as a medical device under REGULATION (EU) 2017/745.

However, the product complies with relevant CE marking requirements and is produced in accordance with the following standards:

ISO 9001 – Quality Management Systems, ISO 14001 – Environmental Management Systems

The cabinet is designed for use in healthcare environments and meets high standards of hygiene, safety, and structural integrity.

SERVICE LIFE

The expected service life of the bedside cabinet KABIS is 5 (five) years, provided it is used daily and in accordance with the safety instructions, maintenance guidelines, and intended use described in this Instruction for Use. After this period, the cabinet must be recycled. Do not dispose of the cabinet with household waste (see chapter "DISPOSAL").

INFORMATION ABOUT MANUFACTURER

Manufacturer: UAB "Kasko Group" (registered trade mark Kid-Man®)

Address: A. Vivulskio 12D, Vilnius, LT-03221, Lithuania

Phone: +370 5 265 0000

E-mail: info@kid-man.com

Website: www.kid-man.com

INFORMATION ABOUT PRODUCT

Product: Bedside cabinet

Model: KABIS

REF: KABIS

INTENDED PURPOSE AND INDICATIONS

Bedside cabinet is intended to be used in hospitals, care homes, and other patient environments. Thanks to possibility of adjusting height and tilt angle of table top it is perfect for eating, reading and etc.

INTENDED USERS

The bedside cabinet KABIS is intended to assist patients and healthcare professionals in their daily clinical or caregiving routines by providing accessible and organized storage as well as a functional overbed table surface.

The maximum load capacity of the table top is 20 kg when adjusted in a horizontal position. If the table top is tilted or positioned at an angle, the load must not exceed 5 kg to ensure safe usage.

CONTRA-INDICATIONS

There are no contraindications known, when using the bedside cabinet as intended.

SIDE EFFECTS

No side effects have been identified. The product is considered safe for use in accordance with the instructions provided.

RESIDUAL RISKS

The bedside cabinet has been designed to meet safety standards, but some residual risks may remain during daily use:

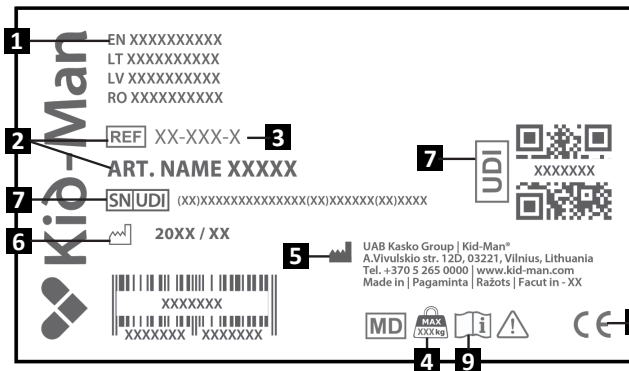
1. Pinching risk – Possible during lifting or swinging of the overbed table if hands are placed near hinges.
2. Dual-sided drawer – May slide through if pulled carelessly, leading to item spillage.
3. Overloading – Exceeding weight limits (20 kg tabletop flat, 5 kg when angled or on side tray) can cause instability.
4. Unlocked castors – Using the cabinet without engaging brakes may lead to unintentional movement.
5. Surface staining – Spills not cleaned promptly can leave marks or reduce hygiene.
6. Wear over time – Frequent use may loosen parts or degrade surfaces without regular maintenance.

These risks are considered acceptable under ISO 14971:2019. Preventive measures such as proper use, weight limits, and maintenance help minimize potential harm.

PRODUCT LABEL

The identification label is attached to the product (see picture).

1. Product
2. Model
3. Size (seat width)
4. Maximum load
5. Manufacturer / contact details
6. Date of manufacture
7. UDI / SN code
8. CE mark
9. Consult instructions for use


CONTENT OF PACKAGE

The bedside cabinet is delivered mostly pre-assembled for your convenience. Only the handles require installation before use.

- i** If you see signs of damage, please notify the transport company immediately upon delivery.
- i** Please check the content of the package, and if any of the parts listed below are missing, contact your local dealer.


Content of package:

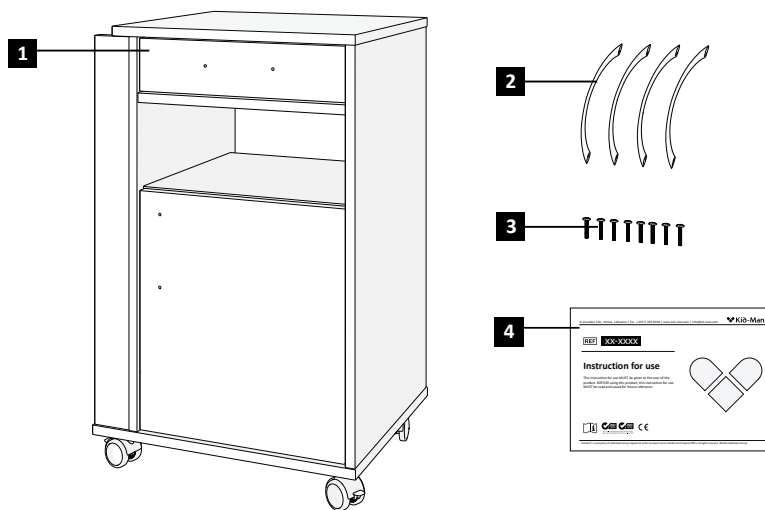1. 1 x bedside cabinet
2. 4 x handles
3. 8 x screws
4. Instruction for use

COMPONENTS AND FUNCTIONS**Overview**

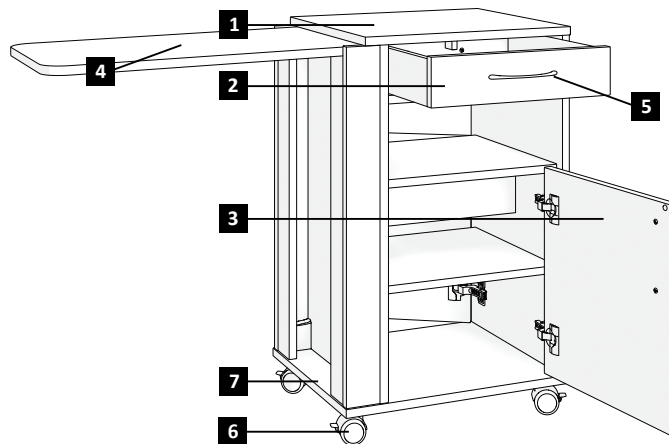1. Tabletop – Flat surface for placing items (main tabletop max load: 20 kg; side tray max load: 5 kg).
2. Drawer – Opens from both sides for flexible access.
3. Lower cabinet with shelf – Door opens from both sides; provides enclosed storage.
4. Pull-out side tray – Auxiliary surface for meals or reading (max load: 5 kg).
5. Handles – Can be mounted on either side for easy maneuvering.
6. Castors – Four wheels with brakes, for safe and stable positioning.
7. Base frame – Provides structural support and overall stability.

**i**

Complexations of your bedside cabinet can differ a little from a bedside cabinet described in this manual due to continuous development of the product.

SETTING UP THE BEDSIDE CABINET

The bedside cabinet is delivered fully assembled, with the exception of the handles.

To complete setup:

1. Attach the four handles to the cabinet, two on the drawer, two on the door;
2. Use the eight screws provided — two screws per handle;
Tighten all screws securely using a suitable cross-head screwdriver. (Picture 1)
After installing the handles, the cabinet is ready for use.

WARNING

Danger of trapping your finger!

CAUTION

For your safety, make sure casters are set in braking position once throughout assembly.

CAUTION

Do not overtighten to avoid damaging the surface.

OPERATION

Side Table Operation

To use the pull-out side table, lift the tray vertically by its integrated handle. Then gently lower the tabletop until it reaches the horizontal position, resting securely on the built-in support points. (Picture 2)

CAUTION

Ensure the table is fully supported before placing items on it (max load: 5 kg).

Braking

The bedside cabinet is equipped with brakes on the caster frame.

For maximum safety, it is recommended to engage all four brakes whenever the cabinet is stationary. (Picture 3)

CAUTION

Always check stability before using the overbed or side table surfaces.

MAINTENANCE

To ensure safe and reliable operation of the bedside cabinet, the following maintenance actions should be carried out regularly by the user or qualified personnel. Below is the recommended maintenance.

	Before use	Once a week	4-6 time per year	Once a year
Visual check	X			
Clean the bedside cabinet		X		
Check castor locking function		X		
Check handles and pull-out side tray mechanism			X	
Have bedside cabinet checked by an authorized dealer				X

WARNING

Repairs or adjustments to the bedside cabinet must be performed only by authorized service personnel, and must not be carried out while the cabinet is in use by a patient.

CLEANING AND DISINFECTION

Clean the bedside cabinet with a soft cloth and mild, non-abrasive detergent.
Disinfect using approved surface disinfectants suitable for plastics.

Follow the disinfectant manufacturer's instructions. Avoid steam, autoclaving, or high heat. Clean before first use, between patients, and regularly.

STORAGE

The bedside cabinet should be stored in a dry, clean environment, protected from direct sunlight and excessive humidity.
Recommended storage temperature: from 0 °C to +50 °C.

Avoid placing heavy items on top of the cabinet during storage to prevent deformation or damage.

RE-USE

The bedside cabinet is suitable for re-use. The following actions should be taken: Cleaning and disinfection (see chapter “CLEANING AND DISINFECTION”); Inspection (see chapter “MAINTENANCE”). Reuse should only occur if the cabinet passes all hygiene and safety checks.

DISPOSAL

Be environmentally aware and dispose of your bedside cabinet properly. Disposal is subject to national and local regulations. To have bedside cabinet properly recycled, please contact your local community, disposal facility or our service department.

SPECIFICATIONS

Model	KABIS
Weight (kg)	33.3
Maximum safe load (kg)	20
Maximum safe load on overbed table (kg)	5
General height (cm)	80
General width (cm)	40
General length (cm)	50
Wheel height (cm)	6.7

INCIDENTS

Any serious incident that has occurred in relation to the device should be reported to UAB Kasko Group | Kid-Man® and the competent authority of the Member State in which the user and/or patient is established.

CONTACTS:

Manufacturer: UAB “Kasko Group” (registered trade mark Kid-Man®)

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