


# Kid-Man® latex examination gloves

Intendent use:

- Instructon for use
- This Instruction for use **MUST** be given to the user of the product. **BEFORE** using this product, this Instruction for use **MUST** be read and saved for future reference

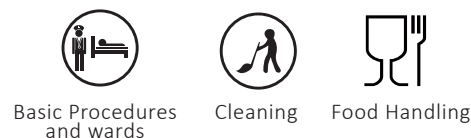

## INTRODUCTION

- The instruction below should be used in conjunction with detailed information on the packaging.
- Disposable examination and protective gloves made of latex; powder-free; non-sterile.  
Caution: Suitable for food contact (No more than 30 minutes of contact time and 40 degrees of contact temperature). Contains natural rubber latex – may cause allergic reactions. Powder-free and DEHP/DOP-free. Compliant with MD Regulation (EU)2017/745 (Class 1), EN455-1:2020, EN455-2:2015, EN455-3:2015, EN455-4:2009, PPE Regulation (EU)2016/425 (CAT III), EN ISO 21420:2020, and EN ISO 374-1:2016+A1:2018, EN ISO 374-5:2016. Check for the damage before use. Do not use damaged gloves.
- Donning & doffing gloves in a proper way is a skill that needs to be practised by healthcare workers and others that use gloves. Donning must be performed in the correct order to prevent transmission of infections. Keep hands clean before donning gloves. When removing the gloves, avoid allowing the outer surface of the gloves to come in contact with your skin, because the surface may have been contaminated with blood and other body fluids. Avoid snapping, as this may cause contaminants to splash into your eyes or mouth or onto your skin or other people nearby.

## COMPONENTS / HAZARDOUS COMPONENTS

Contains natural rubber latex, which can cause allergic reactions (including Type I allergies).  
Discontinue use and consult a physician if allergic symptoms appear.

## LABELING

Gloves are category III, Personal Protective Equipment as per Annex I of the Regulation 2016/425 and comply to standards: EN ISO 21420:2020, EN ISO 374-2:2019.

- Each unit displays the following information: product reference, batch number and expiry date.
- Symbols used in compliance with standard EN ISO 15223: Symbols to be used with medical device labels, labelling and information to be supplied.

**CE 2777**

**Notified Body responsible for certification and ongoing conformity: SATRA Technology Europe Ltd, Bracetown Business Park, Clonee, Dublin, D15 YN2P, Ireland**



Resistance to bacteria and fungi- pass. Resistance to virus- pass.\*The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen.

## KEY FEATURES

- Made of natural rubber latex, providing excellent elasticity and comfort
- Excellent tactile sensitivity – ideal for precision tasks
- Powder-free to minimize contamination
- Good viral and bacterial barrier properties
- Textured fingertips for improved grip
- Ambidextrous, with rolled rim for cuff strength
- Excellent fit, easy to don and remove
- The gloves are suitable for all food contact

<b>MEDICAL</b>	2017/745	CLASS I	EN 455	AQL 1.5
<b>PPE</b>	2016/425	CATEGORY III	EN 374	AQL 1.5


- 100 pcs per box
- 10 boxes per master carton
- 600-800 boxes per palete

## BASIC INFORMATION

Size	Palm width	Overall lenght	Thickness		
			Palm	Cuff	Finger
<b>XS</b> 5-6	75 ± 5 mm	240 mm	Min 0,08 mm	Min 0,07 mm	Min 0,10 mm
<b>S</b> 6-7	85 ± 5 mm				
<b>M</b> 7-8	95 ± 5 mm				
<b>L</b> 8-9	110 ± 5 mm				
<b>XL</b> 9-10	120 ± 5 mm				
<b>XXL</b> 10-11	130 ± 5 mm				

### HOW TO PUT ON DISPOSABLE GLOVES


1

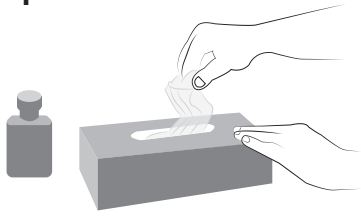


2




3

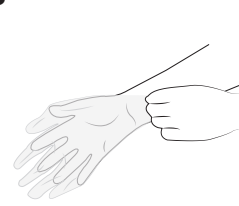

4

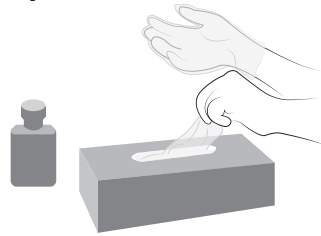

5

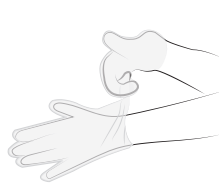

6




### HOW TO PROPERLY REMOVE DISPOSABLE GLOVES


1



2

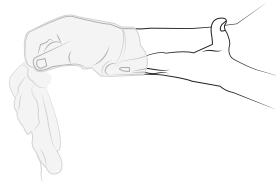

3

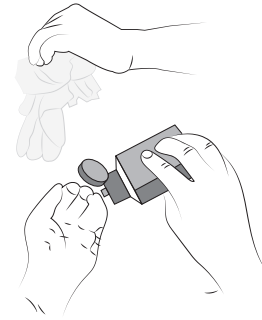

### STORAGE INSTRUCTIONS

Do not expose to direct sunlight, ozone sources or sources of fire. Store in a dry and cool place, at a temperature of 5-38°C. Do not keep in direct vicinity of solvents, oils, fuels and lubricants.

### DISPOSAL



Dispose of in accordance with local regulations for contaminated medical waste.

### SYMBOLS USED ON PACKAGING



Manufacturer



Date of manufacture



Temperature limitation



Consult instructions for use



Lot / batch number



Expiry date



For single use only



Recycling packaging



Product number



Keep dry



Non-sterile



This side up



Medical device



Keep away from sunlight



Suitable for food contact



Handle with care



**UAB Kasko Group | Kid-Man®**  
A.Vivulskio str. 12D, 03221, Vilnius, Lithuania  
www.kid-man.com | info@kid-man.com  
Tel. +370 5 265 0000

Importator / Distributor in Romania : Kid-Man Aid SRL  
Strada Theodor Aman Nr 2-2A, Braila Tel: +40 753 660066

Made in China  
Designed by Kid-Man®

Published by UAB Kasko Group | Kid-Man®

Date of last update 2025-05-13

