# KVADI

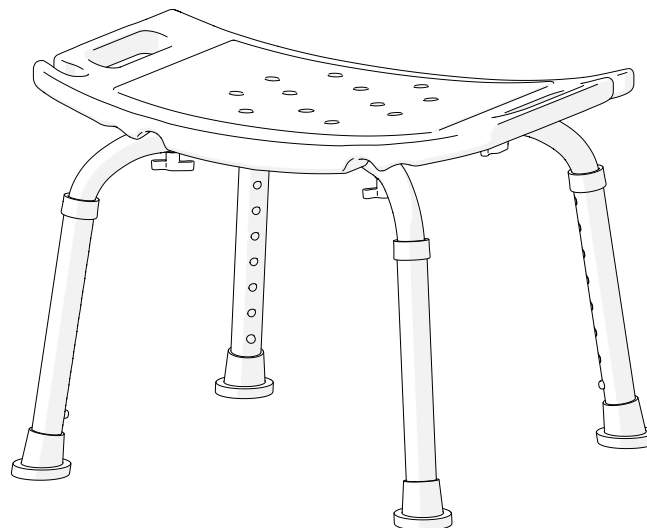
01-5005

**SHOWER CHAIR**  
**BRUSEBADESTOL**  
**SUIHKUTUOLI**

EN

DK

FI

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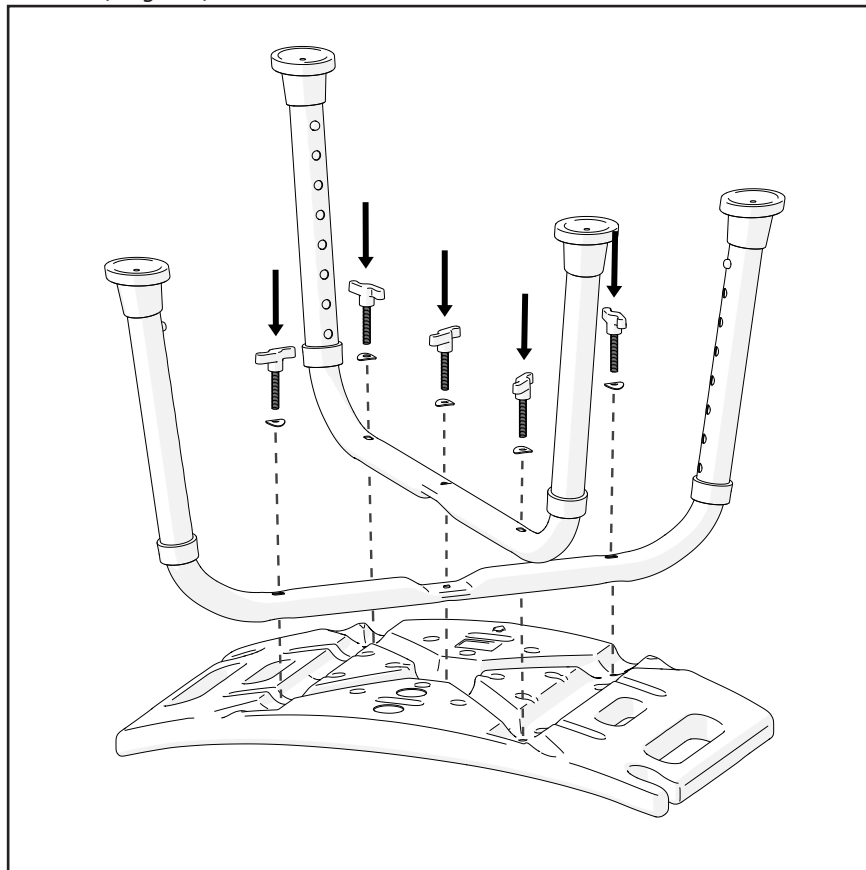
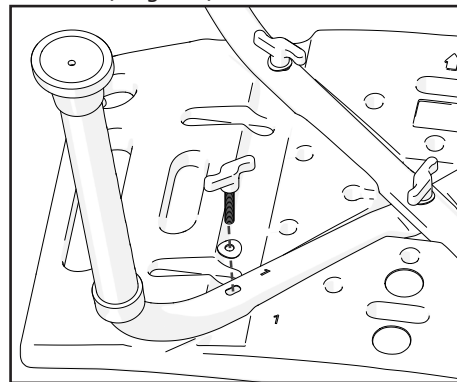
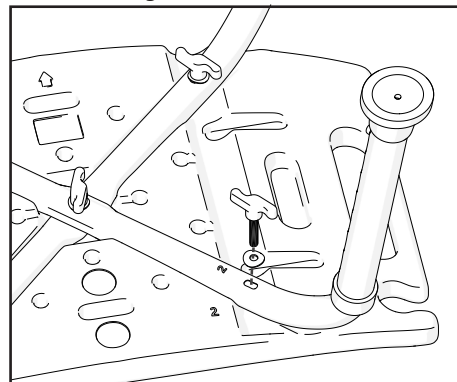
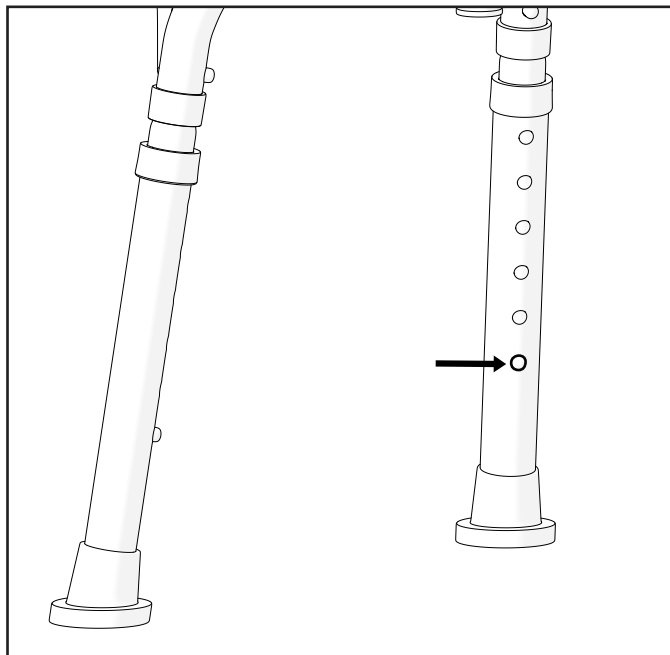
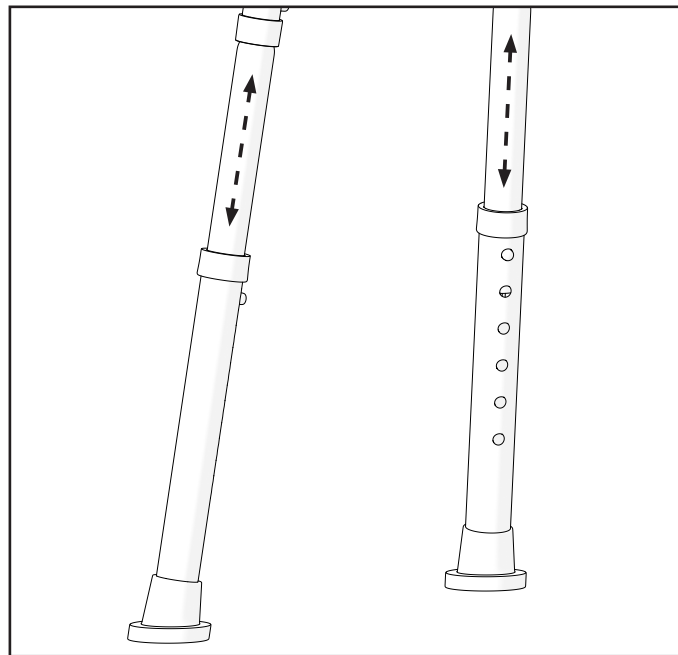
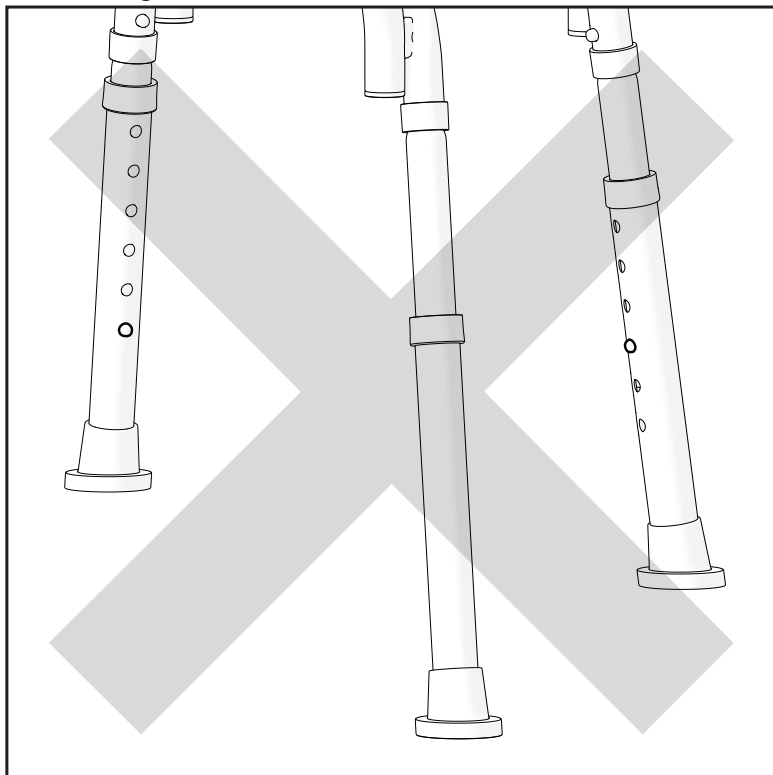
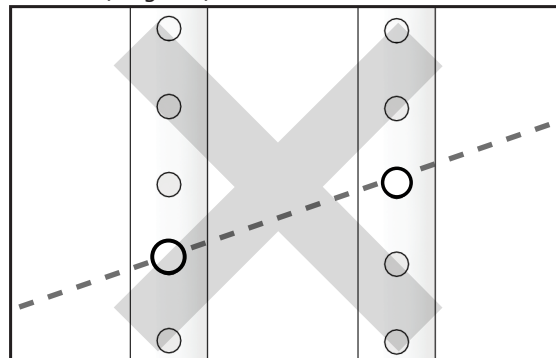
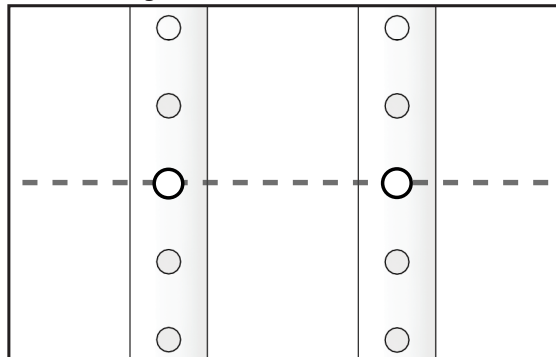
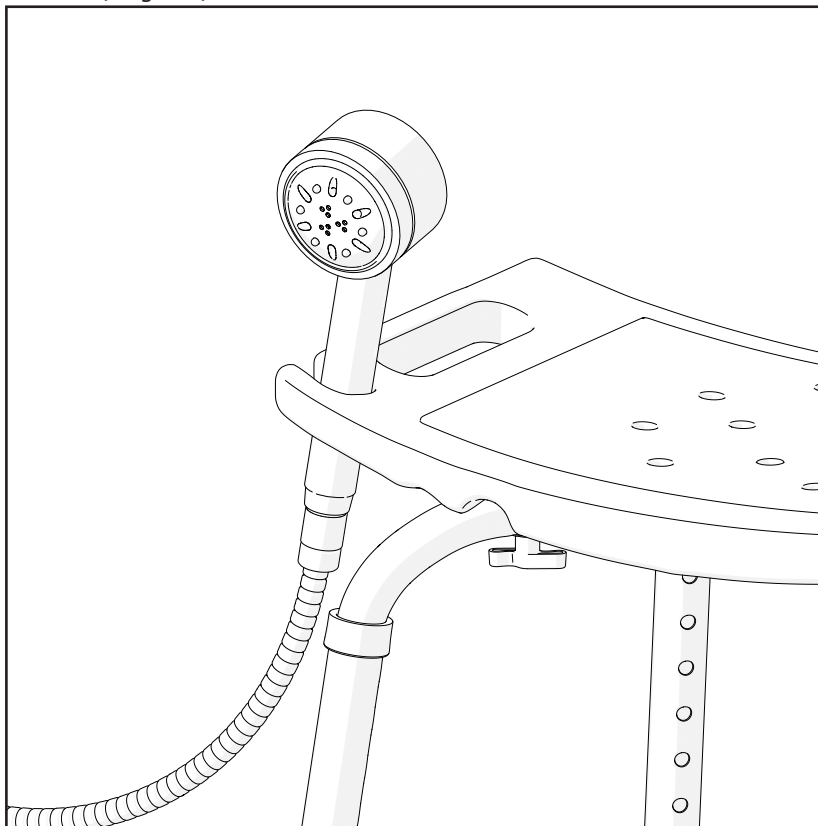
**SYMBOLS**

In this Instruction for Use, warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger or basic information.

Gives basic information, recommendations, and guidance for efficient, trouble-free use.

**WARNING / CAUTION** - to indicate that caution is necessary when operating the device. To indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

**ATTENTION!** This sign is used for very important parts concerning your safety. Please look at the operating instructions.

**SAFETY INFORMATION****WARNING**

**Risk of accidents.** For proper and safe use, we recommend you follow safety instructions while using your shower chair. This chair was designed to be stable installed and ensure stability while seated. Every movement on the shower chair affects the point of gravity, which may cause the chair to tip over.

**CAUTION**

The shower chair is only suitable for single occupancy.

**WARNING**

All service, cleaning and adjustment should be performed while the shower chair is unoccupied.

**WARNING**

It is forbidden to exceed the maximum safe load of shower chair.

**WARNING**

**Risk of burning.** The shower chair must not be permanently exposed to temperatures greater than +40 °C. Otherwise, the frame as well as the seat can heat up, which might cause burns.

**CAUTION**

**Risk of getting fingers caught.** There is always a risk of getting, e.g., fingers or arms, caught in the moving parts of the shower chair.

**WARNING**

**Risk of choking.** Please protect children under the age of five years from easy access to the KVADI shower chair. Due to small elements used in its construction, children could swallow them or choke.

**WARNING**

The design of your shower chair KVADI may differ slightly from the model described in this manual due to ongoing product development.

## GENERAL

### INTRODUCTION

This Instruction for Use contains important information about handling the product. To ensure safety when using the product, read this Instruction for Use carefully and follow the safety instructions.

### PREFACE

Thank you for the confidence shown in Kid-Man®. Every product is factory checked and left us in faultless condition. We wish you much success and we hope UAB Kasko Group | Kid-Man® product will make a positive impact to your daily living.

### WARRANTY

We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB Kasko Group | Kid-Man® (hereinafter called Manufacturer) after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, Manufacturer, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

### LIMITATION OF LIABILITY

Manufacturer accepts no liability for damage arising from:

- Non-compliance with the Instruction for use;
- Incorrect use;
- Natural wear and tear;
- Incorrect assembly or setup by the purchaser or a third-party technical modifications;
- The usage of unapproved 3rd party accessories instead of approved Manufacturer's accessories;
- Unauthorized modifications and/or use of unsuitable spare part.

### STANDARDS AND REGULATIONS

The shower chair KVADI comply with the requirements of:

a) REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009, and repealing Council Directives 90/385/EEC and 93/42/EEC.

### SERVICE LIFE

The expected service life is 5 (five) years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this Instruction for Use.

After then, the device must be recycled. Do not throw your product away with common waste (see chapter "DISPOSAL").

### INFORMATION ABOUT MANUFACTURER

**Manufacturer:** UAB “Kasko Group” (registered trade mark Kid-Man®)  
**Address:** A. Vivulskio 12D, Vilnius, LT-03221, Lithuania  
**Phone:** +370 5 265 0000  
**E-mail:** info@kid-man.com  
**Website:** www.kid-man.com

### INFORMATION ABOUT PRODUCT

**Product:** Shower chair  
**Model:** KVADI  
**REF:** 01-5005

### INTENDED PURPOSE AND INDICATIONS

The shower chair KVADI is intended for use by persons with permanent or temporary mobility disorders who require support when bathing. The chair is designed to be used in a shower area or placed inside a bathtub. KVADI may be used independently or with the help of an attendant. Thanks to its stable frame and non-slip feet, the chair provides a safe and reliable seating solution during bathing. The shower chair is designed for indoor use.

### CONTRA-INDICATIONS

There are no known contraindications when using the shower chair KVADI as intended.

### INTENDED USERS

The shower chair KVADI is intended to be used by persons with disabilities or limited mobility during bathing in a shower or bathtub. The chair has a maximum weight capacity of 135 kg.

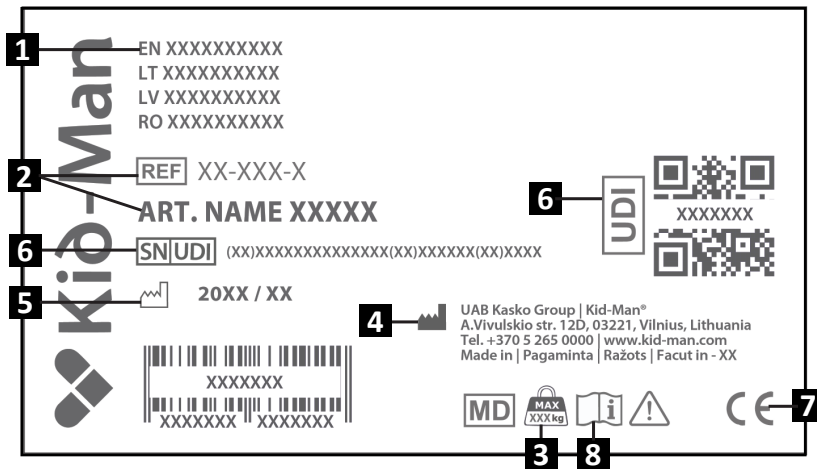
**WARNING**  
**Risk of accidents.**

It is forbidden to exceed the maximum safe load of the shower chair.

PRODUCT LABEL

The identification label is attached to the frame of your product (see picture).

1. Product
2. Model
3. Maximum load
4. Manufacturer / contact details
5. Date of manufacture
6. UDI / SN code
7. CE mark
8. Consult instructions for use

## CONTENT OF PACKAGE

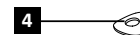
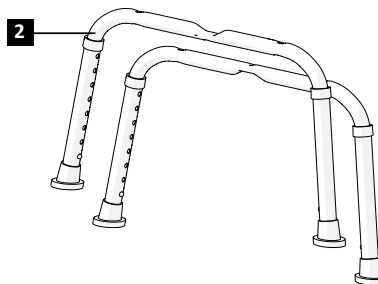
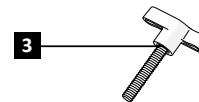
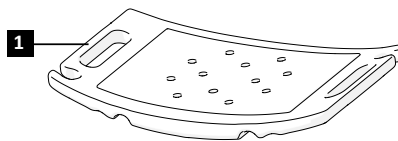
The KVADI shower chair is delivered in cardboard packaging and is ready for use. After unpacking, please keep the packaging if possible, as you may need it later for storage or for returning the product. Once the parcel is received, please check whether it was damaged during transportation.

**i** If you see signs of damage, please make a note of it to the transport company at the same time.

**i** Please check the contents of the package, and if any of the parts listed below are missing, contact your local dealer.


### Contents of the package:

1. 1 x Seat
2. 2 x Curved leg frame
3. 5 x Wing knob screw
4. 5 x Washer
5. Instruction for use

**COMPONENTS AND FUNCTIONS****Overview:**

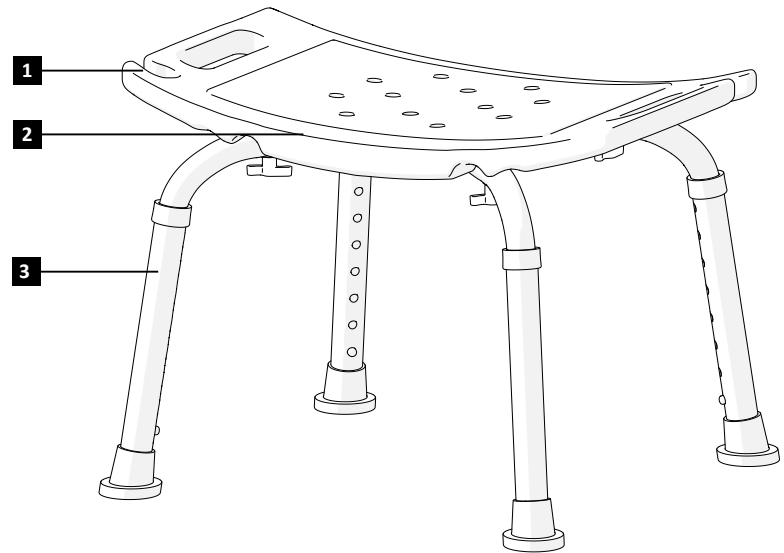1. Holder
2. Seat
3. Leg

The design of the device may differ slightly from the one described in this manual due to continuous product development.

## SETTING UP THE SHOWER CHAIR

To set up the shower chair:

1. Place the seat upside down on a flat, stable surface (Picture 1).
2. Take the two curved leg frames and align them beneath the seat, forming an X shape:
  - One leg frame has a recessed groove marked with a number 1.
  - The other frame, marked with number 2, should sit on top of the first one, fitting into the groove securely.
  - Match the numbers on the leg frames to the corresponding 1 and 2 symbols found on the bottom of the seat.
3. Take the 5 washers and place one on each wing knob screw. Insert the screws into the correct five holes: one in the center and one at each outer corner (Pictures 2-3).
4. Tighten the wing knobs by hand until the leg frames are firmly attached. Do not overtighten.
5. Once assembled, turn the chair upright. Ensure all screws are tight and the chair stands level and steady on all four feet.

Do not mix left and right leg frames!

## OPERATION

### Adjusting the height

Each leg has 8 height levels. Press the spring-loaded button (Picture 4) and slide the inner leg tube to the desired height until the button clicks into a hole (Picture 5).

Make sure all legs are adjusted equally so the chair remains level and stable (Pictures 6-8).

### WARNING

Danger of trapping your finger!

### Holder

On both sides of the chair, there are holders that can be used to hold your shower head (Picture 9).

### MAINTENANCE

To ensure safe and reliable operation, carry out the following checks and maintenance regularly or have it carried out by another person. Below You will find recommended maintenance's schedule.

	Before use	Once a week	Once a month	Once a year
Check that the shower chair is seated correctly and is stable	X			
Visual check		X		
Check rubber tips		X		
Check if spring-loaded buttons are fixated	X			
Have shower chairs checked by an authorized dealer				X

#### WARNING

Repair, cleaning, or adjustment of the product must be performed by an authorized service person while the product is not in use.

#### CAUTION

Only original spare parts can be used for replacement. Please provide model and serial number to dealer or contact manufacturer directly. Manufacturer and dealer are not responsible for any damage occurred to product or user if non original spare parts are used an (or) repair was done in unauthorized facility.

### STORAGE

The product should be stored in a dry environment, away from direct sunlight.

### DISPOSAL

Be environmentally aware and dispose of the product properly. Disposal is subject to national and local regulations. To have the product recycled, please contact your local community, waste disposal facility, or our service department.

## CLEANING AND DISINFECTION

### CLEANING

- It is recommended to use disposable gloves.
- It is recommended to use a pH neutral detergent and warm water for cleaning before disinfecting.
- Use pH neutral detergent and warm water for cleaning .
- Should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent.

### DISINFECTION

- It is recommended to use disposable gloves.
- It is recommended to use chlorine-based disinfectant solution for disinfection.

## ENVIRONMENTAL CONDITIONS

Keep the product under these conditions:  
Temperature limitation -20° C to +40° C;  
Relative humidity 20% to 75%;  
Atmospheric pressure 800 hPa to 1060 hPa.

## RE-USE

The shower chair is suitable for re-use. The following actions should be taken:  
Cleaning and disinfection (see chapter “CLEANING AND DISINFECTION”);  
Inspection (see chapter “MAINTENANCE”).

Before handing the product over to a new user, it should be adjusted and verified according to the service documentation, ensuring it is suitable and safe for the new user’s needs.

**SPECIFICATION**

Model	KVADI
REF	01-5005
Maximum safe load (kg)	135
Weight (kg)	2
Overall length (cm)	43.5
Overall width (cm)	49.5
Overall height (cm)	37.5 - 55
Seat width (cm)	43.5
Seat depth (cm)	30.5
Height positions	8

**INCIDENTS**

**Any serious incident that has occurred in relation to the device should be reported to UAB Kasko Group | Kid-Man® and the competent authority of the Member State in which the user and/or patient is established.**

**CONTACTS:**

**Manufacturer:** UAB “Kasko Group” (registered trade mark Kid-Man®)

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