LIGHTMAN START LIGHTMAN START PLUS

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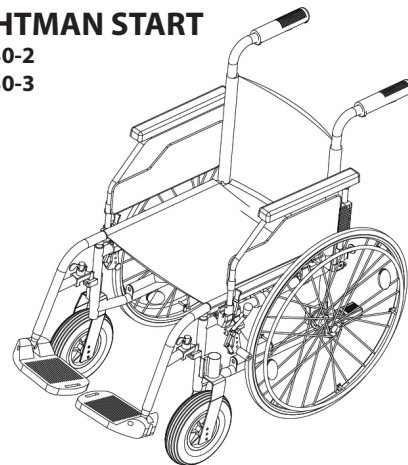
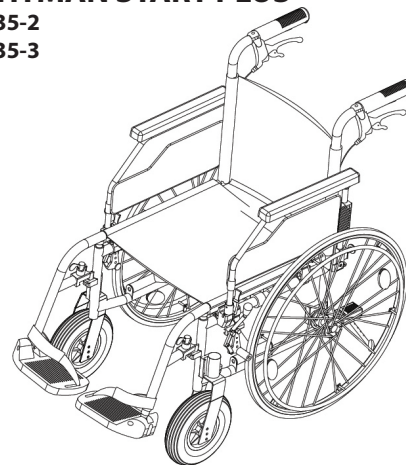
04-035-3

EN**SYMBOLS**

In this Instruction for Use warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger or basic information.

Gives basic information, recommendations and information for efficient, trouble-free use.

Warning / Caution - to indicate that caution is necessary when operating the device. To indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences

Attention! This sign is used for very important parts concerning your safety. Please look at the operating instructions.

SAFETY INFORMATION

WARNING
Any other or incorrect use could lead to hazardous situation

WARNING
Risk of accidents and serious injury. Accidents with resulting serious injury can occur if the wheelchair is wrongly adjusted. Adjustments at the wheelchair always must be carried out by an authorized dealer.

WARNING
Risk due to driving style being unsuitable for the conditions. There is a risk of skidding on wet ground, gravel or uneven terrain. Always adjust your speed and driving style to the conditions (weather, surface, individual ability, etc.).

WARNING

Risk of injury. In a collision you could sustain injury to parts of your body that extend beyond the wheelchair (e.g. feet or hands). Avoid an unbraked collision; Never drive into an object head-on; Drive carefully through narrow passages.

WARNING

Risk due to wheelchair being out of control. At high speed you could lose control of your wheelchair and overturn. Never exceed a speed of 7 km/h. Avoid collisions in general. Do not use escalators. Do not reverse down a gradient

CAUTION

Risk of burning. The wheelchair components can heat up when exposed to external sources of heat. Do not expose the wheelchair to strong sunlight before use. Before usage, check all components that come into contact with your skin for their temperature.

CAUTION

Risk of getting fingers caught. There is always a risk of getting, e.g. fingers or arms, caught in the moving parts of the wheelchair.

WARNING

Risk of accidents. Safety devices that are incorrectly set or no longer working (brakes, anti tipper) can cause accidents. Always check that the safety devices are working before you use the wheelchair and have them checked regularly by the authorized dealer.

WARNING

Risk of accidents. For proper and safe use, we recommend you to follow safety instructions while using your wheelchair. This wheelchair was designed to ensure stability while seated. Every movement on the wheelchair affects the point of gravity, which may cause the wheelchair to tip over.

WARNING

Risk of choke. Please protect children under the age of the five years from easy access to wheelchair. Due to small elements used in the construction, the children can swallow them or choke.

-  **CAUTION**
The wheelchair is only suitable for single occupancy.
-  **CAUTION**
Keep your feet on the footplates when moving. Do not stand on the footplates. Maintain proper balance at all times. Users should not move their centre of gravity out of the seating area.
-  **CAUTION**
We recommend you consult your healthcare professional for advice about transferring to and from the wheelchair. The parking brakes should always be applied when transferring. The wheelchair brakes and parking brakes are not suitable for slowing the wheelchair down.
-  **CAUTION**
Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's toys, etc. Do not reach for items further than your arm will extend.
-  **WARNING**
Risk of accidents. Do not attempt to move up or down an incline with a water, ice or oil film. Do not attempt to stop the wheelchair while on a sloped surface.
-  **WARNING**
Risk of accidents. Do not attempt to ride over curbs or obstacles. Doing so may cause your wheelchair to tip over and cause bodily harm to you or damage to the wheelchair.
-  **WARNING**
Risk of accidents. Never leave an unoccupied wheelchair on an incline or other place which may cause injury to surrounded people or things.
The expected service life is 5 (five) years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this Instruction for Use.
After then, wheelchair has to be recycled. Do not throw your wheelchair away with common waste (see chapter "DISPOSAL").

GENERAL

INTRODUCTION

This Instruction for use contains important information about the handling of the product. To ensure safety when using the product, read the Instruction for use carefully and follow the safety instructions.

LightMan wheelchair is available in two different modifications - LightMan Start and LightMan Start Plus. „Plus“ version is upgraded with drum brakes for assistant control. You may identify type of your wheelchair and it's unique identification number reading a label stucked on the armrest plate and metal cross brace underneath seat.

Eg. Round sticker with number (for example 39) on skirt guard indicates that size (seat width) of your chair is 39 cm.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models. If not otherwise stated, each section in this document refers to all models of the product.

PREFACE

Thank you for the confidence shown in Kid-Man. Every product is factory checked and left us in faultless condition. We wish you much success and we hope UAB Kasko Group | Kid-Man® wheelchair will make a positive impact to your daily living.

WARRANTY

We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB Kasko Group | Kid-Man® (hereinafter called Manufacturer) after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, Manufacturer, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty

LIMITATION OF LIABILITY

Manufacturer accepts no liability for damage arising from:

- non-compliance with the Instruction for use;
- incorrect use;
- natural wear and tear;
- incorrect assembly or set-up by the purchaser or a third-party technical modifications
- the usage of unapproved 3rd party accessories instead of approved Manufacturer's accessories
- unauthorized modifications and/or use of unsuitable spare part

STANDARDS AND REGULATIONS

The wheelchair complies the requirements of:

- a) standard EN 12183:2022 Manual wheelchairs - Requirements and test methods;
- b) REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009 and repealing Council Directives 90/385/EEC and 93/42/EEC.

SERVICE LIFE

The expected service life is 5 (five) years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this Instruction for Use.

After then, wheelchair has to be recycled. Do not throw your wheelchair away with common waste (see chapter "DISPOSAL").

INFORMATION ABOUT MANUFACTURER

Manufacturer: UAB "Kasko Group" (registered trade mark Kid-Man)
Address A.Vivulskio 12D, Vilnius, Lithuania,
Phone +370 5 265 0000
E-mail info@kid-man.com
Webside www.kid-man.com

INFORMATION ABOUT PRODUCT

Product (s) name: LIGHTMAN START & LIGHTMAN START PLUS
Product: LIGHTWEIGHT MANUAL WHEELCHAIR
Model: 04-030-2; 04-030-3; 04-035-2; 04-035-3.

Warning
Risk of accidents and serious injury. The wheelchair is not intended for use in the motor vehicle.

INTENDED PURPOSE AND INDICATIONS

LIGHTMAN START & LIGHTMAN START PLUS wheelchair is propelled manually and should only be used for independent or assisted transport of a disabled person with temporary (eg. broken bones or injury to the legs or feet) or long-term (eg. paralysis, musculoskeletal issues, neurological issues, balance, or gait problems) inability to walk long distances mobility difficulties. It can be used both indoors and outdoors. Wheelchair is propelled by moving push rims, mounted on the rear wheels. Ether by attendant.

i This wheelchair needs to be subscribed and fit to your specific health condition.

CONTRA-INDICATIONS

There are no contraindications known, when using the wheelchair as intended.

INTENDED USERS

LIGHTMAN START & LIGHTMAN START PLUS wheelchair is intended to provide mobility to persons ages 15 and up (adolescents and adults) with a maximum weight capacity of 125 kg or 150 kg (only for seat width 51 cm and 54 cm). The maximum age to use a wheelchair is not defined and depends on individual.

WARNING

Risk of accidents. It is forbidden to exceed the maximum safe load of wheelchair.

The device is indicated to provide mobility to persons limited to sitting position. When used without an assistant, it should only be used by persons who are physically and mentally able to use the wheelchair safely (e.g. to propel themselves, steer, brake).

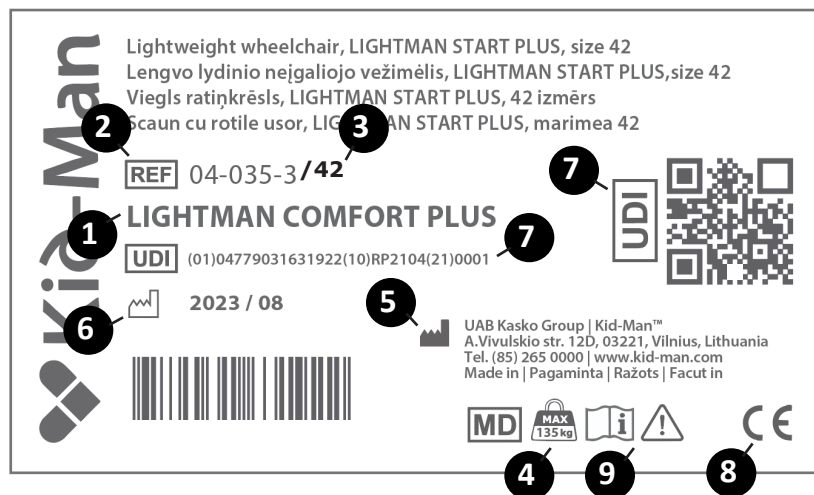
CAUTION

The wheelchair is appropriate for use by others. In order to do this, the wheelchair must first be thoroughly cleaned and disinfected. Then, the all components of the wheelchair must be get checked by an authorised service dealer for general condition, wear and shall get repaired if necessary.

PRODUCT LABEL

**The identification label is attached to the frame of your wheelchair (see picture).
Label is attached to the product.**

1. Name of product
2. Model
3. Size (seat width)
4. Maximum load
5. Manufacturer / contact details
6. Manufacturer date
7. UDI code
8. CE mark
9. Consult instructions for use

CONTENT OF PACKAGE

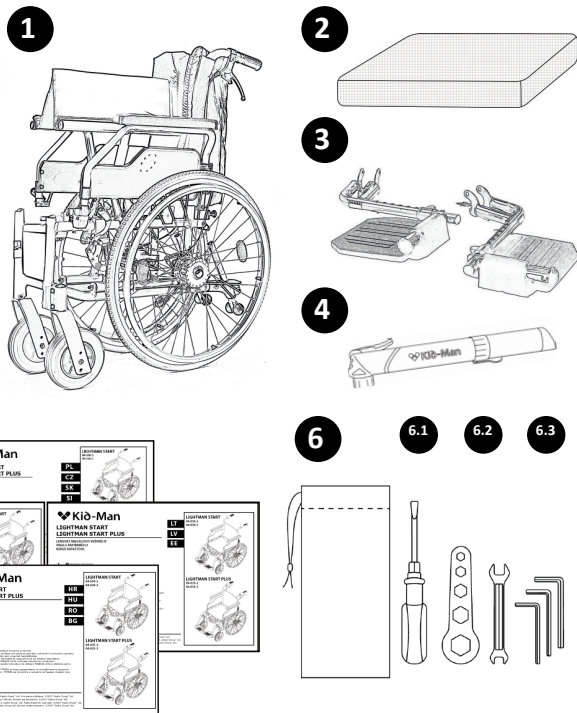
The wheelchairs LightMan Start and LightMan Start Plus are delivered in a cardboard packaging and are almost ready for use. It is only needed to attached the pair of legrest onto the frame. After unpacking, please keep the cardboard packaging if possible. You may need it later for storage or for the return of the wheelchair. Once parcel is received, please check whether it is not damaged during transportation.

If you see signs of damage, please make note to transport company on the same time.

Please check content of package and if any of below parts are missing, get in touch with your local dealer.

Content of package:

1. 1 x folded wheelchair
2. 1 x cushion
3. 1 x pair of legrest
4. 1 x pump (if rear wheels are inflatable)
5. 1 x Instructions for use
6. 1 x set of adjustment tools:
 - 6.1 1 x screwdriver
 - 6.2 1 x spanner (8 and 10 mm)
 - 6.3 1 x adjustment key
 - 6.4 3 x allen key


COMPONENTS AND FUNCTIONS

Overview

This is a wheelchair with crossbar folding mechanism and swing away & removable leg rests.

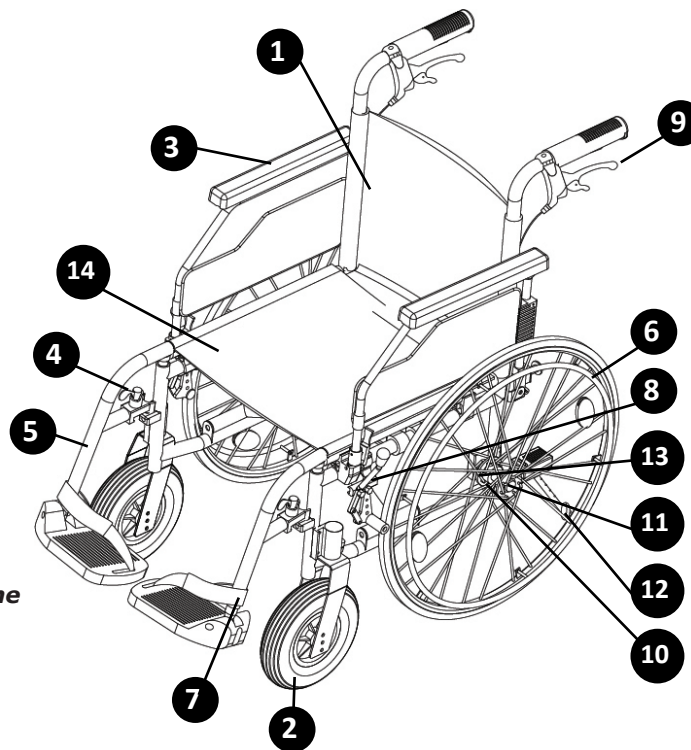- 1 Backrest
 - 2 Front Castor
 - 3 Armrest
 - 4 Legrest swing lever
 - 5 Legrest
 - 6 Rear self-propel wheel
 - 7 Heel strap
 - 8 Brake lever
 - 9 Attendant push handles
 - 10 Adapter block
 - 11 Quick release axle
 - 12 Anti-tipper*
 - 13 Drum brake*
 - 14 Cushion
 - 15 Velcro straps*
- *LightMan Start Plus

i *Complexations of your wheelchair can differ a little from a wheelchair described in this manual due to continuous development of the product.*

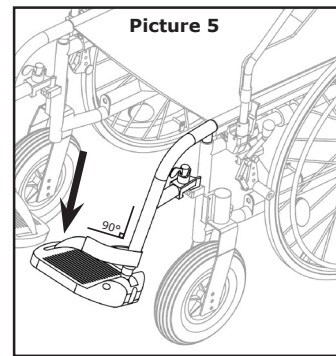
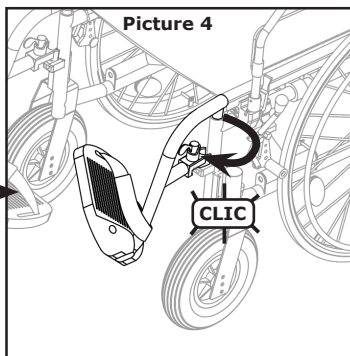
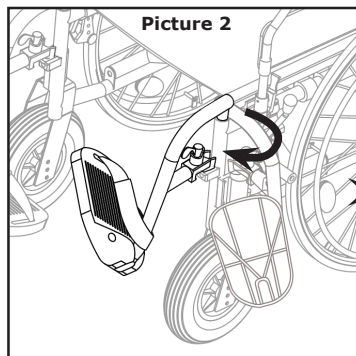
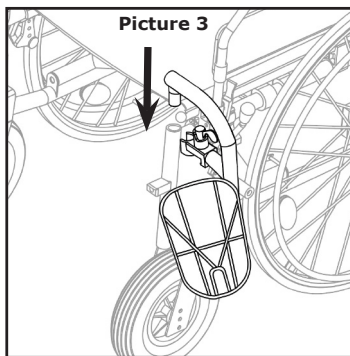
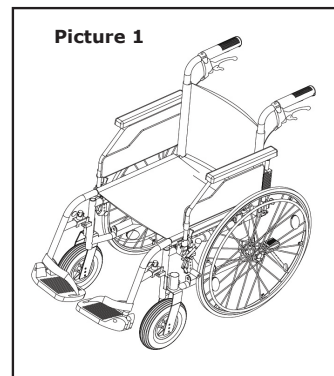
UNFOLDING AND FOLDING THE WHEELCHAIR

To unfold the wheelchair:

- Position the wheelchair next to you;
- With your hand flat, press the seat edge closest to you, so that the seat surface flattens out and the seat pipes are completely set in the plastic holders (Picture 1). The seat cushion must be completely spread out;
- Put Legrest in their places (Picture 2);
 - Turn Legrest to side (Picture 3) to fix right position (Legrest must stand tightly).
 - Set Legrest to appropriate / comfortable position: push Legrest down as shown in Picture 4. Apocopate position of Legrest is shown in Picture 5.


i

You don't need to use energy to much. If You push too much You can damage the Legrest (eg. break it).


To fold wheelchair, first remove Legrest or flip up footplates (Picture 6)

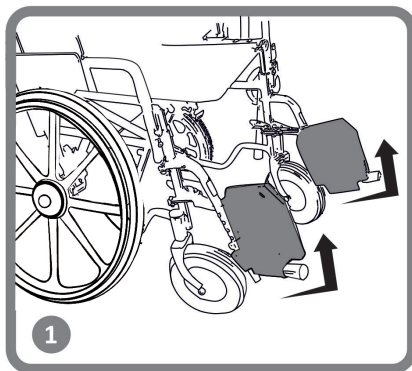
After, place your hands on the seat cushion at the front and back and pull it up (Picture 7)

Picture 6

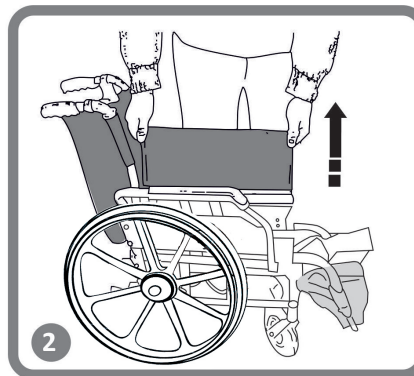Picture 7

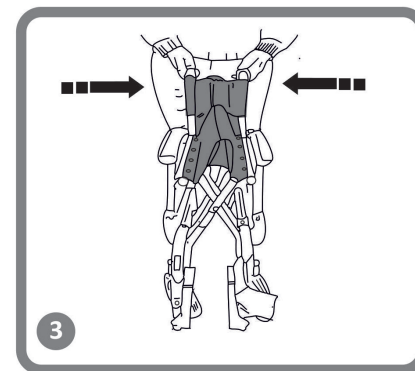i In order to prepare wheelchair for transportation into car, we suggest to take off the rear wheels gently over the full floating axle.

WARNING

Danger of trapping your finger! Be careful not to grasp the seat pipe.

WARNING

Never lift the wheelchair by removable parts (armrests, footrests). Please make sure all removable parts are taken off. The removable parts in the wheelchair are armrest, legrest, rear wheels and cushions. How to remove them, please refer to this Instruction for use. After parts are removed, please to fold the wheelchair. After it is folded, you can easily lift it holding with both hands on the frame. The lifting weight after all parts are removed is around 8-10 kg depending on the size of the chair and model.

OPERATION

i This wheelchair was designed to ensure stability while seated. Every movement on the wheelchair affects the point of gravity, which may cause the wheelchair to tip over. To increase your safety, you may use seatbelt.

WARNING

There are no positions of points intended to carry additional loads.

LEANING FORWARDS

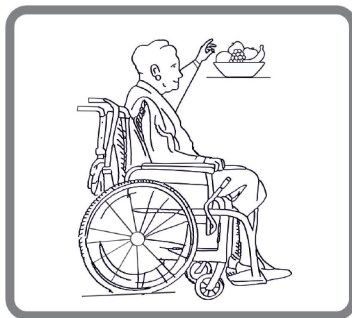
WARNING


Risk of falling out. If you lean forwards out of the wheelchair you could fall out of it. Never bend too far forwards and do not shift forwards in your seat to reach an object. Do not bend forwards between your knees to pick up something off the floor.

REACHING AN OBJECT FROM THE WHEELCHAIR

Only the arms may be extended over the wheelchairs seat. The body and head should stay inside the limits of the seat (see Picture 8).


Picture 8

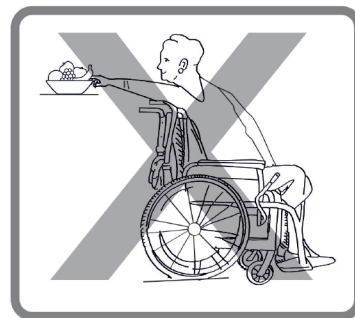MOVING ONTO THE SIDEWALK

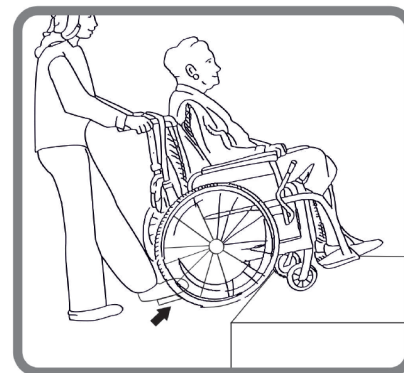We recommend to have assistant to move onto sidewalk. Firstly, accompanying person positions the wheelchair straight in front of the curb side. After, tips the wheelchair backwards, until the steering wheels reach the curb. Lastly, pushes the wheelchair straight forward, until the back wheels hit the curb and when pushes the wheelchair again, until the back wheels go over the curb side. (Picture 9)

Picture 9

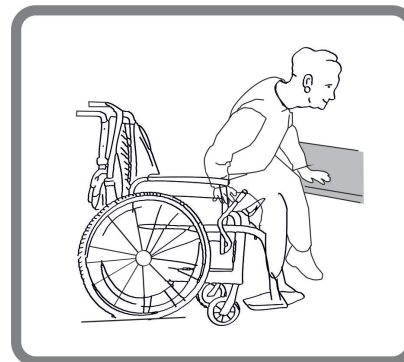
TRANSFER

Transfer can be performed alone, assuming you are strong enough.

Firstly, set the wheelchair as close to the place, which you want to sit on, as possible, whereby the steering wheels point forward.

Activate the fixing brake (Picture 10).

Move your body towards the desired place (Picture 11).


Picture 11

If there is no support beneath you while moving, please use a board if possible. Fold the legrest up and put them aside to make room. Bend your body forward, while leaning on the two armrests, and stand up. Then move your body toward the place, where you want to sit, while distributing your weight to your arms and hands.

If you are capable of getting up and standing on your own, you may change seats by leaning forward. If you have trouble with this movement, we advise you to seek assistance from an accompanying person to ensure an easy and safe transfer. Firstly, engage both brakes and make sure both steering wheels point towards the front. Please bear in mind that you must not support yourself on the brakes. Grab the handle of the armrest firmly and support yourself on your arms, to stand up easily (before, make sure armrest are securely locked in place). The assistant may help you complete the transfer. Position yourself as close to the desired place as possible. When changing your position, please sit as far back as possible, to prevent screws from breaking, seat cushions from getting damaged, or the wheelchair from tipping over. Activate the two brakes; these brakes should in no way be used as support for the transfer. You must never support yourself on the footrests, while sitting or getting up.




STAIRS

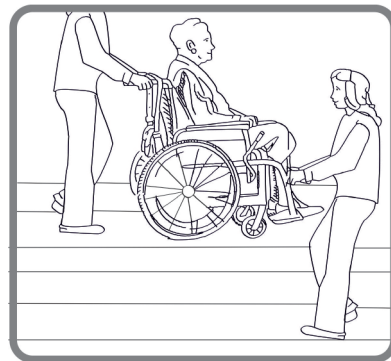We suggest seeking help from two people, one in front of the wheelchair and one behind it, due to the fact that hurdling stairs is more difficult and dangerous. The person behind the wheelchair, tips the wheelchair backwards until it is balanced. Then, pushes the wheelchair against the first step, grabbing the handle firmly, to prepare to lift the wheelchair. The person in front of the wheelchair grabs the sides of the frame in the front of the wheelchair and lifts the chair over the steps, while the other accompanying person steps on the next step, repeating the first process. Do not lower the wheelchair until you hurdled the last step of the person in front of the wheelchair. Carrying a wheelchair down the stairs is the same process as explained previously, in reversed order. (Picture 12)

Picture 12

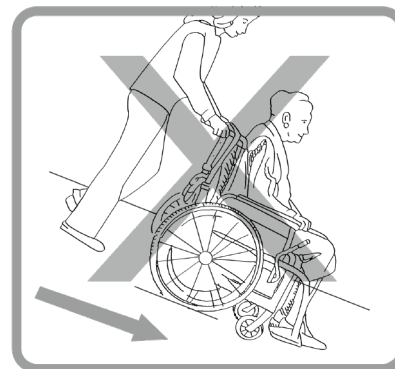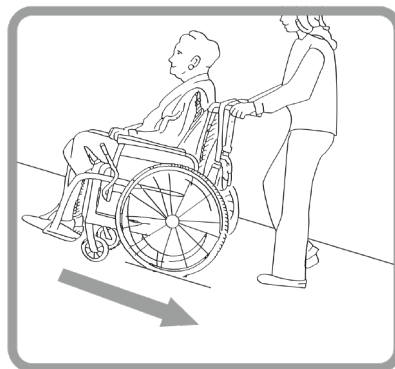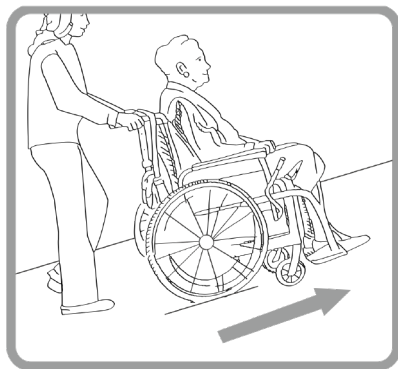ASCENDING/DESCENDING A RAMP


Picture 13

ARMREST OPERATION

Wheelchair is equipped with flip up and detachable armrest (Picture 14). To flip it up, please push plastic lever located in the front of armrest. Keep it pushed and lift armrest gently. We recommend to flip armrest up getting in or out of wheelchair from side.

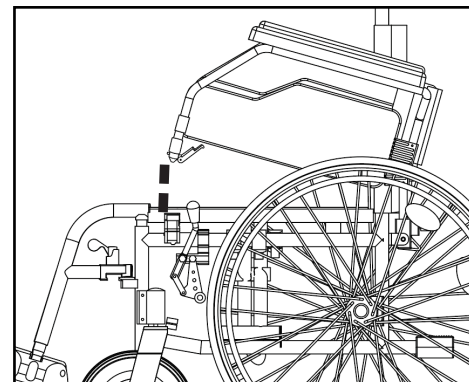While operating wheelchair, armrests should be locked. In case the need to completely remove armrest occurs, firstly flip it up as explained before. Once armrest is flipped up, and plastic handle in the rear of armrest and pull it out. Once pulled out, armrest is unlocked and you may take it out. To put it back, please repeat actions in opposite direction.


Picture 14

SEAT HEIGHT ADJUSTMENT

Thanks to front and rear wheels, you are able to adjust seat height. In order to adjust height of steering caster, untighten screw with allen key, pull it out, set caster on desired height and tighten again. (Picture 15) In order to adjust height of rear wheel, untighten 2 screws using spanner and set rear wheel adapter in needed height. After set in place, tighten it again. For safety reasons, we recommend to keep front and rear wheels in the same height. The wheelbase can be extended by turning the adapter by 180°. This increases the stability against tipping backwards

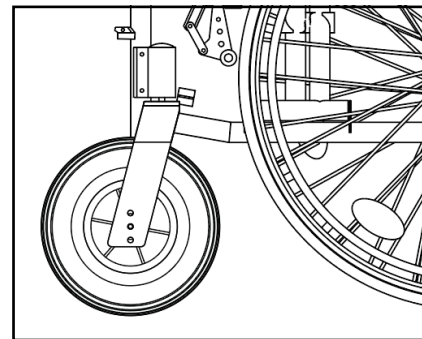i Please note, that rear wheel must be removed while making adjustment of adapter.


Picture 15

REAR WHEEL OPERATION

Your wheelchair has quick release axle, you need no tool to take off the rear wheel.

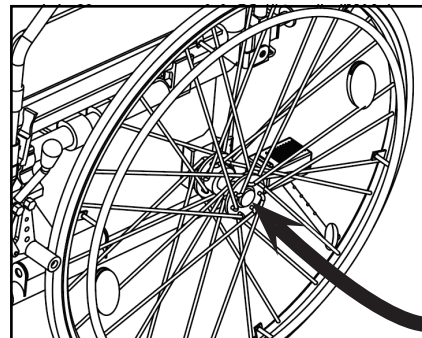Simply hold frame with one hand and push quick release axle located in the centre of rear wheel (Picture 16)
Keep it pushed and pull rear wheel out.
To put it back, repeat action in opposite way.

Picture 16

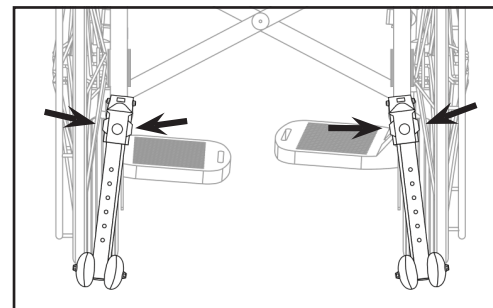
ANTI-TIPPER

LightMan Start Plus wheelchairs are equipped with anti-tip wheels. They are factory in built, thus requires small adjustments only before use. The aim of anti tippers is to prevent the wheelchair from tipping backwards. They can also be used to help tilt the wheelchair by an accompanying person.

The height of the wheels can be adjusted.

To do this, pull the silver-coloured disks towards you and slide the tube upwards or downwards along the bracket (Picture 17).

If you want to take wheels off, unscrew bolt, take it out and pull device towards.

Picture 17

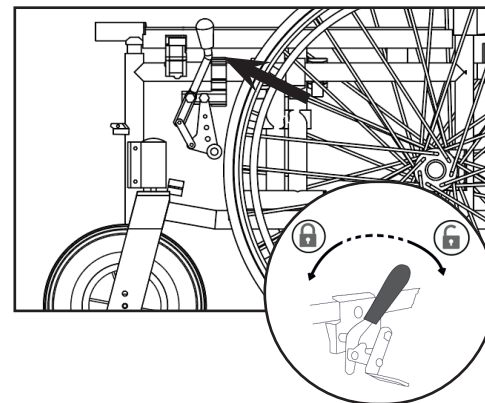
BRAKING

Your wheelchair has brakes mounted onto frame from both sides. (Picture 18)

Once delivered, brakes are adjusted properly to ensure sufficient braking.

In order to apply brakes, please hold braking lever and push it forward. We recommend always use both brakes at the same time to ensure maximum safety. Due to normal wear and tear of rear wheel, braking efficiency may be reduced. In order to have sufficient braking, you must adjust position of brakes. To do so, please use allen key and untighten two screws located in bottom part of brakes. Then, push it forwards or backwards and tighten screws.

In case you move rear wheel adapter by 180° to extend rear wheel base, it is necessary to re-locate brakes. Please use allen key to untighten brake device, push it backwards and tighten again.


Picture 18

LEGREST OPERATION

The footrests must be installed in the front of wheelchair. Firstly, please insert plastic part into metal tube and push it down. After, move legrest inside until you hear snap. (Picture 19) Once locked in position, legrest stand firmly.

i Do not mix left and right legrest!

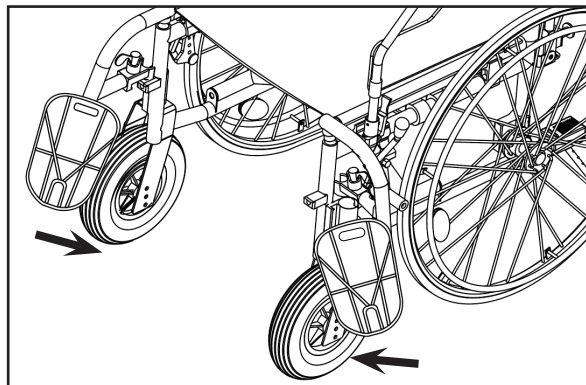
After installation, plastic footplate must be faced inwards. Once leaving wheelchair, legrest must be flipped away or completely removed. Pull plastic lever upwards and move legrest outside. Length of legrest can be adjusted to meet user's needs. Firstly, unscrew bolt using allen key, pull out bolt and tighten it again in needed position.

Picture 19

DRUM BRAKE

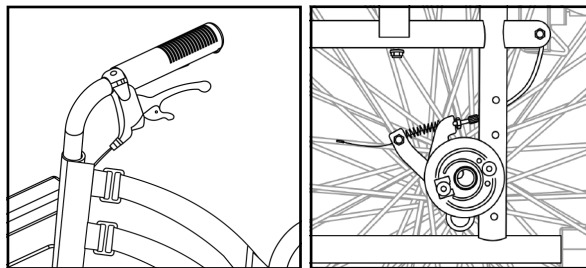
LightMan Start Plus version is equipped with a drum brake, which should only be operated by an accompanying person via the brake lever located on the push handles (Picture 20). The drum brake should be used as both a locking brake and also as a service brake (Picture 20) When used as a service brake it provides well-regulated deceleration. The locking brake function is used for parking. To lock the brake, pull the brake lever and push the locking lever upwards. To release the locking brake, pull the brake lever upwards softly and the locking mechanism releases automatically. To achieve optimum braking efficiency, the braking power is set using the adjusting screw. The braking power is increased by unscrewing the adjusting screw and reduced by tightening it.

Picture 20

VELCRO STRAPS

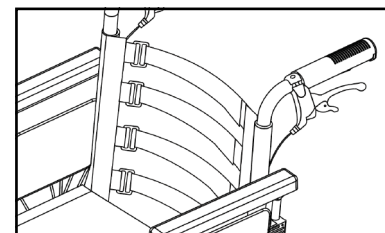LightMan Start Plus model has Velcro straps on backrest, which help to regulate the level of comfort (Picture 21). In order to regulate the level of comfort, take off the cover of the backrest, unstick Velcro straps, set your level of comfort and stick the straps again

Picture 21

TYRE

The front casters of the wheelchair are solid (PU). The rear wheels are pneumatic (04-030-2 series) or solid PU (04-030-3 series).

Recommended air pressure is indicated on the tyre (make sure it is always between min and max values given below):

	Min - Max pressure		
Tire size 24" (37-540mm)	3 – 4,6 bar	300-460 kPa	44-65 psi

For your safety, please check air pressure in tyres periodically (see chapter "MAINTENANCE").

If you notice air pressure is too low, please inflate them using the pump which is included into delivery. In case tyres do not keep the air or there is obvious wear of outer tyre, please inform service company immediately.

WARNING

Use of wheelchair with empty air tyres is forbidden!

MAINTENANCE

To ensure safe and reliable operation, carry out the following checks and maintenance regularly or have it carried out by another person.

Below You will find recommended maintenance's schedule.

	Before use	Once a week	Once a month	Once a year
Check the tire pressure		X		
Check that rear wheels are seated correctly		X		
Visual check		X		
Check the folding mechanism			X	
Check the brakes	X		X	
Clean the front wheels			X	
Have wheelchair checked by an authorized dealer				X

WARNING

Repair, cleaning, or adjustment of wheelchair should be performed by authorized service person while person is not sitting in a wheelchair.

CLEANING AND DISINFECTION

Use the moist cloth to clean frame as well as plastic parts. The use of a mild detergent for more stubborn dirt is possible as well. The wheels can be cleaned with a wet synthetic fiber brush (the use of wire wire brush is not allowed). Seat and back pads can be washed off. Use mild detergent. Use a common non aggressive disinfectant. If you are unsure if the one used by you is suitable for disinfection, please get in touch with local dealer

Sand and seawater can damage the bearings and steel parts can rust if the surface is damaged. Only expose the wheelchair to sand and seawater for short periods and clean it after every trip to the beach.

The wheelchair may be disinfected by spraying or wiping with tested, approved disinfectants.

Spray a soft cleaning and disinfecting product and follow the instructions given by the manufacturer.

STORAGE

The wheelchair should be stored in a dry environment, away from direct sunlight. When in storage the wheelchair can be folded up. The wheelchair should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent. Clean regularly, use a general purpose, neutral detergent and warm water. Please check the suitability of the cleaners or detergent before use. Do not use abrasive cleaners as this will permanently damage the surface and invalidate any warranty.

WARNING

The wheelchair must not be permanently exposed to temperatures greater than +40°C. Otherwise the frame as well as seat, backrest and armrests can heat up, which might cause burns.

WARNING

Never store other items on top of the wheelchair.

After long-term storage (more than 3 months) the wheelchair must be inspected (see chapter „MAINTENANCE“).

Environmental conditions

Keep wheelchair under these conditions:

- Temperature limitation – 20° C to + 40° C;
- Relative humidity 20% to 75%;
- Atmospheric pressure 800 hPa to 1060 hPa.

RE-USE

The wheelchair is suitable for re-use. The following actions should be taken:

Cleaning and disinfection (see chapter „CLEANING AND DISINFECTION“);
Inspection (see chapter „MAINTENANCE“).

The wheelchair should be adapted for the new user in accordance with the service documentation.

DISPOSAL

Be environmentally aware and dispose of your wheelchair properly. Disposal is subject to national and local regulations. To have wheelchair properly recycled, please contact your local community, disposal facility or our service department.

SPECIFICATION

Seat width	390 mm	420 mm	450 mm	480 mm	510 mm	540 mm
LightMan Start	04-030-2/39 04-030-3/39	04-030-2/42 04-030-3/42	04-030-2/45 04-030-3/45	04-030-2/48 04-030-3/48	04-030-2/51 04-030-3/51	04-030-2/54 04-030-3/54
LightMan Start Plus (with drum brakes)	04-035-2/39 04-035-3/39	04-035-2/42 04-035-3/42	04-035-2/45 04-035-3/45	04-035-2/48 04-035-3/48	04-035-2/51 04-035-3/51	04-035-2/54 04-035-3/54
Overall length	1150 mm					
Overall width	Seat width + 200 mm					
Overall height	900 - 1000 mm					
Seat depth	410 mm					
Seat height	470 - 520 mm					
Backrest height	440 mm					
Length of footrest	340 - 450 mm					

Armrest height	230 mm
Size of front casters	200 x 45 mm (3 positions)
Size of rear wheel	610 x 35 mm (4 positions)
Weight	From 15,2 kg
Maximum safe load	125 kg
Frame	Aluminium
Upholstery	Nylon

i All materials used are protected against corrosion. We use only REACH compliant materials and component.

INCIDENTS

Any serious incident that has occurred in relation to the device should be reported to UAB Kasko Group | Kid-Man® and the competent authority of the Member State in which the user and/or patient is established.

Contacts

UAB Kasko Group | Kid-Man® (registered trade mark Kid-Man)

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"Kid-Man" is legitimate trademark of "Kasko Group" Ltd. All rights reserved. ©2026 "Kasko Group" Ltd.