LIGHTMAN TRAVEL

04-018-2

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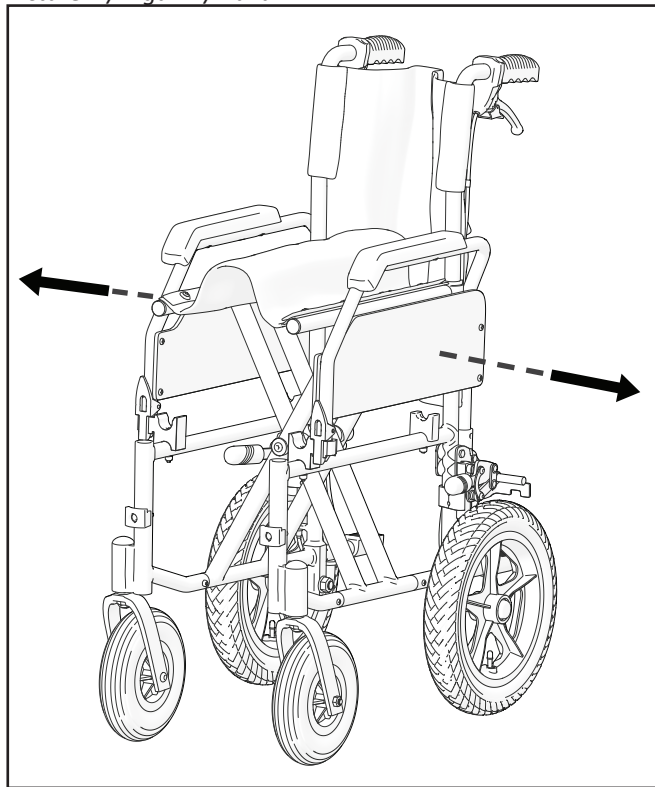
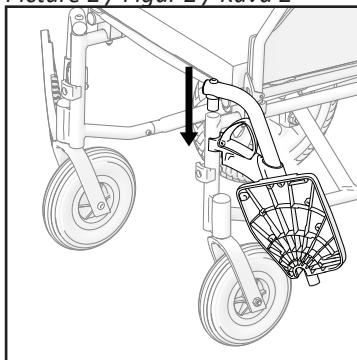
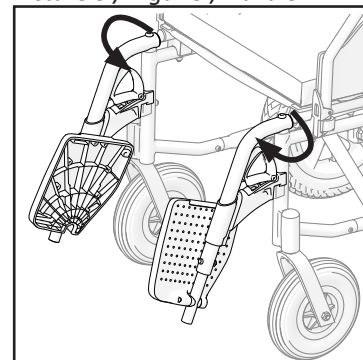
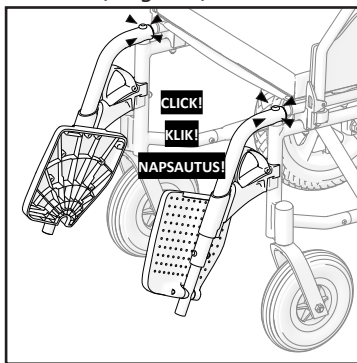
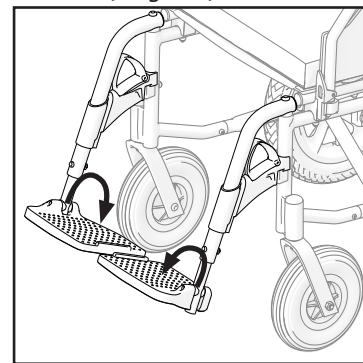
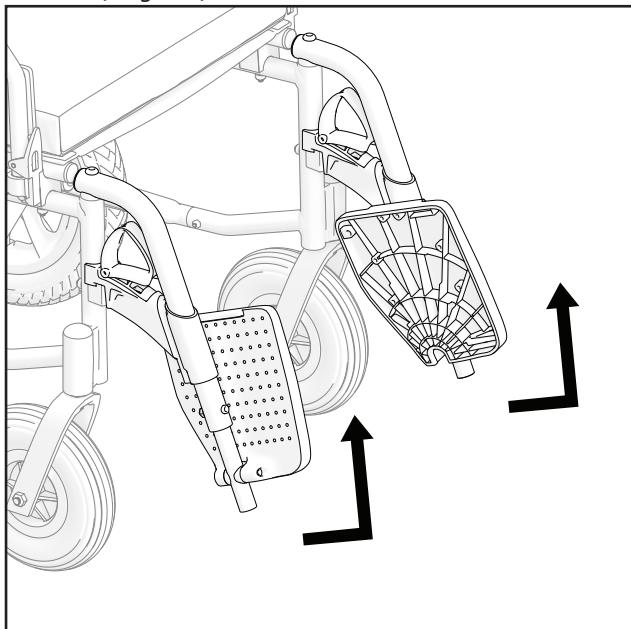
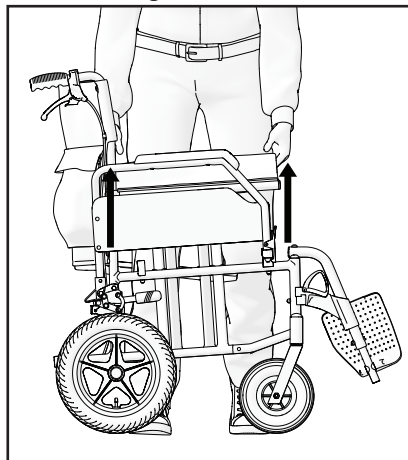
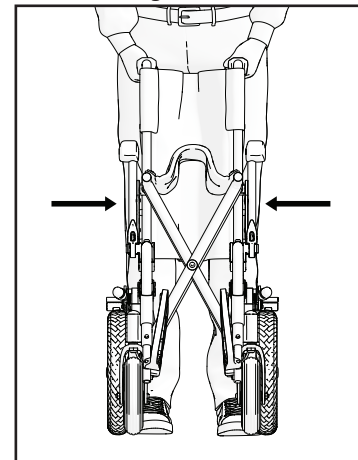
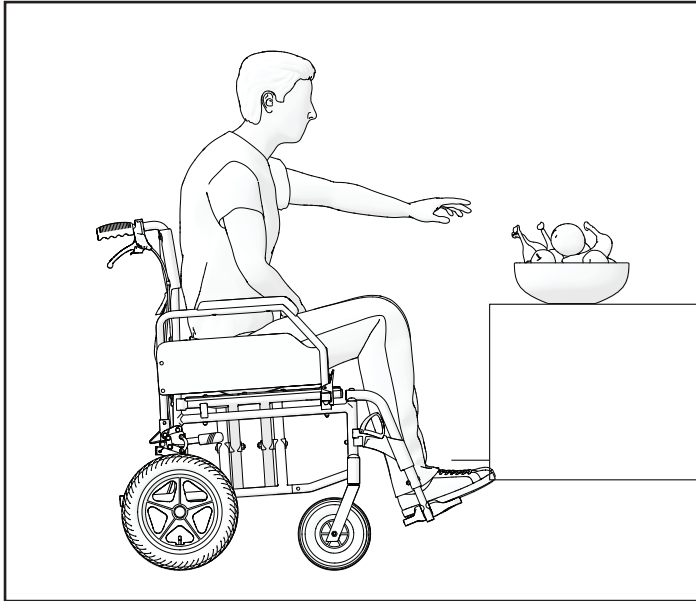
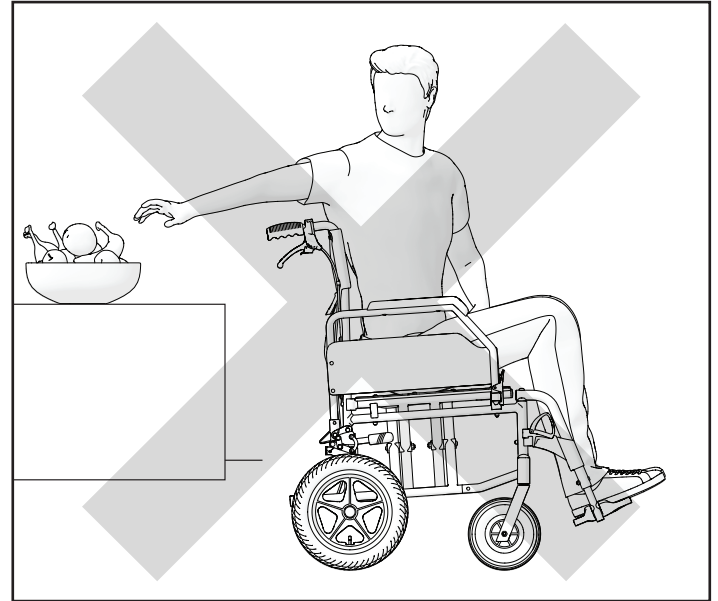
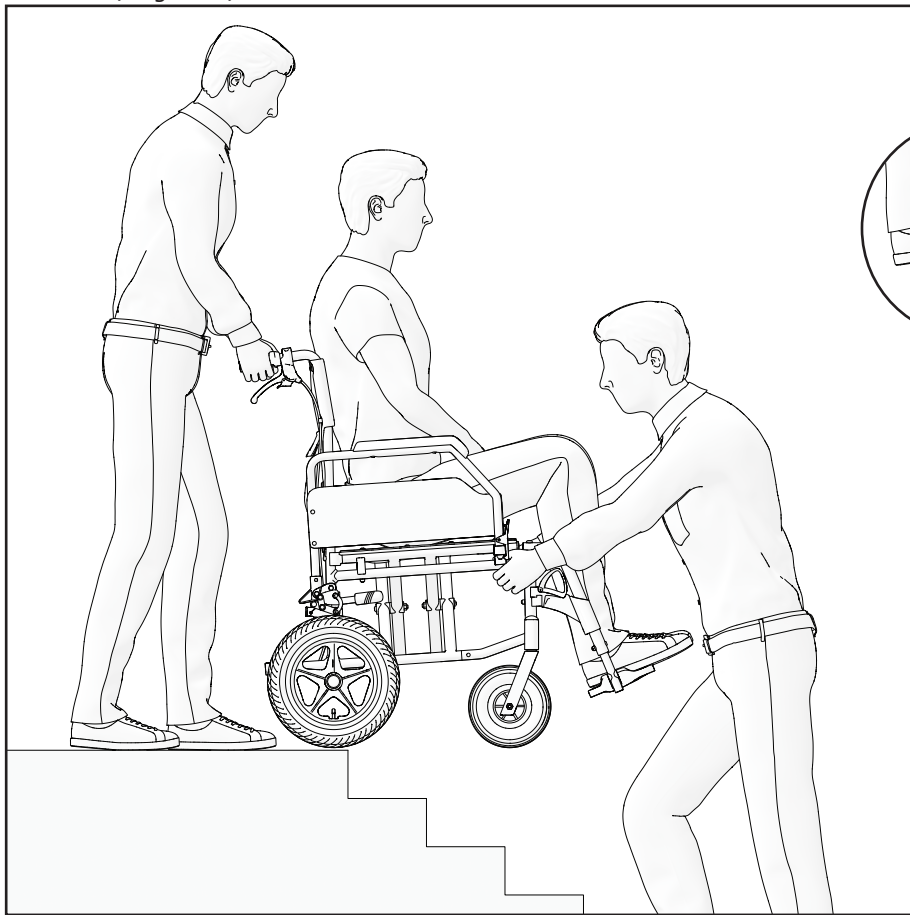
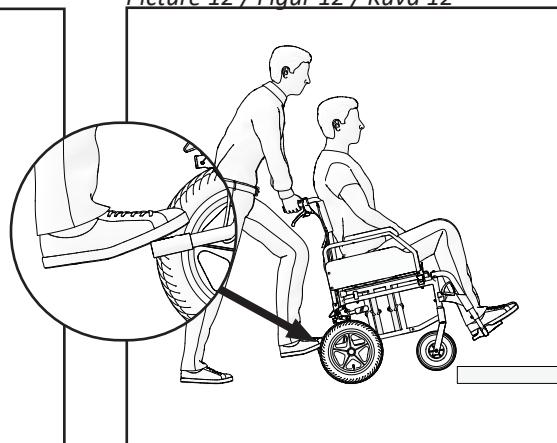
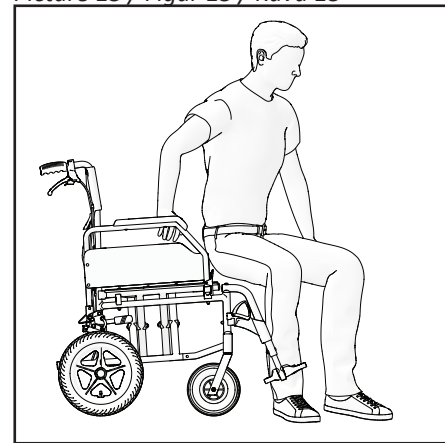
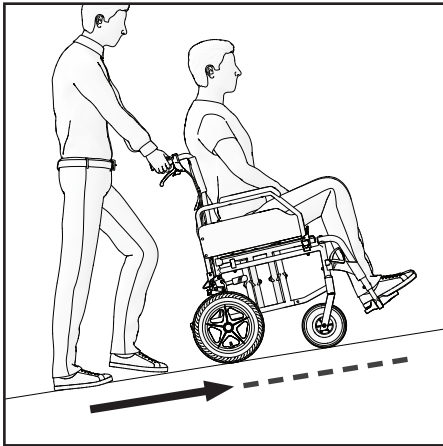
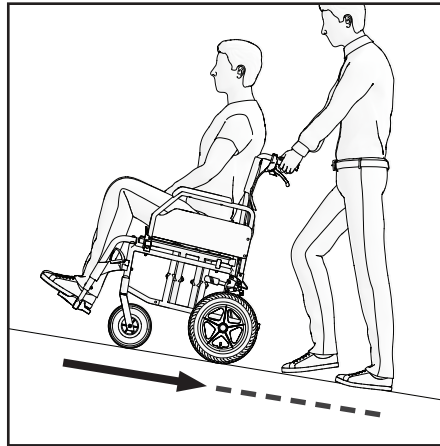
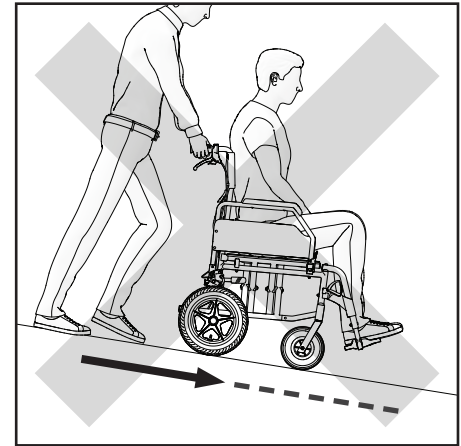
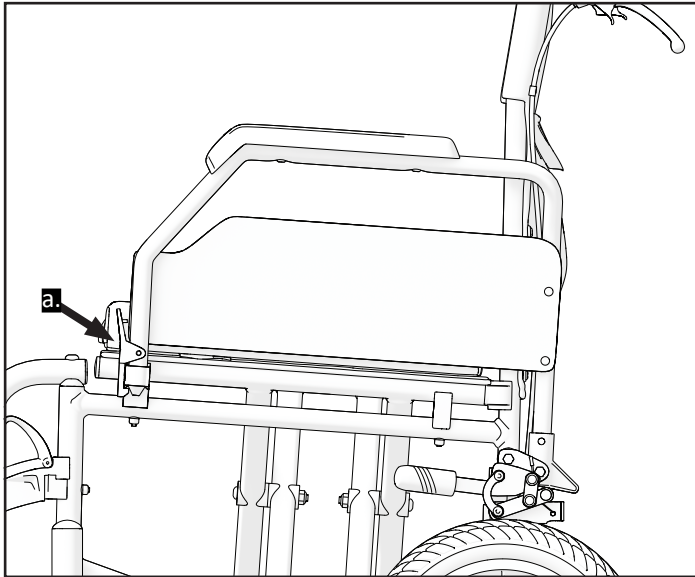
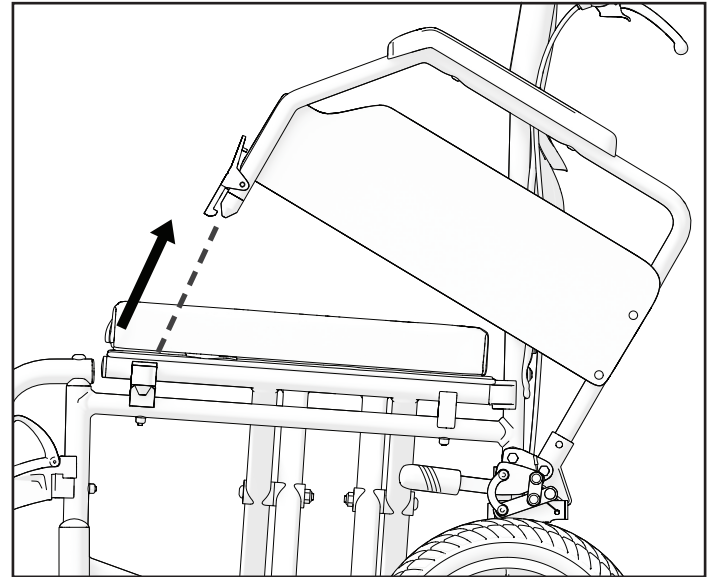
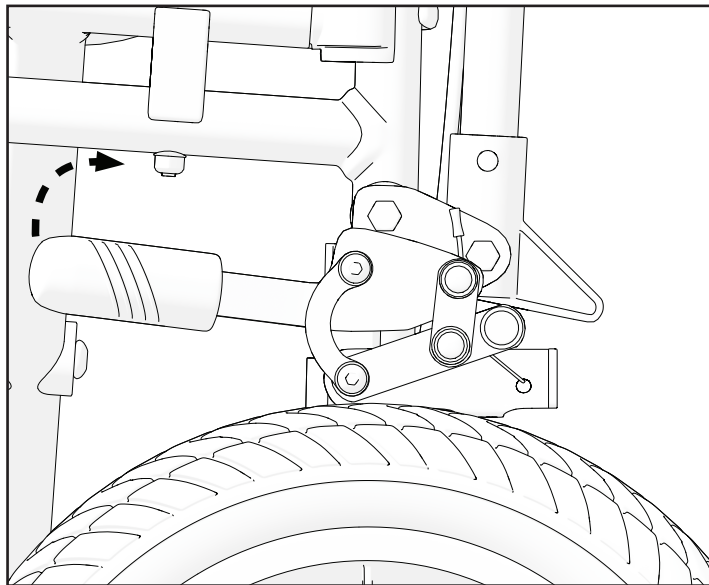
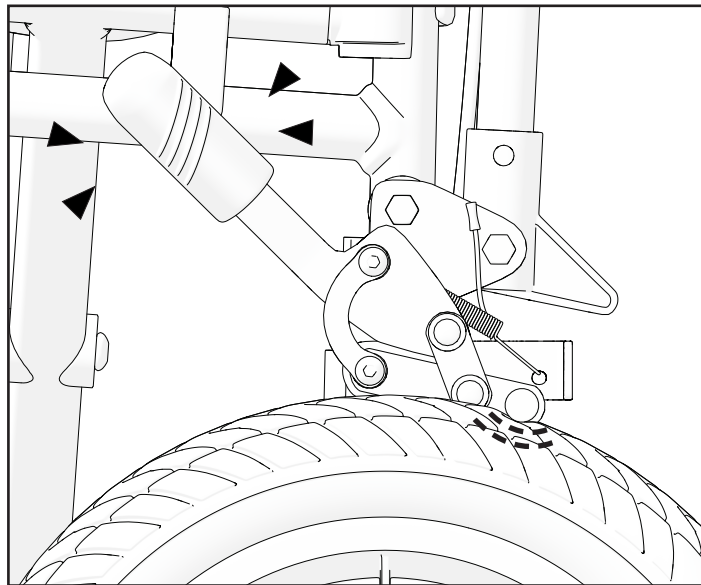
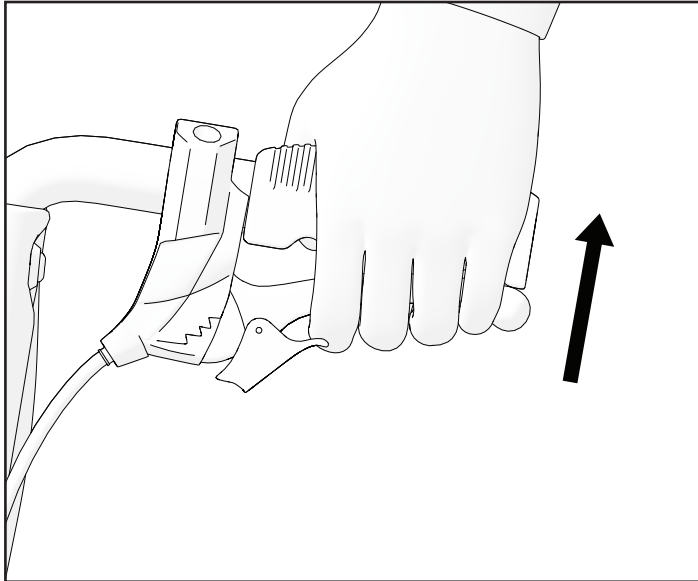
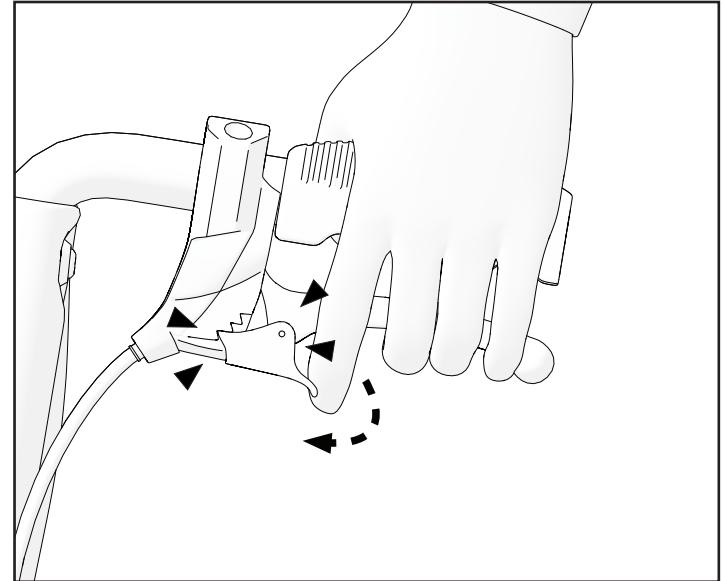
SYMBOLS

**In this Instruction for Use warnings are indicated by symbols.
The warning symbols are accompanied by a heading that indicates the severity of the danger or basic information.**

i Gives basic information, recommendations and information for efficient, trouble-free use.

 **WARNING / CAUTION** - to indicate that caution is necessary when operating the device. To indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

 **ATTENTION!** This sign is used for very important parts concerning your safety. Please look at the operating instructions.

SAFETY INFORMATION

 **WARNING**
Any other or incorrect use could lead to hazardous situation.

 **WARNING**
Risk of accidents and serious injury. Accidents with resulting serious injury can occur if the wheelchair is wrongly adjusted. Adjustments at the wheelchair always must be carried out by an authorized dealer.

 **WARNING**
Risk due to driving style being unsuitable for the conditions. There is a risk of skidding on wet ground, gravel or uneven terrain. Always adjust your speed and driving style to the conditions (weather, surface, individual ability, etc.).

 **WARNING**
Risk due to wheelchair being out of control. At high speed you could lose control of your wheelchair and overturn. Never exceed a speed of 7 km/h. Avoid collisions in general. Do not use escalators. Do not reverse down a gradient.

SAFETY INFORMATION

-  **WARNING**
Risk of burning. The wheelchair components can heat up when exposed to external sources of heat. Do not expose the wheelchair to strong sunlight before use. Before usage, check all components that come into contact with your skin for their temperature.
-  **WARNING**
Risk of getting fingers caught. There is always a risk of getting, e.g. fingers or arms, caught in the moving parts of the wheelchair.
-  **WARNING**
Risk of accidents. Safety devices that are incorrectly set or no longer working (brakes, anti tipper) can cause accidents. Always check that the safety devices are working before you use the wheelchair and have them checked regularly by the authorized dealer.
-  **WARNING**
Risk of accidents. For proper and safe use, we recommend you to follow safety instructions while using your wheelchair. This wheelchair was designed to ensure stability while seated. Every movement on the wheelchair affects the point of gravity, which may cause the wheelchair to tip over. To increase your safety, you may use seatbelt.
-  **WARNING**
Risk of choke. Please protect children under the age of the five years from easy access to wheelchair. Due to small elements used in the construction, the children can swallow them or choke.
-  **CAUTION**
The wheelchair is only suitable for single occupancy.

SAFETY INFORMATION

-  **CAUTION**
Keep your feet on the footplates when moving. Do not stand on the footplates. Maintain proper balance at all times. Users should not move their center of gravity out of the seating area.
-  **CAUTION**
We recommend you consult your healthcare professional for advice about transferring to and from the wheelchair. The parking brakes should always be applied when transferring. The wheelchair brakes are not suitable for slowing the wheelchair down.
-  **CAUTION**
Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's toys, etc. Do not reach for items further than your arm will extend.
-  **WARNING**
Risk of accidents. Do not attempt to move up or down an incline with a water, ice or oil film. Do not attempt to stop the wheelchair while on a sloped surface.
-  **WARNING**
Risk of accidents. Do not attempt to ride over curbs or obstacles. Doing so may cause your wheelchair to tip over and cause bodily harm to you or damage to the wheelchair.
-  **WARNING**
Risk of accidents. Never leave an unoccupied wheelchair on an incline or other place which may cause injury to surrounded people or things.
-  **WARNING**
Risk of accidents and serious injury. The wheelchair is not intended for use in the motor vehicle.

GENERAL

INTRODUCTION

This instruction for use contains important information about the handling of the product. To ensure safety when using the product, read the instruction for use carefully and follow the safety instructions. LightMan TRAVEL wheelchair is available in different sizes. You may identify type of your wheelchair and its unique identification number reading a label stuck on the armrest plate and metal cross brace underneath seat, respectively. *E.g.*, round sticker with number 46 on skirt guard indicates that size (seat width) of your chair is 46 cm.

i Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models. If not otherwise stated, each section in this document refers to all models of the product.

PREFACE

Thank you for the confidence shown in Kid-Man®. Every product is factory checked and left us in faultless condition. We wish you much success and we hope UAB Kasko Group | Kid-Man® product will make a positive impact to your daily living.

WARRANTY

We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in an improper way. Warranty is only valid with fully filled warranty card. UAB “Kasko Group”, after receiving the claimed item from the purchaser within a period of 4 weeks, decides whether the issue is covered under warranty. If the issue is not covered under warranty and customer agrees, UAB “Kasko Group”, if possible, fixes the issue. All costs, including costs of transportation, should be covered by the customer. Warranty repair does not extend warranty.

LIMITATION OF LIABILITY

Manufacturer accepts no liability for damage arising from:

- Non-compliance with the Instruction for use;
- Incorrect use;
- Natural wear and tear;
- Incorrect assembly or set-up by the purchaser or a third-party technical modifications;
- The usage of unapproved 3rd party accessories instead of approved Manufacturer's accessories;
- Unauthorized modifications and/or use of unsuitable spare part.

STANDARDS AND REGULATIONS

The wheelchair complies the requirements of:

- a) standard EN 12183:2022 Manual wheelchairs - Requirements and test methods;
- b) REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009 and repealing Council Directives 90/385/EEC and 93/42/EEC

SERVICE LIFE

The expected service life is 5 (five) years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this instruction for use.

After then, wheelchair has to be recycled. Do not throw your wheelchair away with common waste (see chapter "DISPOSAL").

INFORMATION ABOUT MANUFACTURER

Manufacturer: UAB "Kasko Group" (registered trade mark Kid-Man®)

Address: A. Vivulskio 12D, Vilnius, LT-03221, Lithuania

Phone: +370 5 265 0000

E-mail: info@kid-man.com

Website: www.kid-man.com

INFORMATION ABOUT PRODUCT

Product: TRANSIT WHEELCHAIR

Model: LightMan TRAVEL

REF: 04-018-2/38, 04-018-2/41, 04-018-2/43, 04-018-2/46, 04-018-2/48, 04-018-2/51

INTENDED PURPOSE AND INDICATIONS

The LightMan TRAVEL wheelchair is manually operated and should be used only for transporting a person who has temporary (e.g., broken bones, leg or foot injuries) or long-term (e.g., paralysis, musculoskeletal, neurological, balance, or gait disorders) mobility difficulties. It can be used both indoors and outdoors. The wheelchair does not have propulsion rims; therefore, an accompanying person is required.

i This wheelchair needs to be subscribed and fit to your specific health condition.

INTENDED USERS

LightMan TRAVEL wheelchair is intended to provide mobility to persons ages 15 and up (adolescents and adults) with a maximum weight capacity of 120 kg.

WARNING

Risk of accidents.

It is forbidden to exceed the maximum safe load of wheelchair.

The device is indicated to provide mobility to persons limited to sitting position and with an assistant in presence only.

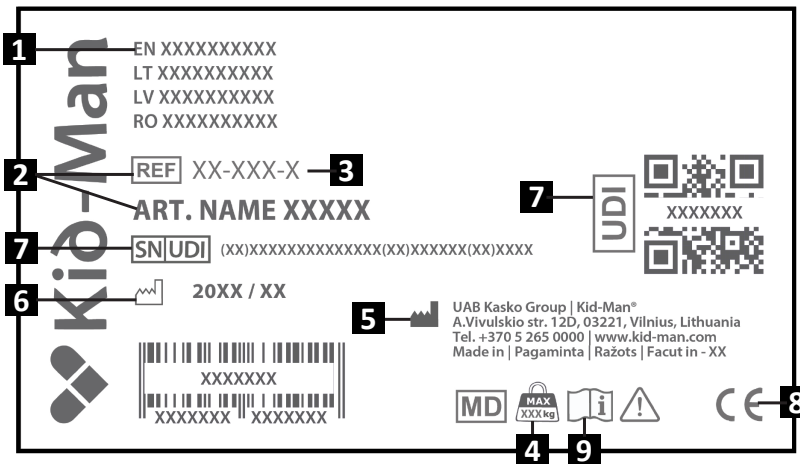
CONTRA-INDICATIONS

There are no contraindications known, when using the wheelchair as intended.

PRODUCT LABEL

The identification label is attached to the frame of your product (see picture).

1. Product
2. Model
3. Size (seat width)
4. Maximum load
5. Manufacturer / contact details
6. Date of manufacture
7. UDI / SN code
8. CE mark
9. Consult instructions for use

CAUTION

The wheelchair is appropriate for use by others. In order to do this, the wheelchair must first be thoroughly cleaned and disinfected. Then, the all components of the wheelchair must be get checked by an authorised service dealer for general condition, wear and shall get repaired if necessary.

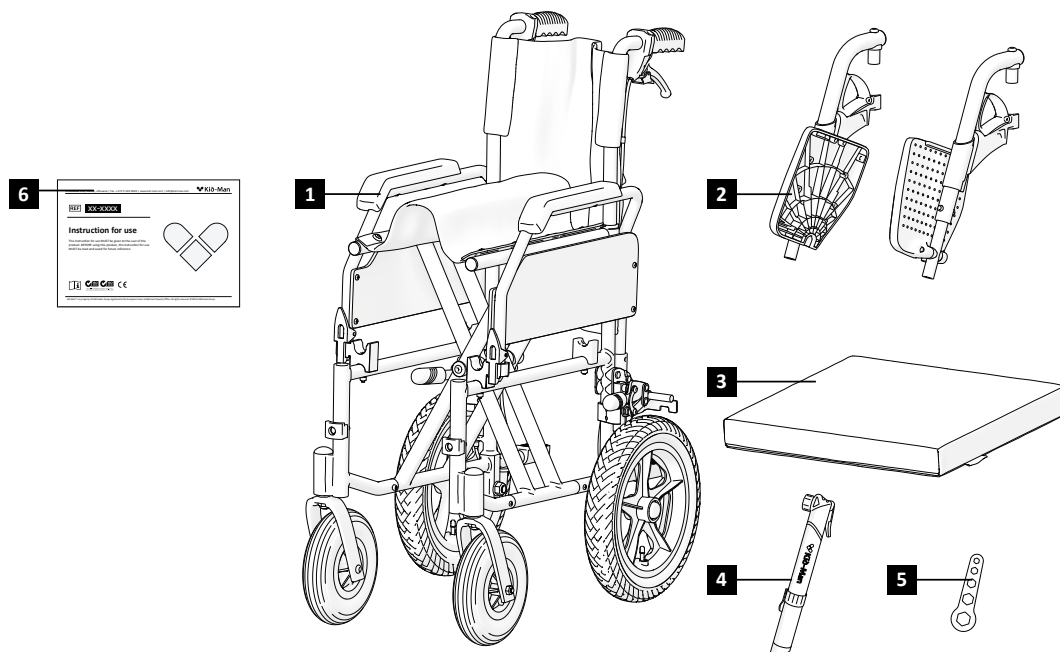
CONTENT OF PACKAGE

The LightMan TRAVEL wheelchair delivered in cardboard packaging and is ready for use. After unpacking, please keep the cardboard packaging if possible. You may need it later for storage or for the return of the wheelchair. Once parcel is received, please check whether it is not damaged during transportation.

- i** If you see signs of damage, please make note to transport company on the same time.
- i** Please check content of package and if any of below parts are missing, get in touch with your local dealer.

Content of package:

1. 1 x Folded wheelchair
2. 1 x Pair of legrests
3. 1 x Cushion
4. 1 x Pump
5. 1 x Adjustment tool
6. Instruction for use


COMPONENTS AND FUNCTIONS

Overview

This is a wheelchair with crossbar folding mechanism and swivelling legrests.

1. Backrest
2. Armrest
3. Cushion
4. Legrest swing lever
5. Footplate
6. Heel strap
7. Attendant push handles
8. Attendant brake levers
9. Parking brake
10. Rear wheel
11. Front Castor

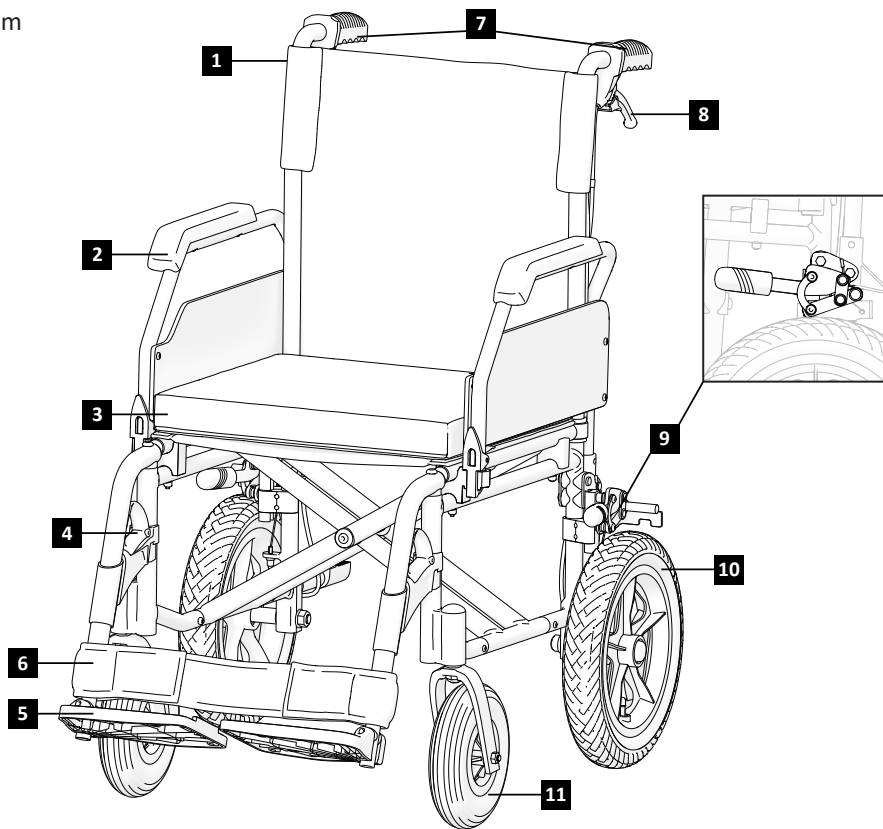i Complexations of your wheelchair can slightly differ from a wheelchair described in this manual due to continuous development of the product.

SETTING UP THE WHEELCHAIR

To **unfold** the wheelchair:

1. Ensure the wheelchair is on a stable, flat surface;
2. With your hand flat, press the seat edge closest to you, so that the seat surface flattens out and the seat pipes are completely set in the plastic holders. The seat must be fully opened;
3. Place the included foam cushion on the seat;
4. Insert the legrests into their designated positions (Picture 2);
5. Turn the legrest to the left or right side (Pictures 3–4) to lock it securely in place (the legrest must stand firmly);
6. Lower the footplates (Picture 5) into a comfortable position.

i You do not need to use energy too much. If You push too much You can damage the legrest (e.g., break it).

To **fold** the wheelchair:

1. Remove the cushion;
2. Remove legrest or flip up footplates (Picture 6);
3. Place your hands at the front and rear of the seat and pull upwards (Picture 7), then press the sides of the frame together to fold the wheelchair (Picture 8).

WARNING

Danger of trapping your finger! Be careful not to grasp the seat pipe.

WARNING

Never lift the wheelchair by removable parts (armrests, footrests). Please make sure all removable parts are taken off. The removable parts in the wheelchair are armrest, legrest, rear wheels and cushions. How to remove them, please refer to this instruction for use. After parts are removed, please to fold the wheelchair. After it is folded, you can easily lift it holding with both hands on the frame. The lifting weight after all parts are removed is around 8-10 kg depending on the size of the wheelchair and model.

OPERATION

i This wheelchair was designed to ensure stability while seated. Every movement on the wheelchair affects the point of gravity, which may cause the wheelchair to tip over. To increase your safety, you may use seatbelt.

WARNING

There are no positions of points intended to carry additional loads.

Leaning forwards

WARNING

Risk of falling out. If you lean forwards out of the wheelchair you could fall out of it. Never bend too far forwards and do not shift forwards in your seat to reach an object. Do not bend forwards between your knees to pick up something off the floor.

Reaching an object from the wheelchair

Only the arms may be extended over the wheelchairs seat. The body and head should stay inside the limits of the seat (Picture 9).

Moving onto the sidewalk

We recommend going up onto the sidewalk with the help of an assistant. Steps for the assistant:

- Position the wheelchair straight, directly in front of the curb.
- Tilt the wheelchair backward and slowly move forward until the front wheels roll over the curb.
- Push the wheelchair straight forward until the rear wheels touch the curb.
- Lift the wheelchair slightly and push it forward until the rear wheels are on the sidewalk (Picture 12).

Note: if the anti-tipper prevents the wheelchair from tilting back far enough, we recommend temporarily removing it until the wheelchair is placed on the sidewalk.

Transfer

You can transfer on your own if you are strong enough. First, place the wheelchair as close as possible to the surface you're transferring to, ensuring the front wheels are facing forward. Apply the brakes (Picture 20). Flip the footrests up and move them aside to create space. Fold back the armrest on the side you're transferring toward. Move toward the desired position (see Picture 13). Lean forward, resting one hand on the lowered armrest and the other on the surface onto which you intend to transfer. Shift your body toward the target surface, distributing your weight between your hands and arms. Do not lean on the brake levers. Using a transfer board is recommended to maintain support under you.

If the movement is too difficult, seek help from an assistant to transfer safely. Never lean on the brakes. While transferring or standing, never rely on the footplates for support.

OPERATION

Stairs

We suggest seeking help from two people, one in front of the wheelchair and one behind it, due to the fact that hurdling stairs is more difficult and dangerous. The person behind the wheelchair, tips the wheelchair backwards until it is balanced. Then, pushes the wheelchair against the first step, grabbing the handle firmly, to prepare to lift the wheelchair. The person in front of the wheelchair grabs the sides of the frame in the front of the wheelchair and lifts the chair over the steps, while the other accompanying person steps on the next step, repeating the first process. Do not lower the wheelchair until you hurdled the last step of the person in front of the wheelchair. Carrying a wheelchair down the stairs is the same process as explained previously, in reversed order (Picture 11).

WARNING

Please do not grab wheelchair over the legrests, due to disassembly possibilities.

ASCENDING / DESCENDING A RAMP WITH ASSISTANCE

Refer to the pictures above for safe wheelchair ascent and descent with assistance — avoid the incorrect method shown in Picture 16.

WARNING

The wheelchair is not applicable as a seat for cars.

OPERATION

Armrest operation

Wheelchair is equipped with flip up armrest. To flip it up, please push plastic lever located in the front of armrest (Picture 17 (a.)). Keep it pushed and lift armrest gently (Picture 18). We recommend to flip armrest up getting in or out of wheelchair from side. While operating wheelchair, armrests should be locked.

Braking

Your wheelchair has brakes mounted onto frame from both sides. Upon delivery, the brakes are properly adjusted to ensure effective braking. To activate the brakes, grasp the brake lever and pull it towards yourself (Pictures 19–20). We recommend always use both brakes at the same time to ensure maximum safety. Due to normal wear and tear of rear wheel, braking efficiency may be reduced. In order to have sufficient braking, you must adjust position of brakes.

OPERATION

Assistant brakes

LightMan TRAVEL is equipped with assistant brakes that may only be operated by an accompanying person using the brake levers located on the push handles. These brakes can be used both as deceleration brakes and as parking brakes. When used for deceleration, they provide smooth and controlled braking (Picture 21). The parking brake function is used when stationary. To engage the parking brakes, pull the brake lever toward yourself and push the locking lever forward (Picture 22). To release the parking brake, pull the brake lever upwards softly and the locking mechanism will release automatically. For optimal braking efficiency, the braking power is set using the adjusting screw. Braking force is increased by loosening the screw and tightening the brake cable.

Legrest operation

Once delivered, legrests are packed in separate small cardboard. In order to install them, please take them out of package. The footrests must be installed in the front of chair. Firstly, please insert plastic part into metal tube and push it down (Picture 2). After, move legrest inside until you hear snap (Pictures 3-4). Once locked in position, legrest stand firmly. Do not mix left and right legrest! The installed plastic footplate must be positioned facing inwards (Picture 5). Once leaving wheelchair, legrest must be flipped away (Picture 6) or completely removed. Pull plastic lever upwards and move legrest outside. Length of legrest can be adjusted to meet user's needs. Firstly, unscrew bolt using Allen key, pull out bolt and tighten it again in needed position.

Tyre

The front casters of the wheelchair are solid (PU). The rear wheels are pneumatic. Recommended air pressure is indicated on the tyre:

	Max pressure	
Tyre 12.5 x 2.25 (57-203)	280kPa	40 psi

For your safety, please check air pressure in tyres periodically (see chapter “MAINTENANCE”). If you notice air pressure is too low, please inflate them using the pump which is included into delivery. In case tyres do not keep the air or their obvious wear of outer tyre, please inform service company immediately.

WARNING

Use of wheelchair with empty air tyres is forbidden!

MAINTENANCE

To ensure safe and reliable operation, carry out the following checks and maintenance regularly or have it carried out by another person. Below You will find recommended maintenance's schedule.

	Before use	Once a week	Once a month	Once a year
Check the tyre pressure		X		
Check that rear wheels are seated correctly	X			
Visual check			X	
Check the folding mechanism			X	
Check the brakes	X			
Clean the front wheels			X	
Have wheelchair checked by an authorized dealer				X

WARNING

Repairs, cleaning, or adjustments of the wheelchair should be performed by an authorized service person while the user is not sitting in the wheelchair.

CAUTION

Only original spare parts can be used for replacement. Please provide model and serial number to dealer or contact manufacturer directly. Manufacturer and dealer are not responsible for any damage occurred to product or user if non original spare parts are used an (or) repair was done in unauthorized facility.

CLEANING AND DISINFECTION

Use the moist cloth to clean frame as well as plastic parts. The use of a mild detergent for more stubborn dirt is possible as well. The wheels can be cleaned with a wet synthetic fibre brush (the use of wire brush is not allowed). Seat and back pads can be washed off. Use mild detergent. Use a common non aggressive disinfectant. If you are unsure if the one used by you is suitable for disinfection, please get in touch with local dealer.

- i** Sand and seawater can damage the bearings and steel parts can rust if the surface is damaged. Only expose the wheelchair to sand and seawater for short periods and clean it after every trip to the beach.

The wheelchair may be disinfected by spraying or wiping with tested, approved disinfectants.

- i** Spray a soft cleaning and disinfecting product and follow the instructions given by the manufacturer.

STORAGE

The wheelchair should be stored in a dry environment, away from direct sunlight. When in storage the wheelchair can be folded up. The wheelchair should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent. Clean regularly, use a general purpose, neutral detergent and warm water. Please check the suitability of the cleaners or detergent before use. Do not use abrasive cleaners as this will permanently damage the surface and invalidate any warranty.

WARNING

The wheelchair must not be permanently exposed to temperatures greater than +40°C. Otherwise the frame as well as seat, backrest and armrests can heat up, which might cause burns.

WARNING

Never store other items on top of the wheelchair.

After long-term storage (more than 3 months) the wheelchair must be inspected (see chapter “MAINTENANCE”).

STORAGE

Environmental conditions

Keep wheelchair under these conditions:

- Temperature limitation – 20° C to + 40° C;
- Relative humidity 20% to 75%;
- Atmospheric pressure 800 hPa to 1060 hPa.

RE-USE

The wheelchair is suitable for re-use. The following actions should be taken:

Cleaning and disinfection (see chapter “CLEANING AND DISINFECTION”);

Inspection (see chapter “MAINTENANCE”).

The wheelchair should be adapted for the new user in accordance with the service documentation.

DISPOSAL

Be environmentally aware and dispose of your wheelchair properly. Disposal is subject to national and local regulations.

To have wheelchair properly recycled, please contact your local community, disposal facility or our service department.

SPECIFICATIONS

Model	LightMan TRAVEL					
REF	04-018-2/38	04-018-2/41	04-018-2/43	04-018-2/46	04-018-2/48	04-018-2/51
Seat width (cm)	38	41	43	46	48	51
Weight (fully assembled) (kg)	13.4	13.6	13.8	14	14.2	14.4
Overall length (cm)	105					
Overall width (cm)	Seat width + 18					
Overall height (cm)	93					
Seat depth (cm)	41					
Seat height (cm)	48 (Without cushion); 52 (With cushion)					
Backrest height (cm)	45					
Length of legrest (cm)	34 - 45					
Armrest height (cm)	25.5 (Without cushion); 21.5 (With cushion)					
Size of front caster (cm)	20 x 5					
Size of rear wheel (cm)	31.7 x 5.7					
Legrest weight (kg)	1.5					
Cushion weight (kg)	0.4 - 0.5					
Maximum safe load	120					
Frame	Aluminium					
Upholstery	Nylon					

INCIDENTS

Any serious incident that has occurred in relation to the device should be reported to UAB Kasko Group | Kid-Man® and the competent authority of the Member State in which the user and/or patient is established.

CONTACTS:

Manufacturer: UAB “Kasko Group” (registered trade mark Kid-Man®)

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