SKUTA

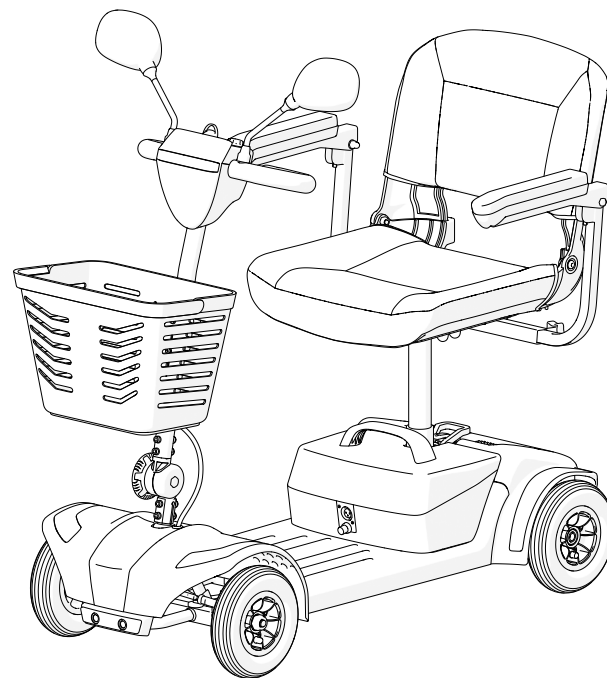
01-3580

MOBILITY SCOOTER MOBILITETSSCOOTER SÄHKÖMOPO

EN

DK

FI

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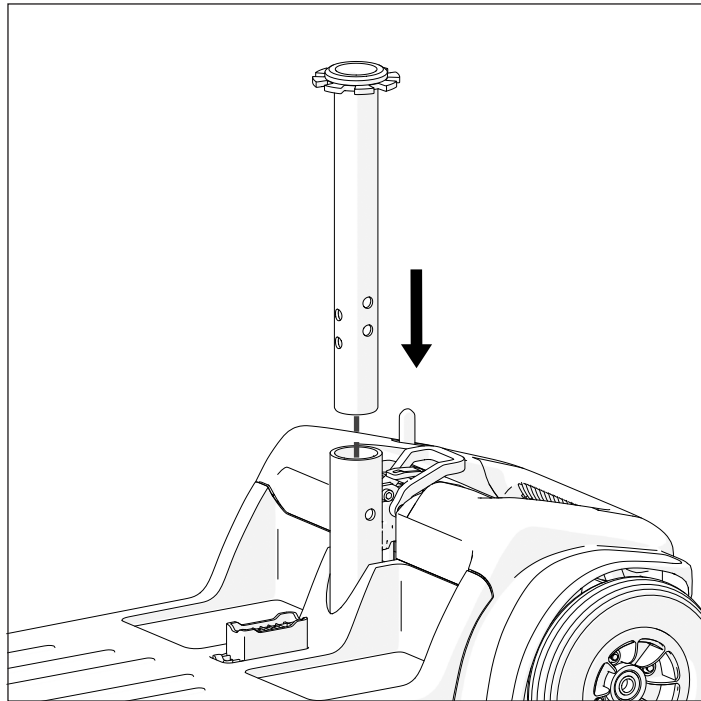
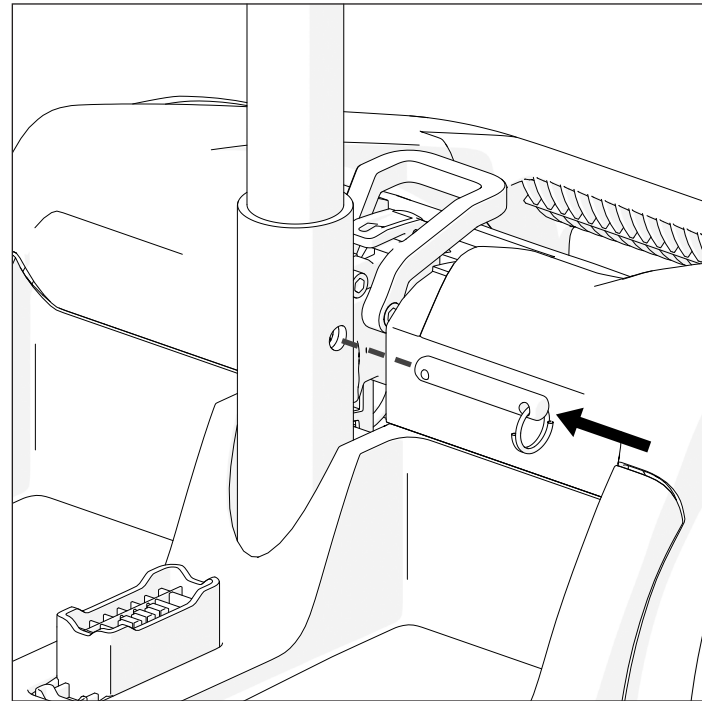
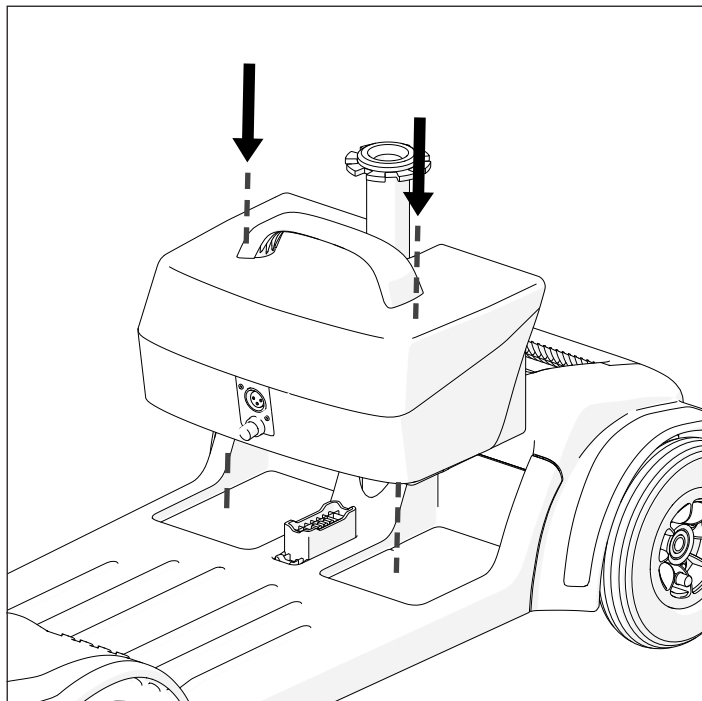
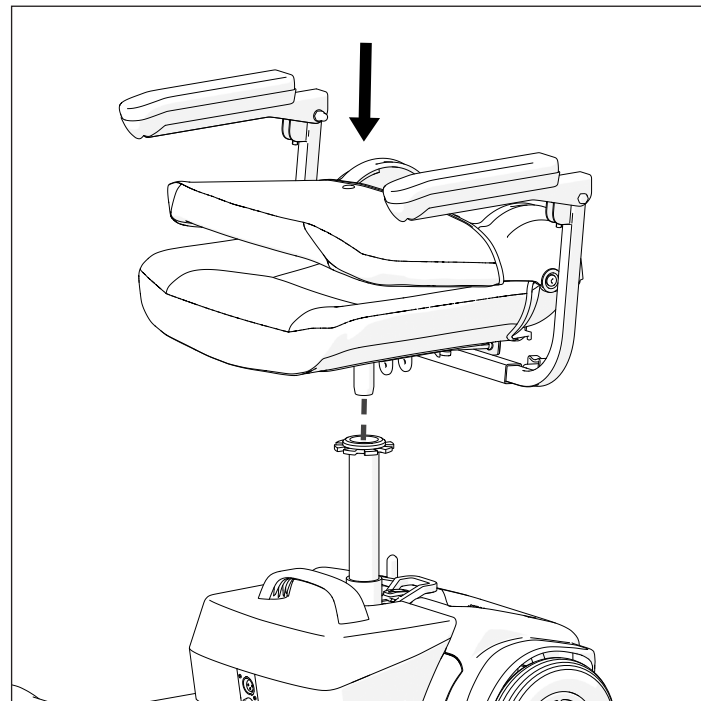
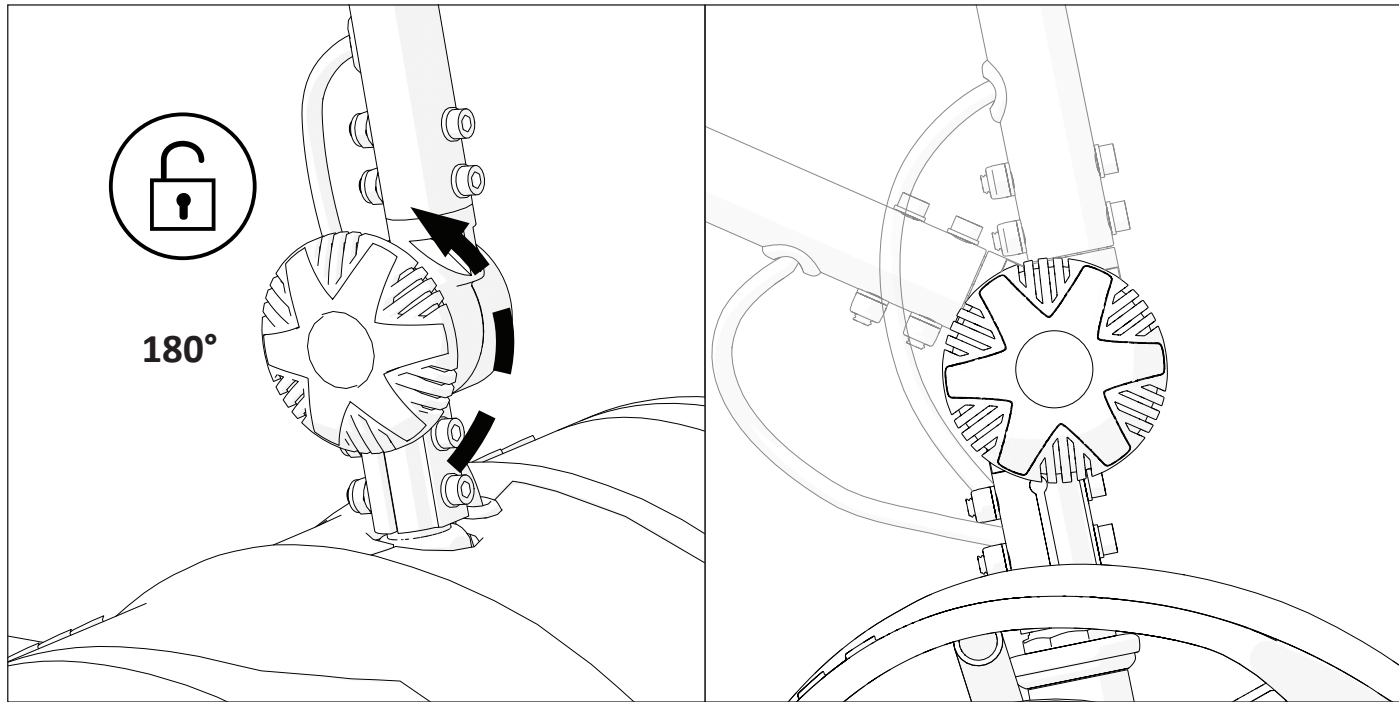
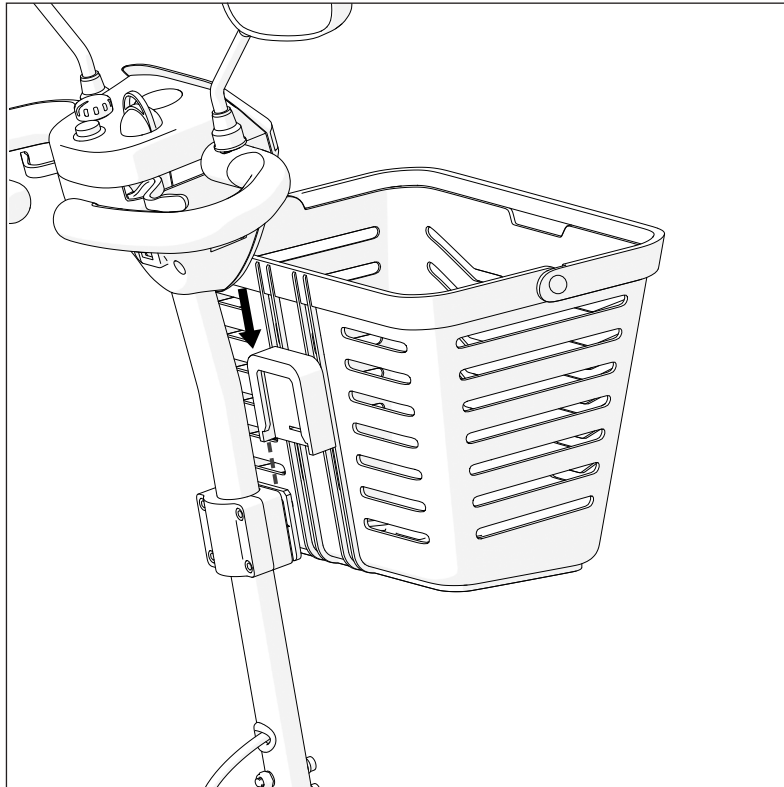
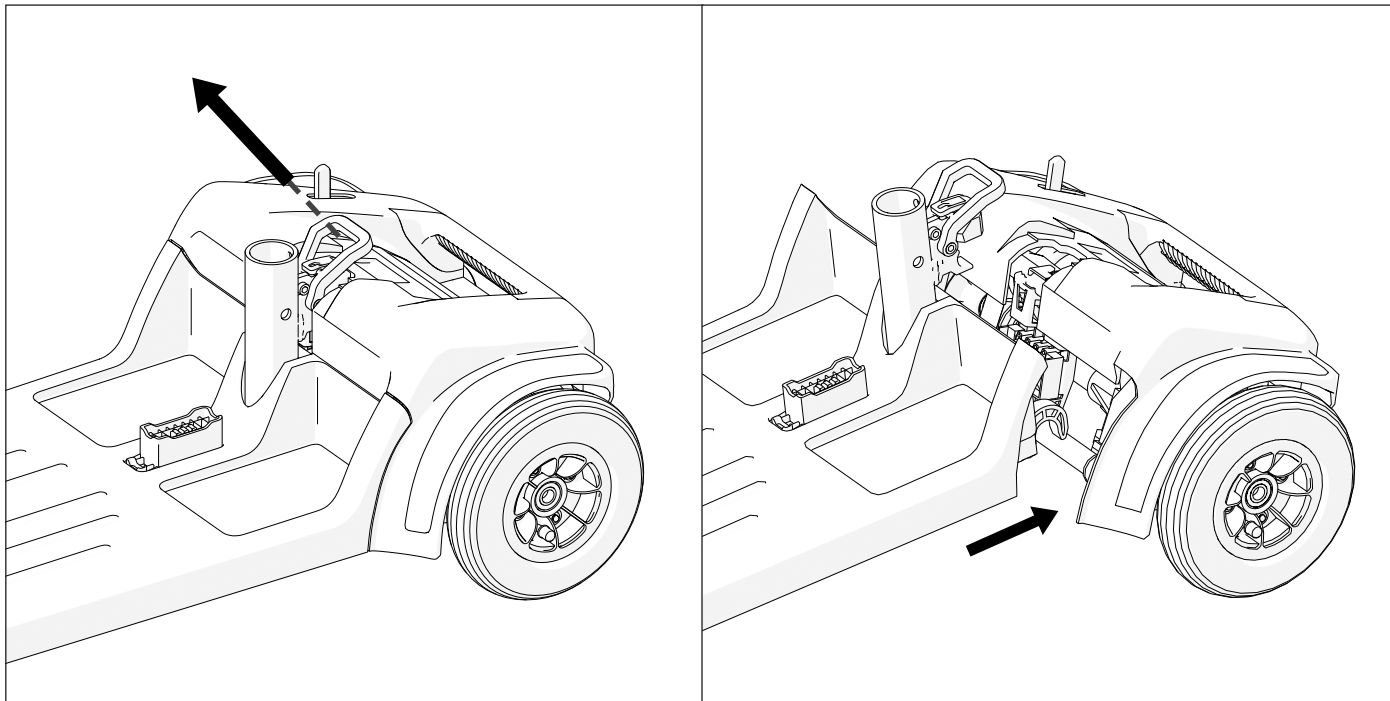
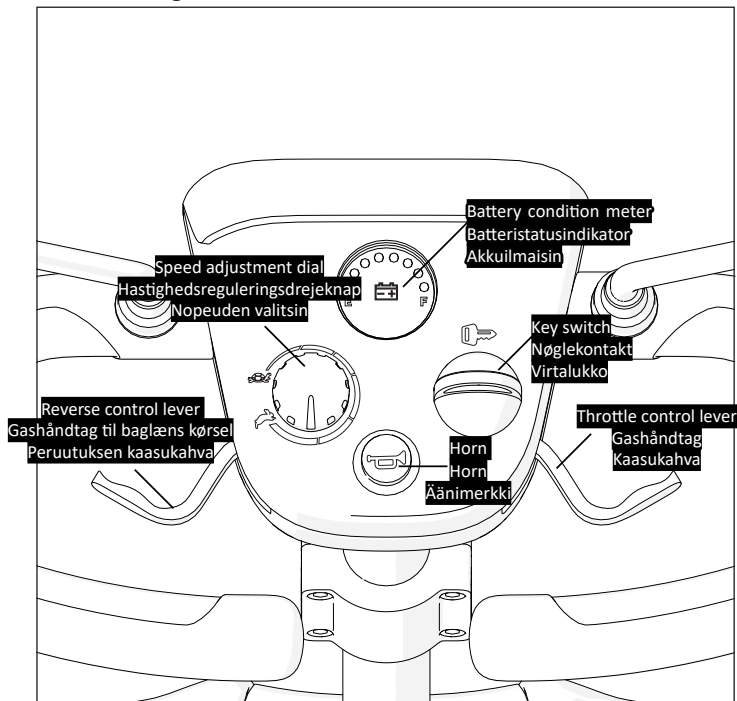
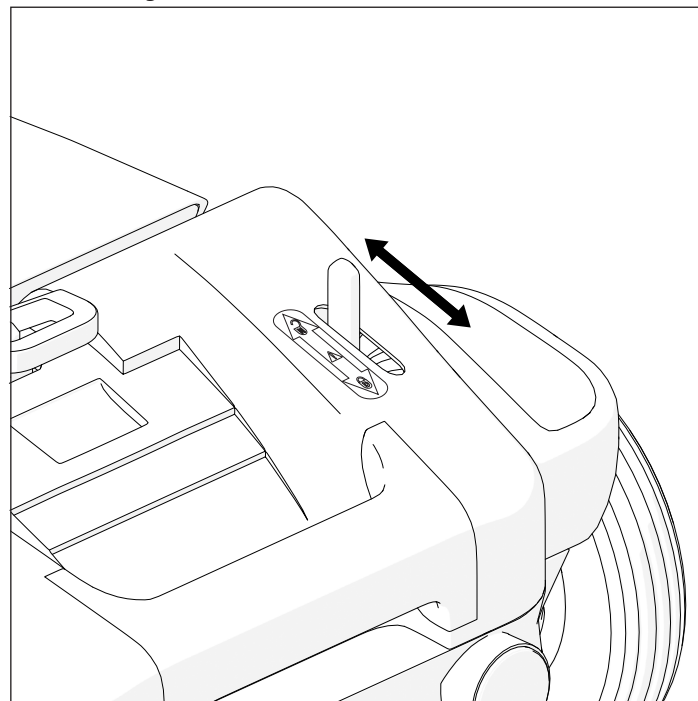
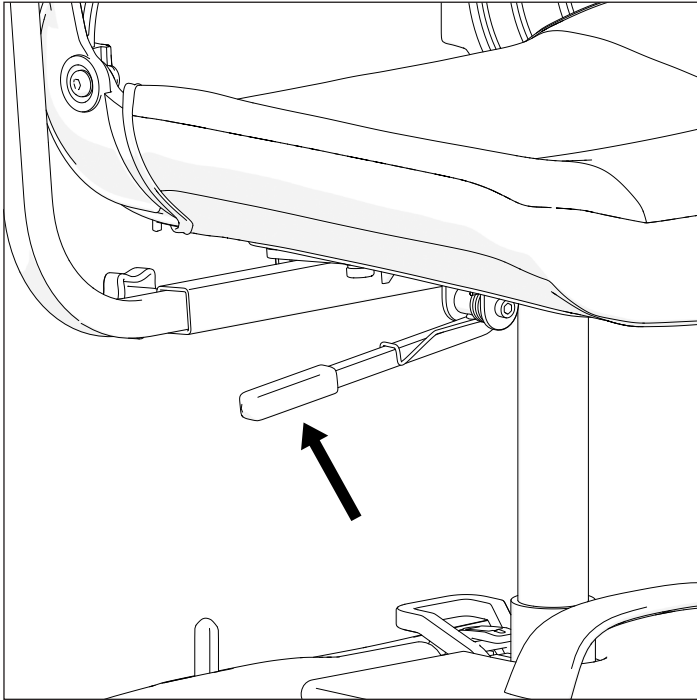
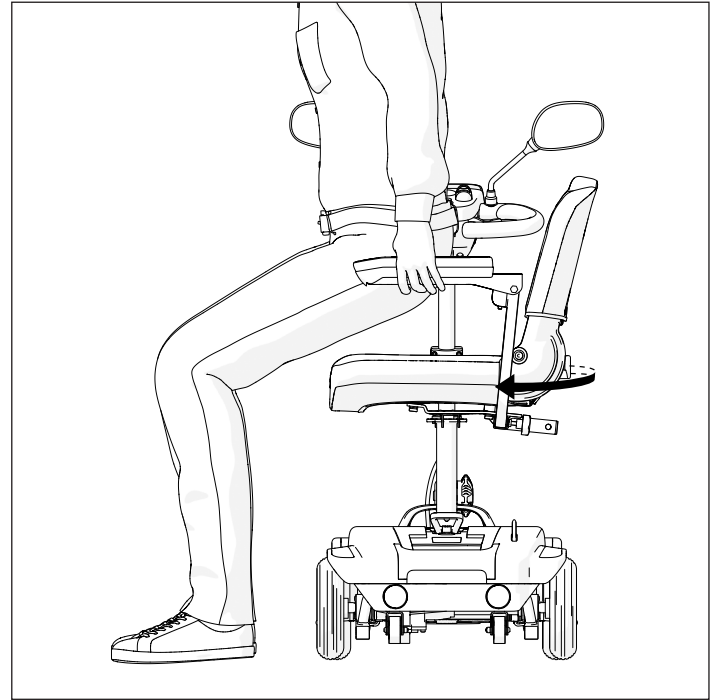
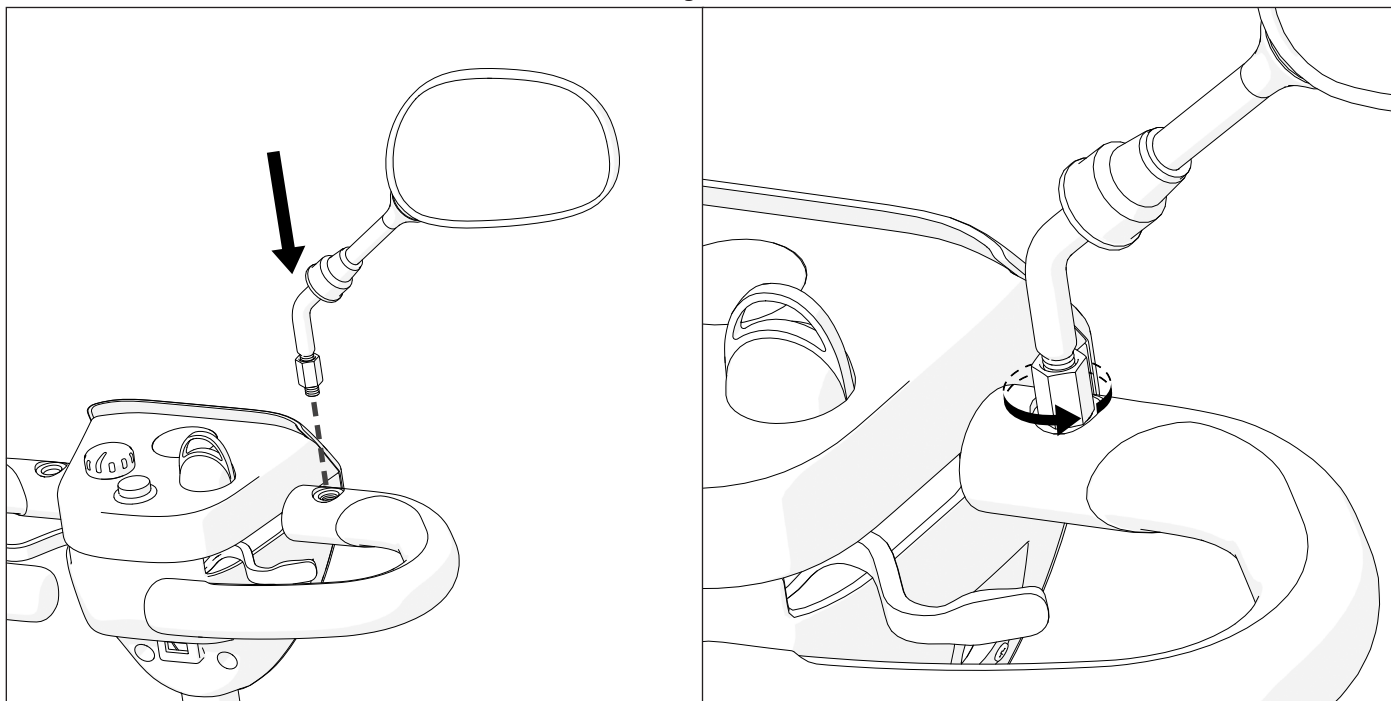
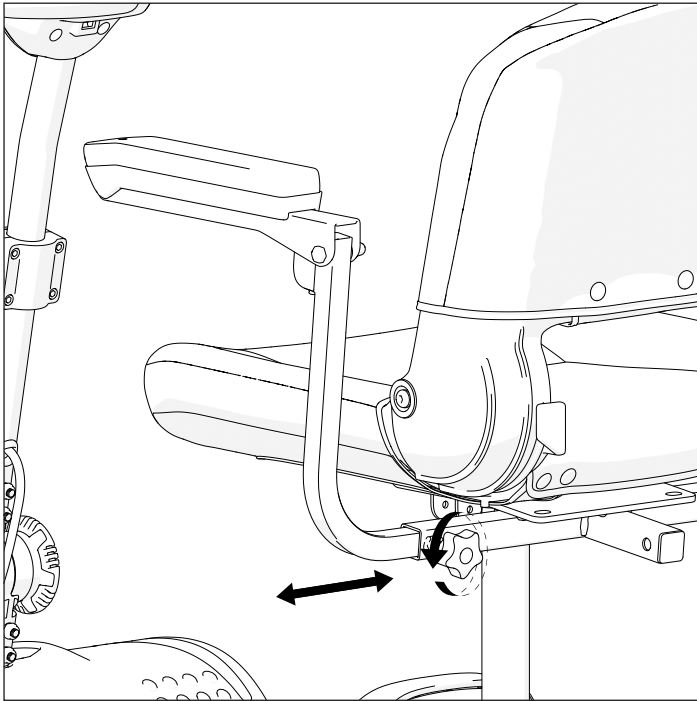
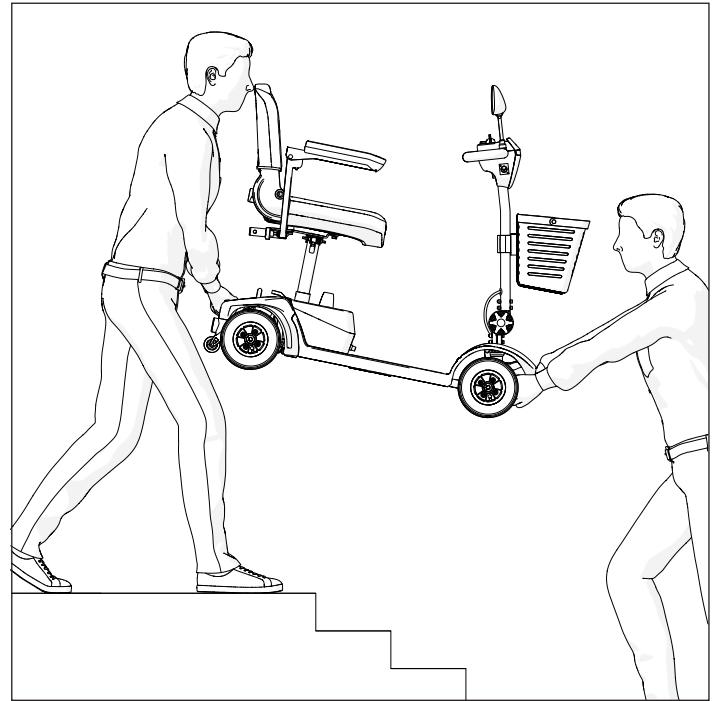
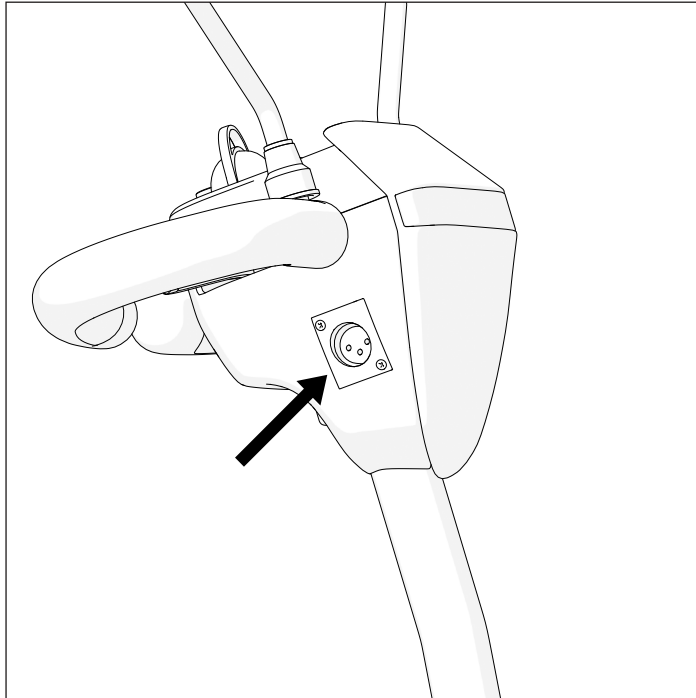
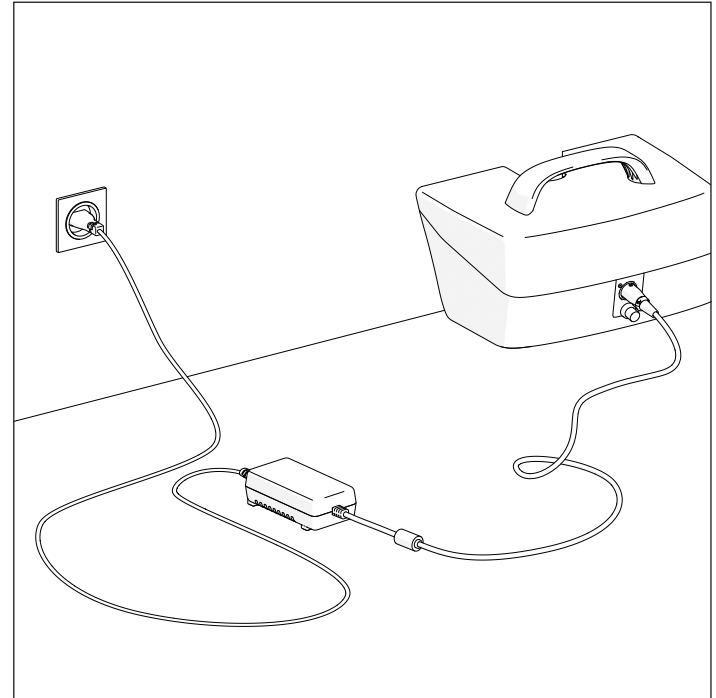
SYMBOLS

**In this Instruction for Use warnings are indicated by symbols.
The warning symbols are accompanied by a heading that indicates the severity of the danger or basic information.**

i Gives basic information, recommendations and information for efficient, trouble-free use.

 **WARNING / CAUTION** - to indicate that caution is necessary when operating the device. To indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

 **ATTENTION!** This sign is used for very important parts concerning your safety. Please look at the operating instructions.

SAFETY INFORMATION

 **WARNING**
Any other or incorrect use could lead to a hazardous situation.

 **WARNING**
The device is a compact mobility scooter designed for both indoor and outdoor use. Exercise caution when cornering, navigating drops, obstacles, and gradients to ensure safe operation.

 **WARNING**
Risk due to driving style being unsuitable for the conditions. There is a risk of skidding on wet ground, gravel or uneven terrain. Always reduce your speed and maintain a stable center of gravity when cornering. Do not corner sharply when driving scooters at higher speeds, and reduce speed before cornering.

 **WARNING**
If the battery gauge drops to red, charge the scooter as soon as possible. Completely discharging the batteries may damage the batteries or shorten the expected life of the batteries.

SAFETY INFORMATION

 **WARNING**
Risk of shock. Keep metal objects away from the battery terminals. Electric shock may occur.

 **WARNING**
Do not allow children to play near or operate the scooter.

 **WARNING**
Risk of accidents. Safety devices that are incorrectly set or no longer working (brakes, battery) can cause accidents. Always check that the safety devices are working before you use the scooter and have them checked regularly by the authorized dealer.

 **WARNING**
Risk of accidents. Read and follow the information in the owner's manual. Do not use this product or any available optional equipment without first thoroughly reading and understanding these instructions. If you cannot understand the warnings, cautions, or instructions, contact your dealer before attempting to use this equipment. Otherwise, injury or damage may occur. Never try to use your mobility scooter beyond its limitations, as described in this manual.

 **WARNING**
Risk of choke. Please protect children under the age of the five years from easy access to scooter. Due to small elements used in the construction, the children can swallow them or choke.

 **CAUTION**
Do not drive your scooter at night without proper lighting.

 **CAUTION**
Do not remove the anti-tip wheels or transit wheels.

 **CAUTION**
Keep your hands and feet away from moving parts while driving. Be aware of loose-fitting clothes that can become caught in the drive wheels.

SAFETY INFORMATION**CAUTION**

Surface temperatures can increase when exposed to external sources of heat.

**WARNING**

Do not connect an extension cord to the battery charger.

**WARNING**

Disassembling the controller, motor, or charger by anyone other than an authorized service agent is prohibited and voids any applicable warranty.

**WARNING**

Do not take your mobility scooter on roads or highways.

**WARNING**

Do not use your mobility scooter if you are under the influence of alcohol, medicines, or other substances that may influence your driving abilities.

**WARNING**

Be cautious when driving your mobility scooter in busy areas or shopping malls.

**WARNING**

Under no circumstances should the mobility scooter be used as a seat in a motor vehicle.

**WARNING**

Do not attempt to lift your mobility scooter by any parts other than the frame (for example, seat or body shrouds)

**WARNING**

Do not allow unsupervised children to play near the scooter while the batteries are charging.

**WARNING**

Keep your hands away from the tyres when driving. Be aware that loose fitting clothing can become caught in the drive tyres.

SAFETY INFORMATION

WARNING

The device is not intended for use in or near electronic security systems such as walk-through metal detectors or electromagnetic anti-theft systems, and that the performance of the device may be degraded in close proximity to such systems.

WARNING

The device is not safe for use in MRI environments, and it should not be used in the vicinity of other advanced medical equipment such as electrocautery, electrosurgical units, and diathermy equipment.

WARNING

Do not modify your mobility scooter in any way; doing so will void the warranty.

WARNING

Do not sit in your mobility scooter while in a moving vehicle.

WARNING

Drive your mobility scooter straight up or down an incline, never backward. Erratic movements or back and forth patterns may increase the chance of tipping. Changes in inclines may decrease stability. Under any circumstances, do not travel on a slope greater than the maximum stability angle for this mobility scooter.

WARNING

Do not operate hand-held transceivers such as CB (citizens band) radios, or turn on personal communication devices, such as cellular phones, while the scooter is powered on.

WARNING

Be aware of nearby transmitters, such as radio or television stations, and avoid close.

GENERAL

INTRODUCTION

This instruction for use contains important information about the handling of the product. To ensure safety when using the product, read the instruction for use carefully and follow the safety instructions. You can identify your scooter model and its unique identification number on the label affixed to the chassis frame beneath the seat.

i Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models. If not otherwise stated, each section in this document refers to all models of the product.

PREFACE

Thank you for the confidence shown in Kid-Man. Every product is factory checked and left us in faultless condition. We wish you much success and we hope UAB Kasko Group | Kid-Man® product will make a positive impact to your daily living.

WARRANTY

We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in an improper way. Warranty is only valid with fully filled warranty card. UAB “Kasko Group”, after receiving the claimed item from the purchaser within a period of 4 weeks, decides whether the issue is covered under warranty. If the issue is not covered under warranty and customer agrees, UAB “Kasko Group”, if possible, fixes the issue. All costs, including costs of transportation, should be covered by the customer. Warranty repair does not extend warranty.

LIMITATION OF LIABILITY

Manufacturer accepts no liability for damage arising from:

- Non-compliance with the Instruction for use;
- Incorrect use;
- Natural wear and tear;
- Incorrect assembly or set-up by the purchaser or a third-party technical modifications;
- The usage of unapproved 3rd party accessories instead of approved Manufacturer's accessories;
- Unauthorized modifications and/or use of unsuitable spare part.

STANDARDS AND REGULATIONS

The scooter complies with the requirements of:

- a) ISO 7176-4: Requirements for energy consumption and theoretical range under standard test conditions;
- b) ISO 7176-8: Requirements and test methods for static, impact, and fatigue strength;
- c) ISO 7176-9: Climatic test requirements;

SERVICE LIFE

The expected lifespan of a mobility scooter motor is typically around three years, meaning that under normal usage and regular maintenance, the motor should operate for approximately three years. Do not throw your scooter away with common waste (see chapter "DISPOSAL").

INFORMATION ABOUT MANUFACTURER

Manufacturer: UAB "Kasko Group" (registered trade mark Kid-Man®)

Address: A. Vivulskio 12D, Vilnius, LT-03221, Lithuania

Phone: +370 5 265 0000

E-mail: info@kid-man.com

Website: www.kid-man.com

INFORMATION ABOUT PRODUCT

Product: ELECTRIC SCOOTER

Model: SKUTA

REF: 01-3580

INTENDED PURPOSE AND INDICATIONS

The intended use of this device is to provide mobility to persons limited to a seated position that have the capability of operating a scooter.

i This scooter needs to be subscribed and fit to your specific health condition.

INTENDED USERS

Scooter is designed for the easy transport of people. It is designed for the use of one person with maximum weight 136 kg, not passengers (including children). Electric scooter is designed for adults, not goods, and not for people under 16 years of age. Electric mobility scooter is a kind of transportation device, which compensates people with mobility difficulties. This scooter is not suitable for hemiplegia or paraplegia and this scooter shall not be used for mentally disabled person

WARNING

Risk of accidents.

It is forbidden to exceed the maximum safe load of scooter.

The device is indicated to provide mobility to persons limited to sitting position and with an assistant in presence only.

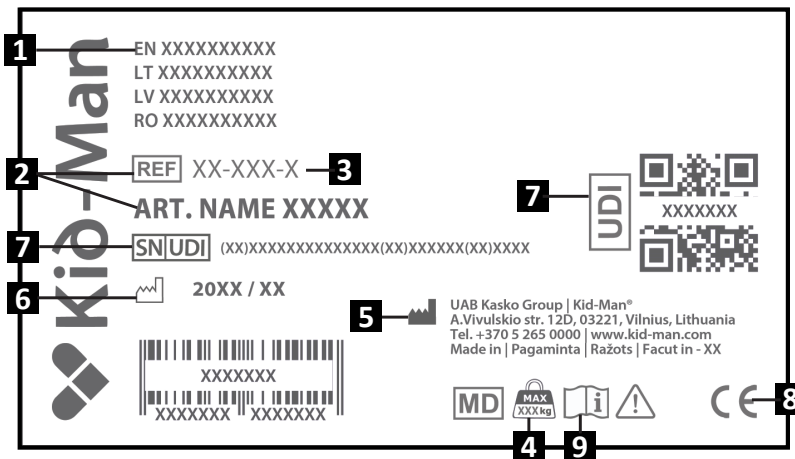
CONTRA-INDICATIONS

There are no contraindications known, when using the scooter as intended.

PRODUCT LABEL

The identification label is attached to the product (see picture).

1. Product
2. Model
3. Size (seat width)
4. Maximum load
5. Manufacturer / contact details
6. Date of manufacture
7. UDI / SN code
8. CE mark
9. Consult instructions for use

CAUTION
The scooter is appropriate for use by others. In order to do this, the scooter must first be thoroughly cleaned and disinfected. Then, the all components of the mobility scooter must be get checked by an authorised service dealer for general condition, wear and shall get repaired if necessary.

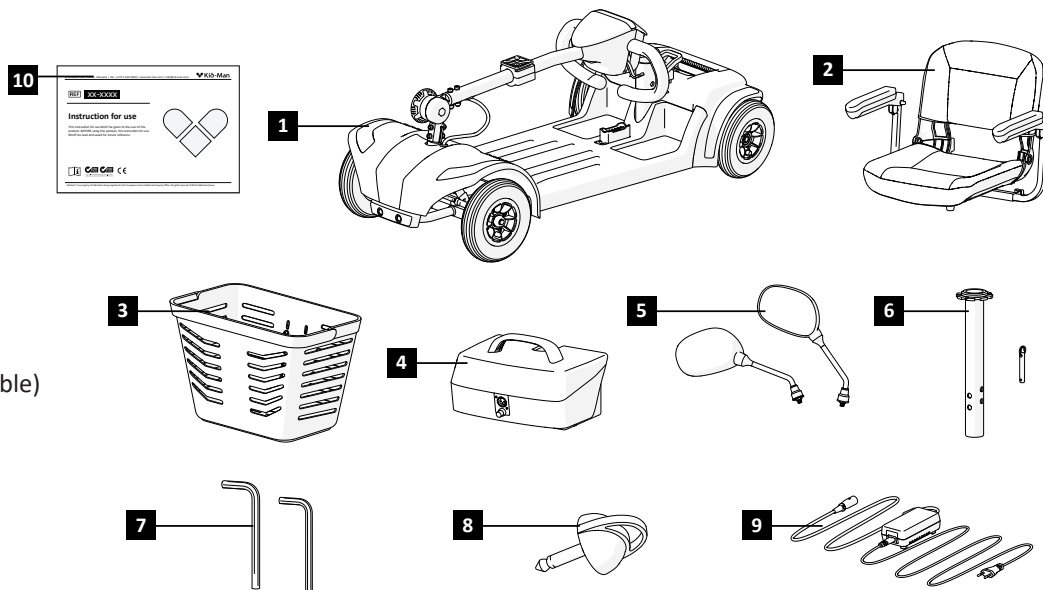
CONTENT OF PACKAGE

The SKUTA mobility scooter delivered in cardboard packaging and is ready for use. After unpacking, please keep the cardboard packaging if possible. You may need it later for storage or for the return of the scooter. Once parcel is received, please check whether it is not damaged during transportation.

- i** If you see signs of damage, please make note to transport company on the same time.
- i** Please check content of package and if any of below parts are missing, get in touch with your local dealer.

Content of package:

1. 1 x Electric scooter
2. 1 x Seat with armrests
3. 1 x Basket
4. 1 x Battery pack (installed or separate)
5. 2 x Side mirrors
6. 1 x Seat post with detent pin
7. 2 x Adjustment tools (if applicable)
8. 2 x Key(s)
9. 1 x Charger
10. Instruction for use

COMPONENTS AND FUNCTIONS

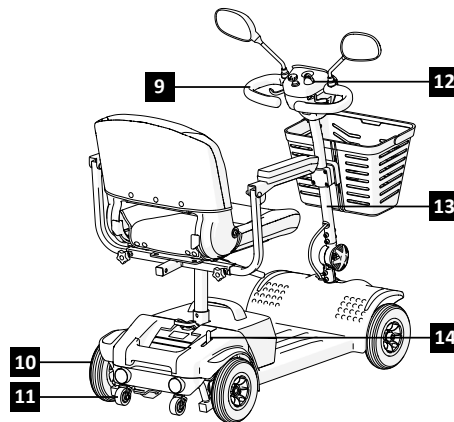
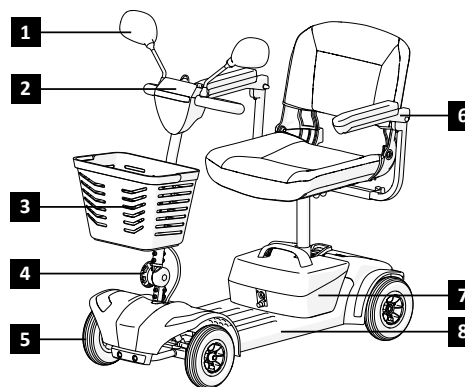
Overview

This is a mobility scooter with a folding frame and ergonomic tiller steering system.

1. Side mirrors
2. Operating console
3. Basket
4. Tiller adjustment knob
5. Front wheel
6. Seat with armrests
7. Battery package
8. Footplate
9. Handgrip
10. Rear wheel / drive wheel
11. Anti-tip wheels
12. Key
13. Tiller
14. Free-wheel lever

i

The configuration of your mobility scooter may slightly differ from the scooter shown in this manual due to ongoing product improvements.

SETTING UP THE SCOOTER

To disassemble scooter:

Your Travel Scooter can be taken apart into five main parts:

1. Seat
2. Rear Frame
3. Battery Pack
4. Front Frame
5. Basket

WARNING

Do not lift more than you can handle. Ask for help if needed.

SETTING UP THE SCOOTER

To assemble the SKUTA mobility scooter:

1. Insert the upper seat post into the lower seat post (Picture 1).
2. Adjust to the desired height and align the adjustment holes.
3. Insert the detent pin fully to secure the post (Picture 2).
4. Install the battery pack by placing it down into position (Picture 3).
5. Attach the seat by lowering it onto the post. Rotate the seat until it clicks and locks in place (Picture 4).
6. Lift the seat back upright until fully extended.
7. Raise the tiller and secure by tightening the tiller adjustment knob clockwise (Picture 5).
8. Insert the basket into the basket holder on the tiller (Picture 6).

Do not use excessive force when assembling the scooter. Applying too much pressure may damage parts such as the frame or locking mechanisms.

To fold the SKUTA mobility scooter:

1. Turn off the scooter and remove the key.
2. Fold down the seat back by pressing it forward.
3. Lift off the seat. If tight, rotate slightly while pulling up (Picture 4).
4. Remove the battery pack by lifting it up (Picture 3).
5. Pull out the detent pin from the seat post (Picture 2).
6. Remove the upper seat post (Picture 1).
7. Unlock the tiller adjustment knob by turning it counterclockwise ~180° and fold the tiller down carefully (Picture 5).
8. Lift up the frame release lever (Picture 7) and move the front frame going forward until the front frame hooks are no longer resting on the rear frame tube. Slowly separate the two parts of the scooter.

**WARNING**

Be careful not to trap fingers or clothing during assembly.
Always check that the tiller is firmly locked before use by gently pushing/pulling it. Retighten the knob if needed.

**WARNING**

Do not lift by seat, battery, basket, or tiller.
Remove all parts first (see manual), then lift by main frame. Weight: ~8 - 10 kg (varies by model).

OPERATION

Before Use

Before using your travel scooter, make sure the battery is fully charged and securely installed. The seat should be properly locked in place, and the tiller adjusted to a comfortable position. Insert the key into the switch to power on the scooter. The battery meter will light up and show the current charge level.
Set the maximum driving speed using the dial on the dashboard - the turtle icon indicates the slowest setting, while the rabbit indicates the fastest (Picture 8).

Driving

To move forward, gently press the right side of the throttle lever with your thumb. For reverse, press the left side. Releasing the throttle will stop the scooter automatically, as the brakes engage when the lever returns to its center position.
Always keep both hands on the tiller and your feet on the foot platform. Never drive while carrying passengers, walking pets on a leash, or hanging items on the tiller, as this may affect safety and control.

Manual push (Freewheel mode)

If you need to push the scooter by hand (for example, moving it short distances indoors), you can engage freewheel mode. This is only possible on level ground and on models equipped with the lever.
Locate the freewheel lever on the rear frame (right side) (Picture 9). Push the lever forward to disengage the motor and brakes. To return to drive mode, pull the lever back into its original position. Always remove the key before switching modes. Never sit on the scooter while in freewheel mode, and do not activate it on an incline.

Automatic power-off

Your scooter has an energy-saving feature. If the key is left in the “on” position but the scooter is not used for about 15 minutes, it will shut down automatically to save battery.
To reactivate it, simply remove the key, reinsert it, and turn the scooter back on.

OPERATION

Getting On and Off

Stand beside the scooter and rotate the seat towards you using the seat swivel lever (Picture 10). Sit down carefully and rotate the seat forward until it locks in place. Always ensure the seat is locked before driving.

Before getting on or off, remove the key from the switch to prevent accidental movement. Make sure your feet are placed safely on the foot platform and that the scooter is on stable, level ground.

To get off safely, first bring the scooter to a complete stop. Always remove the key from the ignition before standing up - this prevents any accidental movement.

Turn the seat to the side using the seat rotation lever, making sure it locks in that position. Then slowly stand up and step off the scooter. Use the armrests to lift yourself up (Picture 11). You can leave the seat turned sideways to make it easier to sit down the next time.

WARNING

Always remove the key before getting on or off to prevent accidental movement.

Adjusting the mirrors (if equipped)

If your scooter has mirrors, you can adjust them for better visibility. Gently lift the cap, loosen the screw, rotate the mirror to the desired angle, and tighten it again (Picture 12).

Adjusting the tiller angle

The tiller can be adjusted to suit your comfort.

Turn the adjustment knob under the tiller counterclockwise to loosen it. Set the tiller to a suitable angle, then tighten the knob again (Picture 5). To fold the tiller fully down, first remove the seat and battery.

Seat adjustments

To change seat height:

1. Remove the seat and battery (Pictures 3 - 4).
2. Pull out the locking pin from the seat post (Picture 2).
3. Raise or lower the seat to the desired height.
4. Align the holes and insert the locking pin securely.
5. Reinstall the battery and seat.

Armrest adjustments

The armrest width can be adjusted inward or outward and lifted up to make getting on and off easier. Loosen the armrest adjustment knobs located under the back of the seat, then tighten them again (Picture 13). Do not slide the armrests inward beyond the "MAX" mark to avoid damaging the seat. You can also remove the armrests by loosening the adjustment knobs.

OPERATION

Braking system

The scooter is equipped with electromagnetic brakes. These brakes engage automatically when you stop pressing the throttle. The braking system only works when the freewheel lever is set to the drive position. If the lever is in **push mode**, the brakes are disengaged. Always ensure the scooter is in drive mode before operating.

Refer to Picture 8

Throttle control lever

The throttle lever controls the forward and reverse movement of your scooter.

- Place your right hand on the right handgrip and your left hand on the left handgrip.
- To move forward, gently press the right side of the throttle lever with your right thumb.
- To move in reverse, press the left side of the throttle lever with your left thumb.
- To stop, release the lever. It will return automatically to the center “stop” position, and the electromagnetic brakes will engage.

CAUTION

Always bring the scooter to a complete stop before changing direction.

Key switch

- Insert the key into the key switch and turn it to the ON position to power up the scooter.
- To power down the scooter, turn the key to the OFF position and remove it.

i Even though the key can remain in the switch when powered off, it is recommended to remove the key when not in use to prevent unauthorized operation.

Speed adjustment dial

This dial limits the maximum speed of your scooter:

- Turn the dial clockwise to increase top speed (rabbit symbol = fastest).
- Turn the dial counterclockwise to reduce top speed (turtle symbol = slowest).

This setting allows you to adjust the scooter’s responsiveness based on your environment or preference.

Horn button

Press the horn button to activate the warning horn. The scooter must be powered on for the horn to function. Use the horn to alert pedestrians or other road users of your presence when necessary.

OPERATION

Stairs

If you need to move the mobility scooter over stairs, always ask for assistance from two people - one at the front and one at the rear (Picture 14). Disassemble the scooter into smaller parts (seat, battery pack, frames) for easier handling. When lifting the fully or partially assembled scooter:

1. The person at the back should tilt the scooter slightly and firmly hold the rear frame or handles.
2. The person at the front should grasp the lower part of the front frame, never the basket or tiller.
3. Carefully lift and step one stair at a time. Do not lower the scooter until the front person reaches the next step.

Going downstairs follows the same steps in reverse.

WARNING

Never attempt to take the scooter up or down stairs while riding it.

WARNING

The mobility scooter is not intended to be used as a seat in vehicles. Always transfer to a proper car seat and secure the scooter separately.

WARNING

Avoid unsafe lifting methods or unsupported handling, as shown in the incorrect example (if applicable).

OPERATION

Tyre

Both the front and rear wheels of the mobility scooter are solid PU tyres, each with a diameter of 190 mm. These tyres are maintenance-free and do not require inflation.

WARNING

Do not attempt to modify or replace the wheels with pneumatic tyres. Always inspect the tyres for visible damage or wear (see chapter “MAINTENANCE”). If any issues are noticed, contact the service provider immediately.

BASIC TROUBLESHOOTING

If the **battery indicator light** on the display starts **flashing**, your mobility scooter has detected a system fault. Follow the steps below to identify and report the issue:

1. Release the throttle lever (wigwag paddle), then switch the scooter OFF and ON again.
 - If the error clears, continue regular use.
 - If the error persists, proceed to step 2.
2. Switch the scooter OFF and fully recharge the battery.
 - After charging, turn the scooter back ON.
 - If the error still appears, continue to step 3.
3. Count the number of flashes on the battery symbol.
 - The light will flash a number of times, then pause.
 - Count the number of flashes between pauses, and report this code to your dealer or service technician.

Flash Code Reference Table:

Flash Count	Description	Meaning & Action
1	Battery Low	Battery charge is running low. Recharge the batteries.
2	Low Battery Fault	Batteries are empty. Recharge the batteries. Check battery condition, connections, and wiring.
3	High Battery Fault	Battery voltage is too high - may occur if overcharged or descending a long slope. Reduce speed during downhill travel.
4	Overcurrent Timeout / Overheat	Motor may have stalled or overheated. Turn scooter OFF, wait a few minutes, then turn ON again. Check motor and wiring.
5	Park Brake Fault / Freewheel mode	Park brake may be released or faulty. Check brake, switch positions, and wiring. / Scooter in freewheel mode. Set lever to drive mode.
6	Drive Inhibit	A stop function is active, charger is connected, or throttle was not neutral during startup. Disconnect charger, check seat/throttle positions.
7	Throttle/Sensor Fault	Throttle, speed pot, or wiring may be faulty. Check components and connections.
8	Motor Voltage Fault	Possible motor fault. Inspect motor and all related wiring.
9	Controller/Internal Fault	Internal fault in controller. Check all wiring and connections. Contact service if unresolved.

i If the error persists after performing the steps above, do not attempt further operation. Contact your authorized dealer or service technician for inspection.

BATTERIES AND CHARGING

Before charging

WARNING

Read all battery and charger instructions before use. Use only the charger provided with your mobility scooter. Do not attempt to charge using an automotive or unapproved charger.

The battery condition meter on the tiller console shows the battery charge level using a color-coded scale:

Green: Fully charged

Yellow: Moderate charge (continue use with caution)

Red: Low charge - Recharge immediately

For the most accurate reading, check the battery meter while riding on a dry, level surface at full speed.

First charge

Your SKUTA mobility scooter is equipped with two 12V deep-cycle, sealed, maintenance-free batteries. Before first use, fully charge the batteries for at least **8 to 14 hours** using the supplied off-board charger.

Charging your scooter

The charger inhibit function disables scooter movement and meter display during charging. This is normal operation.

You can charge your scooter batteries in two ways:

Method 1 - Charging via Scooter Port (Picture 15):

1. Position the scooter close to a standard electrical outlet.
2. Turn the key switch OFF and remove the key.
3. Plug the 3-pin charger connector into the charging port on the tiller.
4. Plug the charger into the power outlet.
5. When charging is complete, unplug the charger from the outlet, then from the scooter.

BATTERIES AND CHARGING

Method 2 - Charging via Battery Pack (Off-Board) (Picture 16):

1. Lift the battery pack straight up and off the scooter (Picture 3).
2. Place the battery pack near a standard electrical outlet.
3. Plug the charger into the charging port on the battery pack.
4. Plug the charger into the wall outlet.
5. Once fully charged, unplug the charger from the wall and then from the battery.

Charging frequency & habits

Daily use: Charge the scooter after each day's use for 8 - 14 hours. For best performance, leave on for 4 extra hours after the charger indicates full.

Occasional use: If used less than once per week, charge the batteries once a week for at least 24 hours.

Monthly maintenance: Fully charge for 48 hours continuously once per month to maintain battery health and extend life.

Charger indicator lights

Red - Charging in progress

Green - Charging complete (95%) – Trickle charge active

WARNING

DO NOT:

Use extension cords. Charge in wet or unventilated areas. Allow unsupervised children near the charger. Charge the scooter while it is occupied. Open the charger housing. Use damaged cables or connectors. Expose non-weatherproof chargers to rain or snow.

CAUTION

Always - plug the charger directly into a grounded, 3-prong outlet. Use only the supplied charger. Allow the charger to cool before storing if hot. Inspect all charger components before use. Contact your dealer if damage is found.

MAINTENANCE

To ensure safe and reliable operation, carry out the following checks and maintenance regularly or have it carried out by another person. Below You will find recommended maintenance’s schedule.

	Before use	Once a week	Once a month	Once a year
Check brakes and test braking on level surface	X			
Check throttle and tiller cable for damage	X			
Check for flat spots on solid tires	X			
Check armrests for looseness or damage	X			
Check wiring harness, connectors, and insulation		X		
Check charger and controller connectors for corrosion		X		
Check tire pressure (if pneumatic)		X		
Inspect anti-tip wheels for wear and proper clearance			X	
Inspect tire wear and replace as needed			X	
Clean scooter; remove dirt, hair, food, and debris			X	
Check and clean battery terminal connections			X	
Inspect and maintain battery charger and charging port			X	
Check and maintain exterior plastic and metal surfaces			X	
Have scooter fully serviced by authorized provider				X

WARNING

Repairs, cleaning, or adjustments to the mobility scooter must be performed by an authorized technician while the scooter is powered off and unoccupied.

Do not service wheels or electrical components without disconnecting power and removing the key.

CLEANING AND DISINFECTION

Use a damp cloth and mild, non-abrasive detergent to clean the frame, tiller, and trim. Avoid scratching surfaces. Do not use aggressive chemicals or rough cleaning tools. Tires and bumpers may benefit from a light application of rubber or vinyl conditioner. Clean wheels with a wet synthetic fiber brush (never use a wire brush). For disinfection, use a tested, approved non-aggressive disinfectant. Spray or wipe as instructed by the product manufacturer. Allow all electronic areas (console, charger port, rear electronics) to dry thoroughly before using the scooter again.

- i** Sand and seawater can damage bearings and cause corrosion. Only expose the scooter to sand and seawater for short periods and clean it after every trip to the beach.
- i** If necessary, clean your product with an approved disinfectant. Make sure the disinfectant is safe for use on your product before application.
- i** Follow all safety instructions for the proper use of the disinfectant or cleaning agent before applying it to your product.

STORAGE

The mobility scooter should be stored in a dry, clean, and well-ventilated environment, protected from direct sunlight and temperature extremes. If storing for a prolonged period (more than 3 months), follow these recommendations:

- Fully charge the batteries before storage.
- Remove the battery pack from the scooter.
- Periodically recharge the batteries during storage to prevent permanent damage.
- If possible, raise the scooter off the ground using wooden boards to prevent flat spots on the tires.
- Keep the scooter clean and free of dust, using a duster or damp cloth and mild detergent.
- Only use non-abrasive, neutral cleaning products suitable for plastics and metals.
- Do not store any items on top of the scooter

WARNING

Do not expose the mobility scooter to temperatures above +50°C or below -10°C. Excessive heat can soften the tires and cause burns upon contact; freezing temperatures can crack tires and cause discomfort to the user. Do not store in environments with high humidity, as it may weaken the scooter's frame.

STORAGE

Environmental conditions

Keep scooter under these conditions:

- Temperature limitation –10°C to +50°C;
- Relative humidity 30% to 70%;
- Atmospheric pressure 86 kPa to 106 kPa.

RE-USE

The mobility scooter is suitable for re-use. Prior to reuse, the following actions must be performed:

Complete cleaning and disinfection (see chapter “CLEANING AND DISINFECTION”).

Technical inspection and safety check (see chapter “MAINTENANCE”). If necessary, replace batteries and worn parts.

The scooter should be adjusted to suit the new user in accordance with the user manual and service documentation.

DISPOSAL

Be environmentally responsible when disposing of the scooter. The product includes recyclable components such as metal, plastic, electronics, batteries, and foam materials. Follow all local and national disposal regulations. For proper disposal of the scooter and its parts, contact your authorized provider, local waste facility, or municipal recycling service.

INCIDENTS

Any serious incident that has occurred in relation to the device should be reported to UAB Kasko Group | Kid-Man® and the competent authority of the Member State in which the user and/or patient is established.

CONTACTS:

Manufacturer: UAB “Kasko Group” (registered trade mark Kid-Man®)

Address: A.Vivulskio 12D, Vilnius, LT-03221, Lithuania

Phone: +370 5 265 0000

E-mail: info@kid-man.com

Website: www.kid-man.com

Auto-fill contacts via QR code

SPECIFICATIONS

Model	SKUTA
Overall length (with basket) (cm)	112
Overall width (cm)	55 - 69
Overall height (cm)	104
Overall weight (kg)	46
Seat depth (cm)	38
Seat height (cm)	52 - 56
Seat width (cm)	43.5
Backrest height (cm)	37
Armrest to seat distance (cm)	20
Armrest length (cm)	26.5
Armrest width (cm)	4.5
Front/rear wheel diameter (cm)	19 / 19
Minimum turning radius (cm)	250
Maximum safe load (kg)	136
Maximum speed forward (km/h)	8
Obstacle climbing height (cm)	4.5
Minimum door width (cm)	90
Frame	Aluminium and plastic composite
Upholstery	PU/Nylon
Electrical system	24 V DC
Battery	2 x 12 V (connected in series) ; 24V 15Ah (C20) Gel battery.
Motor power (W)	270

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