STEELMAN SUPERB

04-055-3

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FI

EN Instruction for use

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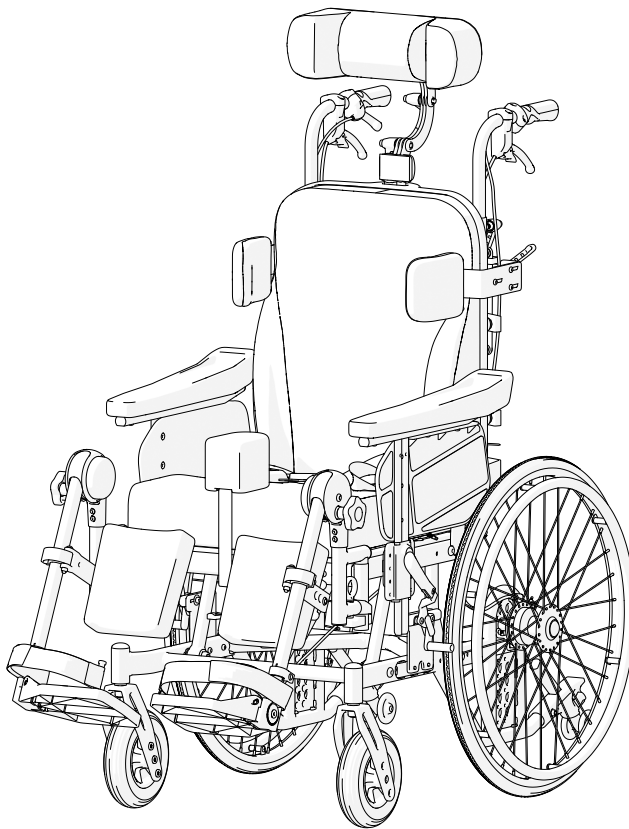
EN This Instruction for use **MUST** be given to the user of the product.

BEFORE using this product, this Instruction for use **MUST** be read and saved for future reference.

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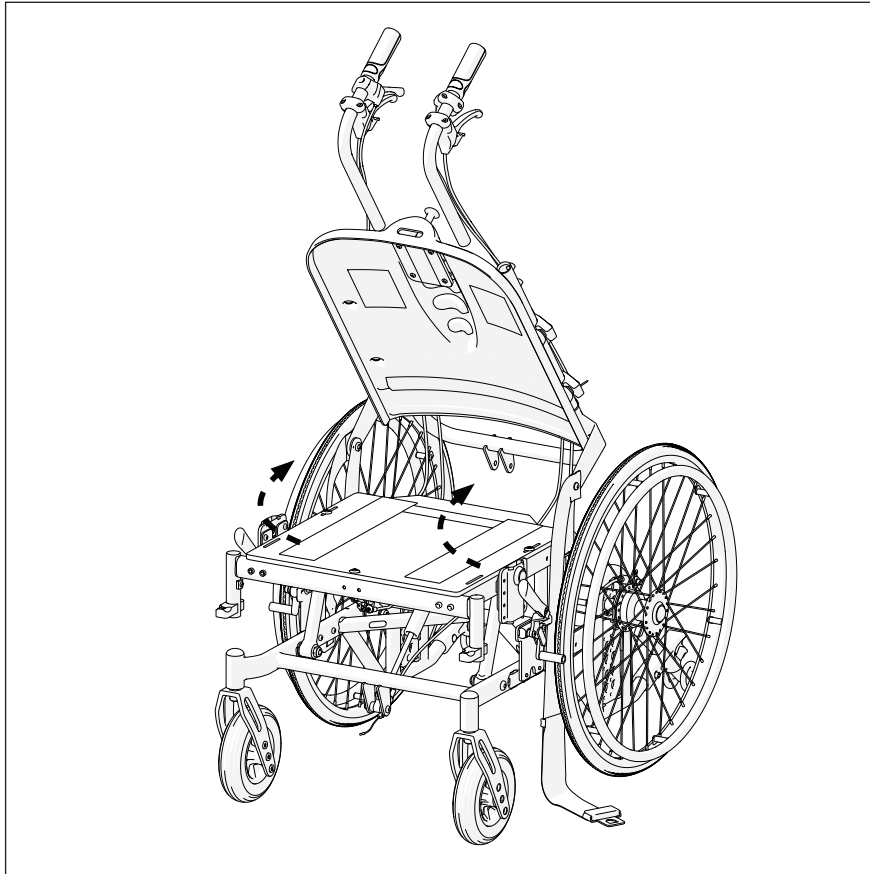
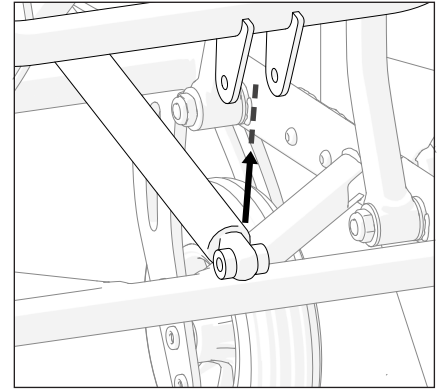
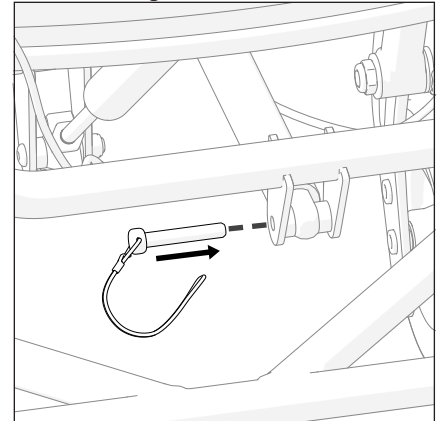
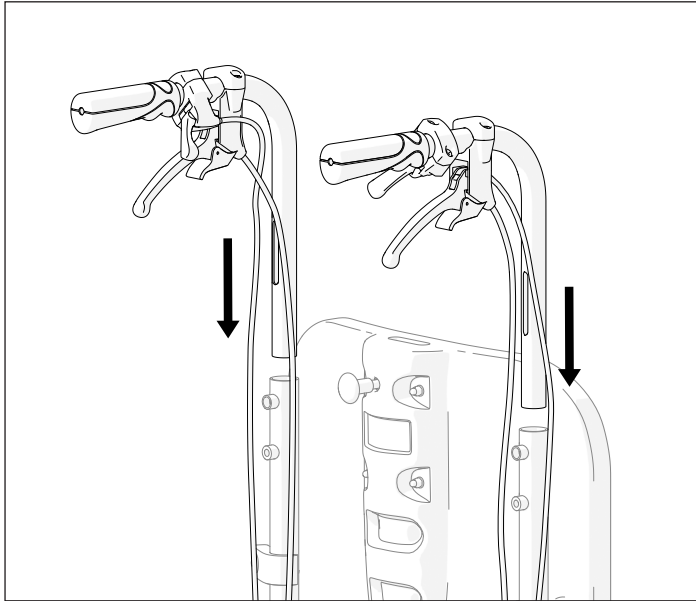
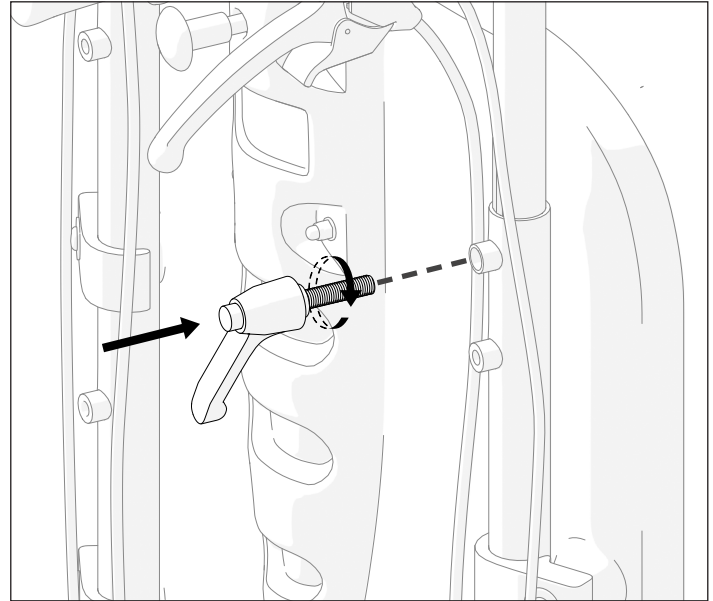
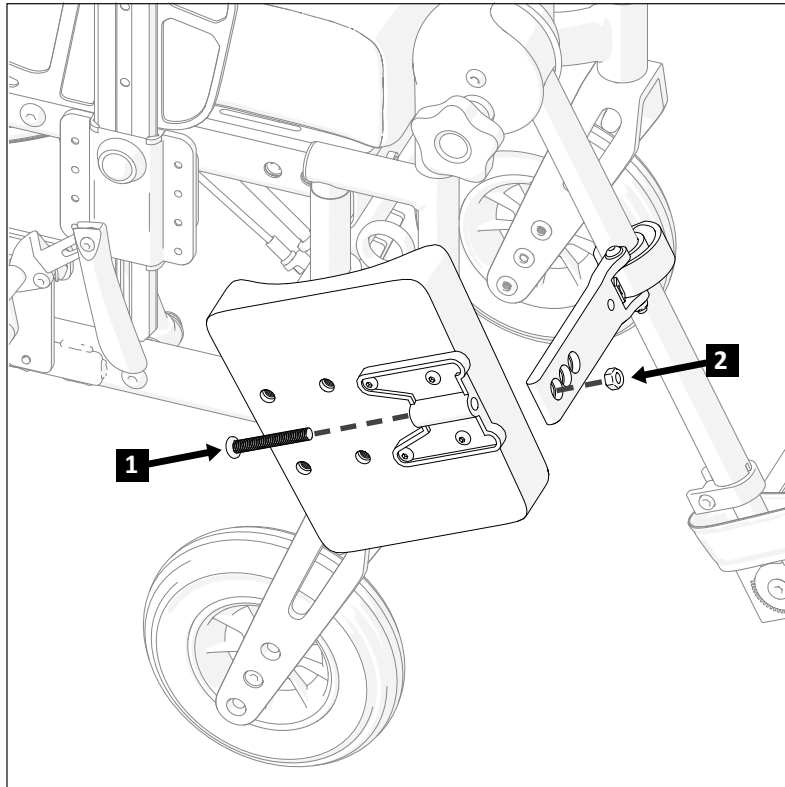
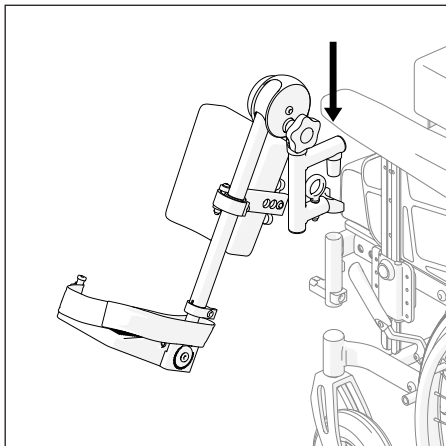
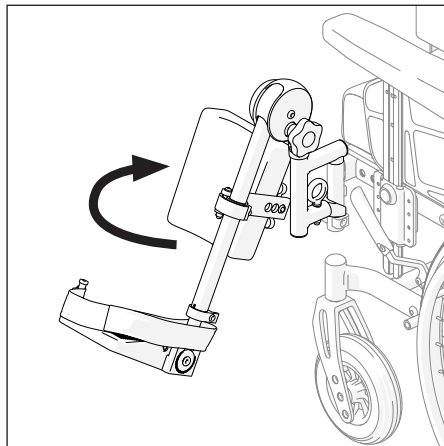
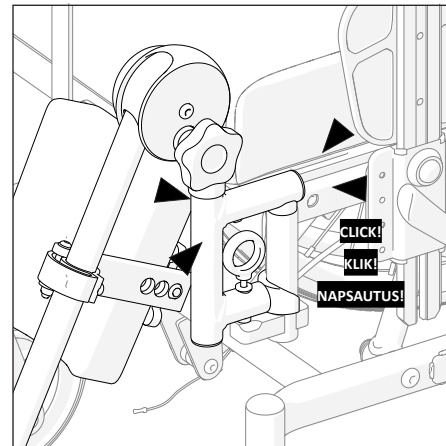
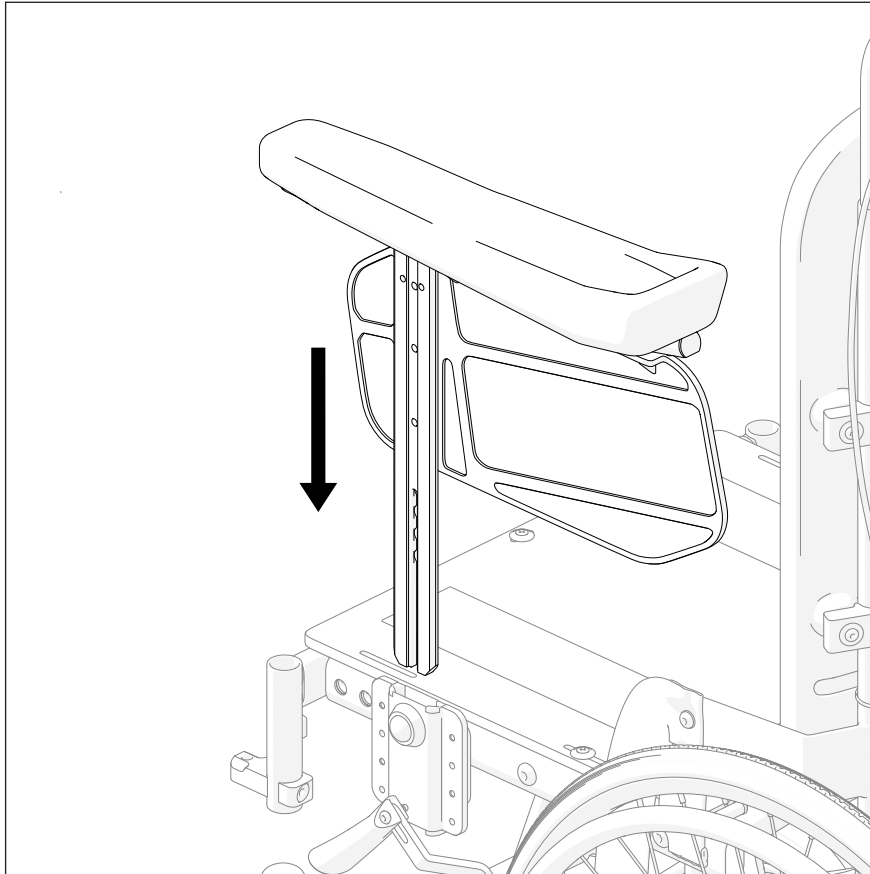
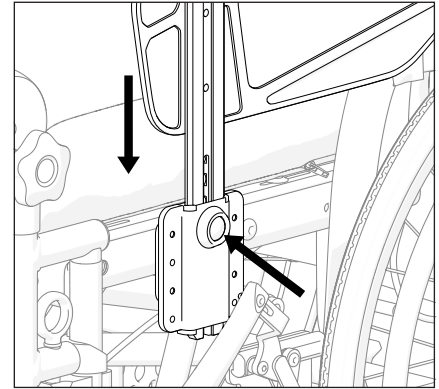
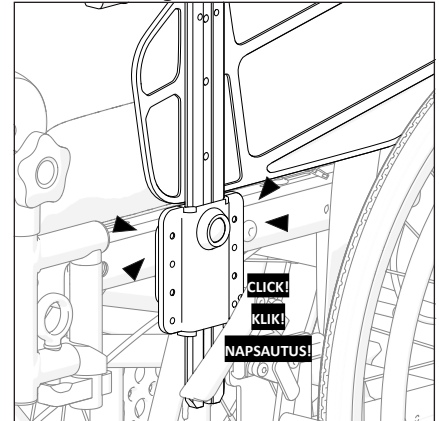
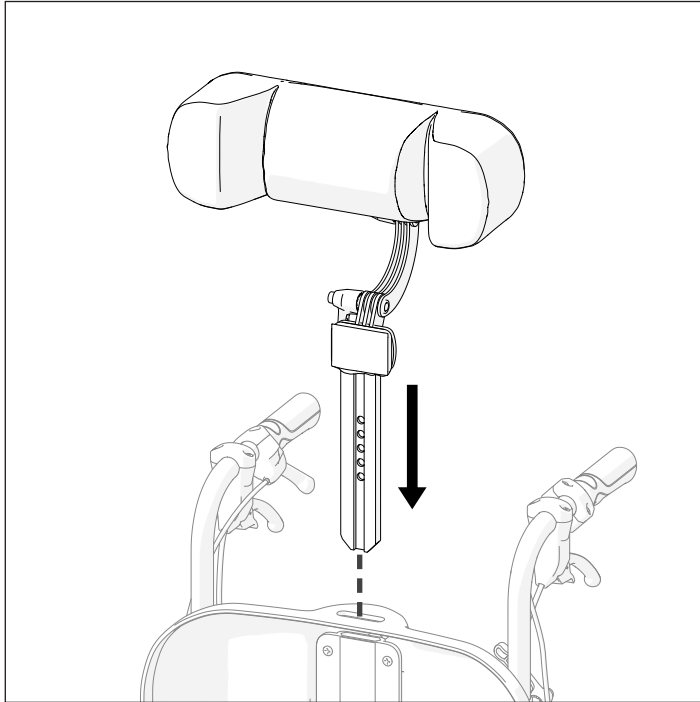
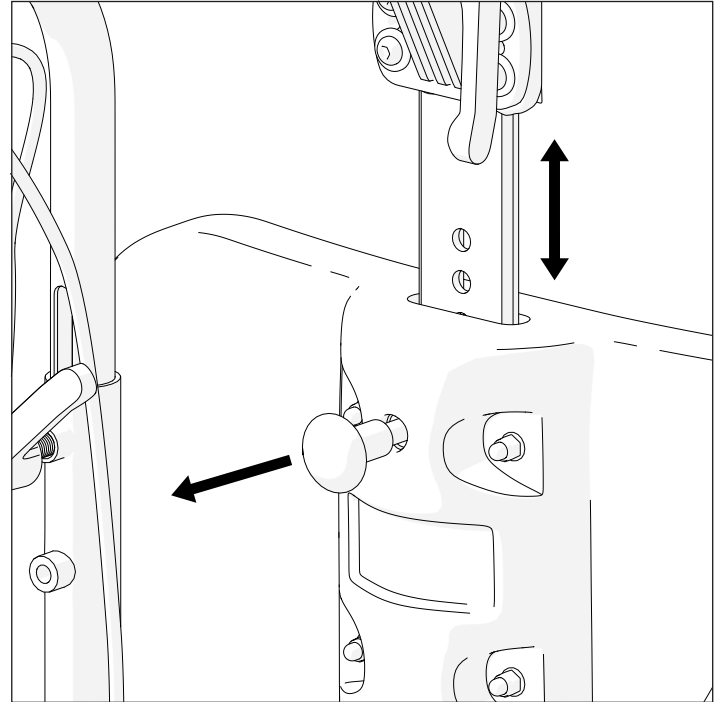
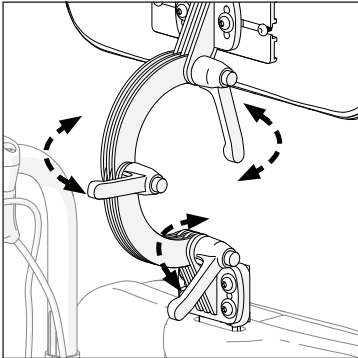
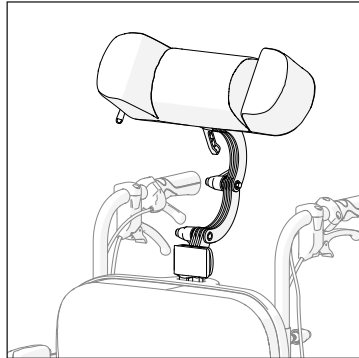
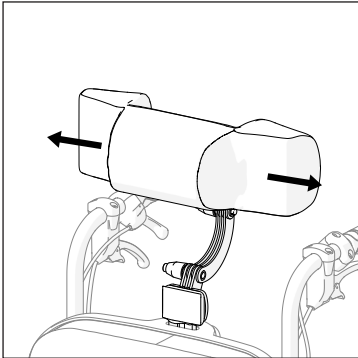
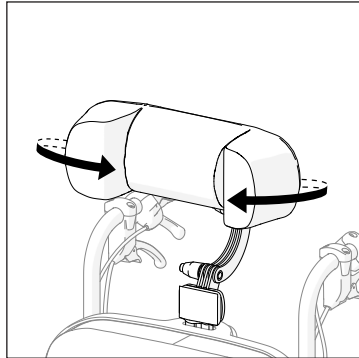
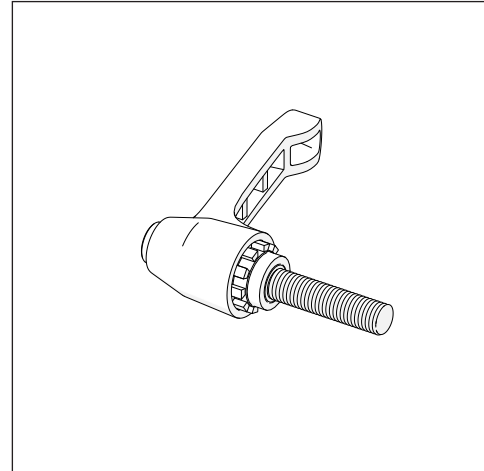
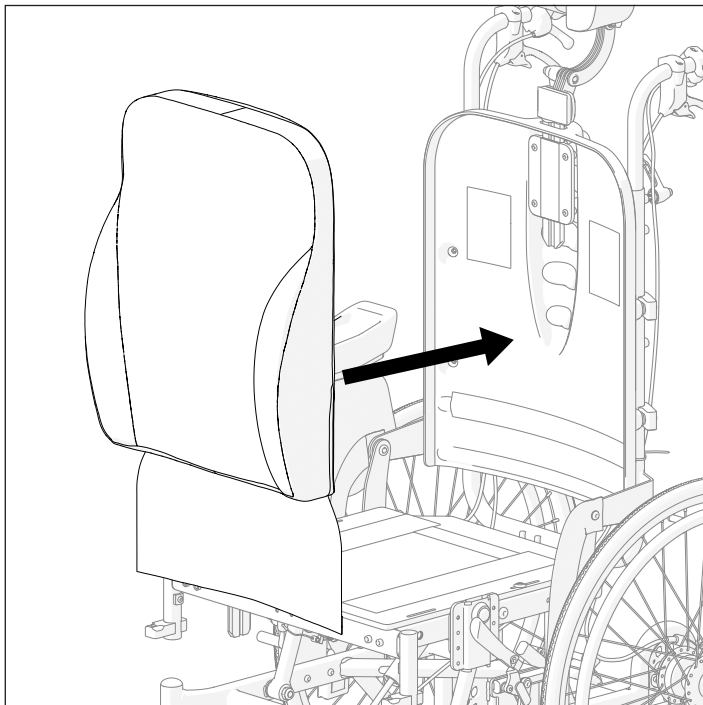
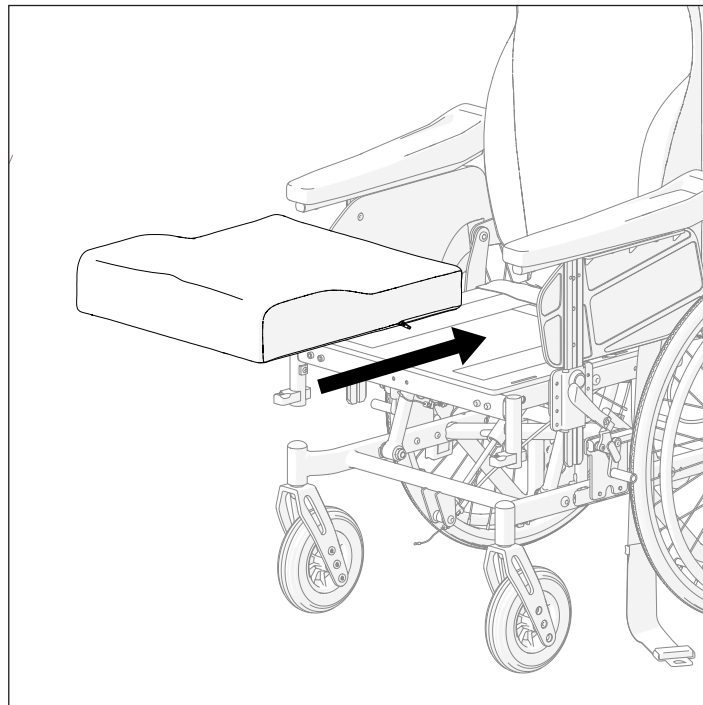
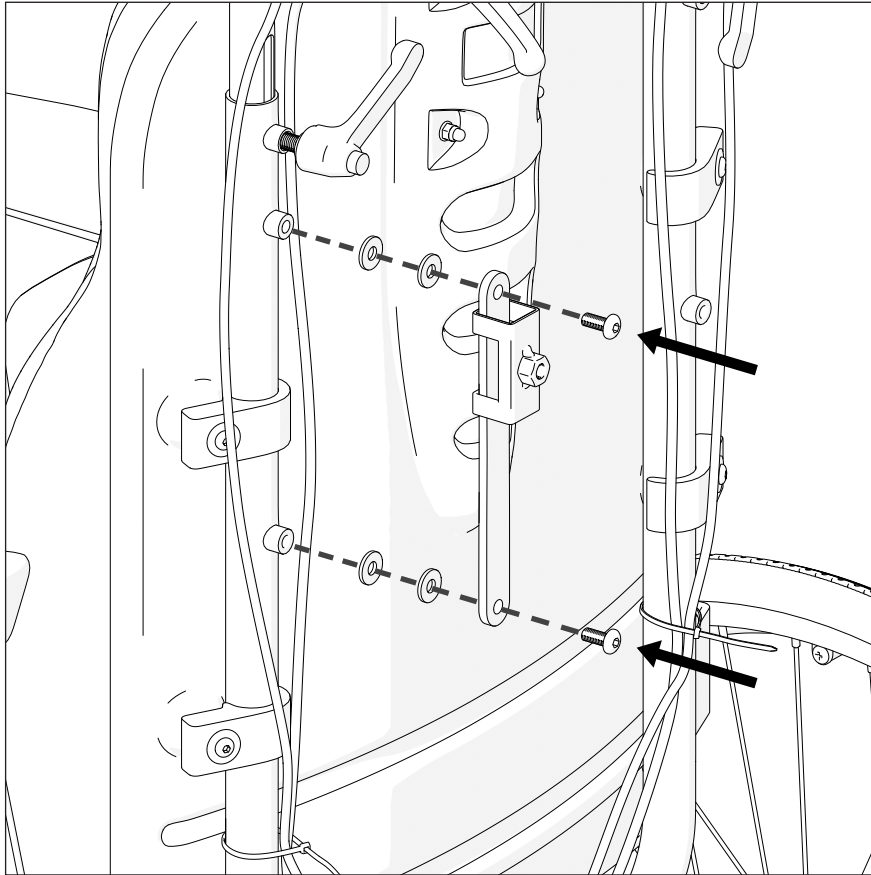
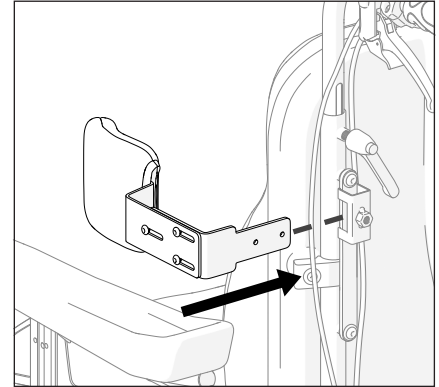
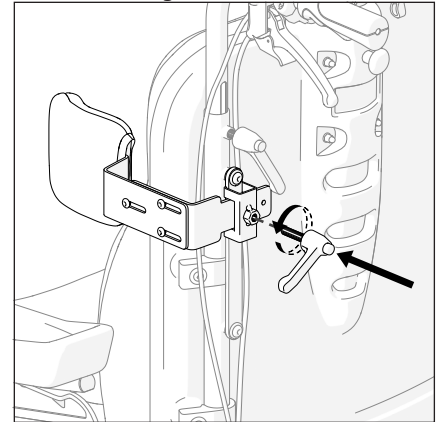
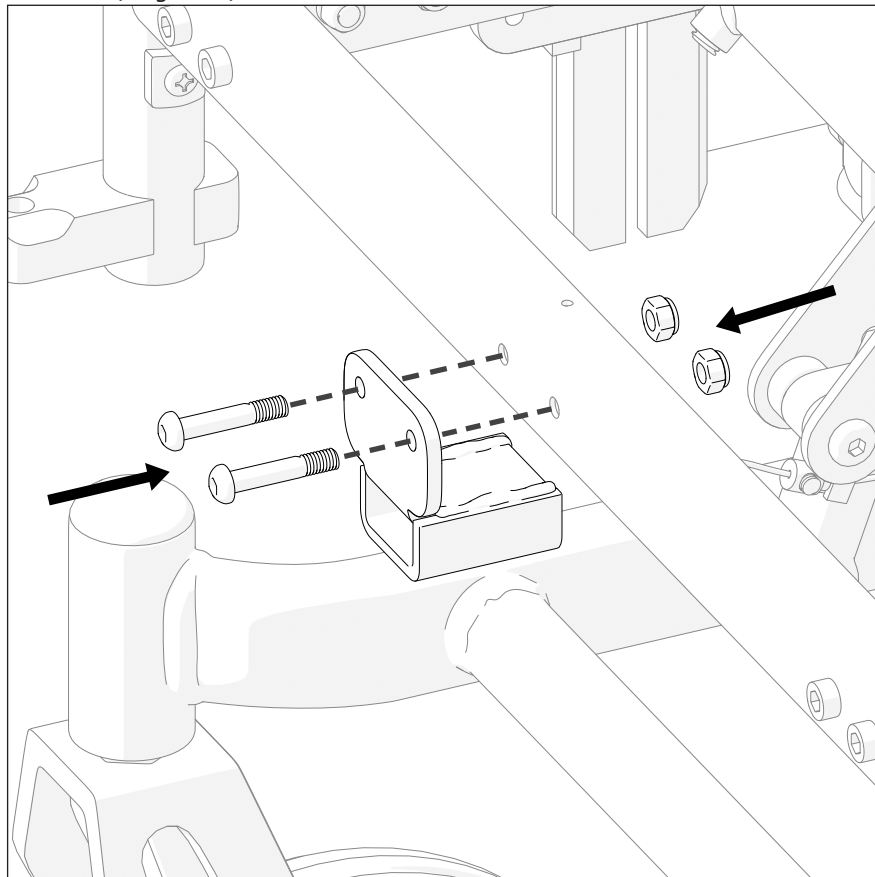
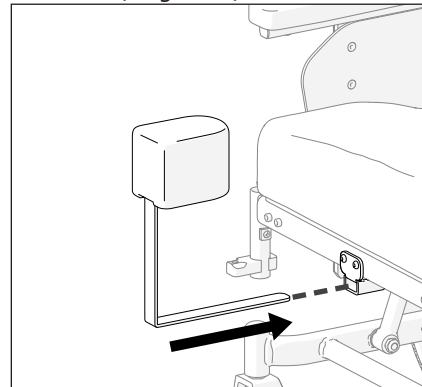
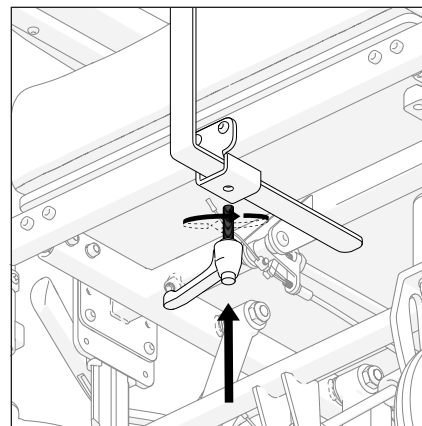
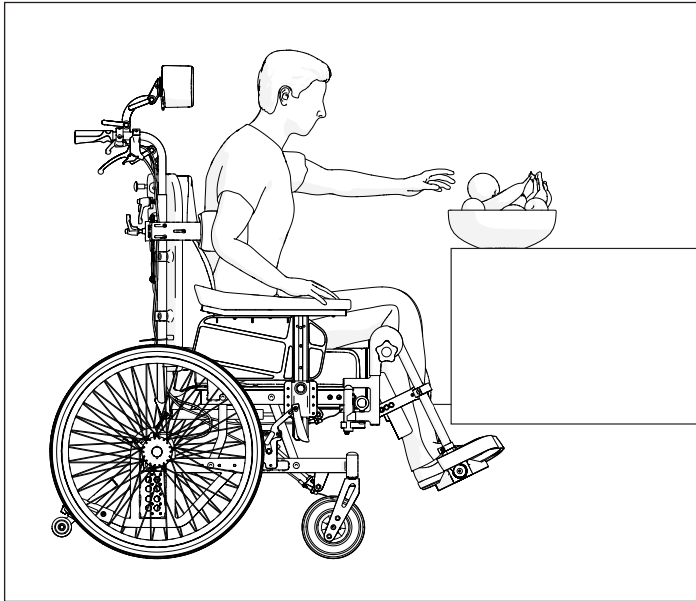
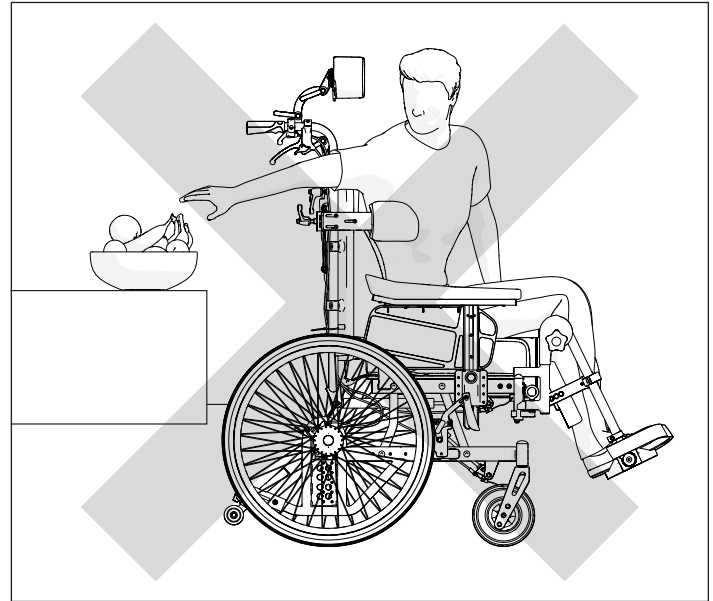
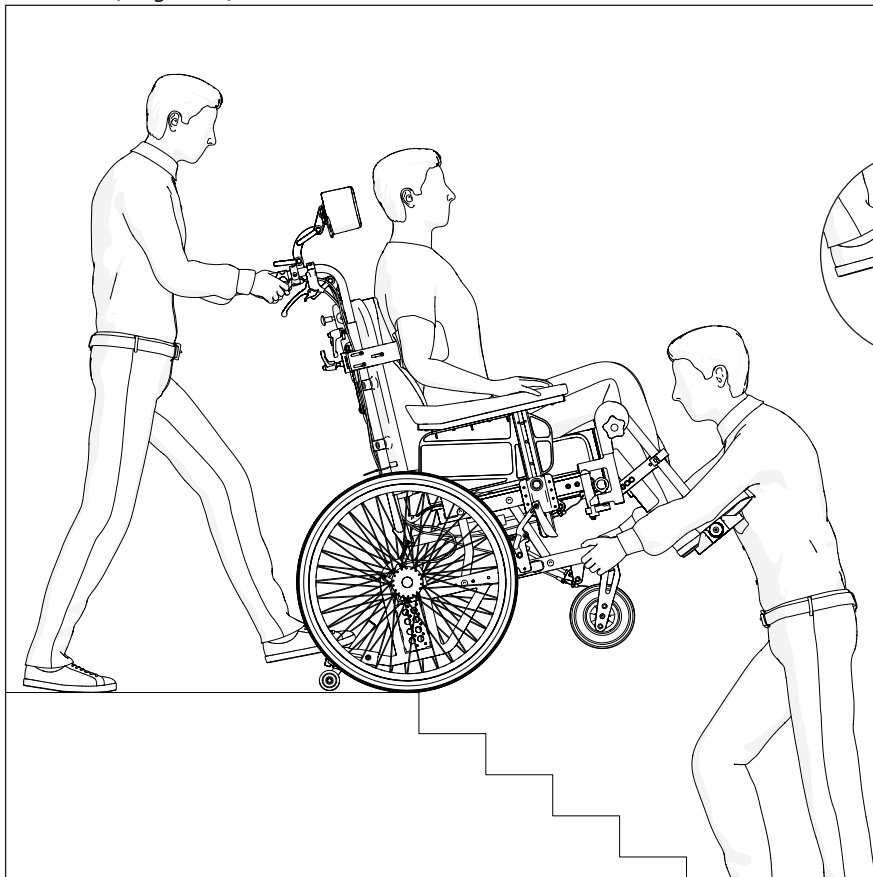
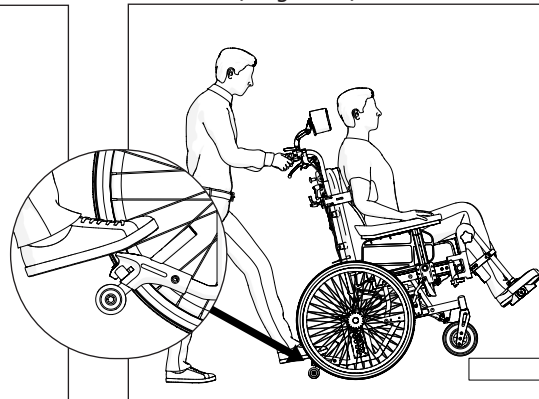
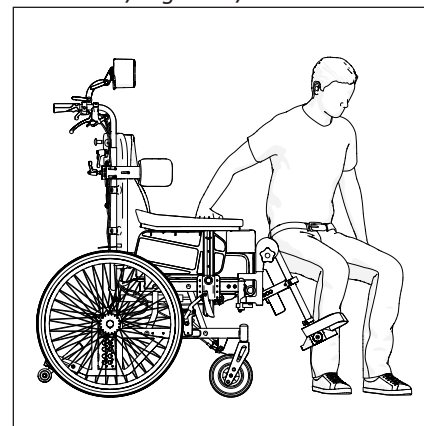
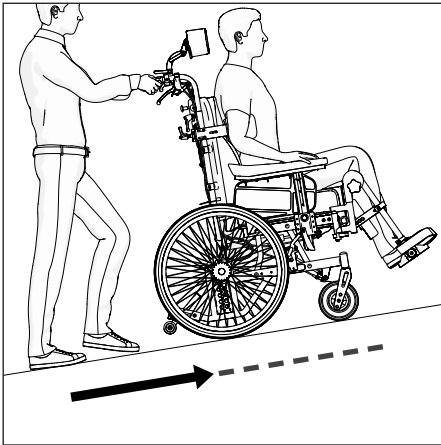
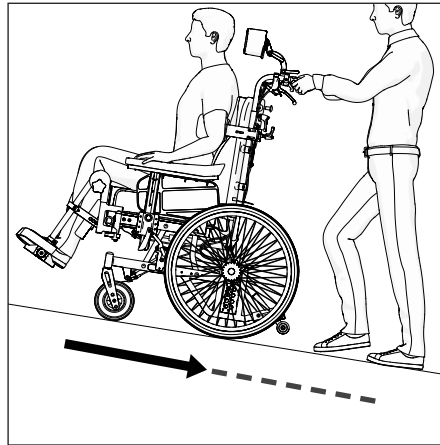
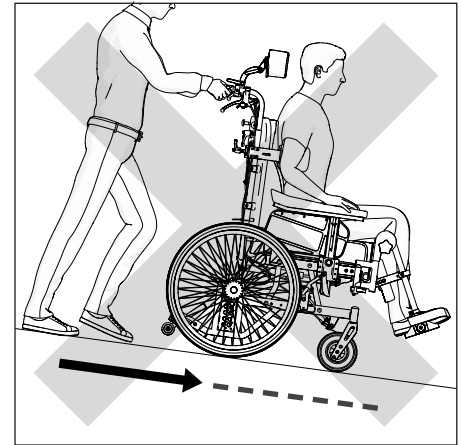
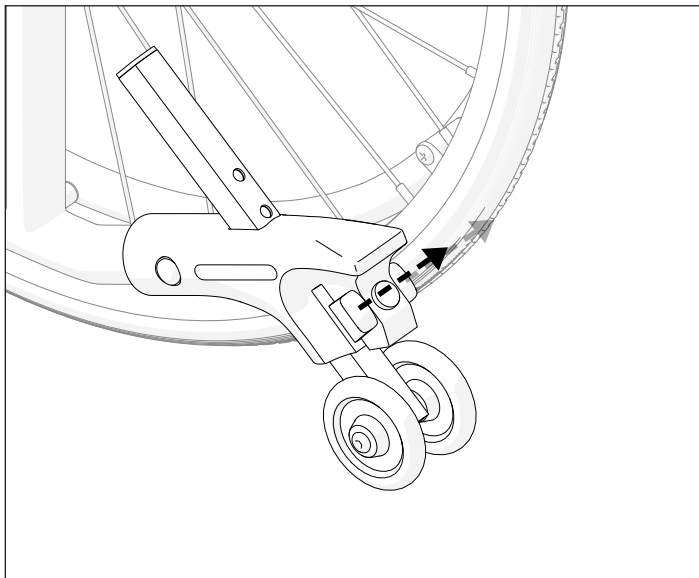
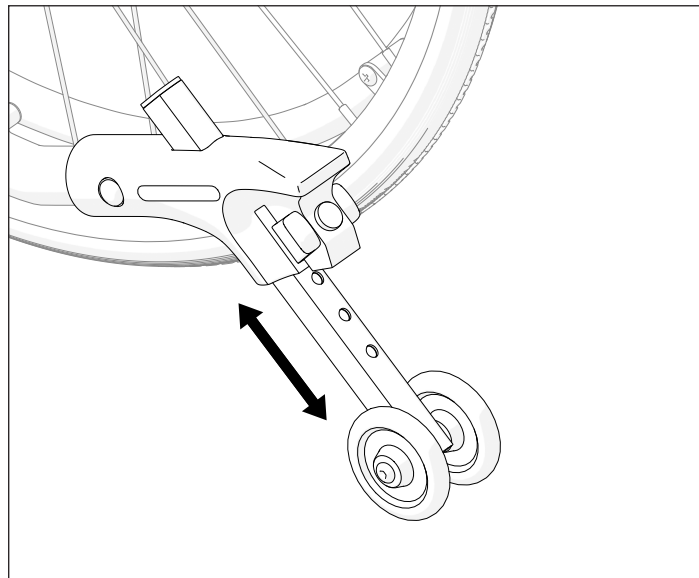
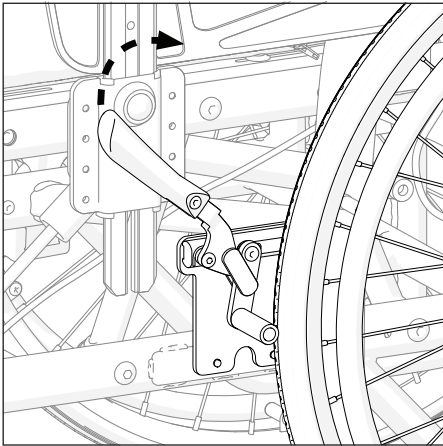
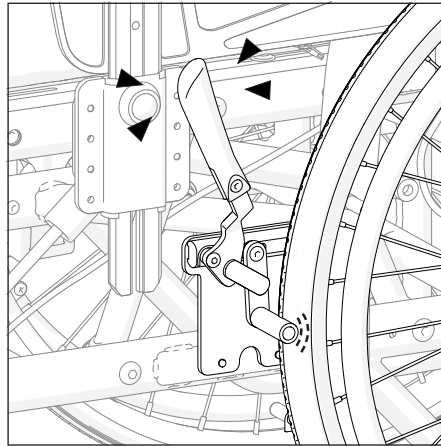
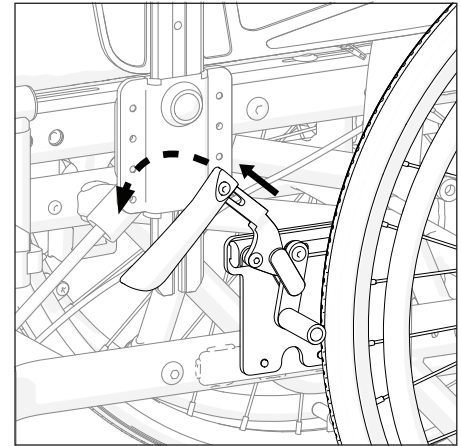
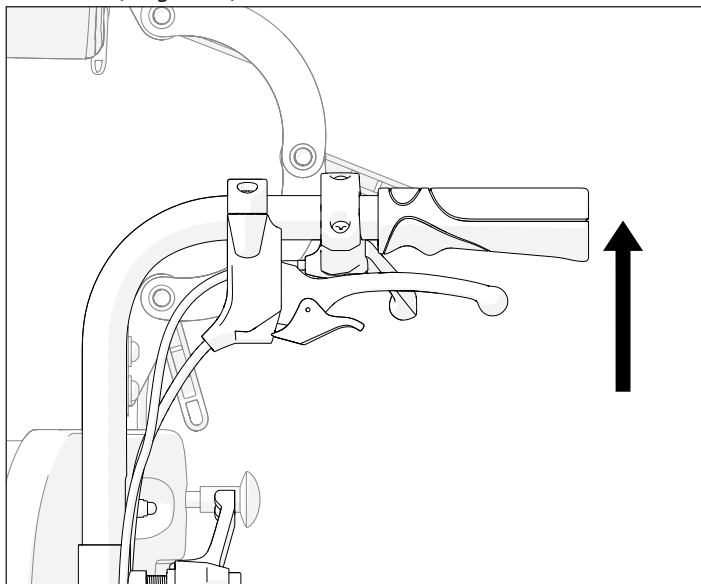
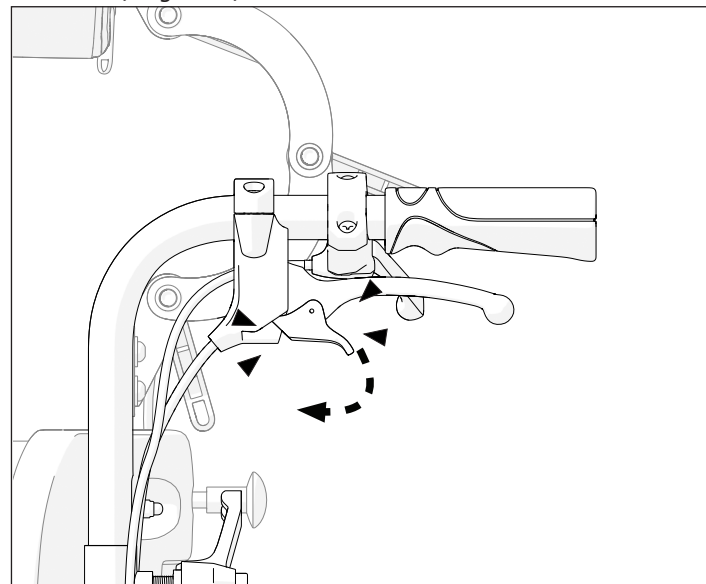
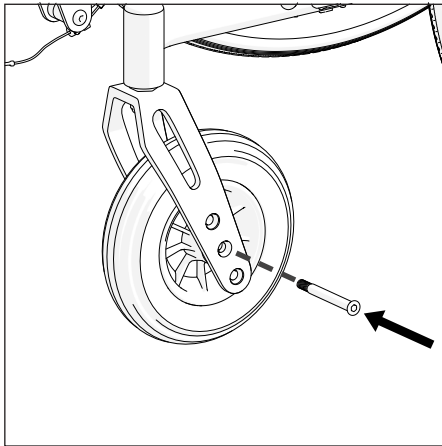
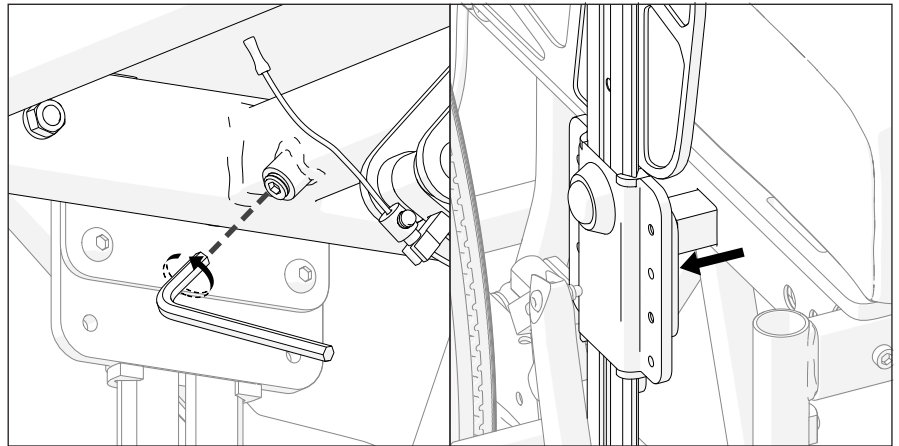
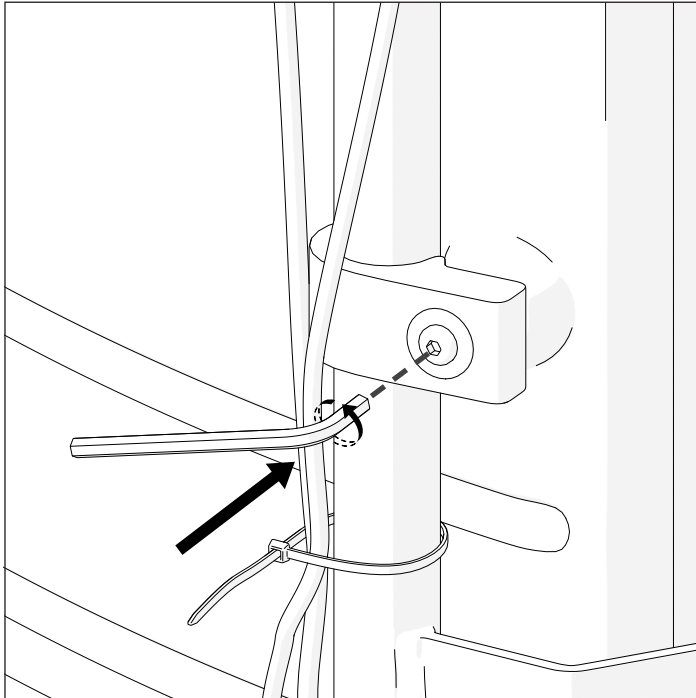
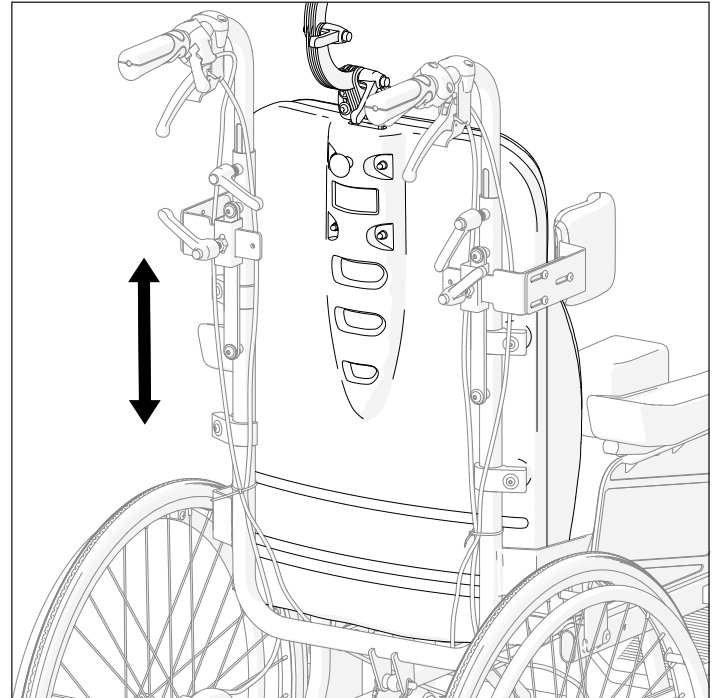
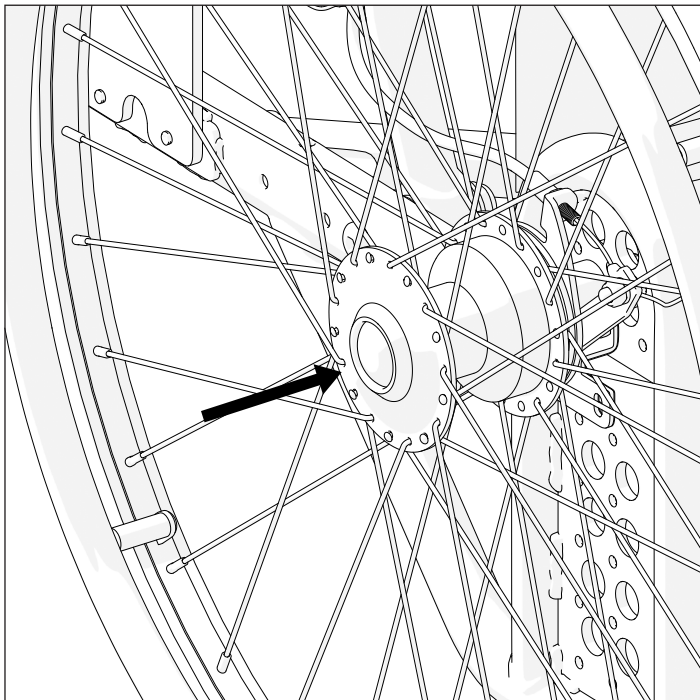
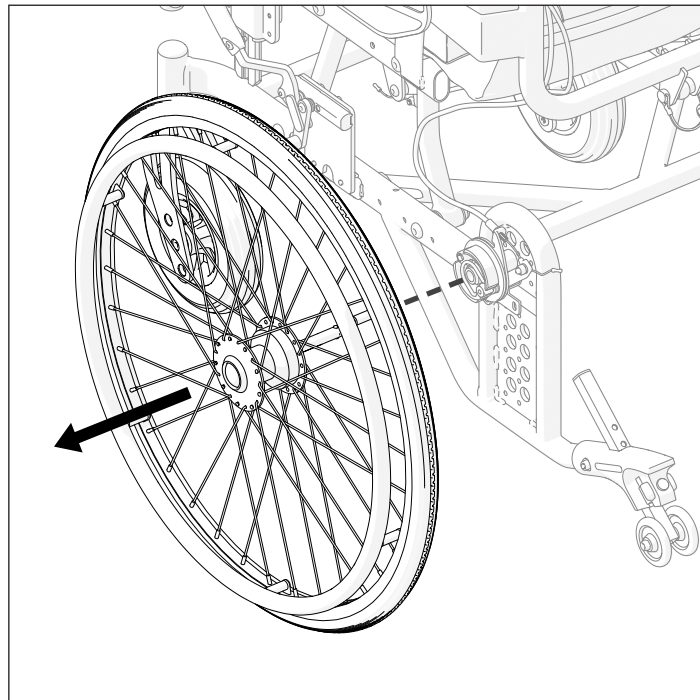
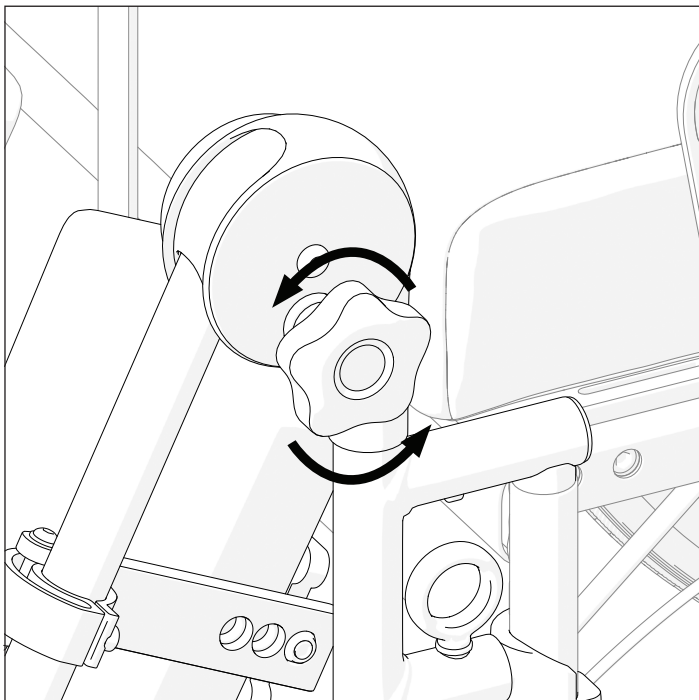
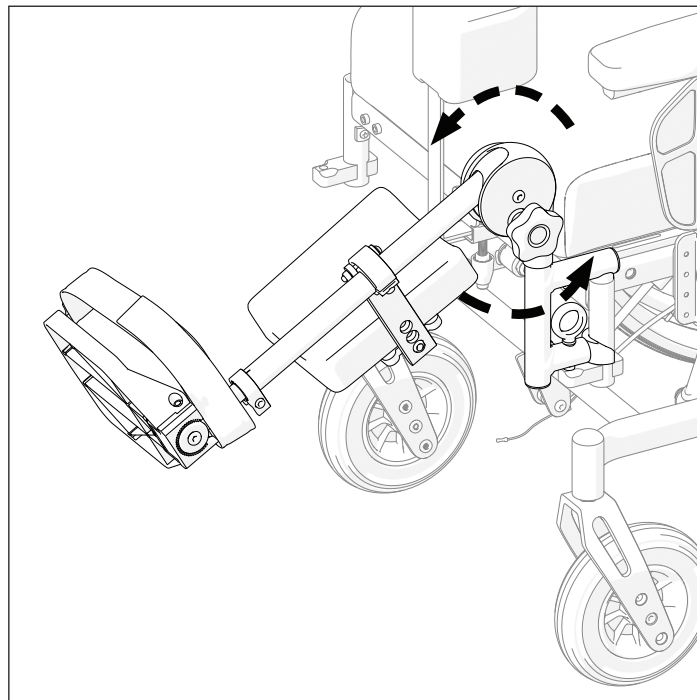
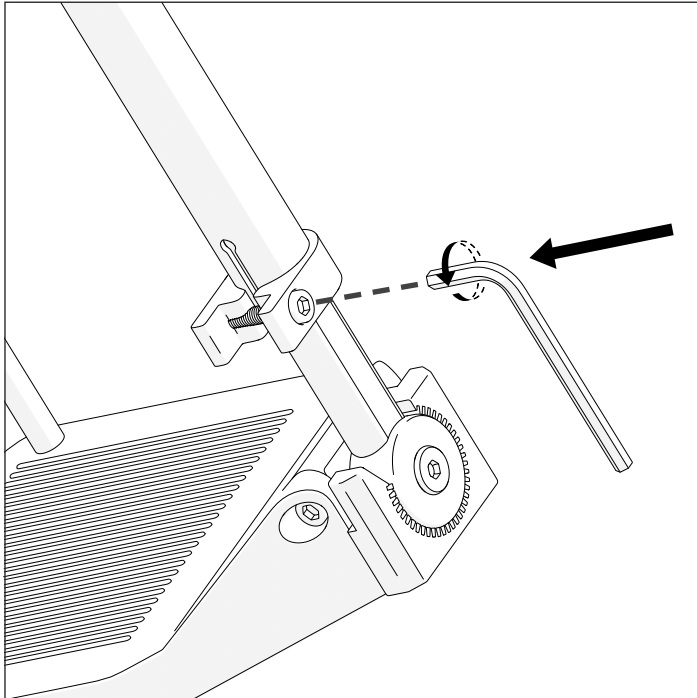
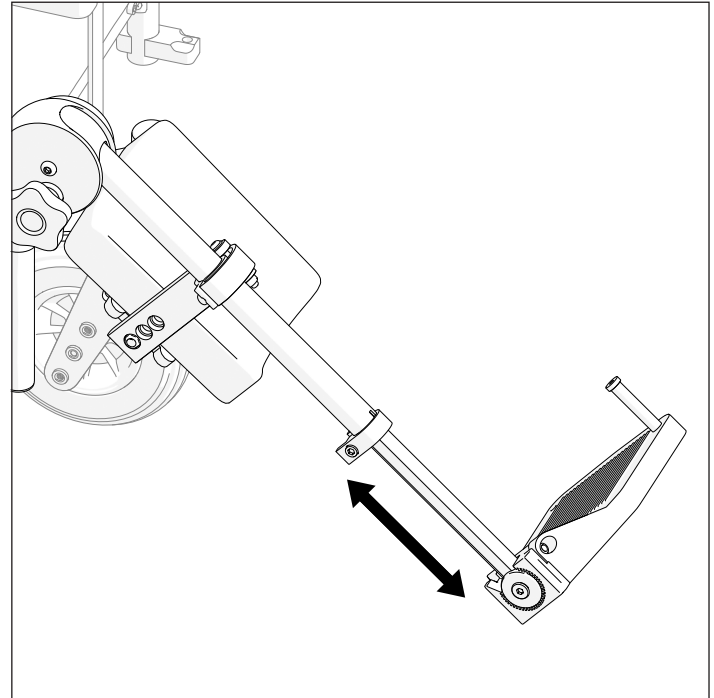
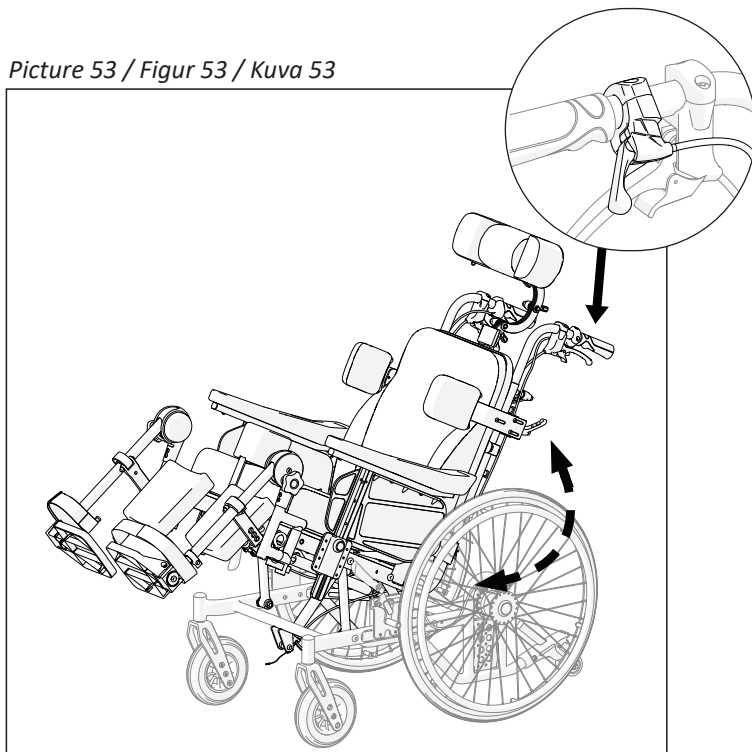
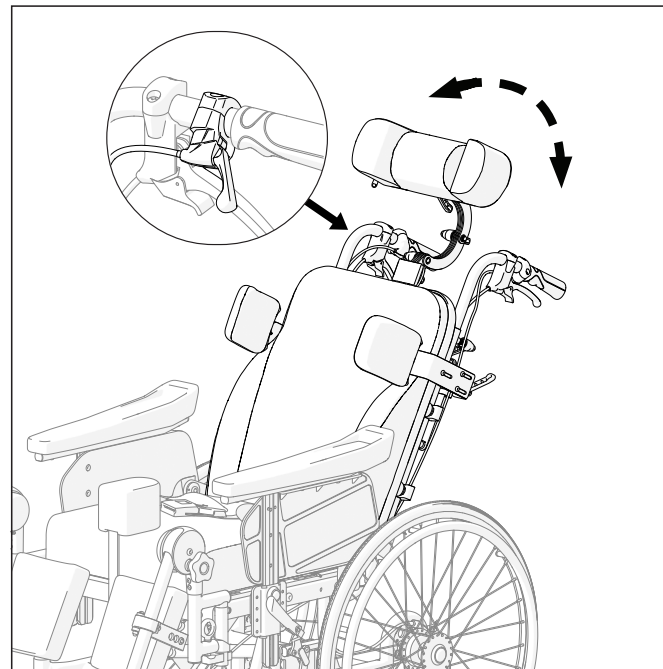
SYMBOLS

In this Instruction for Use warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger or basic information.

 Gives basic information, recommendations and information for efficient, trouble-free use.

 **WARNING / CAUTION** - to indicate that caution is necessary when operating the device. To indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.

 **ATTENTION!** This sign is used for very important parts concerning your safety. Please look at the operating instructions.

SAFETY INFORMATION

 **WARNING**
Any other or incorrect use could lead to hazardous situation.

 **WARNING**
Risk of accidents and serious injury. Accidents with resulting serious injury can occur if the wheelchair is wrongly adjusted. Adjustments at the wheelchair always must be carried out by an authorized dealer.

 **WARNING**
Risk due to driving style being unsuitable for the conditions. There is a risk of skidding on wet ground, gravel or uneven terrain. Always adjust your speed and driving style to the conditions (weather, surface, individual ability, etc.).

 **WARNING**
Risk of injury. In a collision you could sustain injury to parts of your body that extend beyond the wheelchair (e.g. feet or hands). Avoid an unbraked collision; Never drive into an object head-on; Drive carefully through narrow passages.

SAFETY INFORMATION

WARNING

Risk due to wheelchair being out of control. At high speed you could lose control of your wheelchair and overturn. Never exceed a speed of 7 km/h. Avoid collisions in general. Do not use escalators. Do not reverse down a gradient.

CAUTION

Risk of burning. The wheelchair components can heat up when exposed to external sources of heat. Do not expose the wheelchair to strong sunlight before use. Before usage, check all components that come into contact with your skin for their temperature.

CAUTION

Risk of getting fingers caught. There is always a risk of getting, e.g. fingers or arms, caught in the moving parts of the wheelchair.

WARNING

Risk of accidents. Safety devices that are incorrectly set or no longer working (brakes, anti tipper) can cause accidents. Always check that the safety devices are working before you use the wheelchair and have them checked regularly by the authorized dealer.

WARNING

Risk of accidents. For proper and safe use, we recommend you to follow safety instructions while using your wheelchair. This wheelchair was designed to ensure stability while seated. Every movement on the wheelchair affects the point of gravity, which may cause the wheelchair to tip over.

WARNING

Risk of choke. Please protect children under the age of the five years from easy access to wheelchair. Due to small elements used in the construction, the children can swallow them or choke.

SAFETY INFORMATION**CAUTION**

The wheelchair is only suitable for single occupancy.

**CAUTION**

Keep your feet on the footplates when moving. Do not stand on the footplates. Maintain proper balance at all times. Users should not move their centre of gravity out of the seating area.

**CAUTION**

We recommend you consult your healthcare professional for advice about transferring to and from the wheelchair. The parking brakes should always be applied when transferring. The parking brakes are not suitable for slowing the wheelchair down.

**CAUTION**

Be aware of hazards in your environment, such as narrow doorways, steps, household appliances, children's toys, etc. Do not reach for items further than your arm will extend.

**WARNING**

Risk of accidents. Do not attempt to move up or down an incline with a water, ice or oil film. Do not attempt to stop the wheelchair while on a sloped surface.

**WARNING**

Risk of accidents. Do not attempt to ride over curbs or obstacles. Doing so may cause your wheelchair to tip over and cause bodily harm to you or damage to the wheelchair.

**WARNING**

Risk of accidents. Never leave an unoccupied wheelchair on an incline or other place which may cause injury to surrounded people or things.

The expected service life is 5 (five) years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this Instruction for Use. After then, wheelchair has to be recycled. Do not throw your wheelchair away with common waste (see chapter "DISPOSAL").

GENERAL

INTRODUCTION

This Instruction for Use contains important information regarding the proper use of the product. To ensure safe use of the product, please read this manual carefully and follow the safety instructions provided.

Please note that this document may include sections that are not relevant to your specific product, as it applies to all currently available models.

Unless otherwise stated, each section in this document applies to all product models.

PREFACE

Thank you for the confidence shown in Kid-Man®. Every product is factory checked and left us in faultless condition. We wish you much success and we hope UAB Kasko Group | Kid-Man® wheelchair will make a positive impact to your daily living.

WARRANTY

We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB Kasko Group | Kid-Man® (hereinafter called Manufacturer) after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, Manufacturer, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

LIMITATION OF LIABILITY

Manufacturer accepts no liability for damage arising from:

- Non-compliance with the Instruction for use;
- Incorrect use;
- Natural wear and tear;
- Incorrect assembly or set-up by the purchaser or a third-party technical modifications;
- The usage of unapproved 3rd party accessories instead of approved Manufacturer's accessories;
- Unauthorized modifications and/or use of unsuitable spare part.

STANDARDS AND REGULATIONS

The wheelchair complies the requirements of:

- a) standard EN 12183:2022 Manual wheelchairs - Requirements and test methods;
- b) REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009 and repealing Council Directives 90/385/EEC and 93/42/EEC.

SERVICE LIFE

The expected service life is 5 (five) years, presuming that the product is used daily and in accordance with safety instructions, maintenance instructions and intended use, stated in this Instruction for Use.

After then, wheelchair has to be recycled. Do not throw your wheelchair away with common waste (see chapter "DISPOSAL").

INFORMATION ABOUT MANUFACTURER

Manufacturer: UAB "Kasko Group" (registered trade mark Kid-Man®)

Address: A.Vivulskio 12D, Vilnius, LT-03221, Lithuania

Phone: +370 5 265 0000

E-mail: info@kid-man.com

Website: www.kid-man.com

INFORMATION ABOUT PRODUCT

Product: MULTIFUNCTIONAL WHEELCHAIR

Model: SteelMan SUPERB

REF: 04-055-3/39; 04-055-3/44; 04-055-3/49

INTENDED PURPOSE AND INDICATIONS

The SteelMan SUPERB wheelchair is designed for assisted transport of a person with temporary (e.g., fractures or injuries to the legs or feet) or long-term (e.g., paralysis, musculoskeletal conditions, neurological disorders, balance or gait impairments) mobility difficulties. It may be used indoors or outdoors.

This wheelchair **cannot be self-propelled**. Movement is performed **only by an attendant** using the push handles. Its multifunctional features - such as tilt, recline, and adjustable legrests, provide support for users who require enhanced stability, positioning, or postural assistance.

i This wheelchair needs to be subscribed and fit to your specific health condition.

INTENDED USERS

SteelMan SUPERB wheelchair is intended to provide mobility to persons ages 15 and up (adolescents and adults) with a maximum weight capacity of 135 kg. The maximum age to use a wheelchair is not defined and depends on individual.

WARNING

Risk of accidents.

It is forbidden to exceed the maximum safe load of wheelchair.

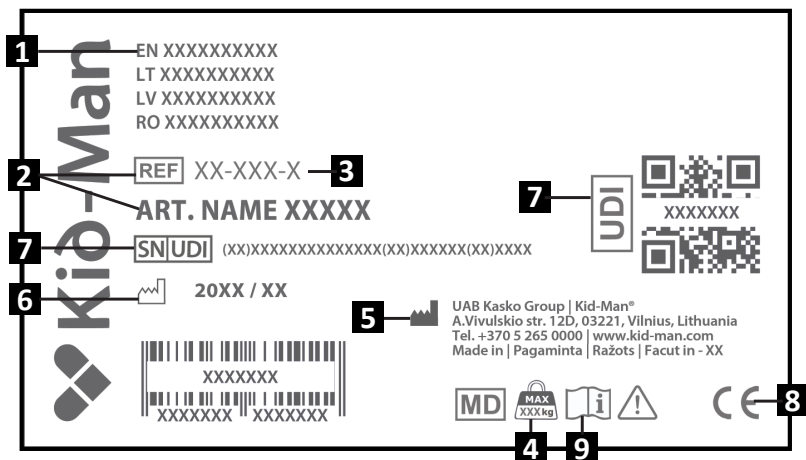
CONTRA-INDICATIONS

There are no contraindications known, when using the wheelchair as intended.

PRODUCT LABEL

The identification label is attached to the frame of your device (see picture).

1. Product
2. Model
3. Size (seat width)
4. Maximum load
5. Manufacturer / contact details
6. Date of manufacture
7. UDI / SN code
8. CE mark
9. Consult instructions for use


CONTENT OF PACKAGE

The wheelchair SteelMan SUPERB is delivered in a cardboard packaging and is ready for use. After unpacking, please keep the cardboard packaging if possible. You may need it later for storage or for the return of the wheelchair. Once parcel is received, please check whether it is not damaged during transportation.

- i** If you see signs of damage, please make note to transport company on the same time.
- i** Please check content of package and if any of below parts are missing, get in touch with your local dealer.

Content of package:

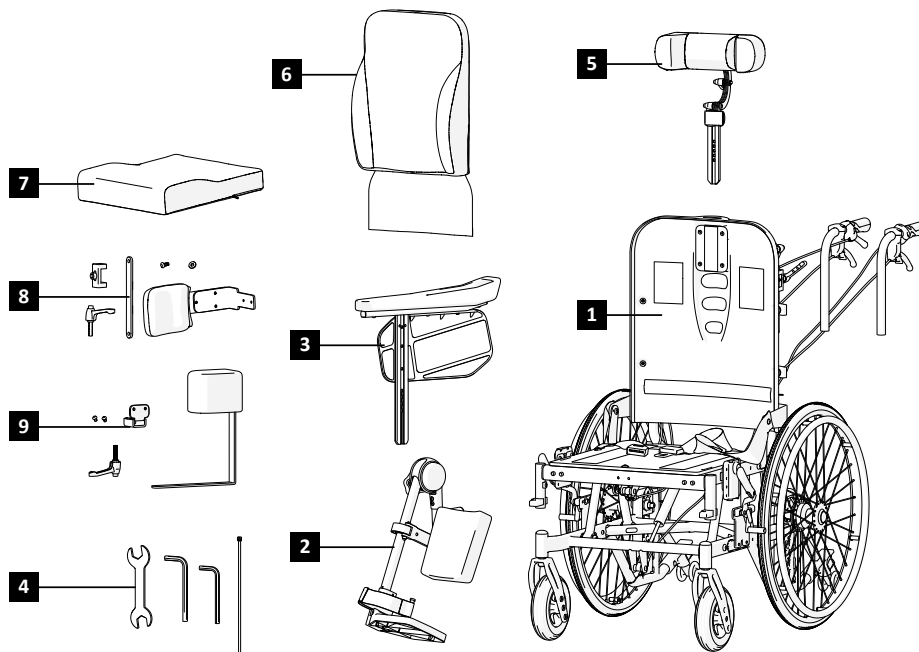1. 1 x Wheelchair frame with push handles and brakes
2. 1 x Pair of legrest
3. 1 x Pair of armrest
4. 1 x Adjustment tools kit
5. 1 x Headrest
6. 1 x Padded backrest
7. 1 x Padded seat
8. 1 x Side support (not included)
9. 1 x Abductor (not included)
10. Instruction for use


COMPONENTS AND FUNCTIONS

Overview

The wheelchair features ergonomic supports and an adjustable seat and backrest angle, which can be easily adjusted using the push handles.

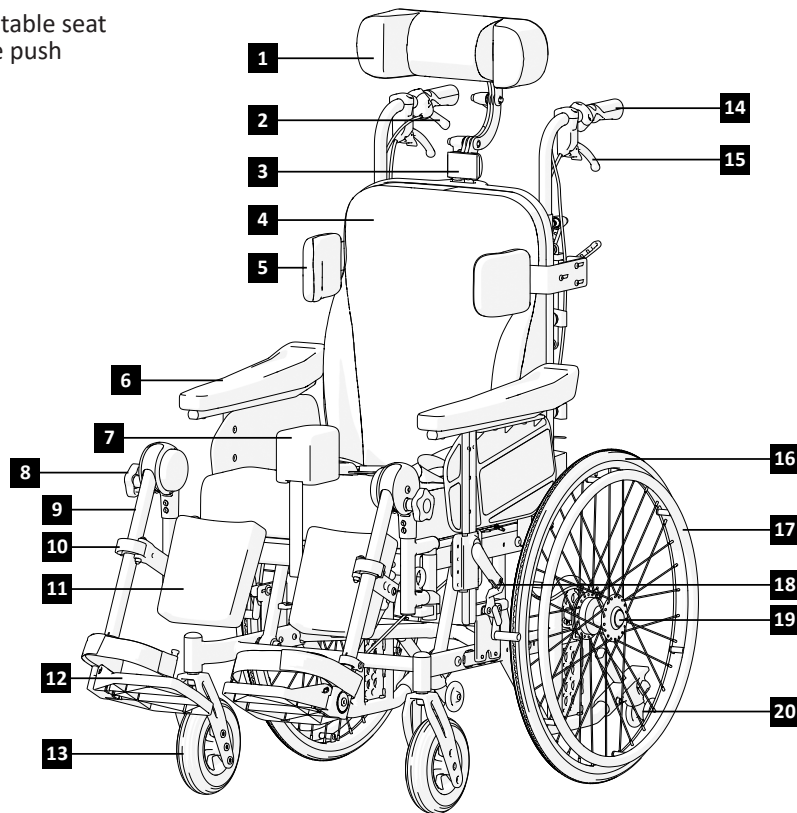1. Headrest
2. Angle control lever
3. Headrest holder
4. Backrest
5. Side supports (not included)
6. Armrest
7. Abductor (not included)
8. Legrest angle adjustment mechanism
9. Legrest
10. Legrest height locking mechanism
11. Calf support
12. Footplate
13. Front caster
14. Push handles
15. Brake levers
16. Rear wheel
17. Push rim
18. Parking brakes
19. Quick release axle
20. Antitiper

i Complexations of your wheelchair can differ a little from a wheelchair described in this manual due to continuous development of the product.

UNFOLDING AND FOLDING THE WHEELCHAIR

After unpacking the product, prepare the wheelchair for use by **following these steps**:

1. Place the wheelchair on a firm, level surface;
2. Adjust the backrest to the vertical position (Picture 1);
3. Lock the linchpin to secure the backrest (Pictures 2 - 3);
4. Insert the push handles with brake levers into their designated positions (Picture 4). Tighten it securely with adjustable ratchet levers (Picture 5);
5. Mount the calf supports onto the legrests. Use adjustment tools to tighten them (Picture 6);
6. Insert the legrests into their designated positions (Pictures 7 - 8). Turn the legrest inward to lock it in place - you should hear a 'click' (Picture 9). Legrest must stand tightly;
7. Lower the footplates into a comfortable position;
8. Insert the armrests into their designated positions (Picture 10). Press the metal button while inserting the armrest into place (Pictures 11 - 12). There should be a 'click' sound to ensure they are locked in place;
9. To insert the headrest, lift the knob on top of the backrest (Picture 14) and slide the headrest into the desired position (Picture 13). Adjust the angle with adjustable ratchet levers (Pictures 15 - 18);
10. Put on the soft cushions of the backrest and seat (Pictures 20 - 21).

Optional parts:

11. Attach side support bracket with given bolts and washers (Picture 22);
12. Insert the **side support** into the bracket (Picture 23);
13. Tighten with adjustable ratchet levers (Picture 24).
14. Attach the abductor bracket with given bolts and nuts (Picture 25);
15. Insert the **abductor** into the bracket (Picture 26);
16. Tighten with adjustable ratchet levers (Picture 27).

i **You don't need to use energy to much.** If You push too much You can damage the legrest or armrest (eg. break it).

To fold the wheelchair, first flip up the footplates and remove the legrests. To remove a legrest, pull its release ring and swing it outward, then lift the legrest off. While pressing the metal button, remove the armrest. Lower the push handles to their lowest position. Remove the headrest by lifting the adjustment knob. Finally, release the linchpin to allow the backrest to fold.

WARNING

Danger of trapping your finger! Be careful not to grasp the seat pipe.

WARNING

Never lift the wheelchair by its removable parts (armrests, legrests or cushion). Remove all detachable parts before lifting - see this Instruction for Use for removal steps. After removing the parts and folding the wheelchair, lift it using both hands on the frame only.

OPERATION

i This wheelchair is designed to provide stable seating, however, any change in the user's position or wheelchair adjustments can shift the center of gravity and may affect stability. To enhance safety, the seatbelt should be used whenever the wheelchair is in motion or when stability is uncertain.

WARNING

There are no positions of points intended to carry additional loads.

LEANING FORWARDS

WARNING

Risk of falling out. If you lean forwards out of the wheelchair you could fall out of it. Never bend too far forwards and do not shift forwards in your seat to reach an object. Do not bend forwards between your knees to pick up something off the floor.

REACHING AN OBJECT FROM THE WHEELCHAIR

Only the arms may be extended over the wheelchairs seat. The body and head should stay inside the limits of the seat (see Pictures 28 - 29).

MOVING ONTO THE SIDEWALK

We recommend going up onto the sidewalk with the help of an assistant. Steps for the assistant:

- Position the wheelchair straight, directly in front of the curb;
- Tilt the wheelchair backward and slowly move forward until the front wheels roll over the curb;
- Push the wheelchair straight forward until the rear wheels touch the curb;
- Lift the wheelchair slightly and push it forward until the rear wheels are on the sidewalk (Picture 31).

Note: if the anti-tipper prevents the wheelchair from tilting back far enough, we recommend temporarily removing it until the wheelchair is placed on the sidewalk.

Tilt wheelchair backward just when antitippers are removed.

TRANSFER

You can transfer on your own if you are strong enough. Position the wheelchair close to the target surface, with front wheels facing forward, and apply the brakes. Flip up the footrests and remove the armrest on the side you're transferring toward. Lean forward and use your hands on the lowered armrest and rear wheel to shift your weight toward the target (see Picture 32). Avoid leaning on the brake levers. A transfer board is recommended for added support.

If the transfer is too difficult, seek assistance. Never rely on the brakes or footplates for support while transferring or standing.

STAIRS

We suggest seeking help from two people, one in front of the wheelchair and one behind it, due to the fact that due to the difficulty and danger of hurdling stairs. The person behind the wheelchair, tips the wheelchair backwards until it is balanced. Then, pushes the wheelchair against the first step, grabbing the handle firmly, to prepare to lift the wheelchair. The person in front of the wheelchair grabs the sides of the frame in the front of the wheelchair and lifts the chair over the steps, while the other accompanying person steps on the next step, repeating the first process. Do not lower the wheelchair until you hurdled the last step of the person in front of the wheelchair. Carrying a wheelchair down the stairs is the same process as explained previously, in reversed order (Picture 30).

WARNING

Please do not grab wheelchair over the legrests, due to disassembly possibilities.

ASCENDING A RAMP

The assistant should maintain a steady, controlled pace when pushing the wheelchair uphill, keeping to a straight path. Sudden changes in direction must be avoided, and the ascent should never be attempted diagonally (Picture 33).

DESCENDING A RAMP

The assistant should regulate the wheelchair's pace by keeping firm control of the push handles, ensuring both speed and direction remain steady throughout (see Pictures 34).

ARMREST OPERATION

The wheelchair is equipped with a removable armrest. If it needs to be removed completely, press the metal button on the side of the armrest and lift it carefully while holding the button down (see Picture 11). Keep it pushed and lift armrest gently. To put it back, please repeat actions in opposite direction. The armrest is height-adjustable - press the metal button to change its position.

While operating wheelchair, armrests should be locked.

ANTI-TIPPER

SteelMan SUPERB wheelchair is equipped with anti-tip wheels. The purpose of the anti-tip device is to prevent the wheelchair from tipping backwards. It may also assist the caregiver in tilting the wheelchair. The height of the wheels can be adjusted. To do this, pull the red-coloured buttons towards you and slide the tube upwards or downwards along the bracket (Pictures 36 - 37) If you want to take wheels off, unscrew bolt, take it out and pull device.

BRAKING

SteelMan SUPERB is equipped with brakes on both sides. The parking brakes are engaged or released using a special handle located on both sides of the wheelchair (Picture 38). For improved comfort and easier access, the brake lever can be pivoted inward when not in use, as shown in Picture 40. To engage the parking brake, pull the handle towards you. To release the parking brake and allow movement, push the handle forward (Picture 39). For greater safety, we recommend always engaging both brakes simultaneously.

Due to normal wear and tear, the braking efficiency of the rear wheels may decrease over time. To ensure proper braking, you must adjust the brake position. To do this, use the provided tool to unscrew the two bolts located at the bottom of the brake. Slightly move the brakes forward or backward and then retighten the bolts.

ATTENDANT BRAKES

SteelMan SUPERB wheelchair is equipped with drum brakes designed for the accompanying person to slow down the wheelchair (especially when going downhill) or to completely lock the rear wheels. The brake lever is mounted on the push handle. To slow down the wheelchair, press the brake lever upward with both hands simultaneously (Picture 41). To fully lock the rear wheels, when the brake lever is in the upper position, push down the locking clip located below the brake lever (Picture 42). The braking intensity of the rear wheel can be adjusted by varying the tension of the brake cable. The more the tension screw is loosened, the stronger the braking force becomes.

SEAT AND BACKREST HEIGHT ADJUSTMENT

Thanks to front and rear wheels, you are able to adjust seat height. In order to adjust the height of the front caster, loosen the screw with an Allen key, pull it out, set the caster to the desired height, and tighten it again (Picture 43). In order to adjust the height of the rear wheel, loosen the axle screw using a spanner and set the rear wheel in the wheel adapter. After set in place, tighten it again.

For safety reasons, we recommend to keep front and rear wheels in the same height. The wheelbase can be extended by turning the adapter by 180°. This increases the stability against tipping backwards.

Please note, that rear wheel must be removed while making adjustment of adapter.

Backrest Height Adjustment

The backrest can be relocated vertically to match the user's posture and seating needs. To adjust: unscrew the four mounting screws securing the backrest brackets (Picture 45). Once loose, slide the backrest upward or downward to the desired height (Picture 46). Then retighten the screws to secure the backrest firmly in the new position.

Always ensure the screws are fully tightened before using the wheelchair. Do not adjust the backrest while the user is seated.

REAR WHEELS

The rear wheels are attached using a quick-release axle. To remove a wheel, hold the wheelchair steady with one hand, take the wheel with the other hand while pressing the quick release axle located in the center of the wheel (Pictures 47 - 48). While keeping the axle pressed, pull the wheel towards you. To reattach the wheel, repeat the steps in reverse order.

LEGREST OPERATION

The legrests are supplied in a separate cardboard box and should be unpacked before installation. Attach them to the front of the wheelchair by inserting the plastic mounting post into the metal bracket. Then, rotate the legrest inward until it clicks securely into place (Pictures 7 - 9).

i Do not mix up the right and left sides!

Each legrest includes padded calf supports for enhanced stability and support (Picture 6). These legrests feature an adjustable angle to provide proper leg positioning and improved comfort (Pictures 49 - 50). The legrest length can also be adjusted as required. Loosen the bolt using the provided tools, set the preferred height, and retighten the bolt securely (Pictures 51 - 52).

HEADREST

The wheelchair is equipped with a multifunctional headrest that can be adjusted to suit the user's posture and support needs. The headrest offers vertical height adjustment, forward/backward (depth) adjustment, angle adjustment, and side cushion width adjustment (Pictures 15 - 18).

All adjustments are made using the adjustable ratchet levers (Picture 19). These levers allow tightening and loosening even in confined spaces. When the lever reaches an obstacle (such as the headrest cushion), pull the lever outward to disengage the internal serrations, reposition the lever to a free angle, and release it. The serrations automatically re-engage, allowing you to continue tightening or loosening the mechanism without rotating the screw itself.

This system ensures precise and secure adjustment of every headrest component.

SEAT AND BACKREST ANGLE ADJUSTMENT

With the help of gas springs, you can adjust the seat and backrest angles. The seat tilt angle is adjustable up to 20° (-5° - 25°), and the backrest angle up to 35° (90° - 125°). These functions are controlled by angle control levers mounted on the push handles. Seat adjustment is operated by the blue handle (Picture 53), and backrest adjustment by the red handle (Picture 54).

SEAT DEPTH AND WIDTH ADJUSTMENT

The seat depth is adjustable and can be locked in any position within a 40 - 50 cm range. To change the seat depth, remove the seat cushion and loosen the four bolts located under the seat. Hold the seat with one hand, slide it forward or backward to the desired position, and then retighten the bolts securely.

The seat width can be increased from 44 cm to 49 cm by moving the armrests outward (each armrest can be adjusted by 2.5 cm). To widen the seat, loosen the bolt located under the seat, slide the armrest outward, and secure it (Picture 44). Repeat the same adjustment on the opposite side. It is recommended to keep both armrests positioned evenly.

The legrest holders must be adjusted using the same principle. Loosen the Allen bolt, slide each legrest holder outward to match the increased seat width, and retighten the bolt. Ensure that both sides are adjusted evenly.

ADDITIONAL PARTS

SIDE SUPPORTS

The side supports are mounted on both sides of the backrest frame and can be positioned according to the user's needs (Pictures 22 - 24). Each support allows length and angle adjustment, ensuring proper lateral stabilization. For easier transfers, the supports can be rotated outward by 180°, providing clear access from the side.

All adjustments and locking functions are performed using the adjustable ratchet levers, which allow tightening even in confined spaces. If a lever reaches an obstruction, pull it outward to reposition it, then continue tightening or loosening as needed.

ABDUCTOR

The abductor is mounted at the front of the wheelchair and is used to support proper leg positioning (Pictures 25 - 27). It can be adjusted horizontally and vertically, allowing it to be individually adapted to the user's needs. Once positioned, all adjustments are secured using the ratcheting clamping lever, which permits tightening even in confined spaces.

MAINTENANCE

To ensure safe and reliable operation, carry out the following checks and maintenance regularly or have it carried out by another person. Below You will find recommended maintenance's schedule.

	Before use	Once a week	Once a month	Once a year
Check that rear wheels are seated correctly		X		
Visual check		X		
Check the folding mechanism			X	
Check the brakes	X		X	
Clean the front casters			X	
Have wheelchair checked by an authorized dealer				X

WARNING

Repair, cleaning, or adjustment of wheelchair should be performed by authorized service person while person is not sitting in a wheelchair.

CLEANING AND DISINFECTION

Use the moist cloth to clean frame as well as plastic parts. The use of a mild detergent for more stubborn dirt is possible as well. The wheels can be cleaned with a wet synthetic fiber brush (the use of wire brush is not allowed). Seat and back pads can be washed off. Use mild detergent. Use a common non aggressive disinfectant. If you are unsure if the one used by you is suitable for disinfection, please get in touch with local dealer.

Sand and seawater can damage the bearings and steel parts can rust if the surface is damaged. Only expose the wheelchair to sand and seawater for short periods and clean it after every trip to the beach.

The wheelchair may be disinfected by spraying or wiping with tested, approved disinfectants.

Spray a soft cleaning and disinfecting product and follow the instructions given by the manufacturer.

STORAGE

The wheelchair should be stored in a dry environment, away from direct sunlight. When in storage the wheelchair can be folded up. The wheelchair should be kept clean and dust free. This can be done with a duster or damp cloth and mild detergent. Clean regularly, use a general purpose, neutral detergent and warm water. Please check the suitability of the cleaners or detergent before use. Do not use abrasive cleaners as this will permanently damage the surface and invalidate any warranty.

WARNING

The wheelchair must not be permanently exposed to temperatures greater than +40°C. Otherwise the frame as well as seat, backrest and armrests can heat up, which might cause burns.

WARNING

Never store other items on top of the wheelchair.

After long-term storage (more than 3 months) the wheelchair must be inspected (see chapter “MAINTENANCE”).

ENVIRONMENTAL CONDITIONS

Keep wheelchair under these conditions:
Temperature limitation -20°C to +40°C;
Relative humidity 20% to 75%;
Atmospheric pressure 800 hPa to 1060 hPa.

RE-USE

The wheelchair is suitable for re-use. The following actions should be taken:
Cleaning and disinfection (see chapter “CLEANING AND DISINFECTION”);
Inspection (see chapter “MAINTENANCE”).

The wheelchair should be adapted for the new user in accordance with the service documentation.

DISPOSAL

Be environmentally aware and dispose of your wheelchair properly. Disposal is subject to national and local regulations. To have wheelchair properly recycled, please contact your local community, disposal facility or our service department.

i All materials used are protected against corrosion. We use only REACH compliant materials and component.

INCIDENTS

Any serious incident that has occurred in relation to the device should be reported to UAB Kasko Group | Kid-Man® and the competent authority of the Member State in which the user and/or patient is established.

CONTACTS:

Manufacturer: UAB “Kasko Group” (registered trade mark Kid-Man®)

Address: A.Vivulskio 12D, Vilnius, LT-03221, Lithuania

Phone: +370 5 265 0000

E-mail: info@kid-man.com

Website: www.kid-man.com

SPECIFICATIONS

Model	SteelMan SUPERB		
REF	04-055-3/39	04-055-3/44	04-055-3/49
Seat width (cm)	39	44	49
Weight (fully assembled) (kg)	41.3	43.3	45.3
Overall length (cm)	122	122	122
Overall width (cm)	61	66	71
Overall height (cm)	127.5 - 140	127.5 - 140	127.5 - 140
Seat depth (cm)	46 - 56		
Seat height (cm)	59 - 64		
Backrest height (cm)	48.5 - 61		
Length of legrest (cm)	42 - 71		
Armrest height (cm)	24 - 30		
Front caster size (cm)	17.5 x 4.4		
Rear wheel size (cm)	60 x 3		
Seat angle	-5° - 25°		
Backrest angle	90° - 125°		
Weight of legrests (kg)	2.9		
Weight of cushion (kg)	0.6		
Maximum safe load (kg)	135		
Frame	Steel frame with powder coating		
Upholstery	Dartex® (PU-coated, breathable, water-resistant)		

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