
**TUALETO PAAUKŠTINIMAS
PAAUGSTINĀTS TUALETES SĒDEKLIS
TUALETIISTME KÖRGENDUS
RAISED TOILET SEAT
ВОЗВЫШЕНИЕ ДЛЯ ТУАЛЕТНОГО СИДЕНИЯ
INALTATOR WC
TOALETNÍ NÁSTAVEC**

**02-7060S4
02-7060S6**

**02-7060L4
02-7060L6**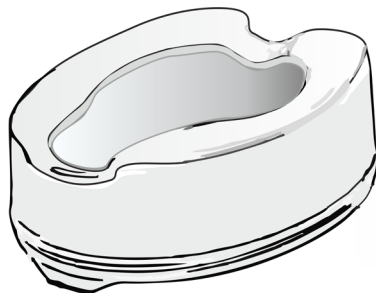

Naudojimo ir priežiūros instrukcija | Lietošanas un kopšanas instrukcija | Kasutusjuhend
User manual | Руководство по эксплуатации | Manual de utilizare | Návod k použití

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
EN INDICATIONS


• Kid-Man raised toilet seats are designed to provide assistance with toileting. It fits most standard toilet seats raising their height by 4" (100 mm) or by 6" (150 mm). The comfortable contoured seat allows for even weight distribution and its wide frontal opening facilitates personal cleaning. Raised toilet seat secures firmly onto the existing toilet seat with adjustable side brackets. Fast and simple „no tool“ installation. Alternatively, raised toilet seats are available with lid to prevent spread of bad odour.

PARTS OF THE PRODUCT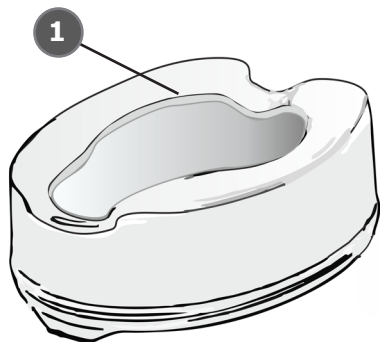

1. 1 x Raised toilet seat

2. 2x Locking lever

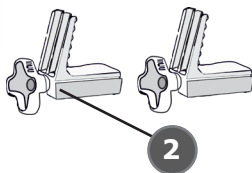
02-7060S4
02-7060S6

1. 1 x Raised toilet seat

2. 2 x Locking lever

3. Lid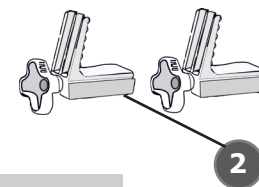


02-7060L4
02-7060L6

SET UP

Check all parts carefully before assembling the product. Do not assemble if any of the parts are damaged. Do not assemble if any of the parts are missing.

If you found all parts, you may then start installation. Before installation ensure that the existing toilet seat assembly is fitted securely and undamaged. Lift the lid of the existing toilet seat prior to installation of the raised toilet seat.

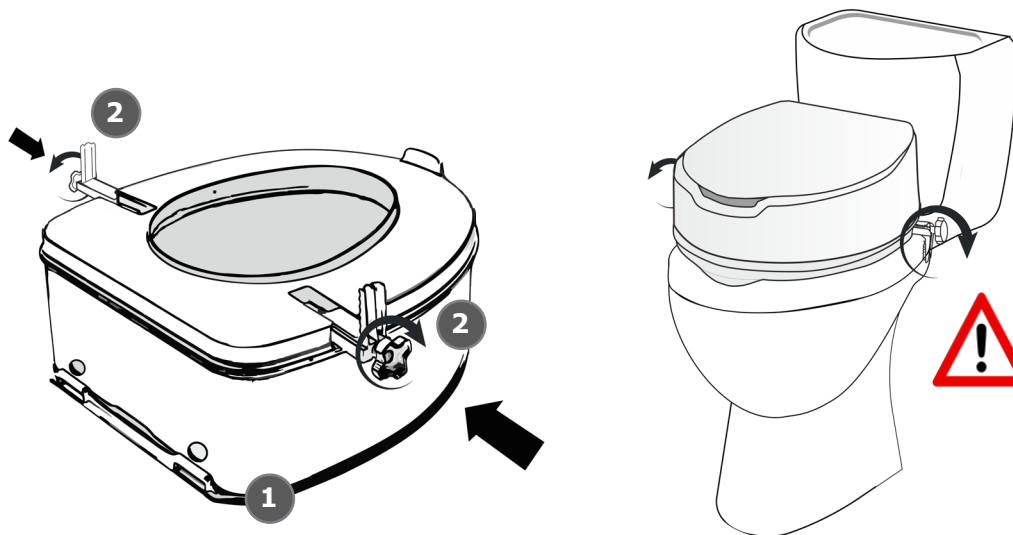Always ensure raised toilet seat is securely installed and fitted to the existing toilet by a competent person prior to use. Do not hold onto the seat as a support when getting on and off the toilet seat. Do not sit on top of the lid of the raised toilet seat. Inspect regularly for wear and damage as well as for secure fit.

CARE AND MAINTENANCE

• Clean regularly, use a general purpose, neutral detergent and warm water. Please check the suitability of the product before use. Do not use abrasive cleaners as this will permanently damage the surface and invalidate any warranty. Dry the seat thoroughly before use.

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EN We issue a warranty of 24 months from date of purchase. Any unauthorized modifications carried out without approval of the manufacturer will make the liability null and void. This includes exceeding the maximum user loads and using product in improper way. Warranty is only valid with fully filled warranty card. UAB „Kasko Group“ after getting claimed item from purchaser within period of 4 weeks decides whether issue is treated under warranty. If issue is not covered under warranty and customer agrees, UAB „Kasko Group“, if possible, fixes issue. All costs, including costs of transportation, should be covered by customer. Warranty repair does not extend warranty.

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